






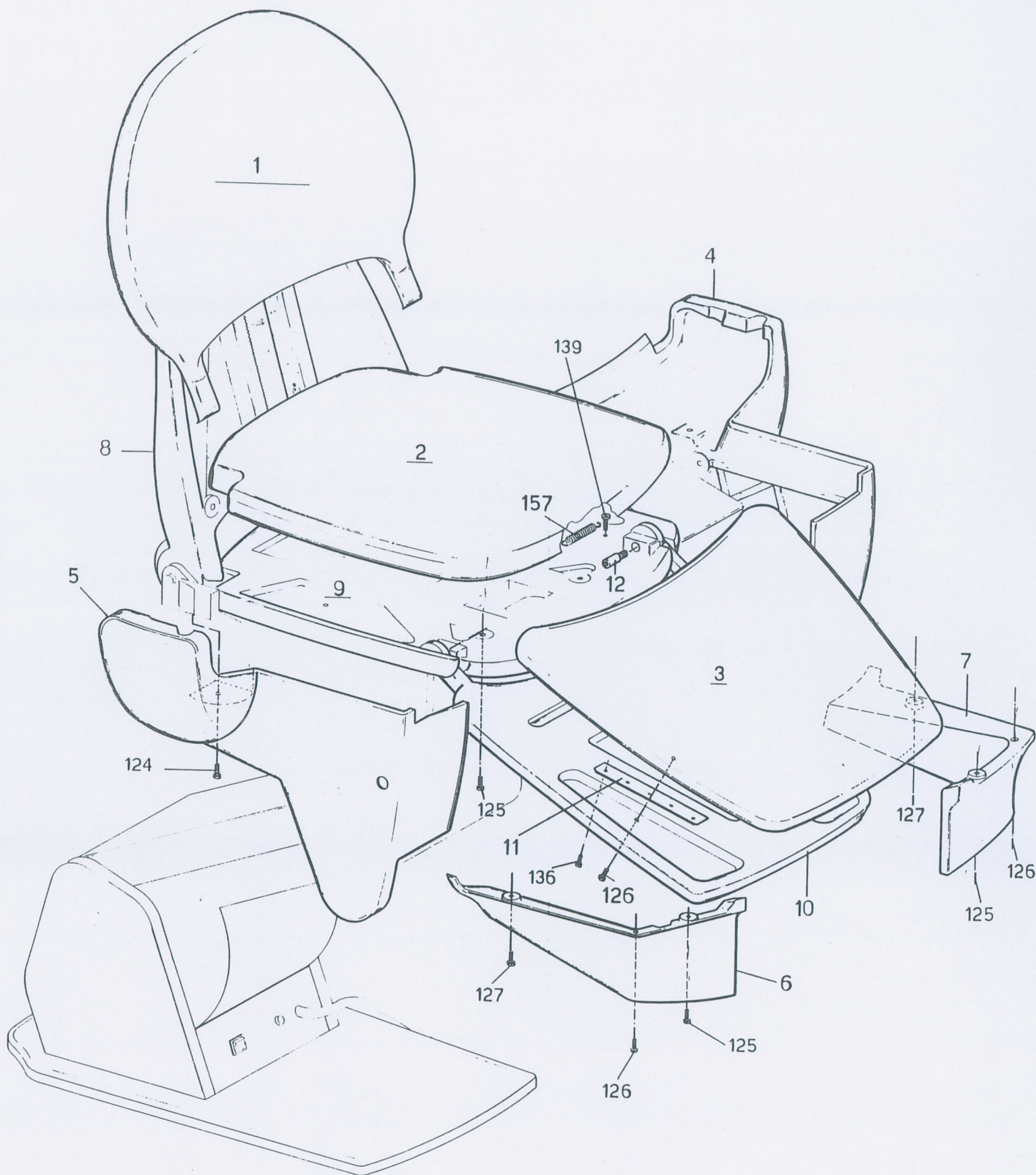


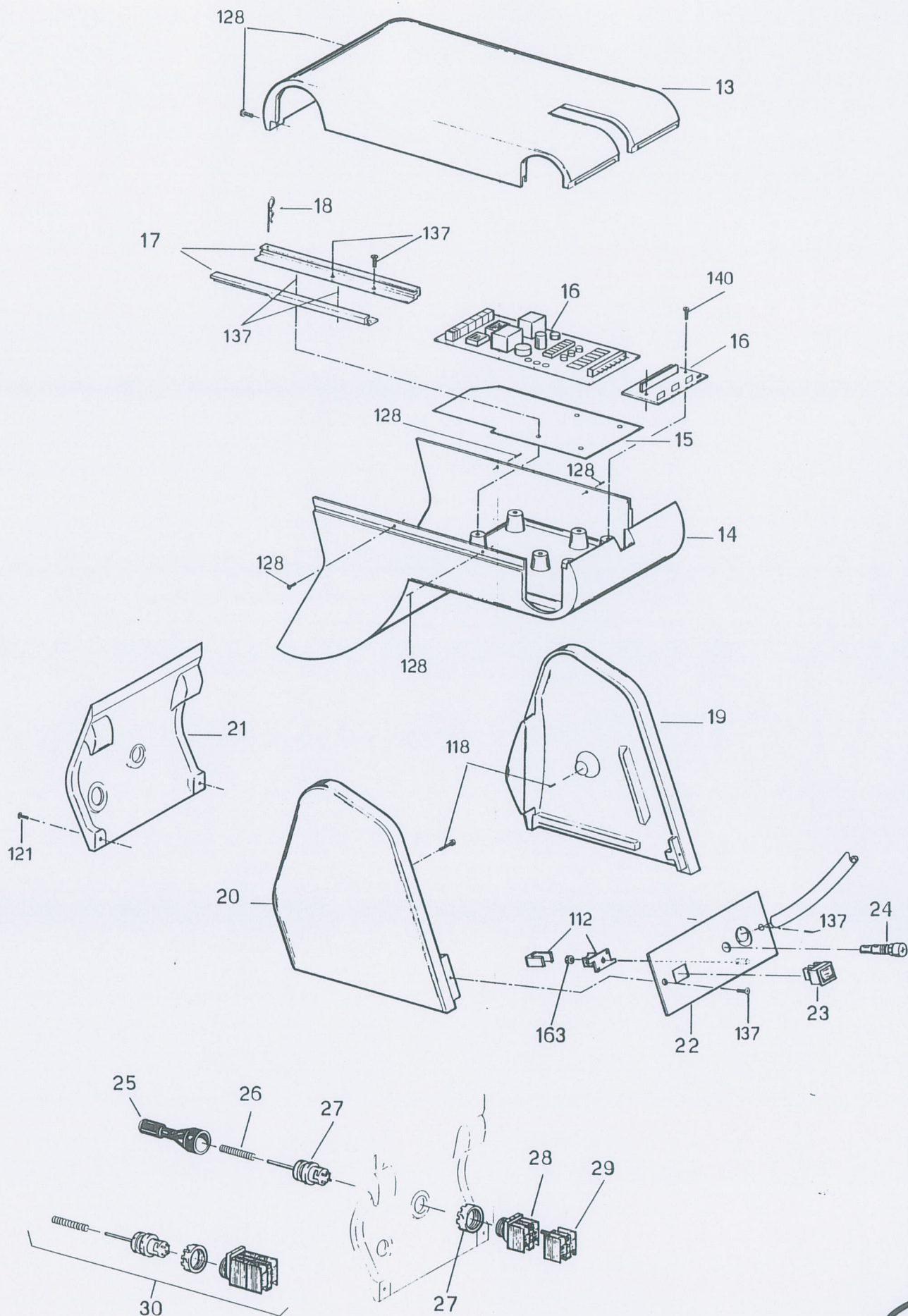

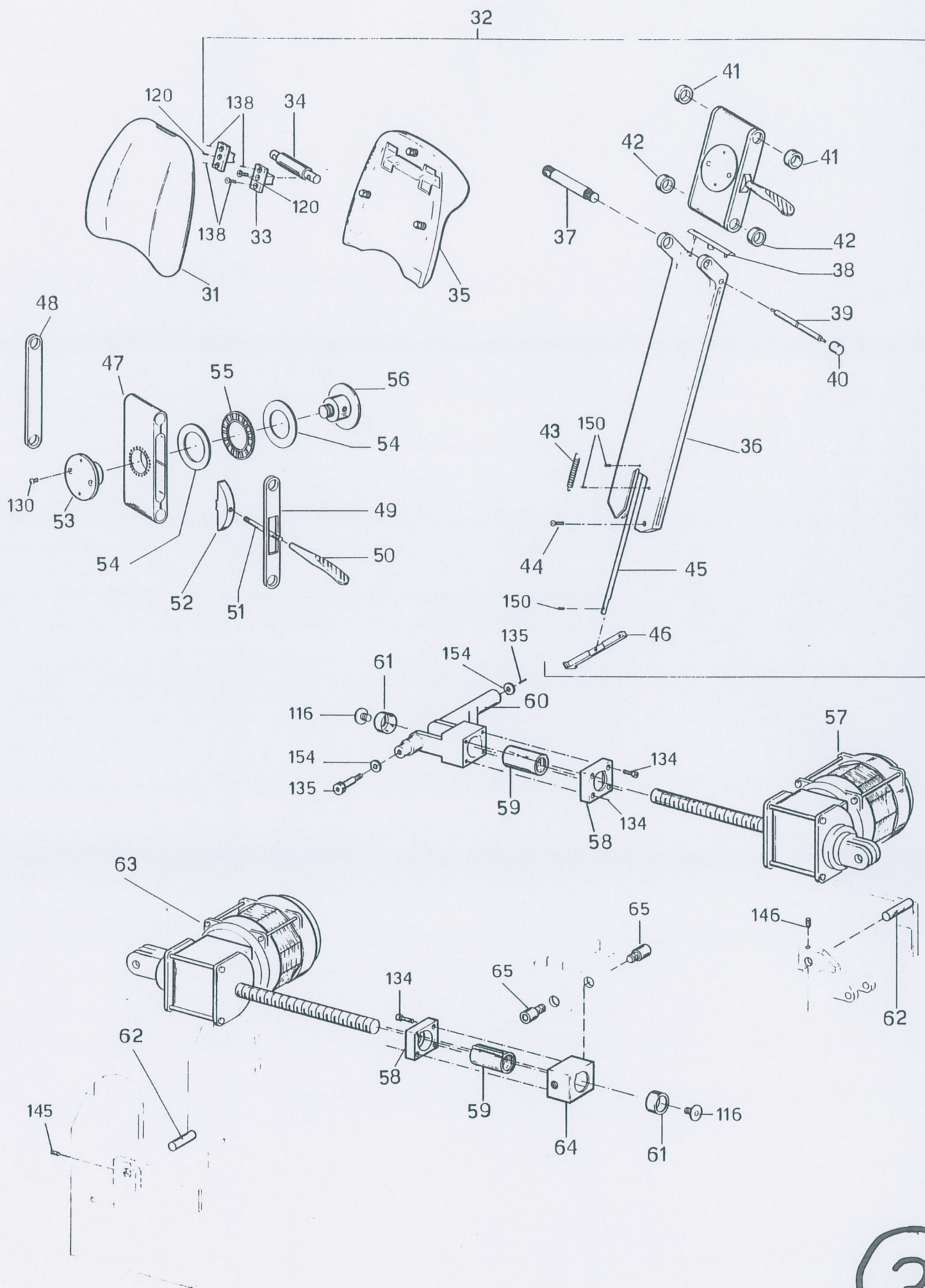

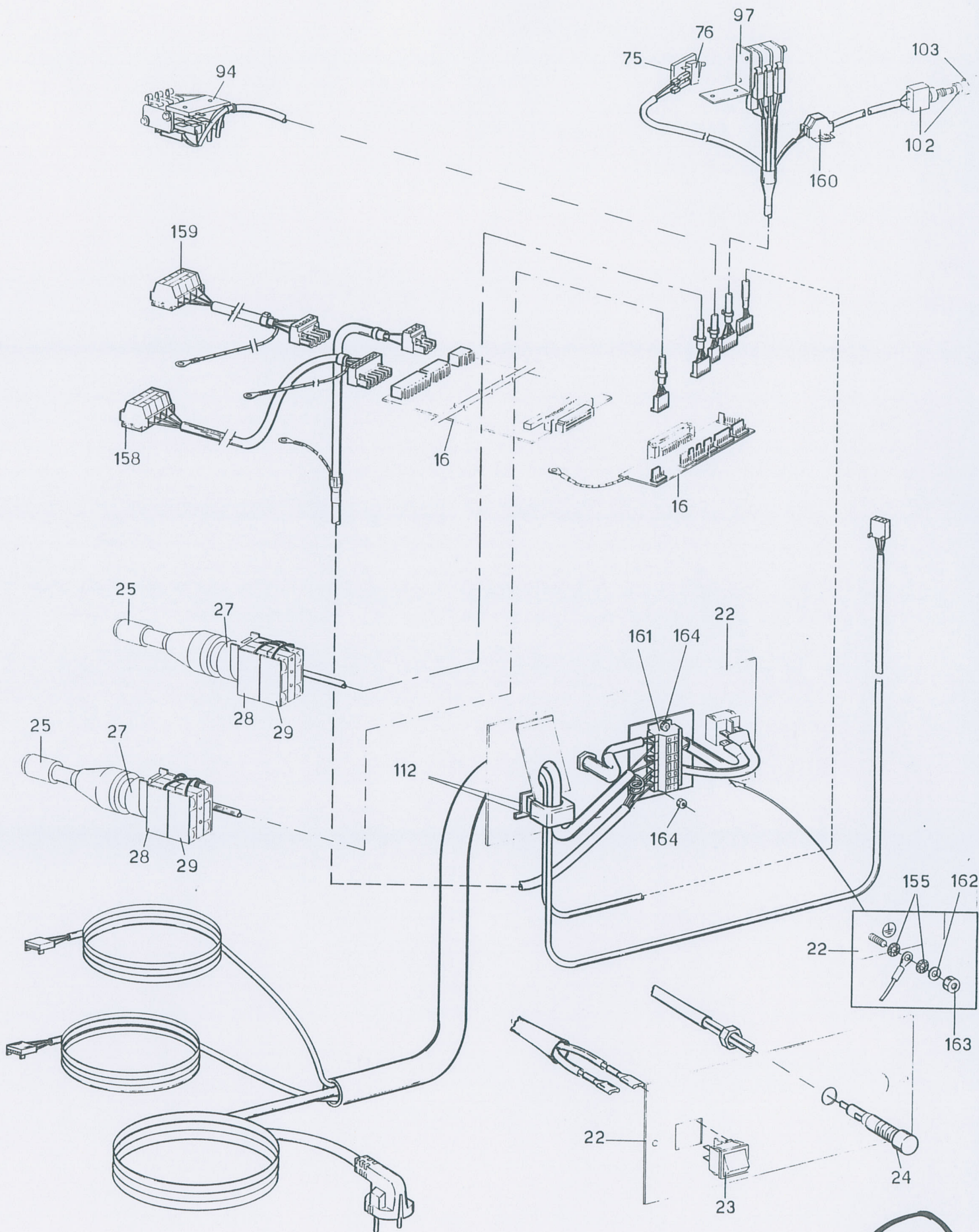

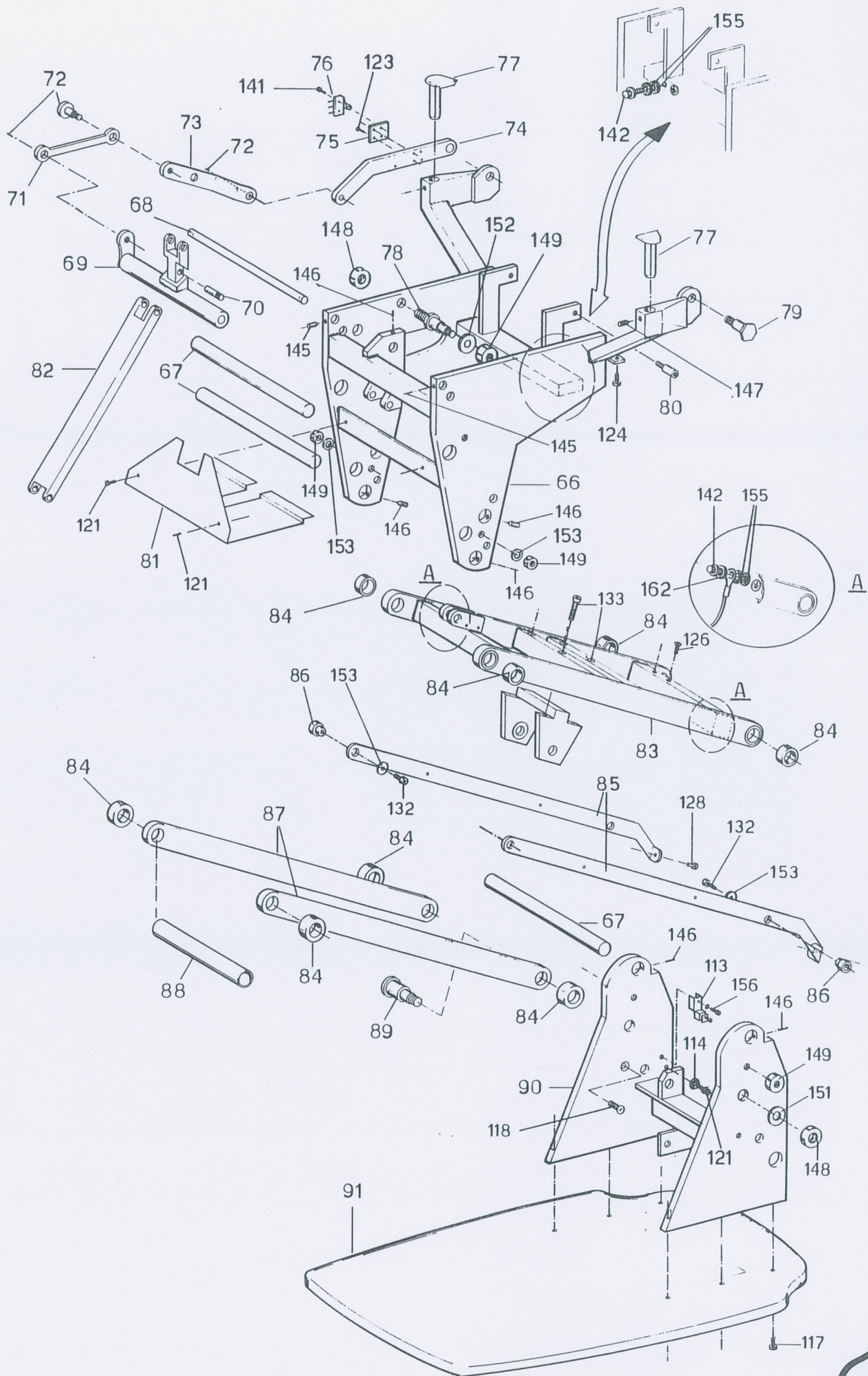

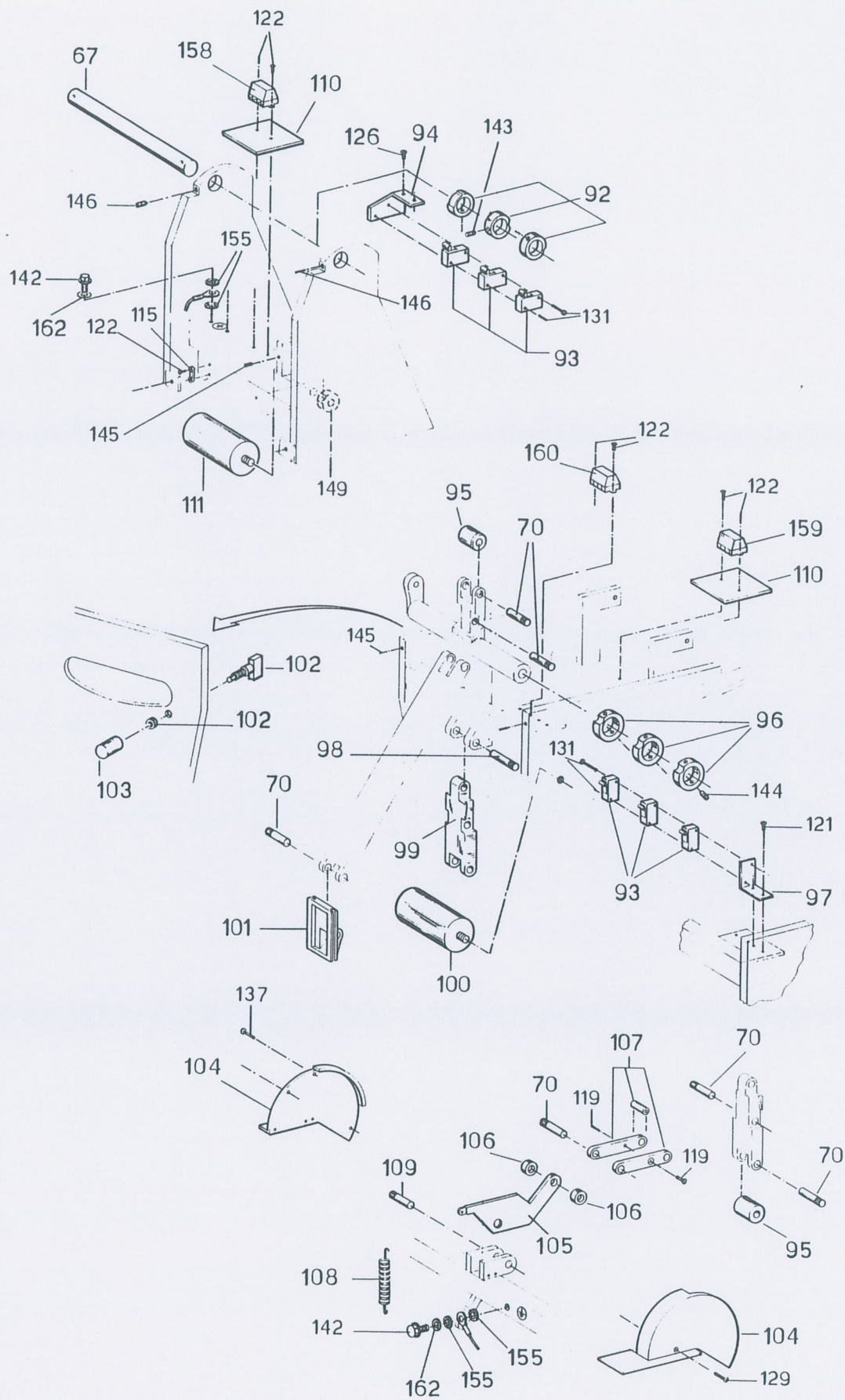

NR.	COOICE/CODE	DESCRIZIONE	DESCRIPTION	
00001	MD 852/A	TARPEZZERIA SCHIENALE INTEGRAL II	INTEGRAL II BACKREST UPHOLSTER	Polster
00002	MD 006/A	TARPEZZERIA SEDILE INTEGRAL II	INTEGRAL II SEAT UPHOLSTERY	—"
00003	MD 012	TARPEZZERIA APF/GAMBE INTEGRAL	INTEGRAL FOOTREST UPHOLSTERY	—"
00004	MC 847	CARTER SCALONE SX INTEGRAL II VERN.	LEFT SEAT COVER, PAINTED, INTE	V
00005	MC 848	CARTER SCALONE DX INTEGRAL II VERN.	RIGHT SEAT COVER, PAINTED, INT	V
00006	1095863	CARTER SOTTO SEDILE DX INTEGRAL II	RIGHT COUNTERSEAT COVER INTEGR	MC-875
00007	1095864	CARTER SOTTO SEDILE SX INTEGRAL II	LEFT COUNTERSEAT COVER INTEGRA	MC-876
00008	MC 851	CONTRO SCHIENALE INTEGRAL II VERN.	COUNTERBACKREST INTEGRAL II	UC-709
00009	MD 005/A	CONTRO SEDILE INTEGRAL II VERN.	COUNTERSEAT INTEGRAL II	UC-710
00010	MD 010	CONTRO APF/GAMBE INTEGRAL VERN.	COUNTERFOOTREST INTEGRAL II	UC-257
00011	1095803	GUIDA SLITTA APF/GAMBE INTEGRAL	SLIDE GUIDEWAY OF INTEGRAL LEG	MC-983
00012	MC A/8	PERNO SNODO SEDILE INTEGRAL ZINC.	SEAT HINGE PIN INTEGRAL	
00013	MC 712	CARTER BRACCIO SUP. INTEGRAL II VERN.	UPPER LIFTING ARM COVER, INTEG	V
00014	MC 721	CARTER BRACCIO INF. INTEGRAL II VERN.	LOWER LIFTING ARM COVER INTEGR	V
00015	1095904	PIASTRA ISOLANTE SCHEDE INTEGRAL II	INTEGRAL II PLATE INSULATING P	MC-898
00016	MD 427	SCHEDE INTEGRAL II A 220V	INTEGRAL II ELECTRONIC PLATE	V
00017	1095905	GUIDE DX E SX SCHEDE INTEGRAL II	INTEGRAL II PLATE RIGHT AND LE	MC-856 (MC-857) and
00018	1095906	COPPIGLIA DI SICUREZZA ART. 20259	SAFETY PIN	
00019	MC 714L	CARTER LATERALE SX INTEGRAL II VERN.	PAINTED INTEGRAL II LEFT SIDE	MC-715
00020	MC 714R	CARTER LATERALE DX INTEGRAL II VERN.	PAINTED INTEGRAL II RIGHT SIDE	MC-714
00021	MC 720	CARTER COMANDI INTEGRAL II VERN.	FOOTCONTROL COVER, PAINTED, IN	V (MC-980 versione)
00022	MC A/5	CARTER ALIMENTAZ. INTEGRAL II VERN.	PAINTED MAIN COVER, INTEGRAL I	MC-978
00023	1095170	INTERRUTTORE GENERALE 6A BIANCO	WHITE MAIN SWITCH 6A	MC-242
00024	1095670	FORTAFUSIBILE 5X20	FUSE-HOLDER 5X20	SC-841
00025	MC 682	CAPPUCCIO MANIPOLATORE INTEGRAL II	FOOTCONTROL CAP	? ngl mv.
00026	MC 1(MO)	MOLLA MANIPOLATORE INTEGRAL II	FOOTCONTROL SPRING, INTEGRAL I	SD-472
00027	1097044	MECCANICA MANIPOLATORE A POSIZIONI	4 POSITIONS MECHANISM FOR JOYS	

NR.	CODICE/COE	DESCRIZIONE	DESCRIPTION	
00028	1095045	FRUTTO 080 EF20V 2NA	JOYSTICK BLOCK 080 EF20V 2NA	
00029	1095046	FRUTTO 080 B02V 2NC	JOYSTICK BLOCK 080 B02V 2NC	
00030	1095806	MANIPOLATORE E MOLLA INTEGRAL II	FOOTCONTROL AND SPRING OF INTE	SD-412
00031	MO 019	TAPPETTERIA APF/TESTA INTEGRAL	INTEGRAL HEADREST UPHOLSTERY	Polsben.
00032	1095499	APF/TESTA INTEGRAL II COMPLETO	COMPLETE HEADREST INTEGRAL II	MC-979-MC-991
00033	1095187	ATTACCO "T" APF/TESTA INTEGRAL ZINC	ZINC-PLATED "T" HEADREST LINK	
00034	1095188	PERNO FORATO APF/TESTA INTEGRAL	DRIILLED PIN, INTEGRAL HEADREST	
00035	1095805	CONTRO APF/TESTA INTEGRAL II	COUNTER/HEADREST INTEGRAL II	
00036	1095196	STAFFA APF/TESTA INTEGRAL	INTEGRAL HEADREST SWARD	
00037	1095189	PERNO ZIGZAGHATO APF/TESTA INTEGRAL	KHURLED PIN, INTEGRAL HEADREST	
00038	1095971	COPRI STAFFA INTEGRAL II	SWARD CAP	
00039	1095195	PERNO BLOCCAGGIO APF/TESTA INTEGRAL	LOCKING PIN FOR INTEGR. II HEAD	SD-416
00040	1095914	FORNELLO APF/TESTA INTEGRAL II	HEADREST KNOB INTEGRAL II	SD-380
00041	1095972	DIST. CILINDRICO A/TESTA INTEGRAL II	CYLINDRICAL SPACER FOR INTEGRA	
00042	1095973	DIST. CONICO A/TESTA INTEGRAL II	CONIC SPACER FOR INTEGRAL II	
00043	1095200	MOLLA APF/TESTA INTEGRAL	HEADREST MECHANISM INTEGRAL	SD-418
00044	1095228	VITE PERNO APF/TESTA INTEGRAL ZINC	CHROMIUM-PLATED LOCKING BOLT F	SD-421
00045	1095197	ASTA RINV.BLOCC. APF/TESTA INTEGRAL	LOCKING LEVER, INTEGRAL II HEA	SD-417
00046	1095198	LEVA BLOCC. APF/TESTA INTEGRAL ZINC	NICKELED LOCKING LEVER, HEADRE	SD-420
00047	1095397	MOLLONE APF/TESTA UNI CROMATO	CHROMIUMPLATED RELEASING BLOCK	
00048	1095401	CARTER MOLLONE SENZA ASOLA A/T UNI	RELEASING BLOCK COVER WITHOUT	
00049	1095402	CARTER MOLLONE CON ASOLA A/T UNI	RELEASING BLOCK COVER WITH HOL	
00050	1095407	LEVA BLOCCAGGIO APF/TESTA UNI	LOCKING HANDLE FOR UPHOLSTERED	SD SC-886
00051	1095420	PERNO LEVA BLOCCAGGIO A/TESTA UNI	LOCKING HANDLE PIN FOR UPHOLST	
00052	1095403	LUNETTA APF/TESTA UNI	HOLE-CAP FOR UPHOLSTERED UNI	
00053	1095306	VITE REGOLAZIONE APF/TESTA UNI CROM	CHROMIUMPLATED ADJUSTING SCREW	
00054	1095C18	KALLA AS 1730	BEAKING AS 17.30	

NR.	CODICE/COE	DESCRIZIONE	DESCRIPTION
00055	1095C01	GABRIA A RULLINI AXK 1730	HOUSING AXK 17.30
00056	1095395	VITE BLOCCAGGIO APP/TESTA UNI CROM.	CHROMIUMPLATED LOCKING SCREW F
00057	1096055	MOTORE SCHIENALE 230V INTEGRAL II	BACKREST MOTOR 230V.FOR INTEGR
00058	MC 709	ATT.MOT.PART. A-B INTEGRAL II ZINC.	NICKELED LINKAGE MOTOR PARTS A
00059	1095801	BUSSEOLA NYLON MOTORE INTEGRAL II	NYLON BUSHING FOR INTEGRAL II
00060	MC 823BIS	ATT.MOTORE SCHIEN. INTEGRAL II ZINC.	NICKELED LINKAGE OF BACKREST M
00061	MC A/B	ARRESTO MECC. MOTORE INTEGRAL II	MECHANIC STOPPING, INTEGRAL II
00062	MC 711	PERNO MOTORE INTEGRAL II	MOTOR PIN, INTEGRAL II
00063	1096054	MOTORE S/D 230V INTEGRAL II	LIFTING MOTOR 230V. FOR INTEGR
00064	MC 708	ATTACCO MOTORE S/D INTEGRAL II ZINC.	NICKELED LINKAGE, INTEGRAL II
00065	MC 710	VITE ATT. MOT. S/D INTEGRAL II ZINC.	SCREW OD LIFTING MOTOR LINKAGE
00066	MC 844	SCALONE INTEGRAL II ZINCATO GIALLO	PAINTED UPPER BODY INTEGRAL II
00067	MC 689	PERNO BRACCIO INTEGRAL II	INTEGRAL II LIFTING ARM PIN
00068	MC 736BIS	PERNO BILANCINO GAMBE INTEGRAL II	PIN FOOTREST LEVER, INTEGRAL I
00069	MC 846	BILANCINO GAMBE INTEGRAL II VERN.	PAINTED FOOTREST LEVER ROD/BAR
00070	MC 736	PERNO INTEGRAL II	INTEGRAL II PIN
00071	MC 838	LEVA MOV.GAMBE C INTEGRAL II ZINC.	PAINTED FOOTREST LEVER INTEGRA
00072	MC 842	BULLONE LEVA RINV. INTEGRAL II ZINC.	BURNISHED JUMP LEVER BOLT INTE
00073	MC 839	LEVA RINVIO INTEGRAL II ZINCATA	PAINTED INTEGRAL II JUMP-LEVER
00074	MC 822	LEVA MOV.GAMBE B INTEGRAL II ZINC.	PAINTED FOOTREST MOVEMENT LEVE
00075	MC 916	PIASTRINA PVC	PVC METAL PLATE
00076	1095703	MICROSWITCH UNIPOLARE XCS-S1	ELECTRICAL HEADREST UNIPOLAR
00077	1095911	TAPPO BRACCIOLO DX E SX INTEGRAL II	RIGHT AND LEFT ARMREST CAP OF
00078	MC 841	PERNO VITE LEV/RIN. INTEGRAL II ZINC.	BURNISHED SCREW PIN IF INTEGRA
00079	MC A/10	BULLONE ATT. SCHIEN. INTEGRAL ZINC.	BURNISHED BOLT OF BACKREST LIN
00080	MC A/7	PERNO ATT. SEDILE INTEGRAL II ZINC.	SEAT LINKAGE PIN, INTEGRAL II
00081	MC A/4	CARTER TRIANGOLO INTEGRAL II VERN.	PAINTED CENTER BOX COVER, INTE

5C-843

5D-353

5D-454

5C-842

5C-842

MC-736

MC-846

MC-738

MC-838

MC-842

MC-839

MC-822

5D-440

MC-840H-MC-942

MC-841

5D-506

5D-491

HR.	CODE/CO	DESCRIPTION	DESCRIPTION	
00082	MC 737	ASTA BILANC. GAMBE INTEGRAL II VERN.	RCG/BAR, FOOTREST LEVER, INTEG	MC-737
00083	MC 837	INSTRUMENT ASTE BRACC. INTEGRAL II ZINC	PAINTED FITTING ARM RCG/BARS,	
00084	MC 690/151	DIST. PANTOGRAFO INTEGRAL II D=22	INTEGRAL II PANTOGRAFI SPACER	
00085	MC 724	ASTE CARTER OX E SX INTEGRAL II ZIN	NICKELED EIGHT AND LEFT COVER	MC-724- MC-725
00086	MC 690/153	DIST. PANTOGRAFO INTEGRAL II	INTEGRAL II PANTOGRAFI SPACER	
00087	MC 699	ASTA INTEGRAL II ZINCATO	INTEGRAL II RCG/BAR	
00088	MC 690	DISTANZIALE ASTE INTEGRAL II ZINC	INTEGRAL II SPACER FOR RCG/BAR	
00089	MC 692	BULLONE ASTA INTEGRAL II ZINCATO	BURNISHED BOLT, INTEGRAL II	MC-692
00090	MC 943	CORFO INFERIORE INTEGRAL II ZINC	PAINTED LOWER BODY INTEGRAL II	
00091	MC 071/A	BASE INTEGRAL II	INTEGRAL II BASE	SD-428
00092	MC 037/A	CAMMA REG. S/D INTEGRAL II ZINCATO	NICKELED ADJUSTMENT CAM INTEGR	
00093	1095148	MICROSWITCH UNITOLARE XGB6-S20	UNITOLAR MICROSWITCH XGB6-S20	SC-849
00094	MC A/3	SUPPORTO MICRO S/D INTEGRAL II	UP/DOWN MICROSWITCH SUPPORT, I	SD-486
00095	1095907	ROLLO INTEGRAL II	INTEGRAL II ROLL	SD-473
00096	MC 846/A	CAMMA REG. SCHITALE INTEGRAL ZINC	NICKELED ADJUSTING CAM OF INTE	SD-484
00097	MC 826	SQUADRETTO MICRO INTEGRAL II	MICROSWITCH SQUARE, INTEGRAL I	SD-449
00098	MC 739	PERNO SNGO ASTE BILANC. INTEGRAL II	HINGE PIN, FOOTREST RCG/BAR,	
00099	MC 735	BRACCIO "C" INTEGRAL II VERNICCIATO	PAINTED INTEGRAL II "C" ARM	
00100	1095603	CONDENSATORE EFC3 12 MF 450V 36X72	CONDENSER 12MF (EFC3 450V 36X7	SD-413
00101	1095815	SLITTA APF/GAMBE INTEGRAL	LEGEST SLIDE INTEGRAL	MC-843
00102	1095626	PULSANTE (GRANDE) 8632/A	BIG PUSH-BUTTON	SD-423
00103	1095910	CAPPUCCIO PULSANTE INTEGRAL II	PUSHBUTTON CAP, INTEGRAL II	MC-755- MC-756
00104	MC 755	CARTER LEVA MOV. GAMBE INTEGRAL II	FOOTREST-LEVER CAP	MC-755- MC-756
00105	MC 817	LEVA MOVIA. GAMBE A INTEGRAL II ZINC	PAINTED "A" FOOTREST MOVEMENT	
00106	MC 833	DISTANZIALE INTEGRAL II	INTEGRAL II SPACER	
00107	MC 732/A	BRACC. MOV. GAMBE E DISTANZIALE VERN.	PAINTED INTEGRAL II FOOTREST	
00108	1095807	MOLLA INTEGRAL II (MC 2MO)	INTEGRAL II SPRING (MC 2MO)	

SD-480

NR.	COOICE/COOL	DESCRIZIONE	DESCRIZIONE
00109	MC 740	PERNO LEVA MOVIM. GAMGE INTEGRAL II I FIN, FOOTREST MOVEMENT LEVER,	
00110	MC A/2	SUPPORTO MORSET. MOTORE INTEGRAL II I TERMINAL-BOARD SUPPORT, INTEGR	
00111	1095600	CONDENSATORE KPC3 18 MF 450V 40X97	CONDENSER KPC3 18 MF 450V
00112	1013059	FERRACAVO PER INTEGRAL	INTEGRAL CABLE-FASTENER
00113	1095464B	MORSETTO MASSA PER INTEGRAL	TERMINAL GROUND FOR INTEGRAL
00114	1095464	OCCIELLO MASSA PER INTEGRAL	EYELET GROUND FOR INTEGRAL
00115	1095908	BLOCCO PER CAVO ALIMENT. INTEGRAL II I "DISQUIT", INTEGRAL II FOOTRES	
00116	1007618	VITE TPSC 10X16 ZINCATI BIANCA	SCREW TSCE 10X16
00117	1007017	VITE TPSC 8X45 ZINCATI BIANCA	SCREW TSCE 8X45
00118	1095275	VITE TPSC 6X20 ZINCATI BIANCA	SCREW TPSC 6X20
00119	1007108	VITE TPSC 5X14 ZINCATI BIANCA	SCREW TSCE 5X14
00120	1095A12	VITE TPSC 6X30 ZINCATI BIANCA	SCREW TPSC 6X30
00121	1007916	VITE TC CROCE 4X12 ZINCATI BIANCA	ZINC-PLATED CROSS SCREW TDC 4X
00122	1007135	VITE TC CROCE 3X12 ZINCATI BIANCA	SCREW TCC DTH 7905 3X12
00123	1007601	VITE TC CROCE 3X8 ZINCATI BIANCA	ZINC-PLATED CROSS SCREW TC 3X8
00124	1007990	VITE TPS CROCE 6X25 CARTER FLEX	CROSS SCREW TPS 6X25 FOR COVER
00125	1007913	VITE TPS CROCE 4X40 ZINCATI BIANCA	ZINC-PLATED CROSS SCREW TPS 4X
00126	1007910	VITE TPS CROCE 4X30 ZINCATI BIANCA	ZINC-PLATED CROSS SCREW TPS 4X
00127	1007912	VITE TPS CROCE 4X16 ZINCATI BIANCA	ZINC-PLATED CROSS SCREW TPS 4X
00128	1007209	VITE TPS CROCE 4X12 ZINCATI BIANCA	ZINC-PLATED CROSS SCREW TPS 4X1
00129	1007208	VITE TPS CROCE 3X8 ZINCATI BIANCA	ZINC-PLATED CROSS SCREW TPS 3X8
00130	1007138	VITE TPS CROCE 3X5 ZINCATI BIANCA	ZINC-PLATED CROSS SCREW TPS 3X5
00131	1007932	VITE TC CROCE 3X35 ZINCATI BIANCA	SCREW TPSC 8X16
00132	1007057	VITE TCCE 8X30 ZINCATI BIANCA	SCREW TCCE 8X30
00133	1007048	VITE TCCE 6X35 ZINCATI BIANCA	SCREW TCCE 6X35
00134	1007036	VITE TCCE 5X20 ZINCATI BIANCA	SCREW TCCE 5X20
00135	1007934	VITE TCCE 8-6X20 GAMPO RETTIFICATO	SCREW TCCE 8-6X20 UNBRAKO

SD-431

SD-504

NR.	CODICE/CODE	DESCRIZIONE	DESCRIPTION
00136	1007724	RAPID-FIX TPS CROCE 6X3/4 (3,9X19)	SCREW TPS RAPID FIX 3,9X19
00137	1007740	RAPID-FIX TPS CROCE 4X5/8 (3,2X16)	RAPID-FIX TPS CR. 4X5/8 (3,2X16)
00138	1007149	RAPID-FIX TPS CROCE 6X5/8 (3,9X16)	RAPID-FIX TPS CROSS 6X5/8 (3,9
00139	1007116	RAPID-FIX TC CROCE 7X3/4 (4,3X19)	RAPID-FIX CROSS 7X3/4 (4,3X19)
00140	1007503	RAPID-FIX TC CROCE 4X1/2 (3,2X13)	ZINCPLATED SCREW RAPID-FIX TC
00141	1007604	VITE AUTOF. TC CROCE 2X3/8 (2,2X9,5)	RAPID-FIX SCREW TC 2X3/8
00142	1007890	VITE FLANGIATA TE 5X10 RAMATA	COPPER PLATED FLANGED TE 5X10
00143	1007200	VITE STEI 5923 5X4 (GRANO P.F.) ZINC	GRAIN UHI 5923 5X4
00144	1007201	VITE STEI 5923 5X5 (GRANO P.F.) ZINC	GRAIN UHI 5923 5X5
00145	1007133	VITE STEI 5923 6X6 (GRANO P.F.) ZINC	GRAIN 6X6 UHI 5923
00146	1007206	VITE STEI 5923 6X10 (GRANO P.F.) ZINC	GRAIN 6X10 UHI 5923
00147	1007906	VITE STEI 5925 6X12 (GR.F.C11.) ZINC	SCREW STEI (GRAIN) 6X12
00148	1095C08	DADO 12X6X1,75 ZINCATO BIANCO	LOCKING-NUT M12X6
00149	1007230	DADO 8X5X1,25 ZINCATO (DOTAZIONE)	ZINC PLATED NUT 8X5X1,25
00150	1007732	SPINA ELASTICA 3X10	ELASTIC PLUG 3X10
00151	1011148	ROSETTA TRANC. OTTONE 12X24X1 GRZ.	BRASS WASHER 12X24X1
00152	1007904	ROSETTA TRANCIAITA OTTONE 8X17X0,6	ZINC-PLATED BRASS WASHER D 8X0
00153	1007094	ROSETTA D=8 ZINCATA BIANCA	WASHER D8
00154	1007228	ROSETTA IN FIERA 8,5X20X1	8,5X20X1 FIBRE WASHER
00155	1007503	ROSETTA DENTELLATA EST. D=5 RAMATA	INDENTED COPPER-PLATED WASHER
00156	1095464C	VITE PER MORSETTO MASSA INTEGRAL	TERMINAL GROUND SCREW FOR INTE
00157	MC 5(MO)	MOLLA PER CAVO INTEGRAL II	INTEGRAL II CABLE SPRING
00158	1096060	MORSETTIERA MOTORE S/D INTEGRAL II	LIFTING MOTOR TERMINAL BOX FOR
00159	1096061	MORSETTIERA MOTORE SCHIE. INTEGRAL II	BACKREST MOTOR TERMINAL BOX FO
00160	1096062	MORSETTIERA PULSANTE INTEGRAL II	PUSHBOTTON TERMINAL BOX FOR
00161	1096063	MORSETTIERA ALIMENTAZ. INTEGRAL II	SUPPLY TERMINAL BOX FOR INTEGR
00162	1007211	ROSETTA D=5 ZINCATA BIANCA	ZINCPLATED WASHER D5

50-504 " UC-437

50-501
50-502

50-503

-----		DESCRIZIONE		DESCRIZIONE		-----*	
NR.	CODICE/CODE						
00163	1007121	DADO 5X5X0,8	ZINCATO BIANCO	ZINCPLATED	NUT M 5X5		
00164	1007204	DADO 4X4X0,7	ZINCATO BIANCO	ZINCPLATED	NUT M4		