

XO[®] 4
RETROFIT XO HD DISPLAY,
WITH SHORT OR LONG ARM

YB-983
Version 2.10

XO CARE A/S
Usserød Mølle
Håndværkersvinget 6 +45 70 20 55 11
DK 2970 Hørsholm info@xo-care.com
Denmark www.xo-care.com



This instruction covers:

XO-616 Retrofit XO HD Display with long arm to fit XO 4 workstations supplied after 2005.
XO-619 Retrofit XO HD Display with short arm to fit XO 4 workstations supplied after 2005.

WARNING:

When making changes and handling electronics in the XO 4 Unit – always use ESD antistatic wristband (UC-600).




Disconnect the OP lamp cables in the Unit stand. Remove the lamp arm.



Remove the outer tube cover.



Remove the upper bushings and O-rings.

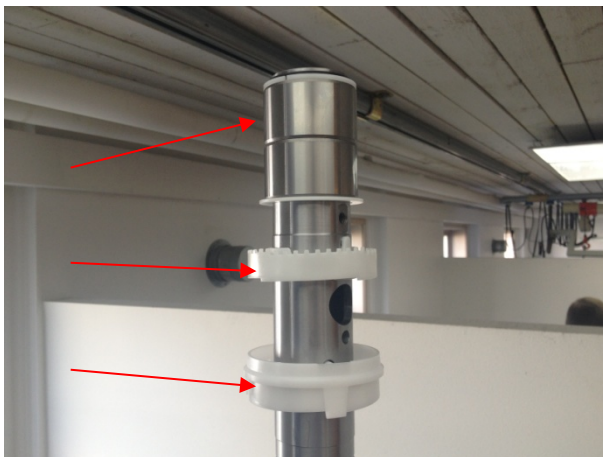

You are now ready to install the new XO HD display incl. long arm.
See Installation Guide YB-990 for further details.