


XO® 4 Sparepart Catalogue

YB-411 Ver. 2017-06-05

Note:

When ordering cables (CA-xxx), please see
part numbers at block schematic YB-600 next page.

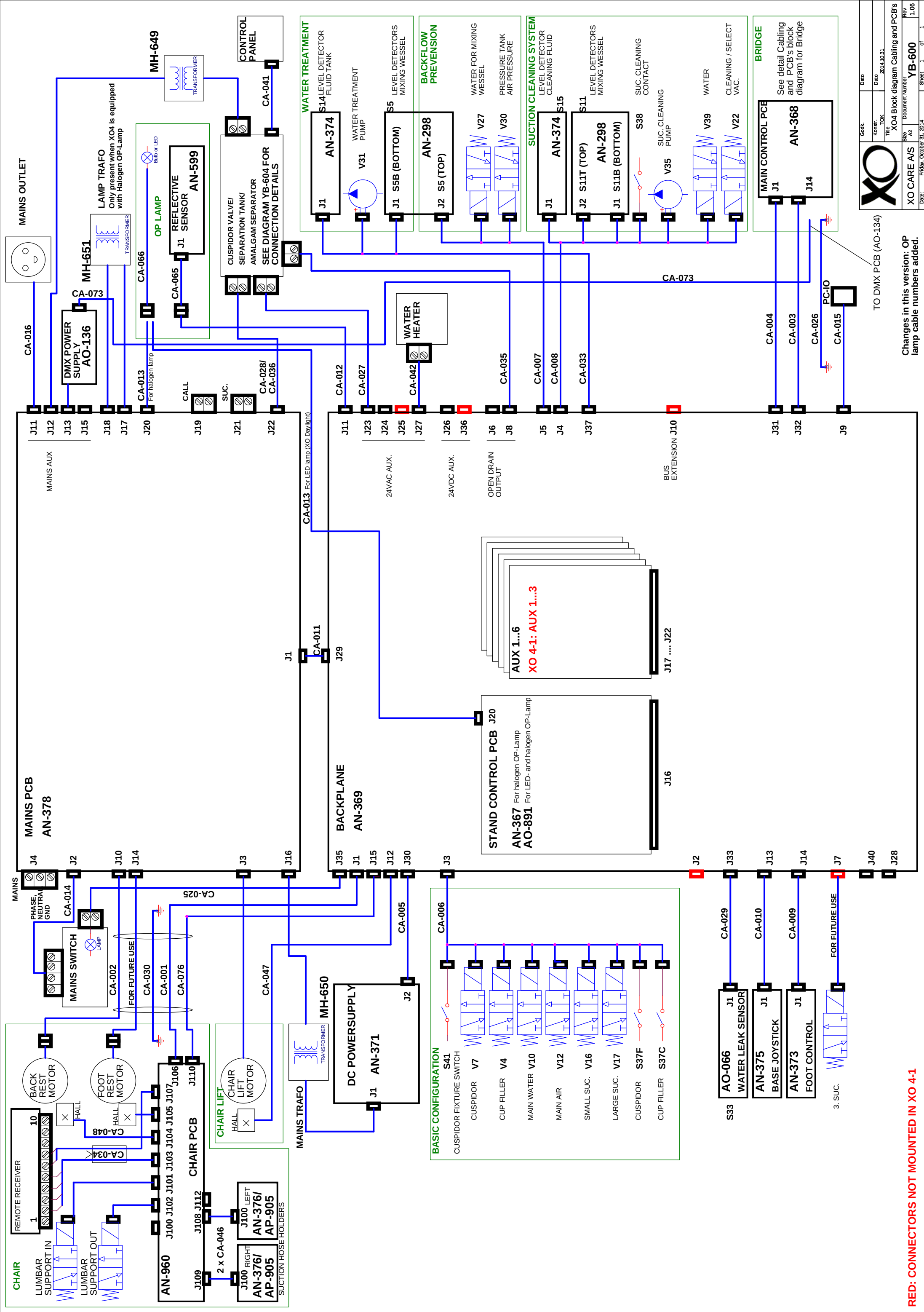

RED: CONNECTORS NOT MOUNTED IN XO 4-1

Changes in this version: OP
lamp cable numbers added.

		Godk.	Dao
		Konstr.	Dao
XO CARE A/S		Title	2014.10.31
Size	A2	Document Number	YB-600
Date:	Friday, October 31, 2014	Sheet	1 of 1

Explanation of "CTBS" – **C**olor **T**o **B**e **S**pecified


Please use below conversion list when "CTBS" is noted in the spare part catalogue part list.

XO Color ID number	XO Description
	XO 4 UNIT color
XB-931	Traffic White (RAL 9016-20)
XB-932	Midnight Black (RAL 9011-20)
XB-933	Bright Aluminium (RAL 9006-20)
XB-934	Calm Beige (RAL 1015-20)

	FABRIC color
XB-150	Black Grey (RAL 7021)
XB-151	Mouse Grey (RAL 7005)
XB-152	Coral Red (RAL 3016)
XB-153	Ultra-Marine Blue (RAL 5002)
XB-154	Fern Green (RAL 6025)
XB-155	Chocolate Brown (RAL 8017)

Example:

Bridge, top cover pos. 17 in Traffic White (previously called Snow white). Please order part no. AP-646 / XB-931 where XB-931 is the color ID number from the table above.

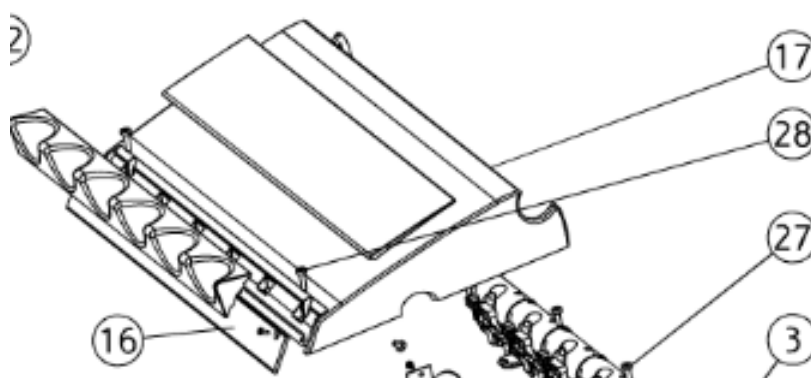

17	AN-463		Bridge, top cover, (XO-901 Sea Blue Metallic)
17	AN-950		Bridge, top cover, (XO-905 XO White,(RAL7047-30))
17	AO-288		Bridge, top cover, (XO-906 XO Titanium)
17	AN-734		Bridge, top cover, (XO-903 Silver Metallic)
17	AP-646		Bridge, top cover, (CTBS)
17	AN-462		Bridge, top cover, (XO-900 XO Grey, RAL 7047-80)

Explanation of "CTBS" – **C**olor **T**o **B**e **S**pecified

The actual colors on the unit and patient chair can be seen at the product label at the unit stand. See example below:

Product label at the unit stand:


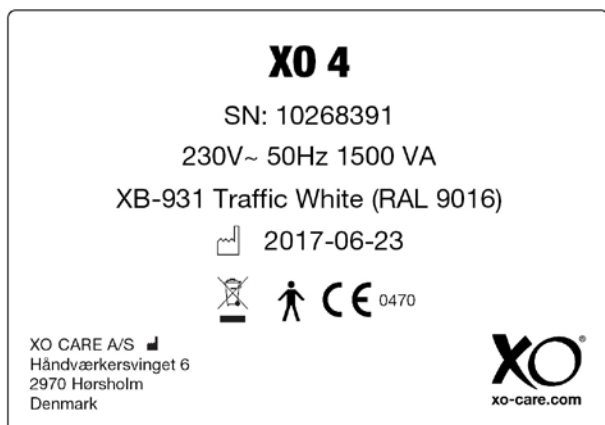


Table of content

XO 4 Stand.	1
XO 4 Stand, suction	59
XO 4 Unit stand, system for suction clean	83
XO 4 Unit stand, system for water clean	87
XO 4 Valve System and Instrument Tubes	93
XO 4 Instrument Bridge	129
XO 4 Arm system for Instrument Bridge	175
XO 4 Peristaltic Pump	205
XO 4 Foot Control	219
XO 4 Patient Chair and Lifting System.	229
XO 4 Lamp and Arm system	297
XO 4 Screen and Arm system	339
XO 4 Operator's Stool	377
XO 4 Care, cleaning and tools	385

Table of content


XO 4 Stand

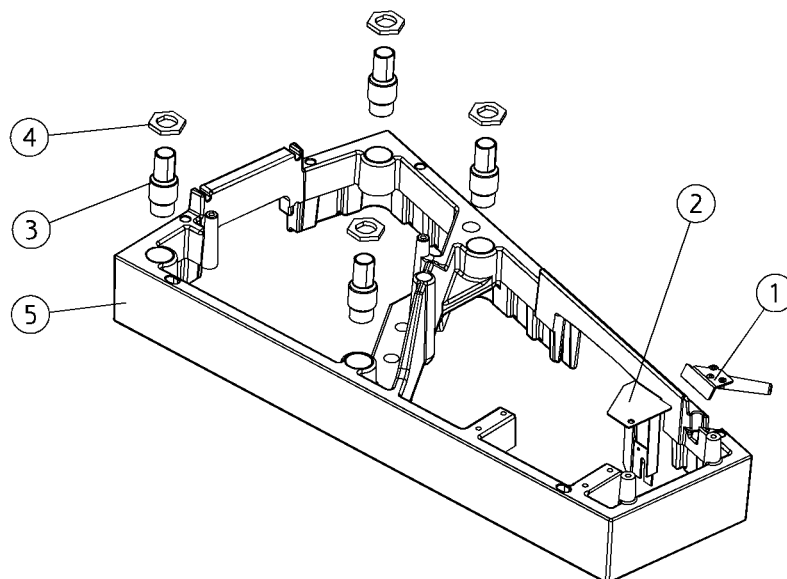


Position	Part NO.	Assembly	Description
1	AN-058	Page 3	Cable holder, foot control
2	KT-053		Upgrade, water leak sensor XO 4
3	MG-010		Adjustment bush, socket
4	MG-019		Adjustment nut, socket
5	AO-717		Foot, (XO-908 (XO-908 Iron Grey (RAL 7011-5)))
5	AN-482		Foot, (XO-909 Iron Grey (RAL 7011-30))
5	AP-677		Foot, (CTBS)

AN-058

Cable holder, foot control

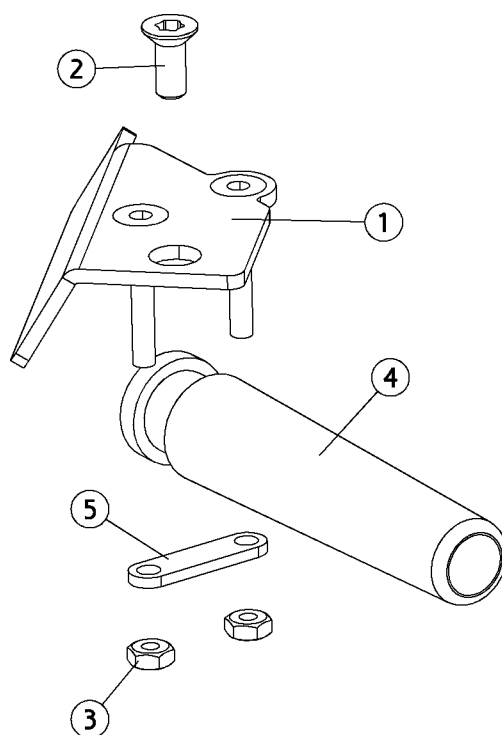


Position	Part NO.	Assembly	Description
1	MG-393		Not a spare part
2	UG-115		Screw, M4x010
3	UG-207		Nut, M3
4	UG-525		Not a spare part
5	UG-526		Not a spare part

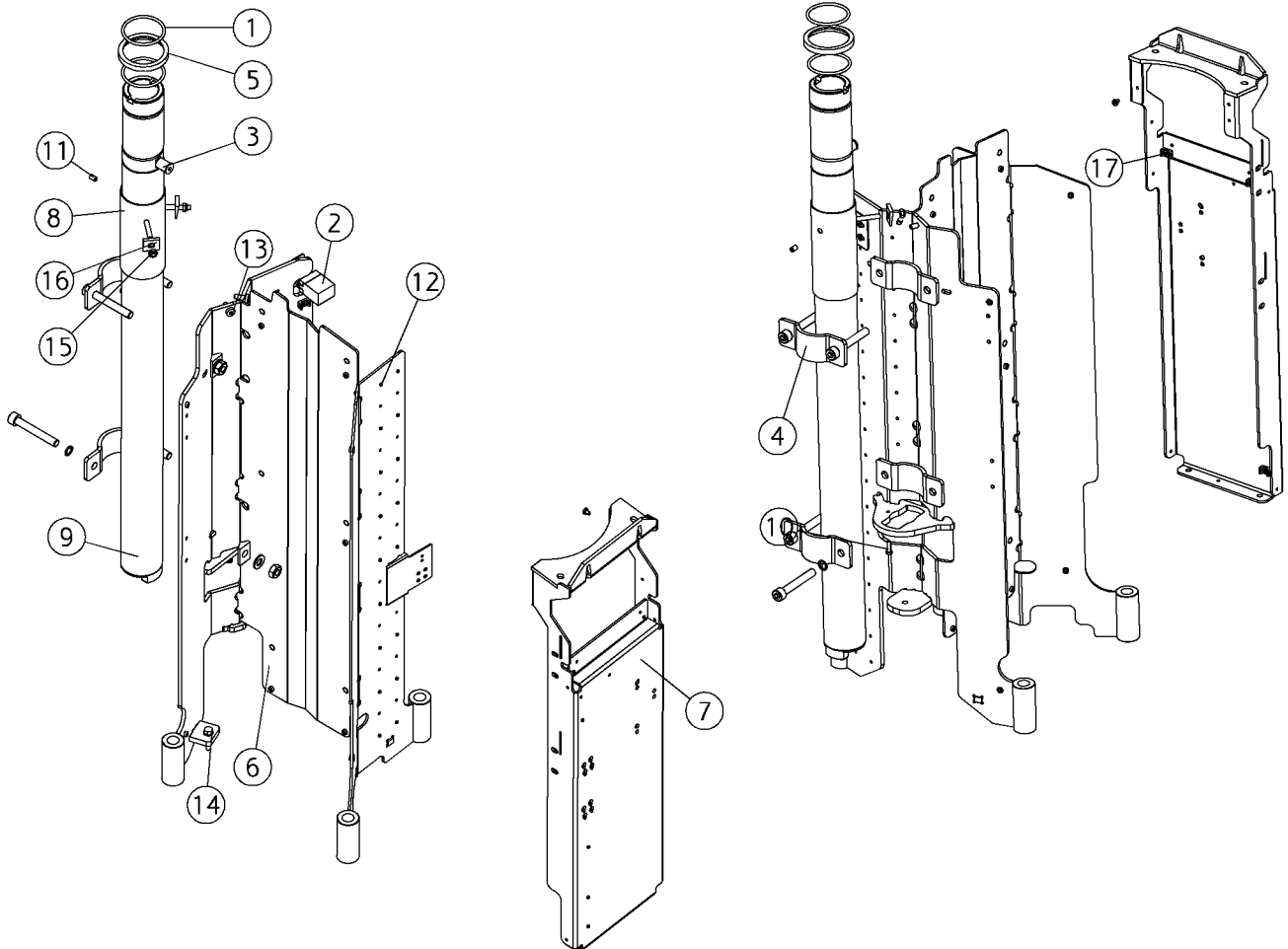

AN-054

Chassis stand


Position	Part NO.	Assembly	Description
1	000573		O-ring 50.34 x 3.53
2	AN-090		Main switch XO 4
3	AN-137	<i>Page 54</i>	Stop,stand tube, rear arm
4	AN-158	<i>Page 6</i>	Holder, stand tube
5	MG-003		Axial bearing, rear arm
6	MG-047		Not a spare part
7	MG-122		Not a spare part
8	MG-551		Adjustment, topplate
9	MG-560		Steel tube, chassis
10	UG-103		Screw, thread making, M4x16
11	UG-105		Pointed screw, M6x10
12	UG-111		Screw, thread making, M4x6
13	UG-113		Screw, thread making, M6x16
14	UG-156		Not a spare part
15	UG-206		Not a spare part
16	UG-251		Not a spare part
17	UG-530		Not a spare part

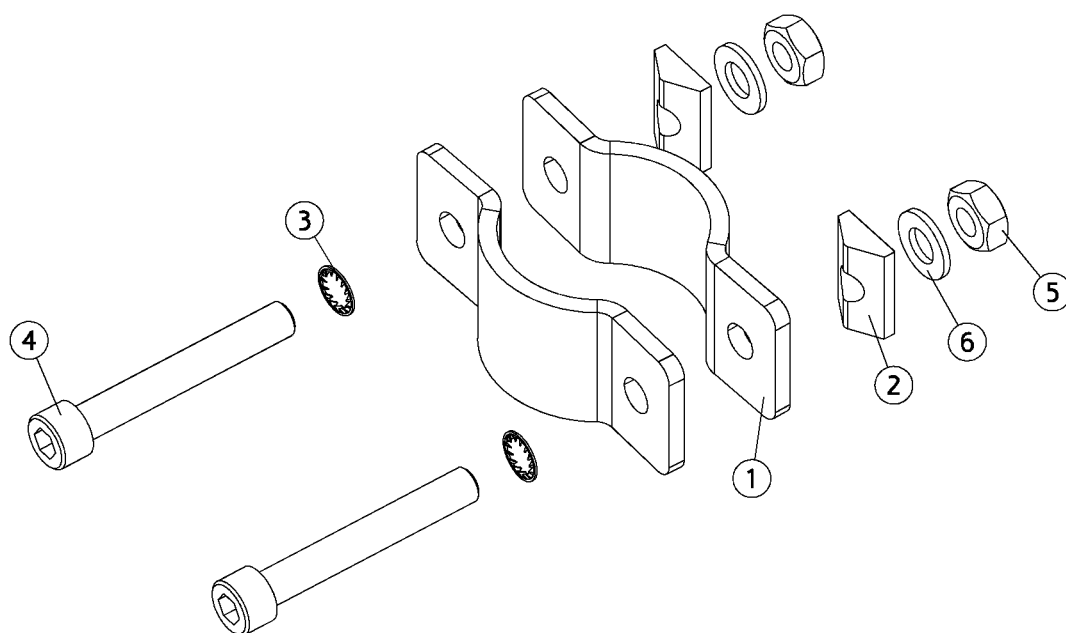

Position	Part NO.	Assembly	Description
1	MG-548		Not a spare part
2	MG-549		Not a spare part
3	UC-170		Not a spare part
4	UG-036		Not a spare part
5	UG-201		Not a spare part
6	UG-259		Washer Ø10.5xØ20.0x2.0

AN-060

Basic config., water/air

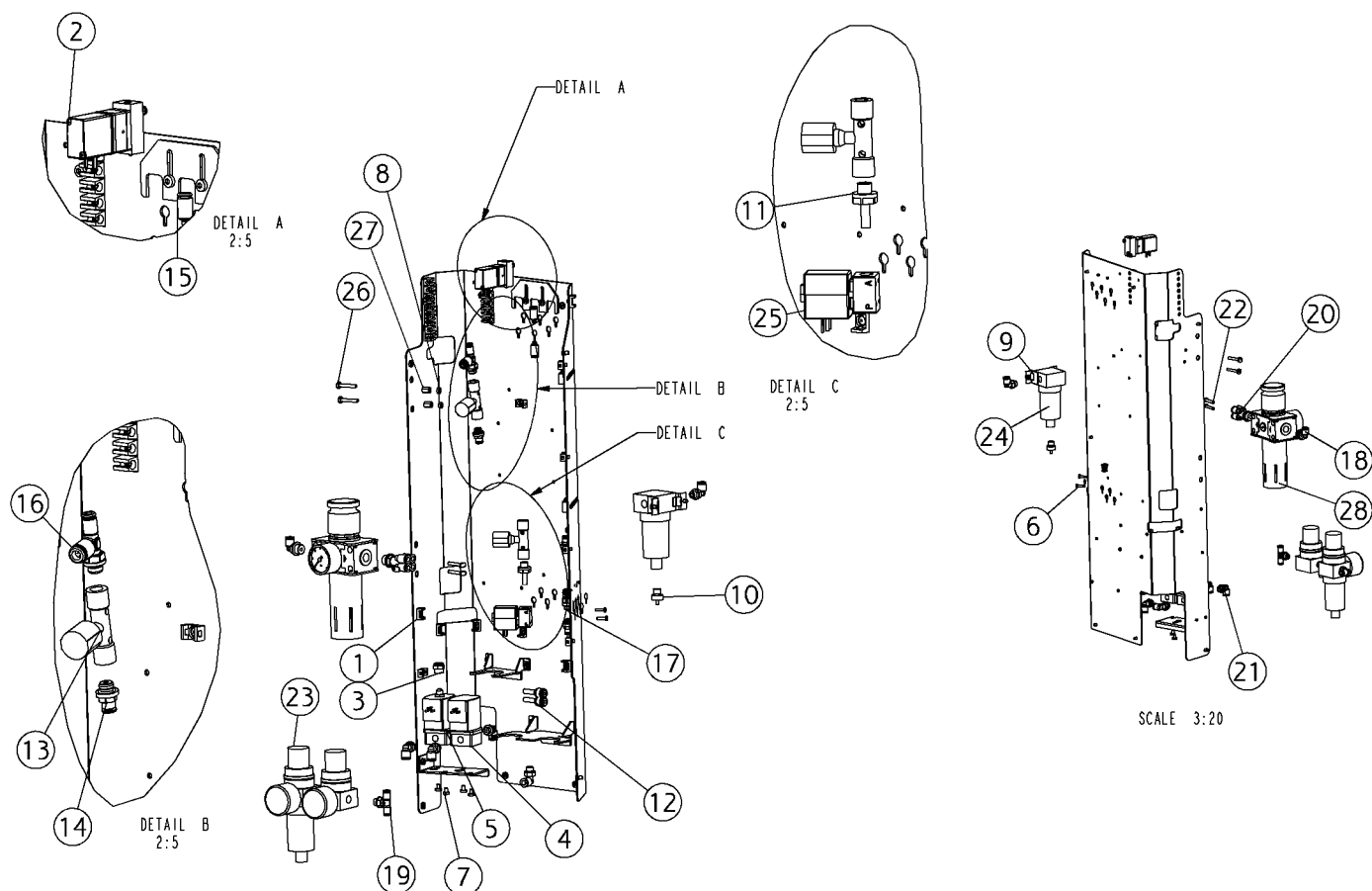




AN-060

Basic config., water/air

Position	Part NO.	Assembly	Description
1	AN-157	<i>Page 9</i>	Mounting plate, v/l
2	AO-778		Valve 2/2 M5, w/adaptor plate
3	UC-038		Filter, exhaust
4	UC-583		Solenoid valve, Ø3, 2 ways
5	UC-584		Solenoid valve, Ø2,5, 3 ways
6	UG-109		Screw, thread making, M3x12
7	UG-111		Screw, thread making, M4x6
8	UG-205		Nut, M4, din 934
9	UG-439		Bracket, filter,exhaust air,XO 4
10	UG-653		Nozzle, oil filter
11	UG-654		Quick coupling Ø6-r1/8 plastic
12	UG-656		Quick fitting, 6/4-4/2
13	UG-658		Water regulator R1/8 - R1/8 VITON
14	UG-659		Fitting, 4mm-R1/8 viton, NSF
15	UG-660		Quick coupling, 4/2-M5 Viton
16	UG-661		Quick coupling 4-6mm, 90-R1/8 Viton
17	UG-662		Quick connect, 6/4-R1/8
18	UG-692		Quick coupling, 6/4-r1/4-90
19	UG-693		Quick coupling, 2X4/2-0-R1/8
20	UG-696		Quick coupling, 4x6/4-r1/4
21	UG-701		Quick coupling, 6/4-r1/8-90
22	UG-719		Quick connector, plug 4/2
23	UG-760		Press. reg./filter, air
24	UG-762		Filter, exhaust air, XO 4
25	UG-800		Valve 2/2 - 1/8 wg
26	UG-993		Not a spare part
27	UG-995		Not a spare part
28	UH-045		Reg./filter, water mkii

AN-157

Mounting plate, v/l

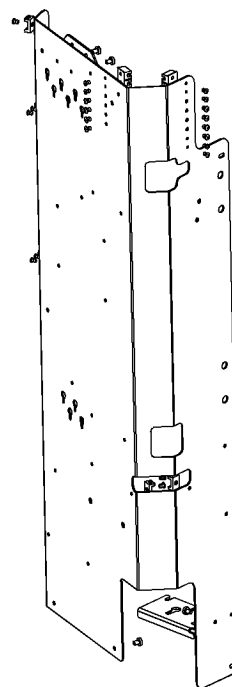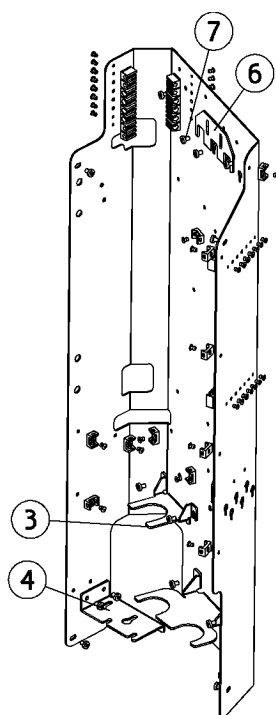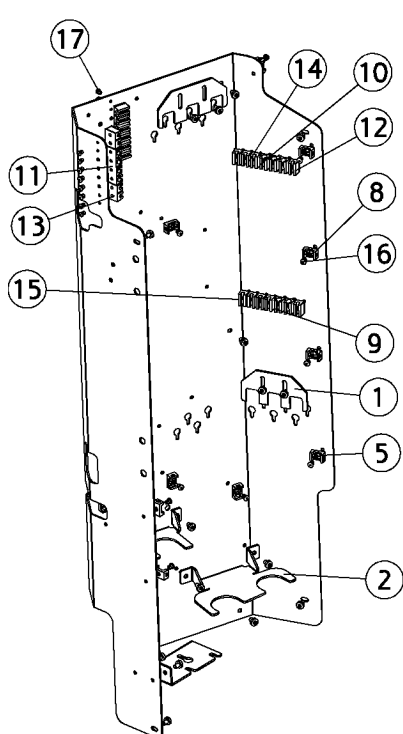



AN-157

Mounting plate, v/l

Position	Part NO.	Assembly	Description
1	MG-400		Valve bracket, bottom
2	MG-562		Not a spare part
3	MG-563		Bracket, water regulator
4	MG-564		Not a spare part
5	MG-565		Not a spare part
6	MG-569		Not a spare part
7	UG-111		Screw, thread making, M4x6
8	UG-530		Not a spare part
9	UG-750		Not a spare part
10	UG-750		Not a spare part
11	UG-750		Not a spare part
12	UG-751		Not a spare part
13	UG-751		Not a spare part
14	UG-751		Not a spare part
15	UG-751		Not a spare part
16	UG-916		Not a spare part
17	UG-917		Not a spare part

AN-061

Back flow prevention

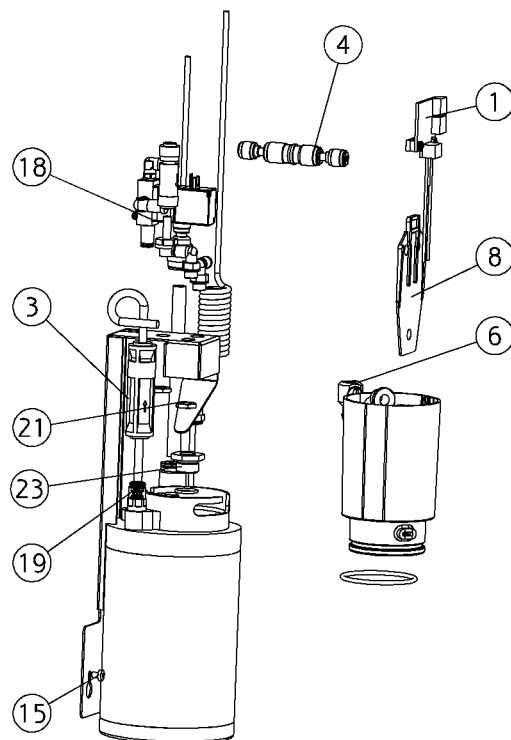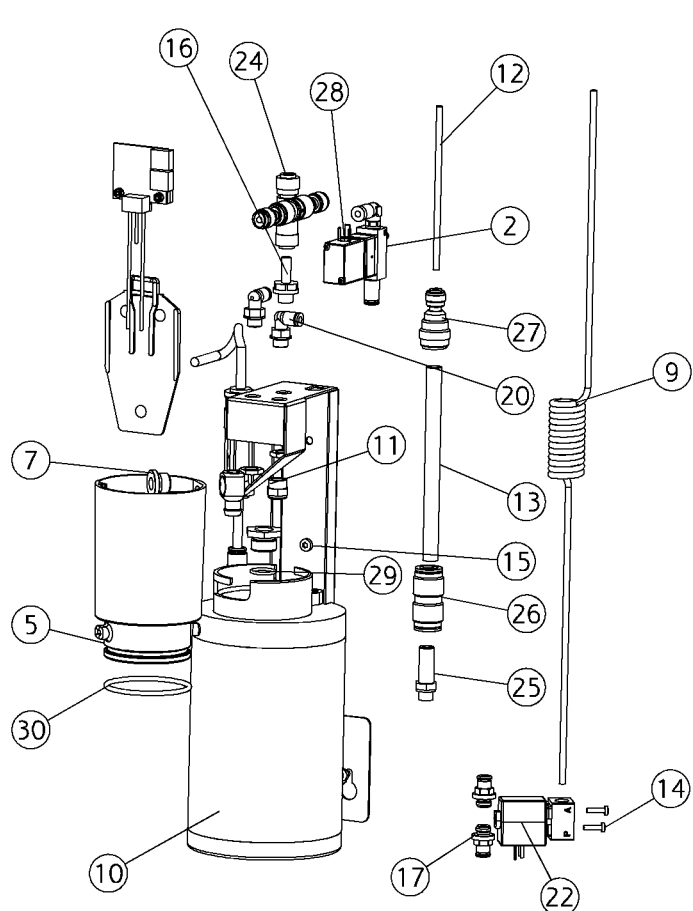


AN-061

Back flow prevention

Position	Part NO.	Assembly	Description
1	AN-298		PCB, vessel, level
2	AO-777		Valve 3/2 M5, w/adaptor plate
3	AO-982		Water filter with tubing
4	AO-983		Non-return valve Ø4/Ø4
5	MG-364		Mixing cup
6	MG-561		Nipple, over run, vessel
7	MG-715		Plug with o-ring Ø8 drilled
8	MG-767		Separation wall,vessel
9	MG-801		Spiral tube 4/2 mm
10	AN-377		Water container
11	UC-038		Filter, exhaust
12	UC-830		Tubing 4/2 blue polyurethan
13	UC-926		Not a spare part
14	UG-109		Screw, thread making, M3x12
15	UG-111		Screw, thread making, M4x6
16	UG-654		Quick coupling Ø6-r1/8 plastic
17	UG-659		Fitting, 4mm-R1/8 viton, NSF
18	UG-660		Quick coupling, 4/2-M5 Viton
19	UG-662		Quick connect, 6/4-R1/8
20	UG-703		Quick coupling, 4/2-r1/8-90
21	UG-799		Nut, stainless steel 1/8
22	UG-800		Valve 2/2 - 1/8 wg
23	UG-809		Valve, back flow prevention
24	UH-057		Non return valve, 4/6 EPDM
25	UH-195		Not a spare part
26	UH-196		Not a spare part
27	UH-197		Not a spare part
28	UH-241		Quick connect, 4/2-M5-90 NSF
29	UH-245		O-ring 14 x 1,78 VITON, SH 75
30	UH-246		O-ring 50 x 3 Viton, 75 SH

AN-079

Unit stand, electronics

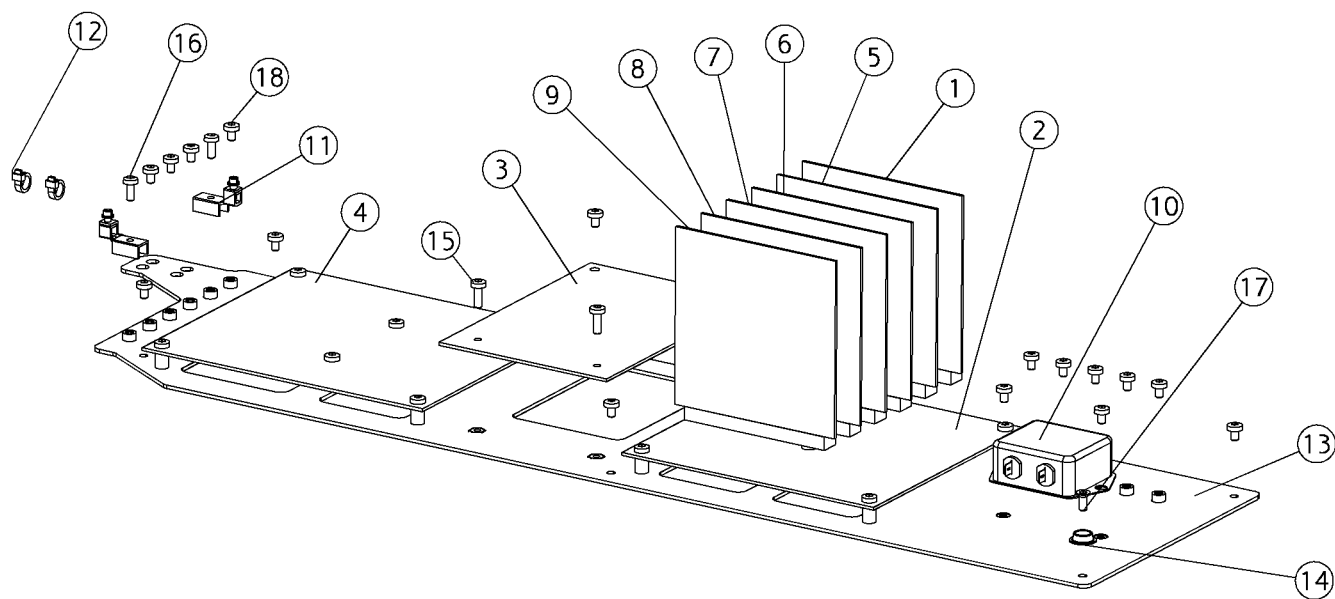


AN-079

Unit stand, electronics

Position	Part NO.	Assembly	Description
1	AO-891		PCB, Stand control with support for LED OP-lampe AN-367
2	AN-369		PCB, Back plane, XO 4
3	AN-371		PCB, Power supply, XO 4
4	AN-378		PCB, Mains, XO 4
5	AN-379		PCB, Video & PC comm/XO SmartLink
6	AN-380		PCB, XO screen
7	AO-184		PCB, Pump control
8	AO-986		PCB, XO widescreen power supply
9	AP-632		PCB, XO HD display psu
10	MH-925		Not a spare part
11	ML-275		Not a spare part
12	ML-281		Cable binder 150 mm
13	MN-472		Not a spare part
14	UC-072		Not a spare part
15	UG-068		Not a spare part
16	UG-110		Screw, thread making, M4x10
17	UG-115		Screw, M4x010
18	UH-092		Not a spare part

AN-043

Cover, tube Ø150

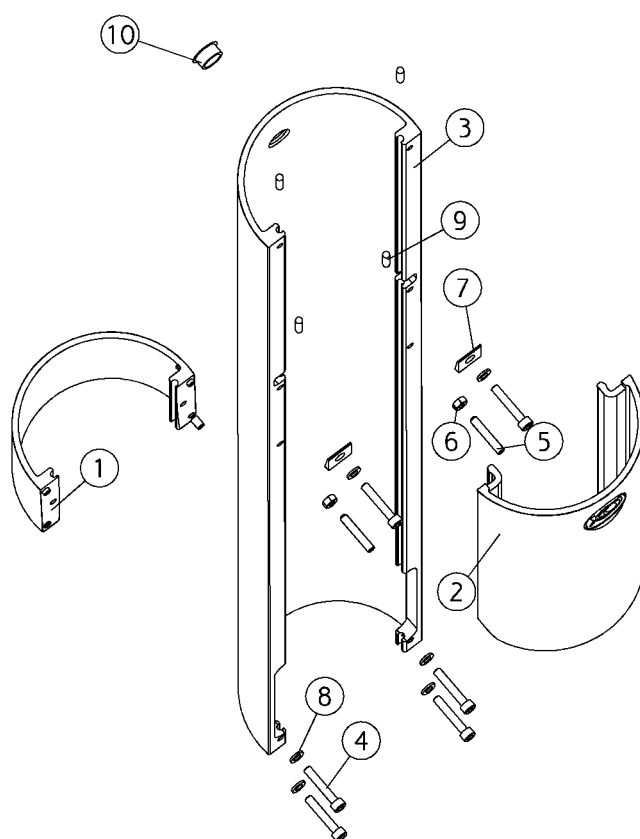


AN-043

Cover, tube Ø150

Position	Part NO.	Assembly	Description
1	AN-044	<i>Page 17</i>	Panel tube, Ø150, L=88, short
2	AN-514	<i>Page 18</i>	Tube Ø150, L=295, demountable
3	AN-955		Tube ext., Ø150, long, (XO-905 XO White,(RAL7047-30))
3	AO-290		Tube ext., Ø150, long, (XO-906 XO Titanium)
3	AN-496		Tube ext., Ø150, long, (XO-900 XO Grey, RAL 7047-80)
3	AP-847		Tube ext., Ø150, L=1109, long, (CTBS)
3	AO-747		Tube ext., Ø150, long, (XO-903 Silver Metallic)
3	AN-497		Tube ext., Ø150, long, (XO-901 Sea Blue Metallic)
4	UG-024		Not a spare part
5	UG-106		Not a spare part
6	UG-210		Not a spare part
7	UG-251		Not a spare part
8	UG-256		Disc, Ø06.4xØ12.0x1.6
9	UG-418		Cord Ø5.34x20
10	UG-980		Cover plug, Ø20, black

AN-044


Panel tube, Ø150, L=88, short

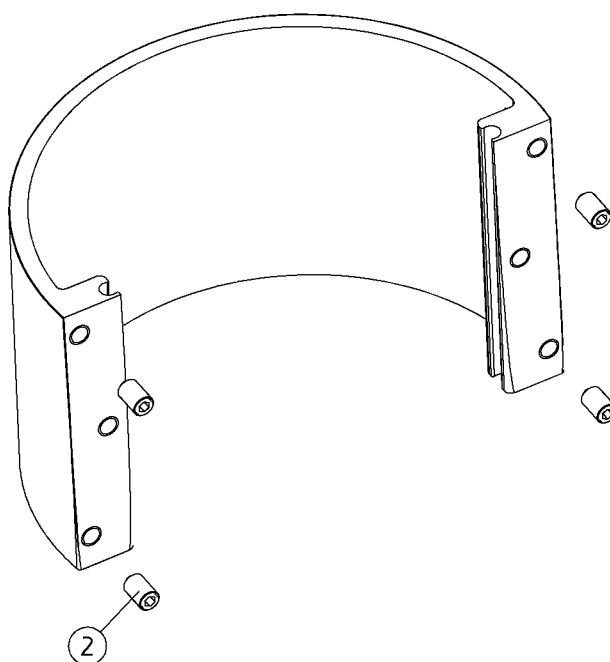

Position	Part NO.	Assembly	Description
1	AP-681		Tube ext., Ø150, L=88, short, (CTBS)
1	AO-720		Tube ext., Ø150, short, (XO-908 Iron Grey (RAL 7011-5))
2	UG-105		Pointed screw, M6x10

AN-514


Tube Ø150, L=295, demountable

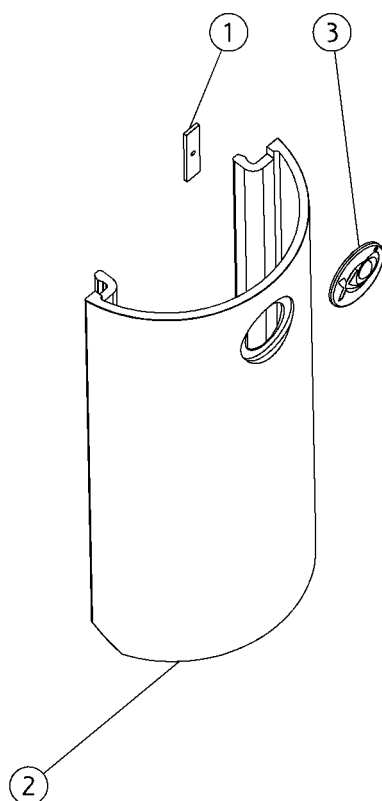

Position	Part NO.	Assembly	Description
1	MG-626		Mounting plate, logo
2	AO-748		Tube, Ø150, (XO-903 Silver Metallic)
2	AO-499		Tube, Ø150, (XO-901 Sea Blue Metallic)
2	AP-849		Tube, Ø150, L=295, (CTBS)
2	AO-291		Tube, Ø150, (XO-906 XO Titanium)
2	AN-956		Tube, Ø150, (XO-905 XO White,(RAL7047-30))
2	AO-498		Tube, Ø150, (XO-900 XO Grey, RAL 7047-80)
3	UH-254		Not a spare part

AN-173


Cuspidor bowl,filter,cover

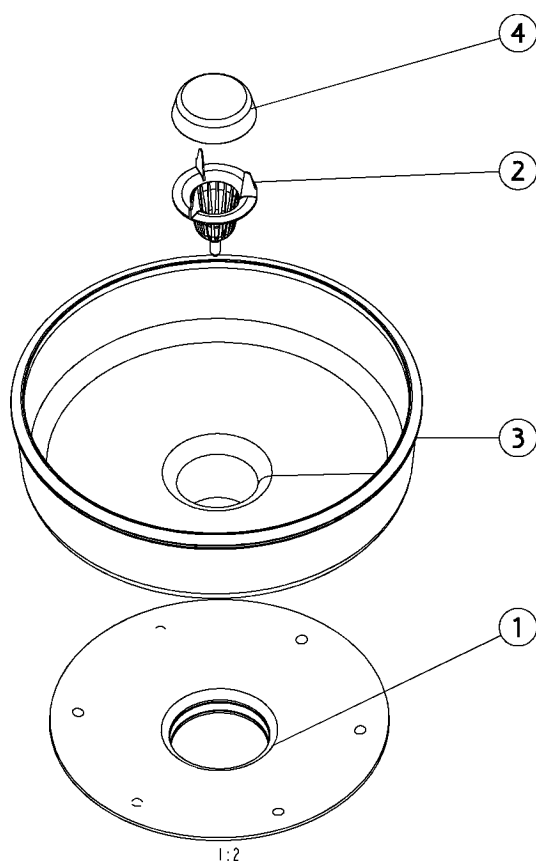

Position	Part NO.	Assembly	Description
1	AO-244	Page 20	Sleeve, cuspidor bowl
2	MG-150		Gold trap, integral 99,XO 4
3	MG-395		Cuspidor bowl, type 2008
4	MG-894		Cap, gold trap, type 2007

AO-244


Sleeve, cuspidor bowl

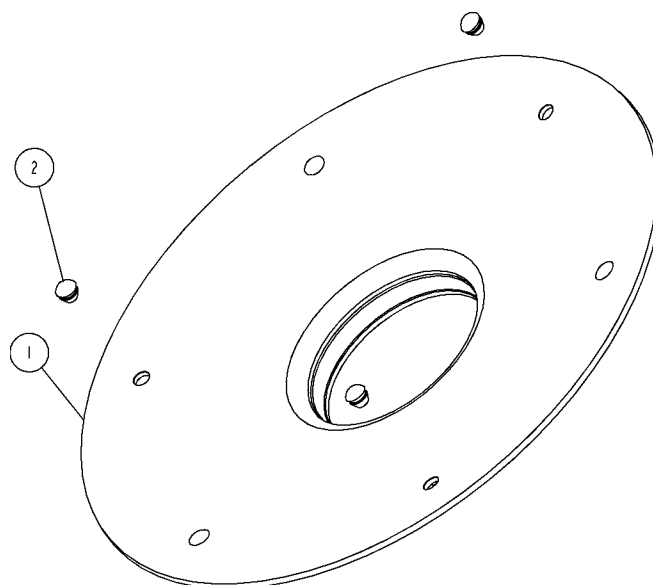

Position	Part NO.	Assembly	Description
1	AO-242		Not a spare part
2	MG-904		Product feet f/protection sleeve

AN-806

Top plate w/o cuspidor

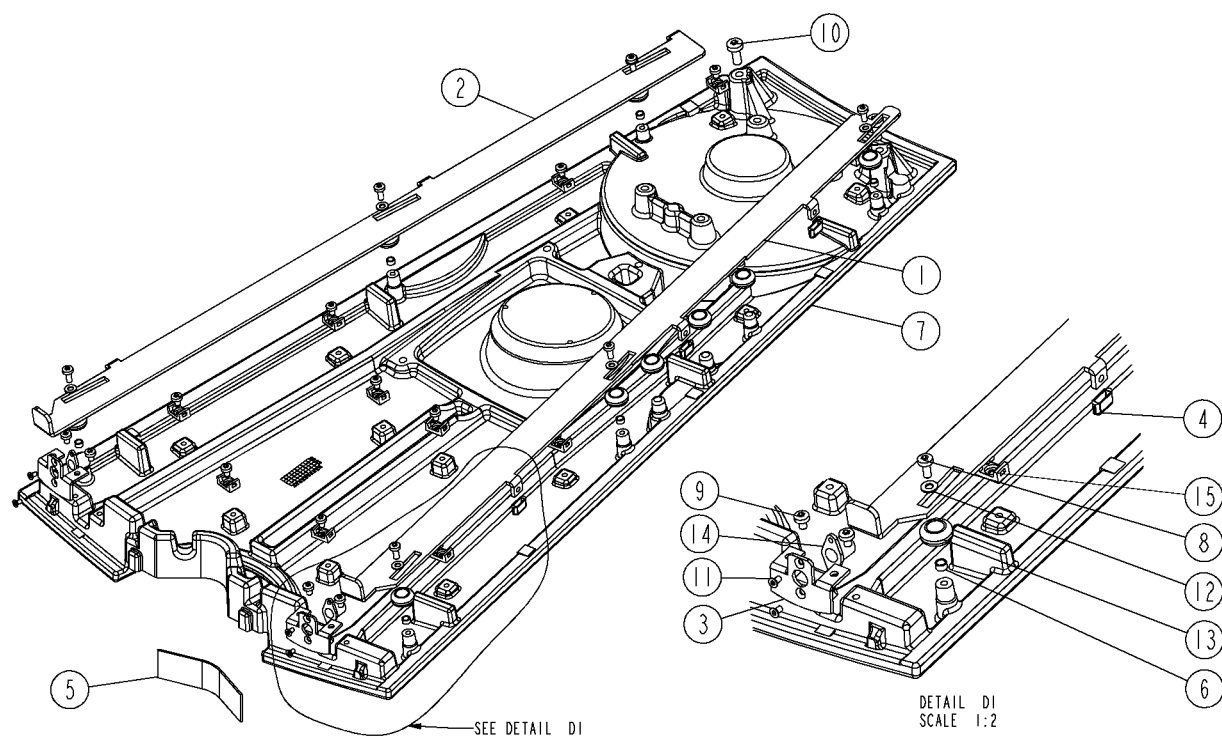


AN-806

Top plate w/o cuspidor

Position	Part NO.	Assembly	Description
1	MG-017		Not a spare part
2	MG-018		Not a spare part
3	MG-022		Lock, topplate
4	MG-060		Not a spare part
5	MG-552		Spring adjustment top plate
6	MG-625		Bushing, topplatte
7	AP-630		Top plate w/o cuspidor, (CTBS)
7	AO-774		Top plate w/o cuspidor, (XO-901 Sea Blue Metallic)
7	AO-773		Top plate w/o cuspidor, (XO-900 XO Grey, RAL 7047-80)
7	AO-775		Top plate w/o cuspidor, (XO-903 Silver Metallic)
7	AO-305		Top plate w/o cuspidor, (XO-906 XO Titanium)
7	AO-007		Top plate w/o cuspidor, (XO-905 XO White,(RAL7047-30))
8	UG-110		Screw, thread making, M4x10
9	UG-111		Screw, thread making, M4x6
10	UG-113		Screw, thread making, M6x16
11	UG-232		Screw M3x6 us
12	UG-254		Disc, Ø04.3xØ09.0x0.8
13	UG-401		Rubber nipple Ø9,5x13x2
14	UG-411		Locking house, topplate
15	UG-530		Not a spare part

AN-059

Top plate stand, mounted

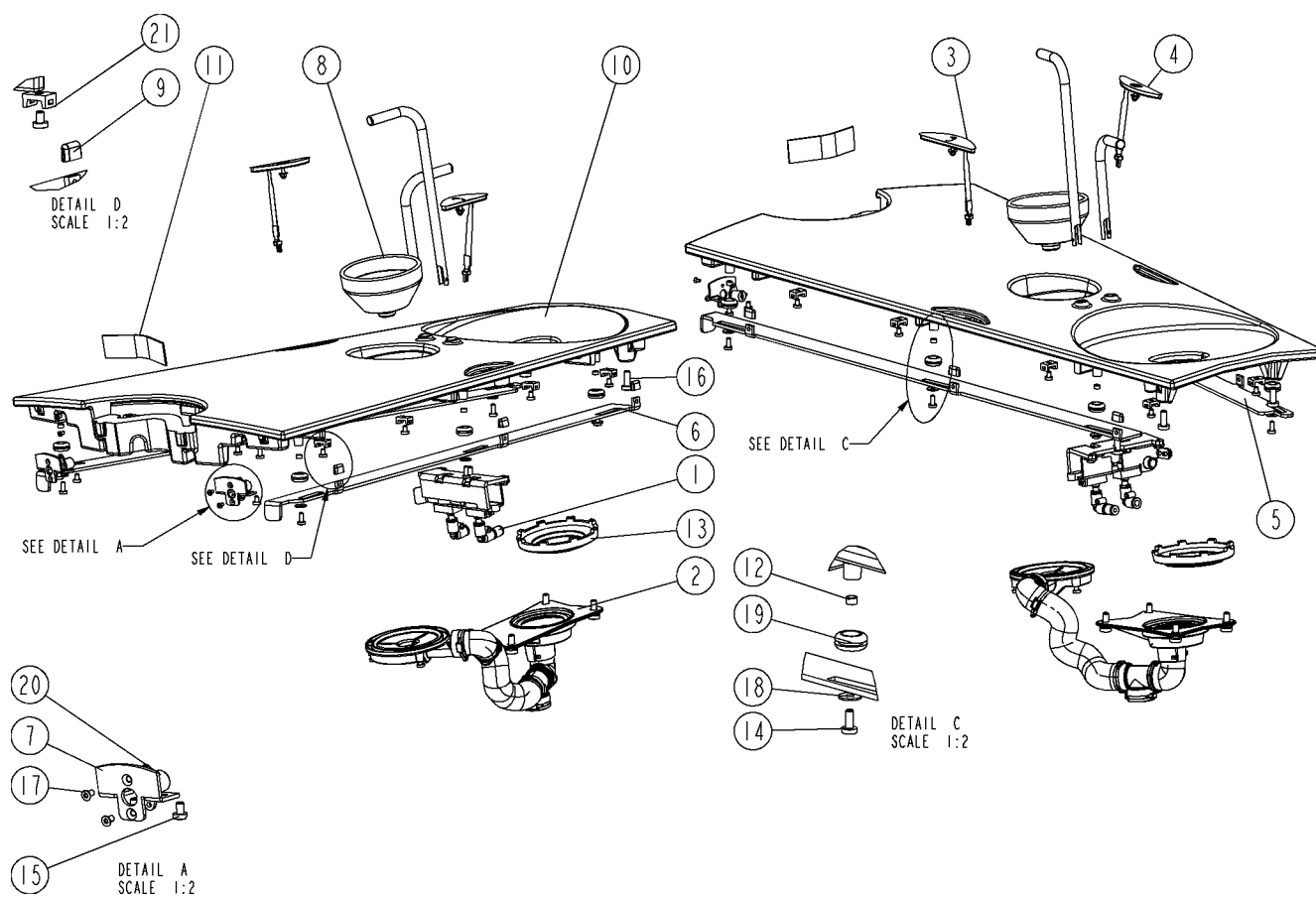



AN-059

Top plate stand, mounted

Position	Part NO.	Assembly	Description
1	AN-186	<i>Page 25</i>	Water connection, top plate
2	AN-521	<i>Page 29</i>	Water reception tank, top plate
3	AO-628		Contact cuspidor,
4	AO-629		Contact, cup filler,
5	MG-017		Not a spare part
6	MG-018		Not a spare part
7	MG-022		Lock, topplate
8	MG-055		Waterglass holder
9	MG-060		Not a spare part
10	KT-009		Kit, top plate, (XO-903 Silver Metallic)
10	KT-044		Kit, top plate, (CTBS)
10	KT-008		Kit, top plate, (XO-901 Sea Blue Metallic)
10	KT-011		Kit, top plate, (XO-906 XO Titanium)
10	KT-007		Kit, top plate, (XO-900 XO Grey, RAL 7047-80)
10	KT-010		Kit, top plate, (XO-905 XO White, (RAL 7047-30))
11	MG-552		Spring adjustment top plate
12	MG-625		Bushing, topplatte
13	MG-901		Not a spare part
14	UG-110		Screw, thread making, M4x10
15	UG-111		Screw, thread making, M4x6
16	UG-113		Screw, thread making, M6x16
17	UG-232		Screw M3x6 us
18	UG-279		Washer, din9021 Ø4,3xØ12x1
19	UG-401		Rubber nipple Ø9,5x13x2
20	UG-411		Locking house, topplate
21	UG-530		Not a spare part

AN-186

Water connection, top plate

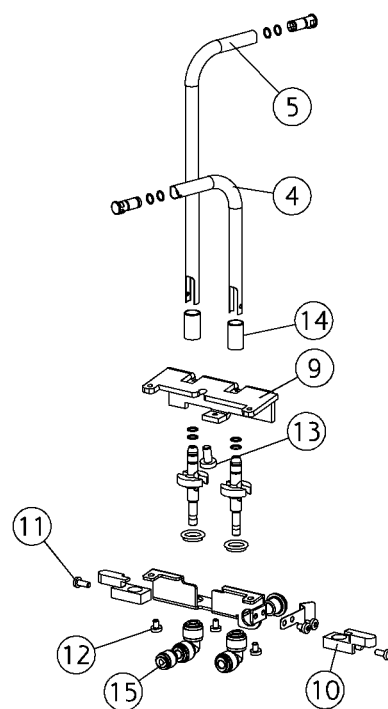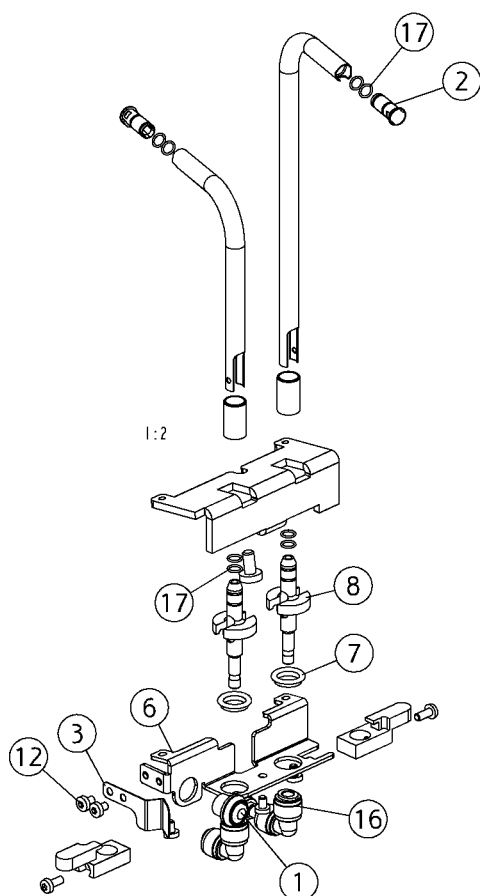



AN-186

Water connection, top plate

Position	Part NO.	Assembly	Description
1	CA-037		Switch +cable, suction disin.
2	MG-409		Nozzle, cuspidor/cup filler
3	MG-645		Spring, cuspidor flush pipe
4	MG-646		Flush tube, spittoon
5	MG-647		Tube for cup filler
6	MG-648		Not a spare part
7	MG-649		Not a spare part
8	MG-650		Conn. piece, rinsing tube
9	MG-651		Not a spare part
10	MG-652		Not a spare part
11	UG-110		Screw, thread making, M4x10
12	UG-111		Screw, thread making, M4x6
13	UG-120		Not a spare part
14	UG-472		Not a spare part
15	UG-656		Quick fitting, 6/4-4/2
16	UG-657		Quick coupling, angle, 4/6
17	UH-247		O-ring 6 x 1 Viton, 75 SH

AN-520

Water connection, top plate

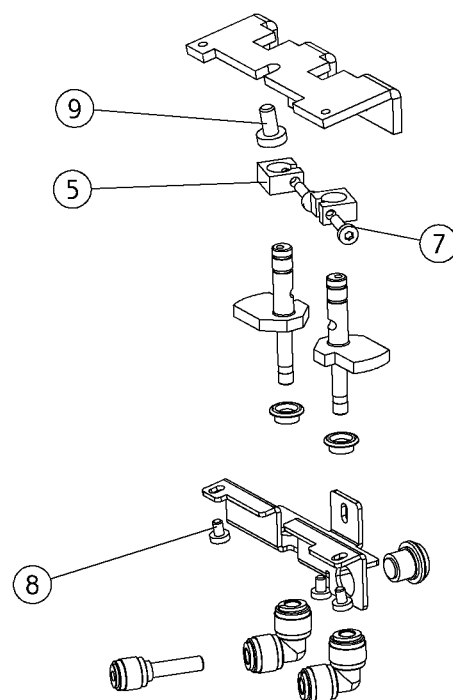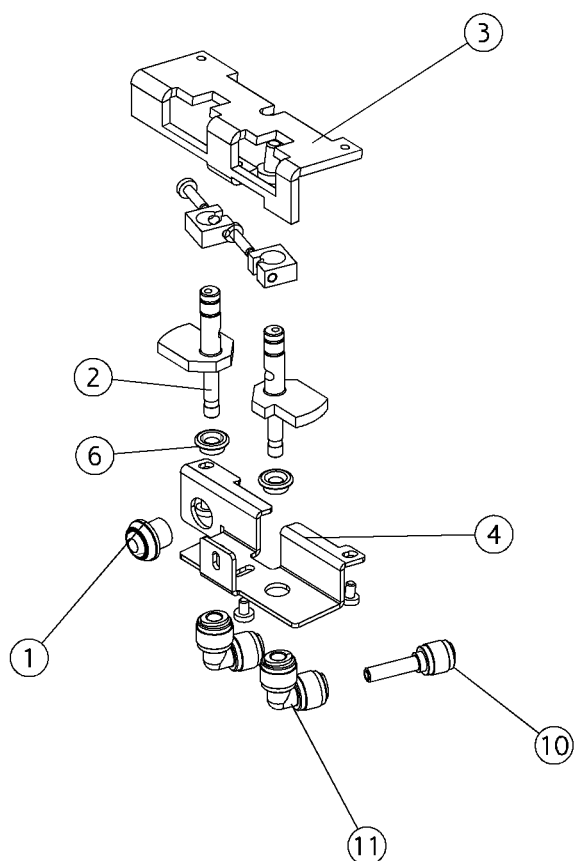


AN-520

Water connection, top plate

Position	Part NO.	Assembly	Description
1	CA-037		Switch +cable, suction disin.
2	MG-590		Not a spare part
3	MG-591		Not a spare part
4	MG-592		Not a spare part
5	MG-593		Not a spare part
6	MG-597		Not a spare part
7	UG-103		Screw, thread making, M4x16
8	UG-111		Screw, thread making, M4x6
9	UG-120		Not a spare part
10	UG-656		Quick fitting, 6/4-4/2
11	UG-657		Quick coupling, angle, 4/6
12	UH-247		O-ring 6 x 1 Viton, 75 SH

AN-521


Water reception tank,top plate

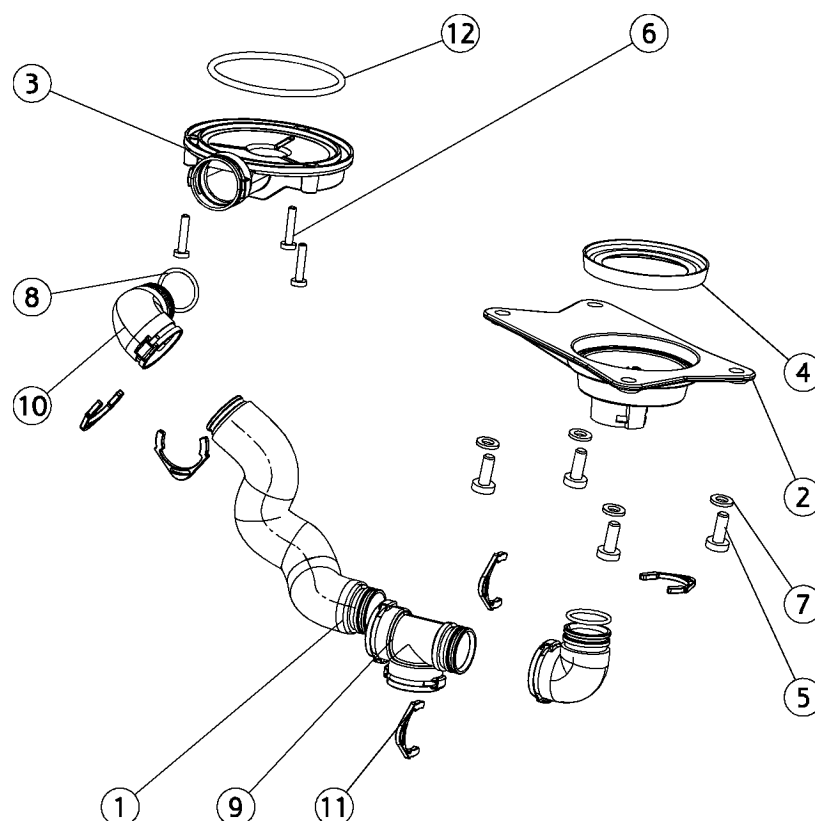

Position	Part NO.	Assembly	Description
1	AN-073	Page 30	Tube, drain, top plate
2	MG-887		Drain, cuspidor, type 2007
3	MG-888		Not a spare part
4	MG-889		Gasket, cuspidor bowl, type 2007, XO 4
5	UG-113		Screw, thread making, M6x16
6	UG-229		Not a spare part
7	UG-256		Disc, Ø06.4xØ12.0x1.6
8	UG-320		O-ring, 20x2
9	UG-723		Dürr conn.t-piece, 2-female, 1-male
10	UG-724		Dürr connect, angle-90
11	UG-730		Dürr connect, clips, blue
12	UH-009		Not a spare part

AN-073


Tube, drain, top plate

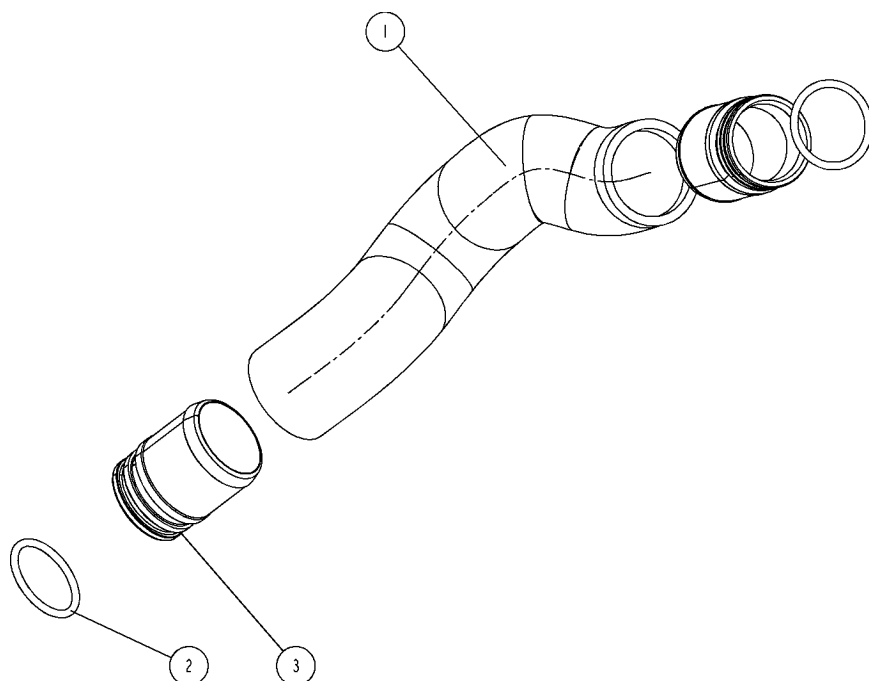

Position	Part NO.	Assembly	Description
1	MS-356		Hose Ø24, black
2	UG-320		O-ring, 20x2
3	UG-726		Dürr connect, Ø25, male

AN-051

Inside service cover, empty

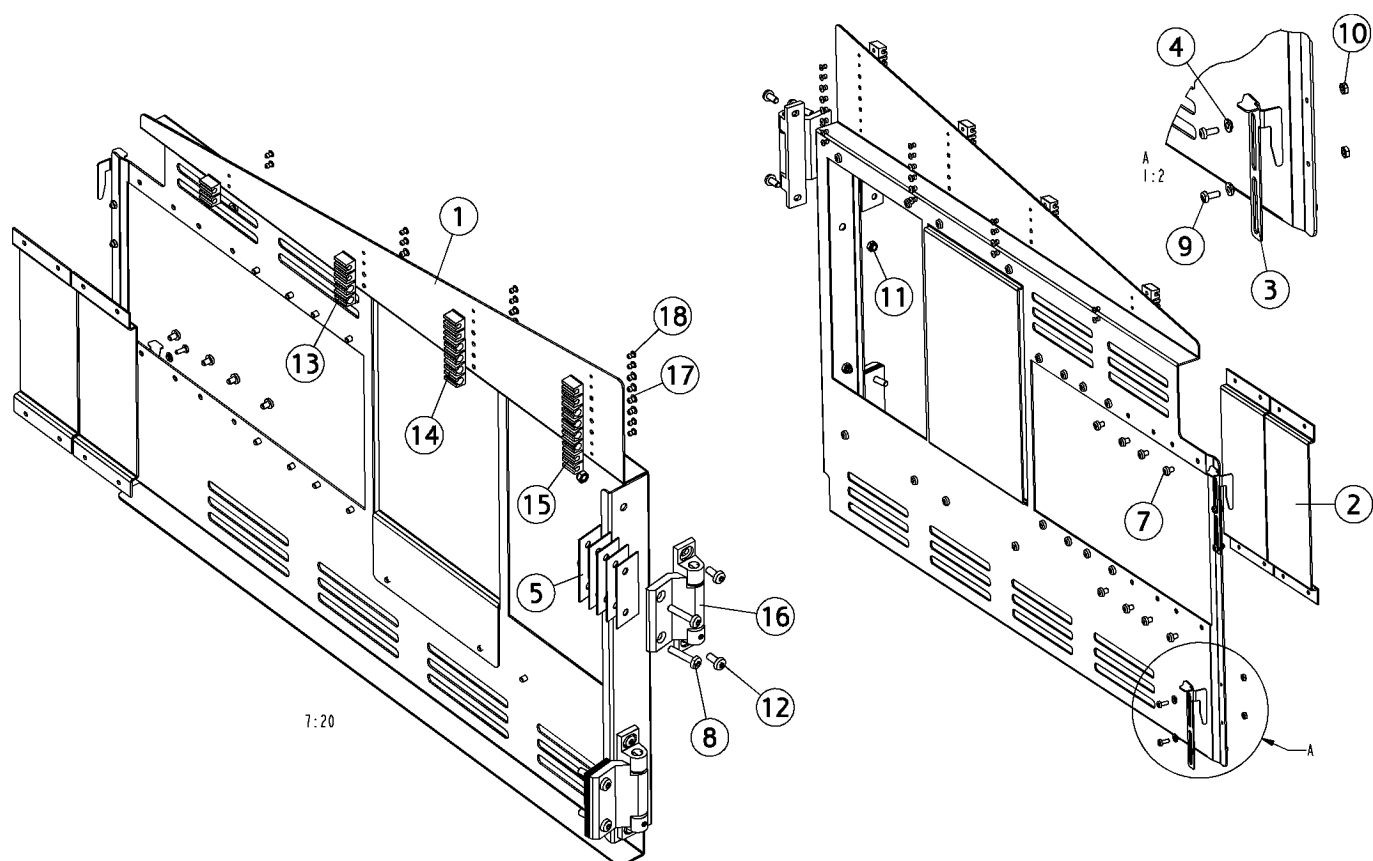


AN-051

Inside service cover, empty


Position	Part NO.	Assembly	Description
1	MG-132		Not a spare part
2	MG-505		Not a spare part
3	MG-574		Not a spare part
4	MG-575		Not a spare part
5	MG-577		Not a spare part
6	MG-642		Not a spare part
7	UG-111		Screw, thread making, M4x6
8	UG-112		Not a spare part
9	UG-139		Screw. M3x8
10	UG-207		Nut, M3
11	UG-208		Locknut M5
12	UG-222		Not a spare part
13	UG-750		Not a spare part
14	UG-750		Not a spare part
15	UG-751		Not a spare part
16	UG-900		Not a spare part
17	UG-916		Not a spare part
18	UG-917		Not a spare part

AP-629

XO water softener assembly

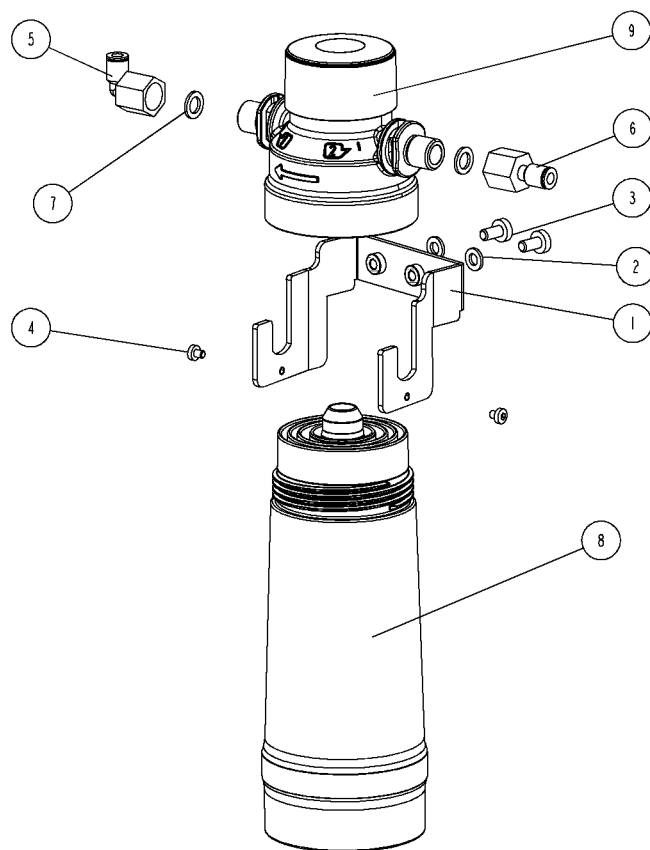

Position	Part NO.	Assembly	Description
1	MN-022		Not a spare part
2	UG-256		Disc, Ø06.4xØ12.0x1.6
3	UH-150		Not a spare part
4	UH-151		Not a spare part
5	UH-152		Quick coupling 3/8-Ø6 tube 90deg
6	UH-153		Quick coupling 3/8-Ø6 tube straight
7	UH-154		Gasket Ø15/Ø9,5x1,5, fiber
8	UH-200		XO water softener, filter cartridge
9	UH-201		XO water softener, filter head

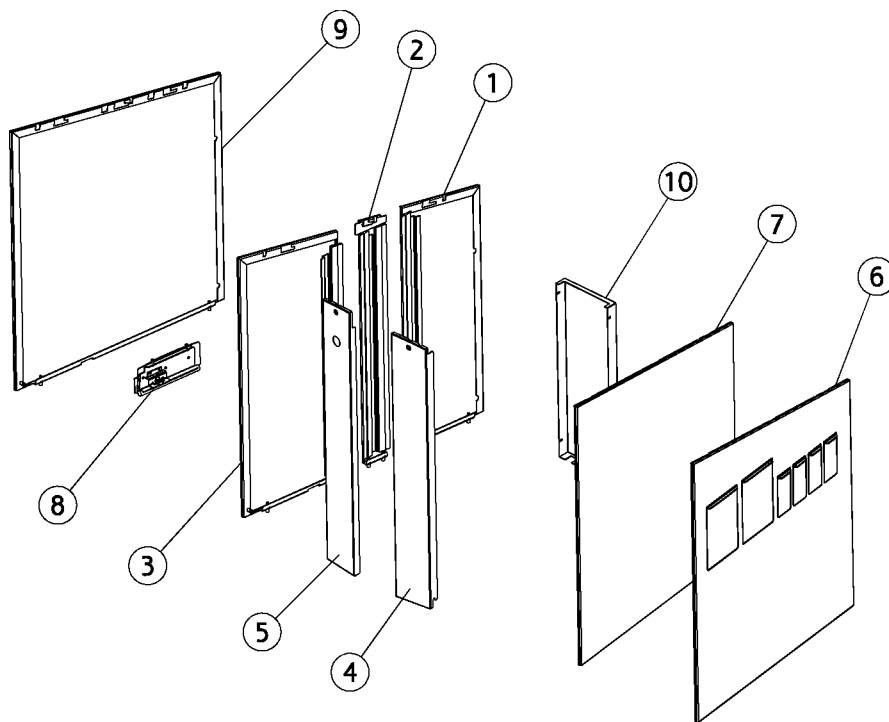

AN-037

Covers stand

Position	Part NO.	Assembly	Description
1	AO-087	Page 37	Panel, right front, (XO-903 Silver Metallic)
1	AO-086	Page 37	Panel, right front, (XO-901 Sea Blue Metallic)
1	AP-485	Page 37	Panel, right front, (CTBS)
1	AO-649	Page 37	Panel, right front, (XO-906 XO Titanium)
1	AO-088	Page 37	Panel, right front, (XO-905 XO White,(RAL7047-30))
1	AO-085	Page 37	Panel, right front, (XO-900 XO Grey, RAL 7047-80)
2	AO-393	Page 38	Panel, right, middle, (XO-906 XO Titanium)
2	AO-092	Page 38	Panel, right, middle, (XO-903 Silver Metallic)
2	AO-093	Page 38	Panel, right, middle, (XO-905 XO White,(RAL7047-30))
2	AP-486	Page 38	Panel, right, middle, (CTBS)
2	AO-091	Page 38	Panel, right, middle, (XO-901 Sea Blue Metallic)
2	AO-089	Page 38	Panel, right, middle, (XO-900 XO Grey, RAL 7047-80)
3	AO-650	Page 39	Panel, right, rear, (XO-906 XO Titanium)
3	AP-487	Page 39	Panel, right, rear, (CTBS)
3	AO-095	Page 39	Panel, right, rear, (XO-901 Sea Blue Metallic)
3	AO-096	Page 39	Panel, right, rear, (XO-903 Silver Metallic)
3	AO-094	Page 39	Panel, right, rear, (XO-900 XO Grey, RAL 7047-80)
3	AO-097	Page 39	Panel, right, rear, (XO-905 XO White,(RAL7047-30))
4	AO-104	Page 40	Panel, rear, left, (XO-903 Silver Metallic)
4	AP-837	Page 40	Panel, rear, left, (CTBS)
4	AO-103	Page 40	Panel, rear, left, (XO-901 Sea Blue Metallic)
4	AP-488	Page 40	Panel, rear, left, (CTBS)
4	AP-611	Page 40	Panel, rear, left, w/sign XO 4-6, (CTBS) (XB-931 Traffic White)
4	AO-098	Page 40	Panel, rear, left, (XO-900 XO Grey, RAL 7047-80)
4	AO-651	Page 40	Panel, rear, left, (XO-906 XO Titanium)
4	AO-107	Page 40	Panel, rear, left, (XO-905 XO White,(RAL7047-30))
5	AO-109	Page 42	Panel, rear, right, (XO-901 Sea Blue Metallic)
5	AP-839	Page 42	Panel, rear, right, (CTBS)
5	AO-111	Page 42	Panel, rear, right, (XO-905 XO White,(RAL7047-30))
5	AO-652	Page 42	Panel, rear, right, (XO-906 XO Titanium)
5	AP-489	Page 42	Panel, rear, right, (CTBS)
5	AO-110	Page 42	Panel, rear, right, (XO-903 Silver Metallic)


AN-037

Covers stand

Position	Part NO.	Assembly	Description
5	AO-108	<i>Page 42</i>	Panel, rear, right, (XO-900 XO Grey, RAL 7047-80)
6	AP-490	<i>Page 44</i>	Panel, left, w/ doors, (CTBS)
6	AO-113	<i>Page 44</i>	Panel, left, w/ doors, (XO-901 Sea Blue Metallic)
6	AO-634	<i>Page 44</i>	Panel, left, w/ doors, (XO-905 XO White,(RAL7047-30))
6	AO-112	<i>Page 44</i>	Panel, left, w/ doors, (XO-900 XO Grey, RAL 7047-80)
6	AO-138	<i>Page 44</i>	Panel, left, w/ doors, (XO-906 XO Titanium)
6	AO-114	<i>Page 44</i>	Panel, left, w/ doors, (XO-903 Silver Metallic)
7	AO-271		Left panel w/o doors, (XO-906 XO Titanium)
7	AO-119		Left panel w/o doors, (XO-905 XO White,(RAL7047-30))
7	AO-117		Left panel w/o doors, (XO-901 Sea Blue Metallic)
7	AO-118		Left panel w/o doors, (XO-903 Silver Metallic)
7	AO-116		Left panel w/o doors, (XO-900 XO Grey, RAL 7047-80)
8	AO-124	<i>Page 47</i>	Cover,plate, socket w/o chair,
9	AO-125	<i>Page 43</i>	Panel right, w/o chair, (XO-900 XO Grey, RAL 7047-80)
9	AO-247	<i>Page 43</i>	Panel right, w/o chair, (XO-906 XO Titanium)
9	AO-126	<i>Page 43</i>	Panel right, w/o chair, (XO-901 Sea Blue Metallic)
9	AO-128	<i>Page 43</i>	Panel right, w/o chair, (XO-905 XO White,(RAL7047-30))
9	AO-127	<i>Page 43</i>	Panel right, w/o chair, (XO-903 Silver Metallic)
10	AN-404		Panel, front, (XO-900 XO Grey, RAL 7047-80)
10	AN-405		Panel, front, (XO-901 Sea Blue Metallic)
10	AN-917		Panel, front, (XO-905 XO White,(RAL7047-30))
10	AO-252		Panel, front, (XO-906 XO Titanium)
10	AP-403		Panel, front, (CTBS)
10	AN-706		Panel, front, (XO-903 Silver Metallic)

AN-038


Panel, right front

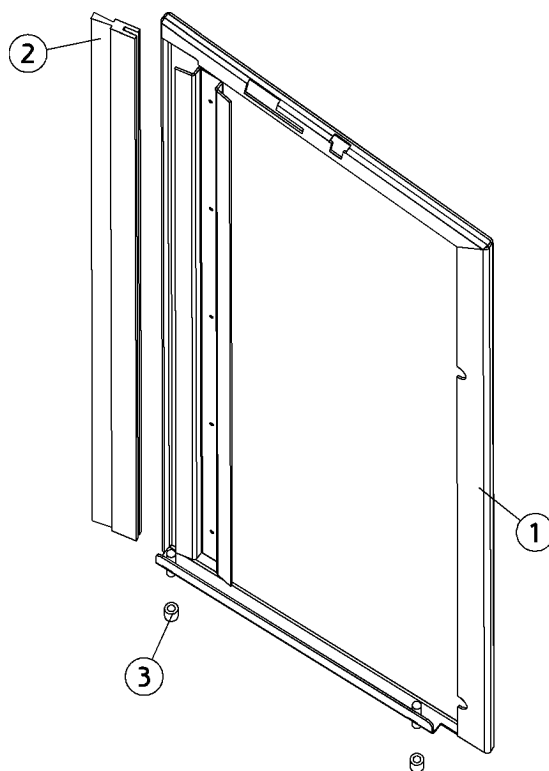

Position	Part NO.	Assembly	Description
1	AN-408		Panel, side, right, front, (XO-900 XO Grey, RAL 7047-80)
1	AN-995		Panel, side, right, front, (XO-905 XO White,(RAL7047-30))
1	AN-409		Panel, side, right, front, (XO-901 Sea Blue Metallic)
1	AO-254		Panel, side, right, front, (XO-906 XO Titanium)
1	AN-708		Panel, side, right, front, (XO-903 Silver Metallic)
1	AP-473		Panel, side, right, front, (CTBS)
2	UG-417		Brush pa6 black Ø0,15/15
3	UG-912		Hose pvc transp. Ø5xØ9

AN-039


Panel, right, middle

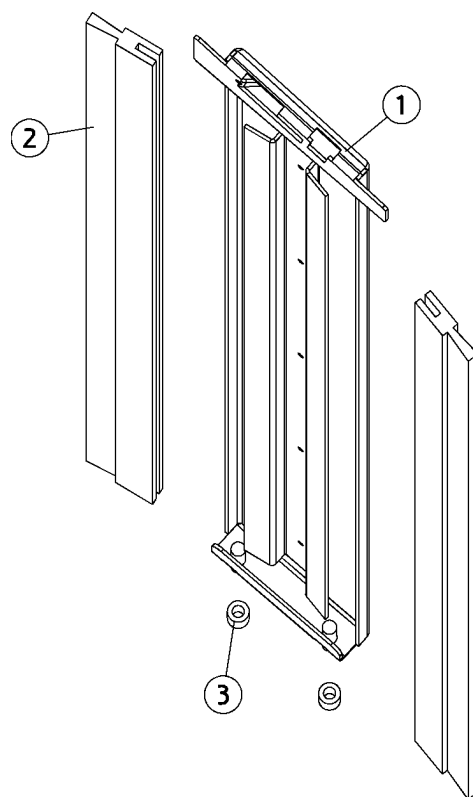

Position	Part NO.	Assembly	Description
1	AN-410		Panel, side, right, centre, (XO-900 XO Grey, RAL 7047-80)
1	AP-474		Panel, side, right, centre, (CTBS)
1	AN-411		Panel, side, right, centre, (XO-901 Sea Blue Metallic)
1	AN-709		Panel, side, right, centre, (XO-903 Silver Metallic)
1	AN-920		Panel, side, right, centre, (XO-905 XO White,(RAL7047-30))
1	AO-255		Panel, side, right, centre, (XO-906 XO Titanium)
2	UG-417		Brush pa6 black Ø0,15/15
3	UG-912		Hose pvc transp. Ø5xØ9

AN-040


Panel, right, rear

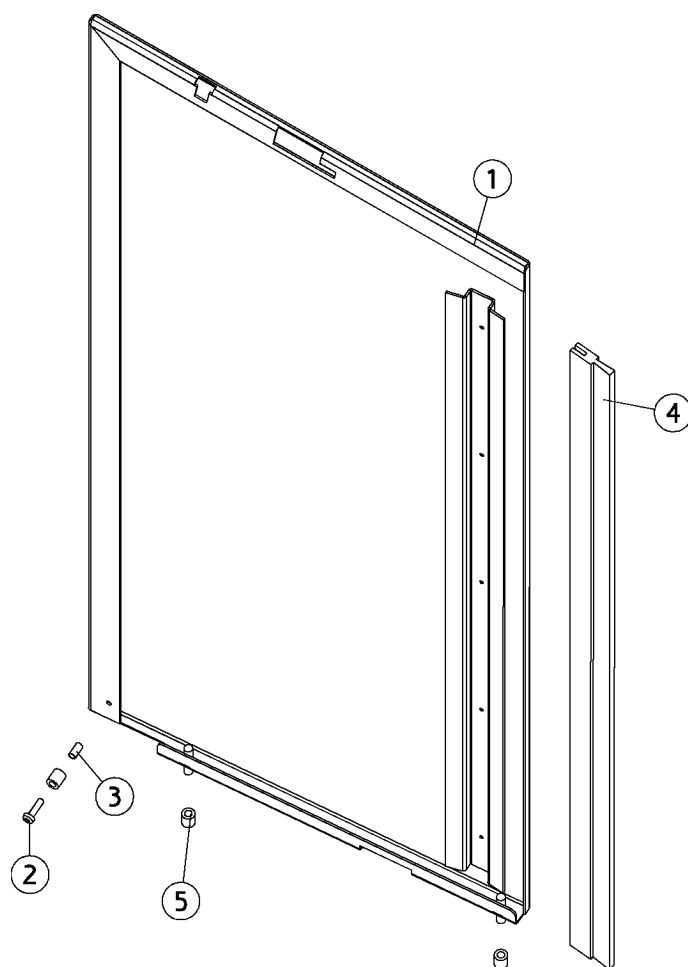

Position	Part NO.	Assembly	Description
1	AP-472		Panel, side, right, rear, (CTBS)
1	AN-989		Panel, side, right, rear, (XO-905 XO White,(RAL7047-30))
1	AN-407		Panel, side, right, rear, (XO-901 Sea Blue Metallic)
1	AN-406		Panel, side, right, rear, (XO-900 XO Grey, RAL 7047-80)
1	AO-253		Panel, side, right, rear, (XO-906 XO Titanium)
1	AN-707		Panel, side, right, rear, (XO-903 Silver Metallic)
2	UG-103		Screw, thread making, M4x16
3	UG-402		Spacer Ø4,2xØ5x10 brass
4	UG-417		Brush pa6 black Ø0,15/15
5	UG-912		Hose pvc transp. Ø5xØ9

AN-041


Panel, rear, left

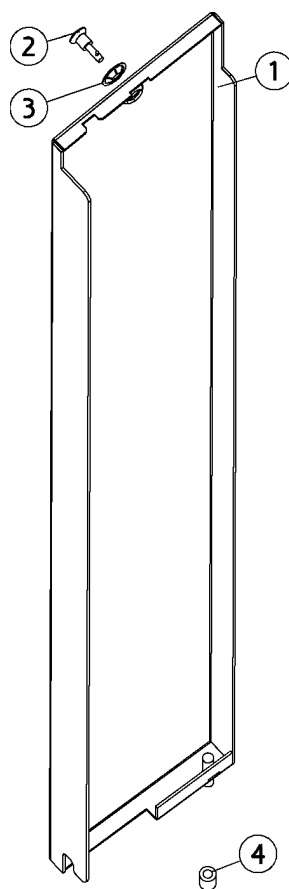

Position	Part NO.	Assembly	Description
1	AN-704		Rear panel, left, (XO-903 Silver Metallic)
1	AN-401		Rear panel, left, (XO-901 Sea Blue Metallic)
1	AN-915		Not a spare part (XO-905 XO White,(RAL7047-30))
1	AO-250		Rear panel, left, (XO-906 XO Titanium)
1	AN-400		Rear panel, left, (XO-900 XO Grey, RAL 7047-80)
2	UG-412		Lock pin, panel
3	UG-413		Lock washer, panel
4	UG-912		Hose pvc transp. Ø5xØ9

AP-488


Panel, rear, left,

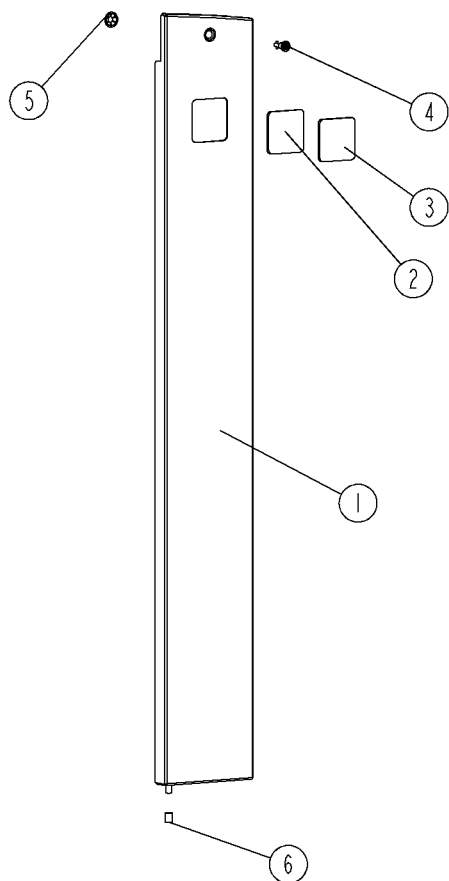

Position	Part NO.	Assembly	Description
1	AP-470		Rear panel, left, (CTBS)
2	MN-058		Product sign XO 4-2, metal
3	MN-059		Product sign XO 4-6, metal
4	UG-412		Lock pin, panel
5	UG-413		Lock washer, panel
6	UG-912		Hose pvc transp. Ø5xØ9

AN-042


Cover back right,

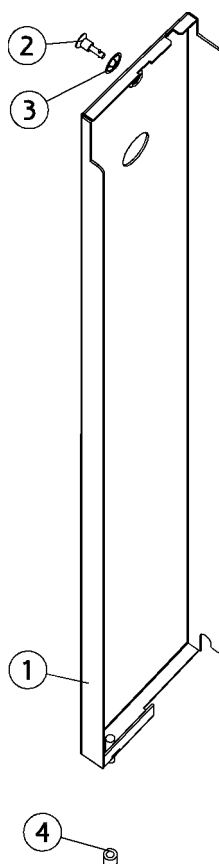


Position	Part NO.	Assembly	Description
1	AN-402		Rear panel, right, (XO-900 XO Grey, RAL 7047-80)
1	AO-251		Rear panel, right, (XO-906 XO Titanium)
1	AN-705		Rear panel, right, (XO-903 Silver Metallic)
1	AN-403		Rear panel, right, (XO-901 Sea Blue Metallic)
1	AN-988		Rear panel, right, (XO-905 XO White,(RAL7047-30))
1	AP-471		Rear panel, right, (CTBS)
2	UG-412		Lock pin, panel
3	UG-413		Lock washer, panel
4	UG-912		Hose pvc transp. Ø5xØ9

AO-129

Panel right, w/o chair

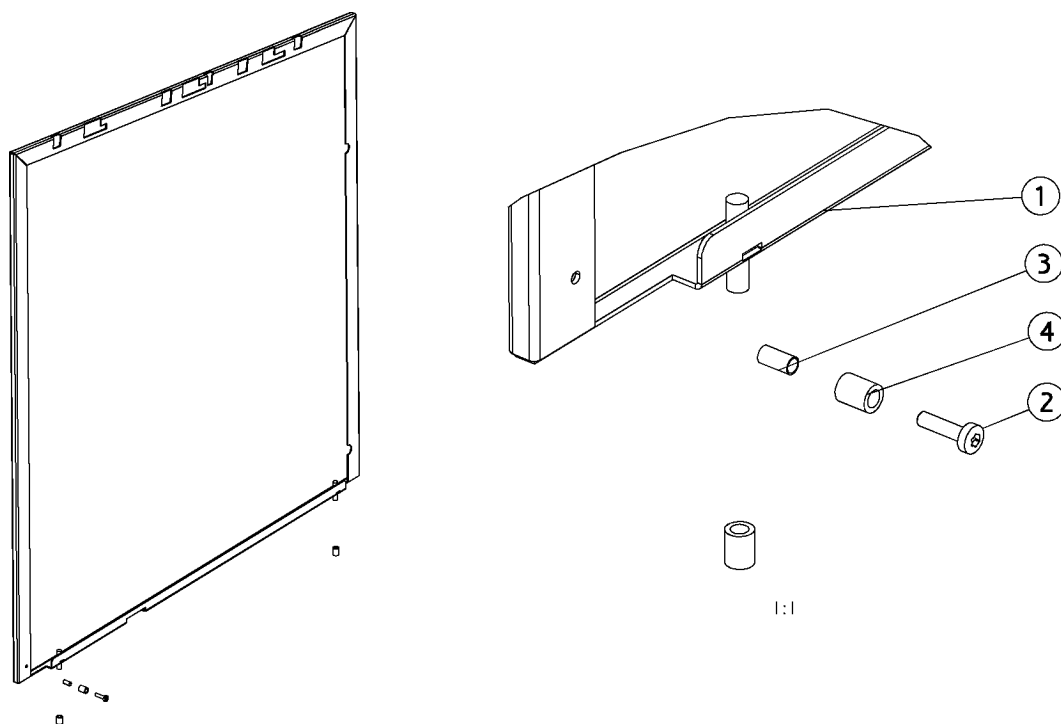

Position	Part NO.	Assembly	Description
1	AO-647		Panel right, w/o chair, (XO-900 XO Grey, RAL 7047-80)
1	AP-449		Panel right, w/o chair, (CTBS)
1	AO-303		Panel right, w/o chair, (XO-906 XO Titanium)
1	AO-648		Panel right, w/o chair, (XO-901 Sea Blue Metallic)
1	AO-695		Panel right, w/o chair, (XO-903 Silver Metallic)
1	AN-970		Panel right, w/o chair, (XO-905 XO White,(RAL7047-30))
2	UG-103		Screw, thread making, M4x16
3	UG-402		Spacer Ø4,2xØ5x10 brass
4	UG-912		Hose pvc transp. Ø5xØ9

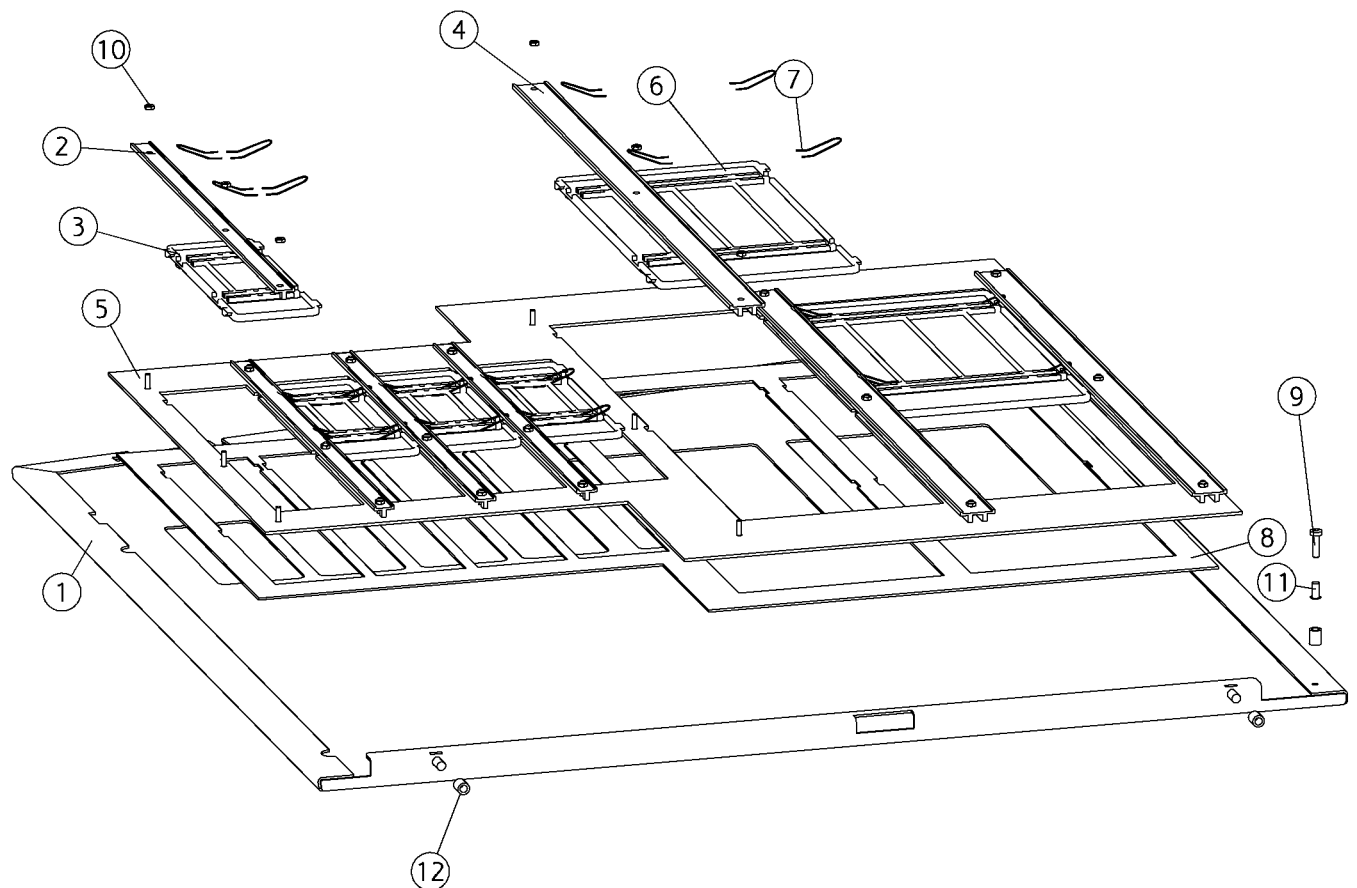


AN-078

Left side panel incl.doors

Position	Part NO.	Assembly	Description
1	AP-475		Panel left, (CTBS)
1	AN-412		Panel left, (XO-900 XO Grey, RAL 7047-80)
1	AN-413		Panel left, (XO-901 Sea Blue Metallic)
1	AN-710		Panel left, (XO-903 Silver Metallic)
1	AN-921		Panel left, (XO-905 XO White,(RAL7047-30))
1	AN-253		Panel left, (XO-901 Sea Blue Metallic)
1	AO-256		Panel left, (XO-906 XO Titanium)
1	AN-252		Panel left, (XO-900 XO Grey, RAL 7047-80)
2	MG-144		Not a spare part
3	AP-701		Not a spare part (CTBS)
3	AN-992		Not a spare part (XO-909 Iron Grey (RAL 7011-30))
3	AN-991		Not a spare part (XO-905 XO White,(RAL7047-30))
4	MG-281		Not a spare part
5	MG-282		Not a spare part
6	AP-702		Not a spare part (CTBS)
6	AN-994		Not a spare part (XO-909 Iron Grey (RAL 7011-30))
6	AN-993		Not a spare part (XO-905 XO White,(RAL7047-30))
7	MG-284		Not a spare part
8	MN-139		Not a spare part
9	UG-103		Screw, thread making, M4x16
10	UG-207		Nut, M3
11	UG-402		Spacer Ø4,2xØ5x10 brass
12	UG-912		Hose pvc transp. Ø5xØ9

AN-646


Panel right, w/o chair

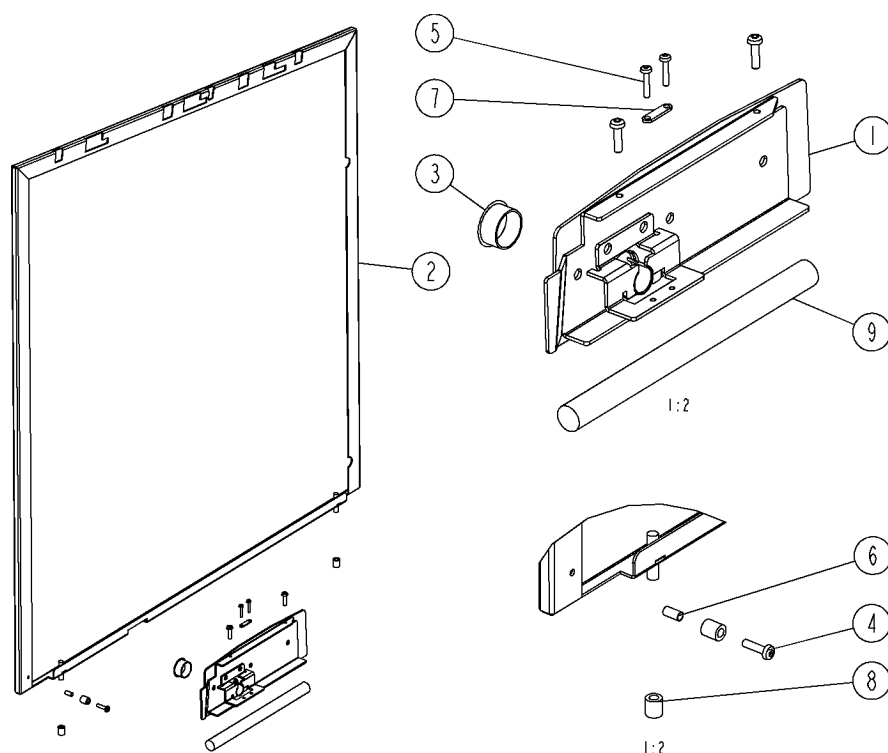


Position	Part NO.	Assembly	Description
1	AN-649		Cover supporting leg, (XO-908)
2	AO-647		Panel right, w/o chair, (XO-900 XO Grey, RAL 7047-80)
2	AP-449		Panel right, w/o chair, (CTBS)
2	AO-303		Panel right, w/o chair, (XO-906 XO Titanium)
2	AO-648		Panel right, w/o chair, (XO-901 Sea Blue Metallic)
2	AO-695		Panel right, w/o chair, (XO-903 Silver Metallic)
2	AN-970		Panel right, w/o chair, (XO-905 XO White,(RAL7047-30))
3	UC-580		Not a spare part
4	UG-103		Screw, thread making, M4x16
5	UG-223		Not a spare part
6	UG-402		Spacer Ø4,2xØ5x10 brass
7	UG-526		Not a spare part
8	UG-912		Hose pvc transp. Ø5xØ9
9	UG-970		Not a spare part

AO-124

Cover,plate, socket w/o chair,

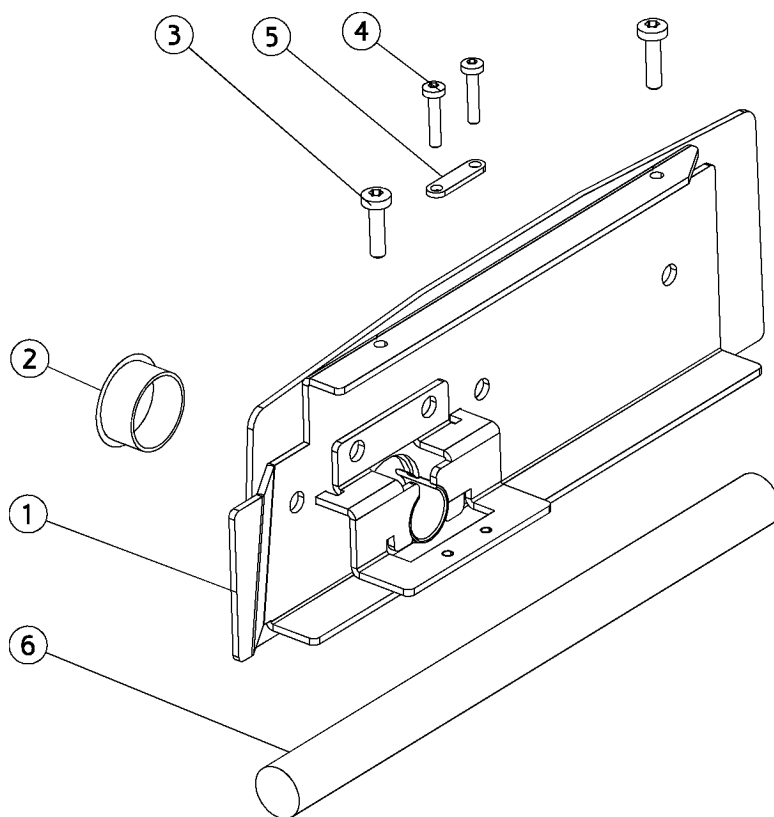


Position	Part NO.	Assembly	Description
1	AN-649		Cover supporting leg, (XO-908)
2	UC-580		Not a spare part
3	UG-103		Screw, thread making, M4x16
4	UG-223		Not a spare part
5	UG-526		Not a spare part
6	UG-970		Not a spare part

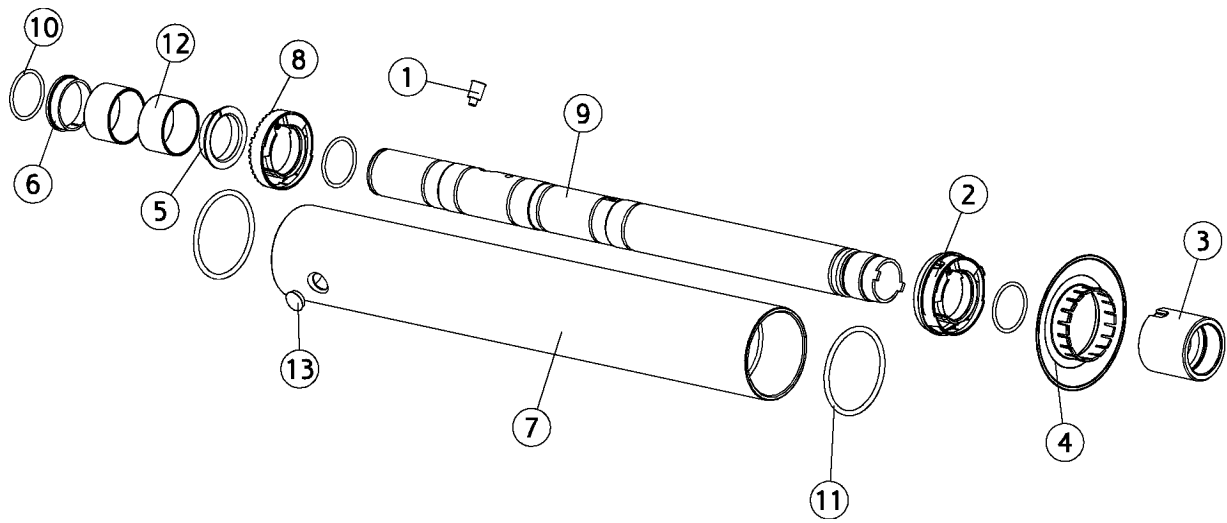

AN-056

Stand tube lamp, basic

Position	Part NO.	Assembly	Description
1	AN-137	<i>Page 54</i>	Stop,stand tube, rear arm
2	AN-138	<i>Page 53</i>	Guide, stand tube
3	MG-021		Connector sleeve, stand/lamp tube
4	AN-437		Cover plug, rear arm, bridge, (XO-901 Sea Blue Metallic)
4	AN-721		Cover plug, rear arm, bridge, (XO-903 Silver Metallic)
4	AO-267		Cover plug, rear arm, bridge, (XO-906 XO Titanium)
4	AN-932		Cover plug, rear arm, bridge, (XO-905 XO White,(RAL7047-30))
4	AP-416		Cover plug, rear arm, bridge, (CTBS)
4	AN-436		Cover plug, rear arm, bridge, (XO-900 XO Grey, RAL 7047-80)
5	MG-351		Bearing bushing 1
6	MG-352		Bearing bushing 2
7	AN-731		Tube bridge, lamp arm, (XO-903 Silver Metallic)
7	AN-457		Tube bridge, lamp arm, (XO-901 Sea Blue Metallic)
7	AP-431		Tube bridge, lamp arm, (CTBS)
7	AN-456		Tube bridge, lamp arm, (XO-900 XO Grey, RAL 7047-80)
7	AO-285		Tube bridge, lamp arm, (XO-906 XO Titanium)
7	AN-947		Tube bridge, lamp arm, (XO-905 XO White,(RAL7047-30))
8	MG-537		Guide Ø100, stand tube
9	MG-791		Tube,screen-lamp v2
10	MR-076		O-ring 47.22 x 3.53
11	UG-306		O-ring Ø085.09xØ05.34
12	UG-410		Bearing ring, Ø55xØ60x35
13	UG-421		Cover plug Ø20, black

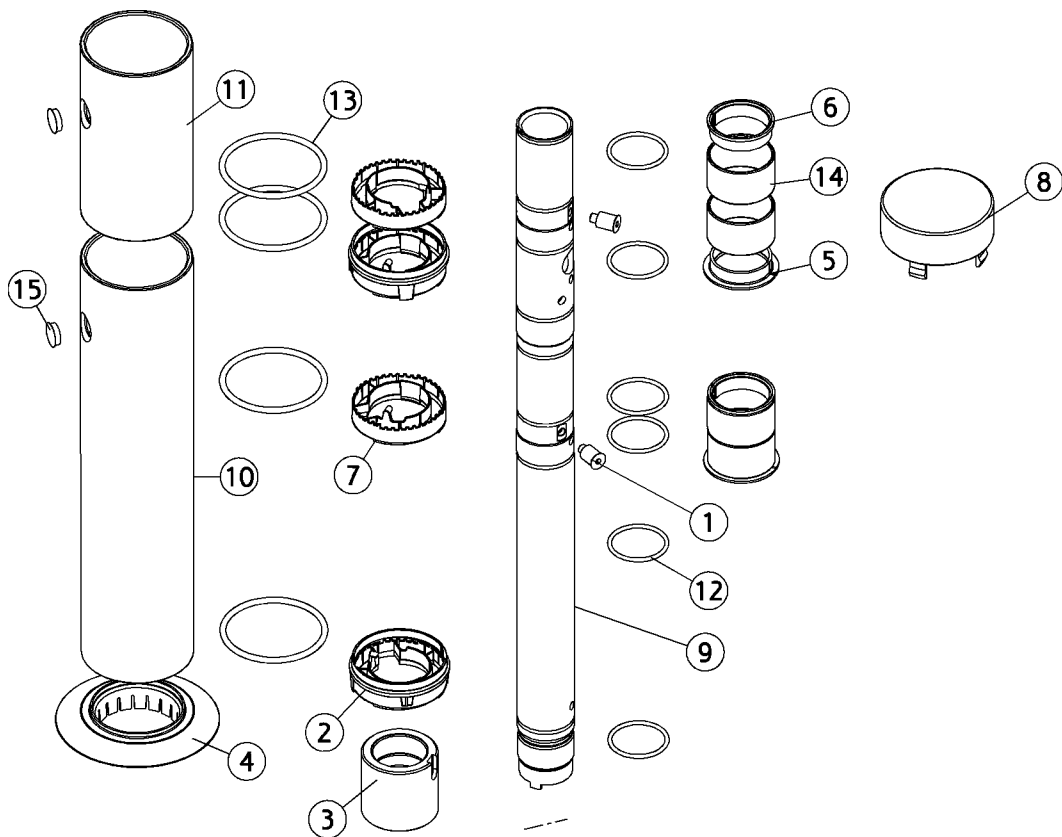

AN-136

Tube,stand,screen/lamp

Position	Part NO.	Assembly	Description
1	AN-137	<i>Page 54</i>	Stop,stand tube, rear arm
2	AN-138	<i>Page 53</i>	Guide, stand tube
3	MG-021		Connector sleeve, stand/lamp tube
4	AN-437		Cover plug, rear arm, bridge, (XO-901 Sea Blue Metallic)
4	AN-721		Cover plug, rear arm, bridge, (XO-903 Silver Metallic)
4	AO-267		Cover plug, rear arm, bridge, (XO-906 XO Titanium)
4	AN-932		Cover plug, rear arm, bridge, (XO-905 XO White,(RAL7047-30))
4	AP-416		Cover plug, rear arm, bridge, (CTBS)
4	AN-436		Cover plug, rear arm, bridge, (XO-900 XO Grey, RAL 7047-80)
5	MG-351		Bearing bushing 1
6	MG-352		Bearing bushing 2
7	MG-537		Guide Ø100, stand tube
8	AN-634		Cover plug, screen w/o lamp, (XO-901 Sea Blue Metallic)
8	AO-302		Cover plug, screen w/o lamp, (XO-906 XO Titanium)
8	AP-448		Cover plug, screen w/o lamp, (CTBS)
8	AN-902		Cover plug, screen w/o lamp, (XO-903 Silver Metallic)
8	AN-969		Cover plug, screen w/o lamp, (XO-905 XO White,(RAL7047-30))
8	AN-633		Cover plug, screen w/o lamp, (XO-900 XO Grey, RAL 7047-80)
9	MG-791		Tube,screen-lamp v2
10	AN-809		Tube,bridge-screen arm, (XO-900 XO Grey, RAL 7047-80)
10	AN-811		Tube,bridge-screen arm, (XO-903 Silver Metallic)
10	AN-971		Tube,bridge-screen arm, (XO-905 XO White,(RAL7047-30))
10	AP-450		Tube,bridge-screen arm, (CTBS)
10	AN-810		Tube,bridge-screen arm, (XO-901 Sea Blue Metallic)
10	AO-306		Tube,bridge-screen arm, (XO-906 XO Titanium)
11	AN-814		Tube,screen-lamp arm, (XO-903 Silver Metallic)
11	AN-813		Tube,screen-lamp arm, (XO-901 Sea Blue Metallic)
11	AN-812		Tube,screen-lamp arm, (XO-900 XO Grey, RAL 7047-80)
11	AO-307		Tube,screen-lamp arm, (XO-906 XO Titanium)
11	AP-451		Tube,screen-lamp arm, (CTBS)
11	AN-972		Tube,screen-lamp arm, (XO-905 XO White,(RAL7047-30))
12	MR-076		O-ring 47.22 x 3.53


AN-136

Tube,stand,screen/lamp

Position	Part NO.	Assembly	Description
13	UG-306		O-ring Ø085.09xØ05.34
14	UG-410		Bearing ring, Ø55xØ60x35
15	UG-421		Cover plug Ø20, black

AN-138


Guide, stand tube

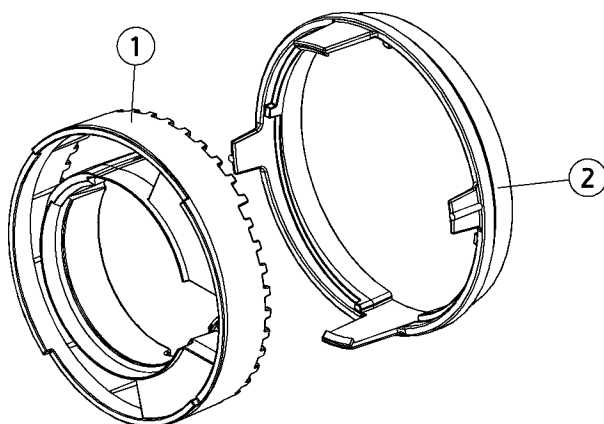

Position	Part NO.	Assembly	Description
1	MG-537		Guide Ø100, stand tube
2	MG-538		Union, guide, Ø100 stand tube

AN-137


Stop,stand tube, rear arm

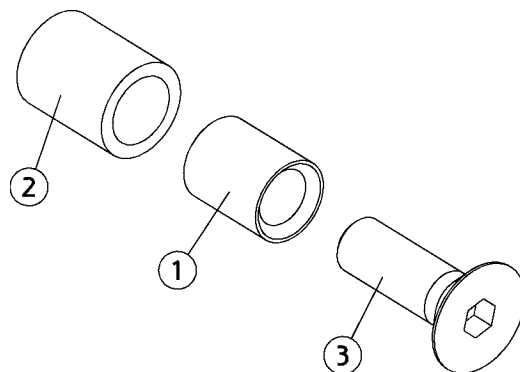

Position	Part NO.	Assembly	Description
1	MG-539		Not a spare part
2	UC-829		Not a spare part
3	UG-033		Not a spare part

AN-518

X-ray adaptor, general parts

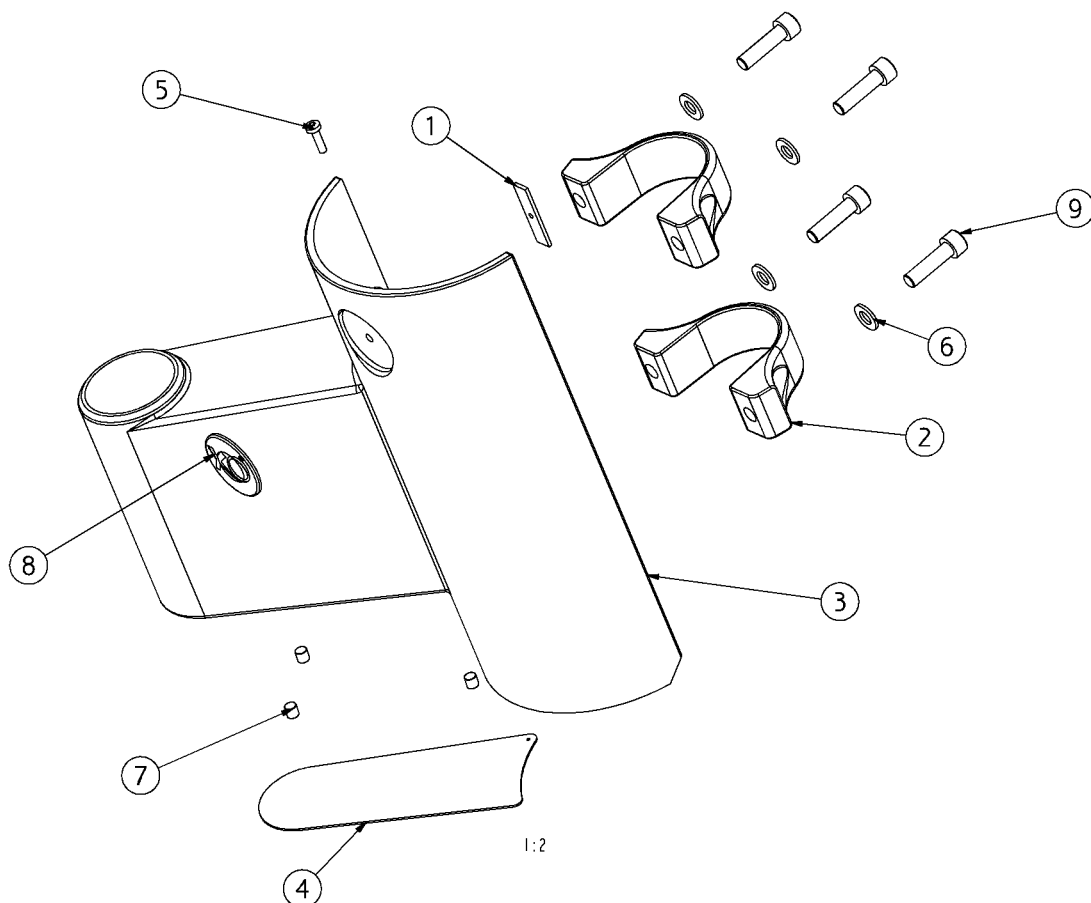


AN-518

X-ray adaptor, general parts

Position	Part NO.	Assembly	Description
1	MG-626		Mounting plate, logo
2	MG-629		Not a spare part
3	AO-300		X-ray adaptor, (XO-906 XO Titanium)
3	AP-851		X-ray adaptor, (CTBS)
3	AN-536		X-ray adaptor, (XO-901 Sea Blue Metallic)
3	AN-535		X-ray adaptor, (XO-900 XO Grey, RAL 7047-80)
3	AN-967		X-ray adaptor, (XO-905 XO White,(RAL7047-30))
3	AN-749		X-ray adaptor, (XO-903 Silver Metallic)
4	AN-537		Bottom plate,x-ray adp, (XO-900 XO Grey, RAL 7047-80)
4	AO-301		Bottom plate,x-ray adp, (XO-906 XO Titanium)
4	AP-447		Bottom plate,x-ray adp, (CTBS)
4	AN-750		Bottom plate,x-ray adp, (XO-903 Silver Metallic)
4	AN-968		Bottom plate,x-ray adp, (XO-905 XO White,(RAL7047-30))
4	AN-538		Bottom plate,x-ray adp, (XO-901 Sea Blue Metallic)
5	UG-103		Screw, thread making, M4x16
6	UG-252		Not a spare part
7	UG-414		Magnet Ø6,35x6,35 neodymium
8	UG-441		Logo Ø40
9	UG-933		Not a spare part

AN-531


Bush, Ø32,1

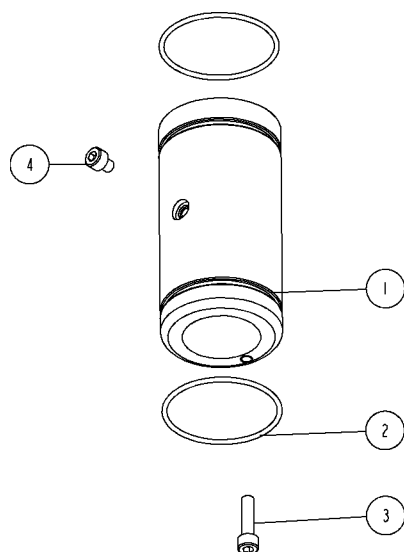

Position	Part NO.	Assembly	Description
1	MG-628		Bush, x-ray adapter
2	MR-310		O-ring 45 x 2
3	UA-574		Not a spare part
4	UG-070		Not a spare part


AN-847

Assembly, bushes for xray

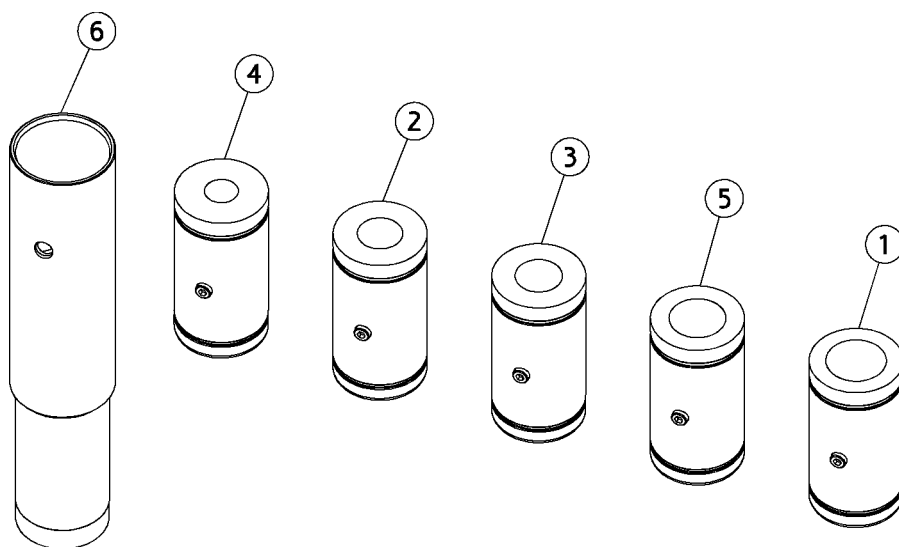

Position	Part NO.	Assembly	Description
1	AN-531	<i>Page 57</i>	Bush, Ø32,1
2	AN-532		Bush, Ø24,1
3	AN-533		Bush, Ø25,1
4	AN-534		Bush, ø??
5	AN-843		Bush, Ø30,1
6	AO-383		Extension 160mm, x-ray, (XO-903 Silver Metallic)
6	AO-381		Extension 160mm, x-ray, (XO-900 XO Grey, RAL 7047-80)
6	AO-082		Extension 160mm, x-ray, (XO-905 XO White,(RAL7047-30))
6	AO-384		Extension 160mm, x-ray, (XO-901 Sea Blue Metallic)
6	AO-314		Extension 160mm, x-ray, (XO-906 XO Titanium)
6	AP-458		Extension 160mm, x-ray, (CTBS)


XO 4 Stand, suction

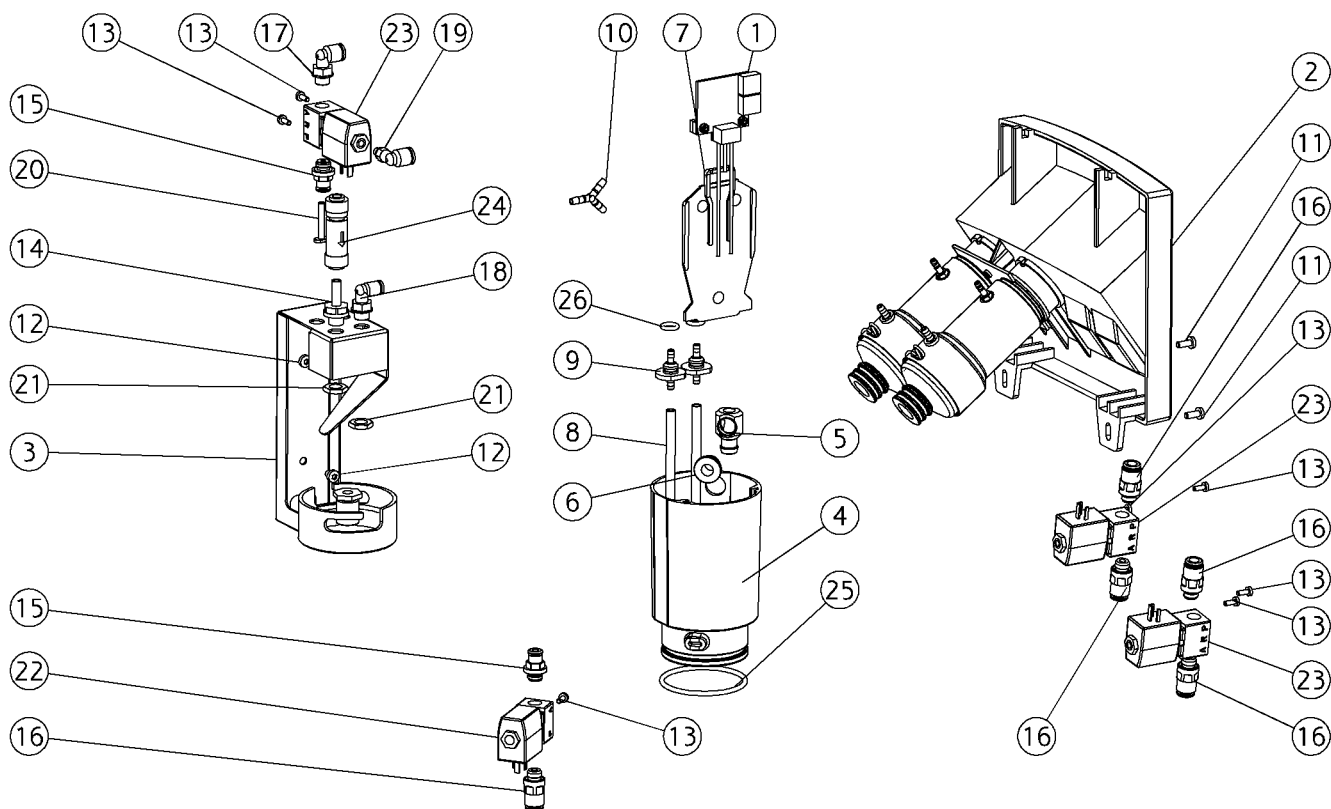

AN-077

Suction, flushing unit


Position	Part NO.	Assembly	Description
1	AN-298		PCB, vessel, level
2	AN-062	<i>Page 62</i>	Suction valves
3	MG-348		Bracket, suction disinfection
4	MG-364		Mixing cup
5	MG-561		Nipple, over run, vessel
6	MG-715		Plug with o-ring Ø8 drilled
7	MG-767		Separation wall,vessel
8	MN-476		Not a spare part
9	MN-483		Not a spare part
10	UC-127		Y-shaped connection piece
11	UG-110		Screw, thread making, M4x10
12	UG-111		Screw, thread making, M4x6
13	UG-139		Screw. M3x8
14	UG-654		Quick coupling Ø6-r1/8 plastic
15	UG-659		Fitting, 4mm-R1/8 viton, NSF
16	UG-662		Quick connect, 6/4-R1/8
17	UG-701		Quick coupling, 6/4-r1/8-90
18	UG-703		Quick coupling, 4/2-r1/8-90
19	UG-706		Quick connect, 6/4-M5-90
20	UG-719		Quick connector, plug 4/2
21	UG-799		Nut, stainless steel 1/8
22	UG-800		Valve 2/2 - 1/8 wg
23	UG-801		Valve 3/2 - 1/8 wg
24	UH-057		Non return valve, 4/6 EPDM
25	UH-246		O-ring 50 x 3 Viton, 75 SH
26	UH-309		Not a spare part

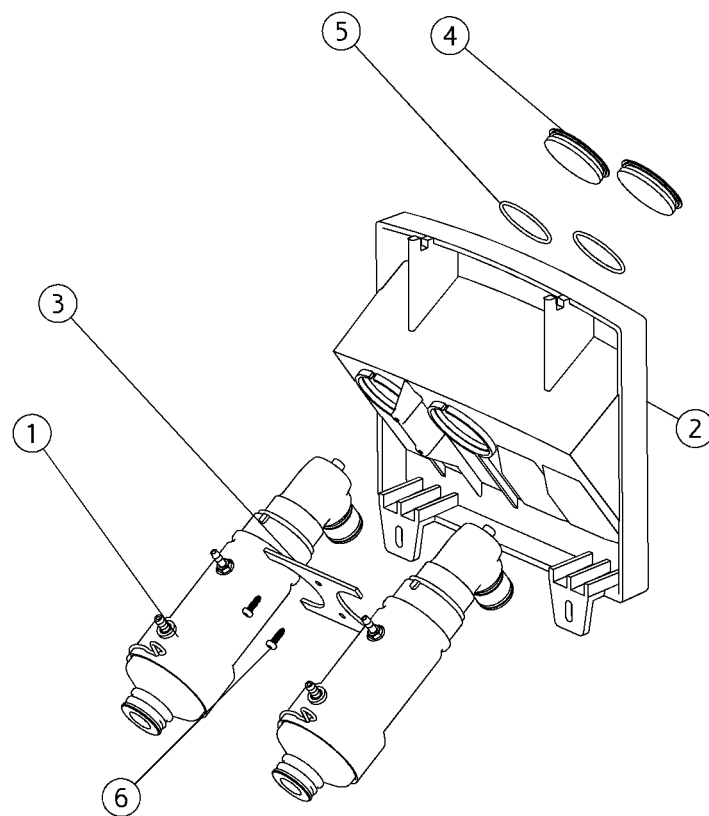

AN-062

Suction valves


Position	Part NO.	Assembly	Description
1	AN-072	<i>Page 64</i>	Suction valve, suction select
2	AO-725		Front, suction valve, (XO-903 Silver Metallic)
2	AO-197		Front, suction valve, (XO-905 XO White,(RAL7047-30))
2	AO-272		Front, suction valve, (XO-906 XO Titanium)
2	AO-444		Front, suction valve, (XO-900 XO Grey, RAL 7047-80)
2	AO-445		Front, suction valve, (XO-901 Sea Blue Metallic)
2	AP-460		Front, suction valve, (CTBS)
3	MG-155		Bracket, suction valves
4	MG-822		Not a spare part
5	UC-478		Not a spare part
6	UG-238		Screw, plastic M3x10

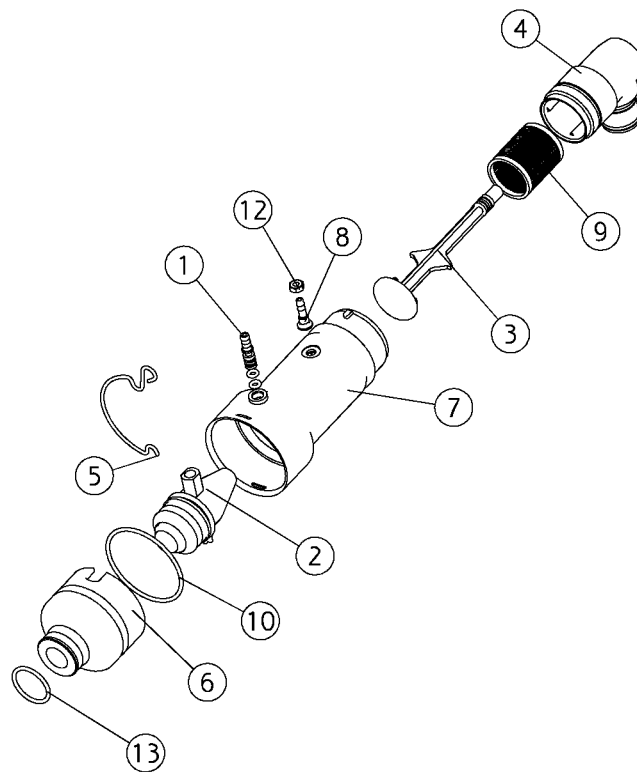

AN-072

Suction valve, suction select


Position	Part NO.	Assembly	Description
1	AC-033		Tube connector, suction
2	AE-124		Holder with bellow, selectvac
3	AN-069	<i>Page 66</i>	Filter holder suction XO 4
4	AN-070	<i>Page 67</i>	Filter tube selectvac XO 4
5	MC-105		Retaining spring, select vac
6	MG-278		Filterhouse, small, suc.valve
7	MG-477		House, filter, select valve
8	MN-164		Tube connector, stainless
9	MR-075		Filter tube
10	MR-210		O-ring 45 x 2
11	UC-039		O-ring Ø5 x 1.5
12	UG-205		Nut, M4, din 934
13	UG-320		O-ring, 20x2

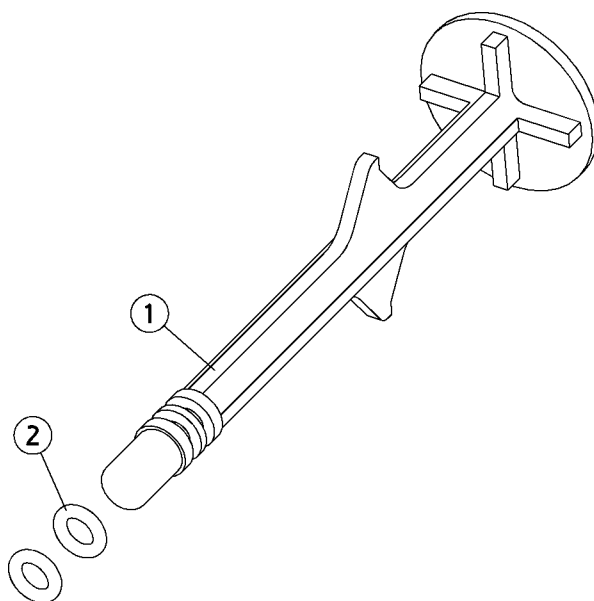

Position	Part NO.	Assembly	Description
1	MG-478		Not a spare part
2	UC-040		O-ring Ø4.48 x 1.78

AN-070


Filter tube selectvac XO 4

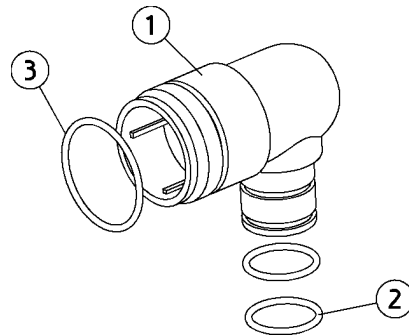

Position	Part NO.	Assembly	Description
1	MG-479		Filter holder,suction valve
2	UC-063		O-ring Ø17.17 x 1.78
3	UC-064		O-ring Ø28.30 x 1.78

AN-068


Suction tube Ø11 w. nipples,l=1650mm

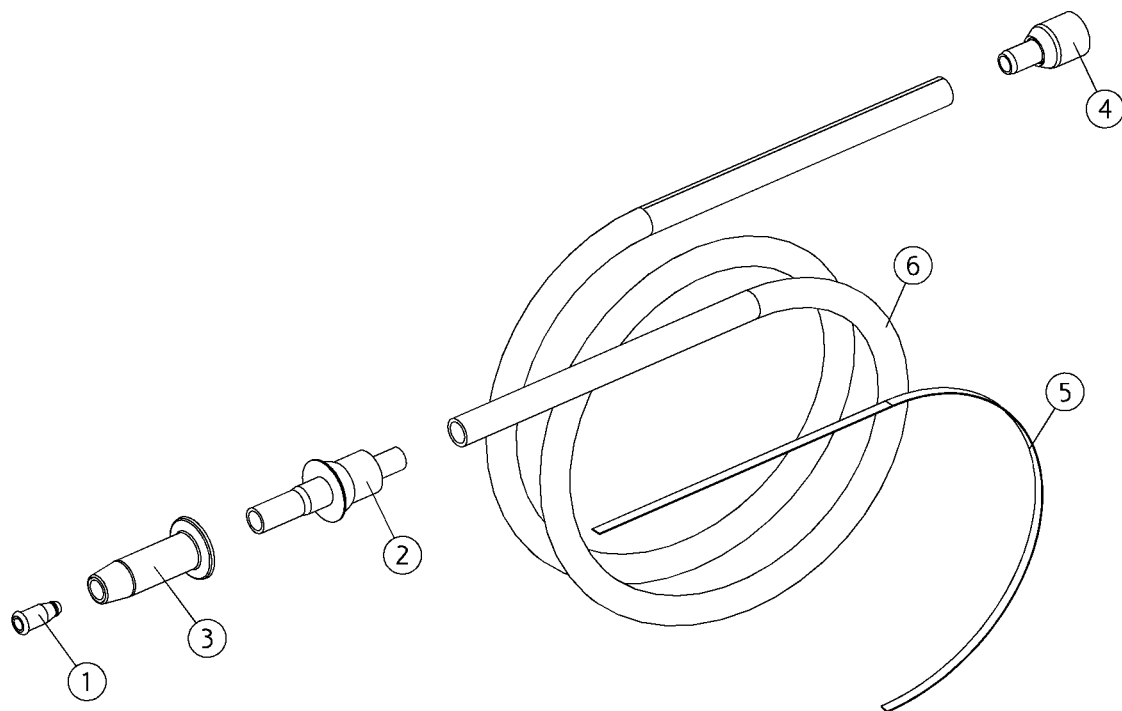


Position	Part NO.	Assembly	Description
1	MC-263		Reduction, suction Ø11 - 7
2	MG-476		Suction nipple, Ø11,
3	MG-481		Jacket, suction tube nipple Ø11
4	MG-483		Nipple, suction hose Ø11,
5	UC-692		Strapex ribbon 6 mm, white
6	UH-194		Hose, suction, Ø12, grey, (RAL 7011)

AN-067

Suction tube Ø16 w. nipples,l=1650mm

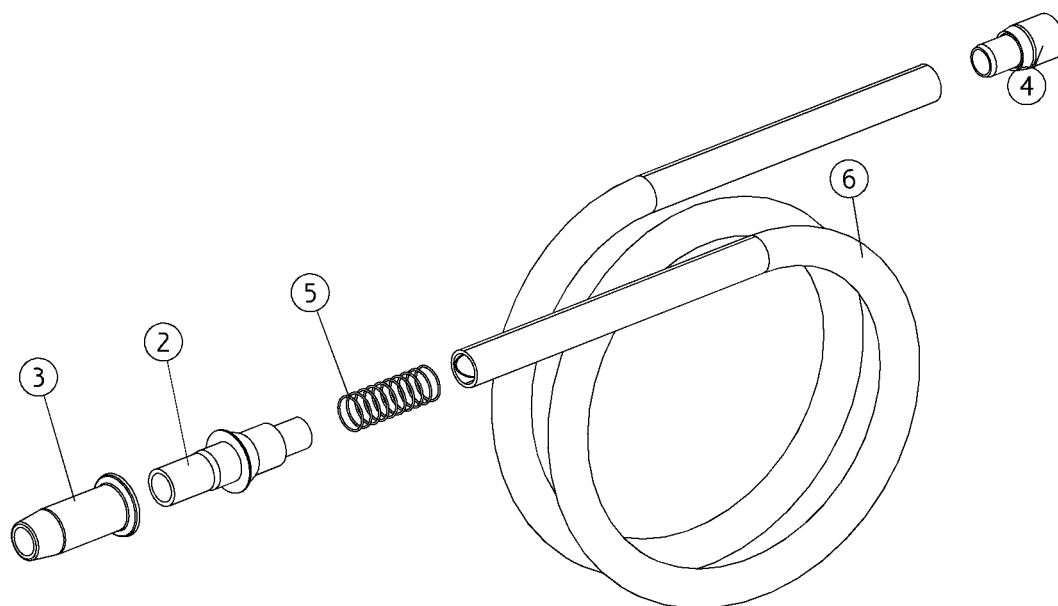


Position	Part NO.	Assembly	Description
1	ME-160		Innertubing for Ø16, suc.tube
2	MG-475		Suction nipple,Ø16,
3	MG-480		Jacket,suction tube nipple Ø16
4	MG-482		Nipple, suction hose Ø16,
5	UC-852		O-ring Ø18 x 1 black
6	UH-193		Hose, suction, Ø17, grey (RAL 7011)

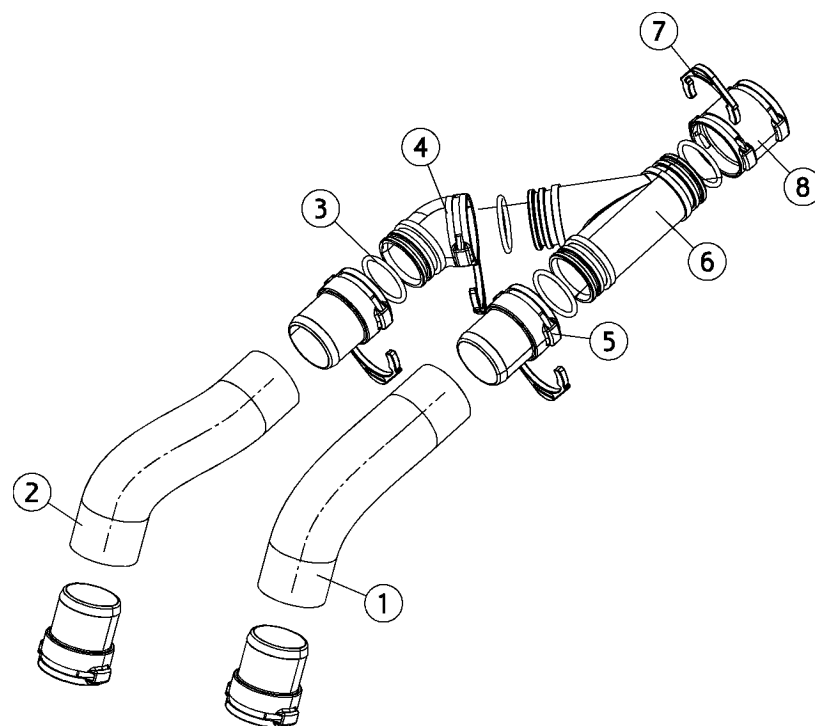

Position	Part NO.	Assembly	Description
1	MS-356		Hose Ø24, black
2	MS-356		Hose Ø24, black
3	UG-320		O-ring, 20x2
4	UG-721		Dürr connect, joint link-45
5	UG-725		Dürr connect, Ø25, female
6	UG-728		Dürr connect, y-piece
7	UG-730		Dürr connect, clips, blue
8	UG-731		Dürr connect, 2-female

AN-066

Cuspidor valve XO 4

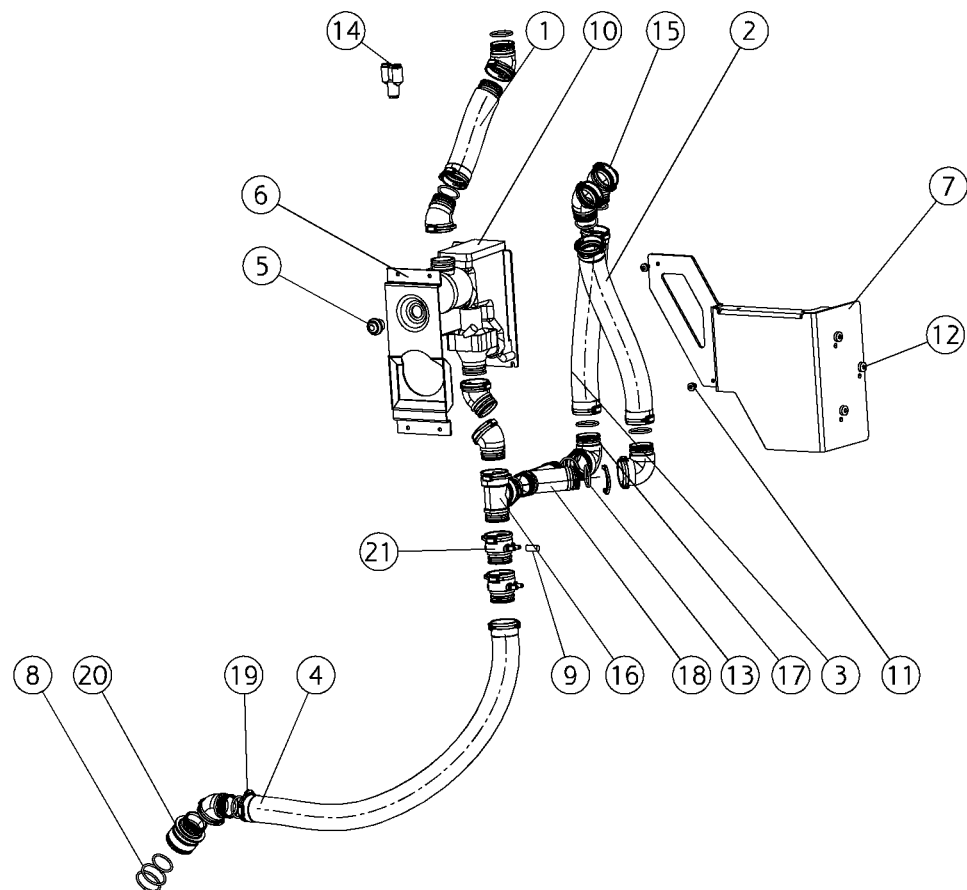


AN-066

Cuspidor valve XO 4

Position	Part NO.	Assembly	Description
1	AN-074	<i>Page 73</i>	Tube, Ø25, cuspidor valve
2	AN-075	<i>Page 74</i>	Tube Ø25, cuspidor valve
3	AN-076	<i>Page 75</i>	Tube, Ø25, cuspidor valve
4	AN-183	<i>Page 76</i>	Tube, Ø25, cusp.valve, suction
5	CA-037		Switch +cable, suction disin.
6	MN-148		Not a spare part
7	MN-209		Not a spare part
8	MR-212		Not a spare part
9	UC-742		Cover plug Ø4 mm
10	UC-823		Cuspidor valve, Dürr
11	UG-111		Screw, thread making, M4x6
12	UG-222		Not a spare part
13	UG-320		O-ring, 20x2
14	UG-691		Quick coupling, Y piece 4/2
15	UG-721		Dürr connect, joint link-45
16	UG-723		Dürr conn.t-piece,2-female,1-male
17	UG-724		Dürr connect, angle-90
18	UG-728		Dürr connect, y-piece
19	UG-730		Dürr connect, clips, blue
20	UG-732		Not a spare part
21	UG-736		Dürr connect, Ø25 male/female,2 suction

AN-074


Tube, Ø25, cuspidor valve

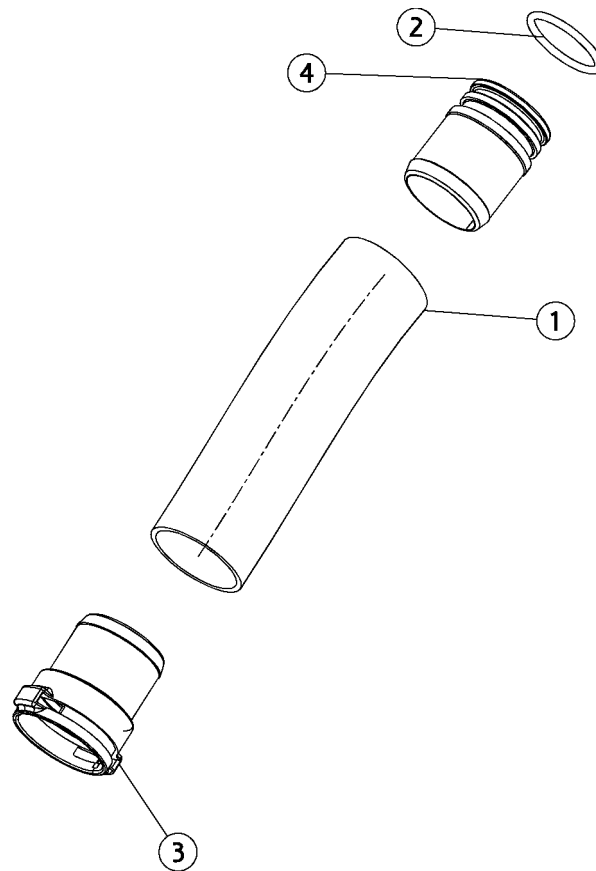

Position	Part NO.	Assembly	Description
1	MS-356		Hose Ø24, black
2	UG-320		O-ring, 20x2
3	UG-725		Dürr connect, Ø25, female
4	UG-726		Dürr connect, Ø25, male

AN-075


Tube Ø25, cuspidor valve

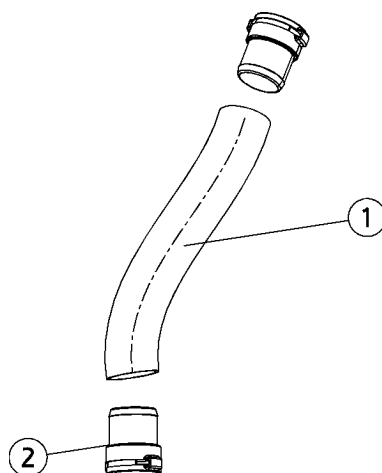

Position	Part NO.	Assembly	Description
1	MS-356		Hose Ø24, black
2	UG-725		Dürr connect, Ø25, female

AN-076


Tube, Ø25, cuspidor valve

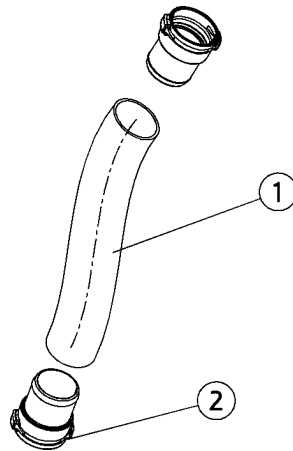

Position	Part NO.	Assembly	Description
1	MS-356		Hose Ø24, black
2	UG-725		Dürr connect, Ø25, female

AN-183


Tube, Ø25, cusp.valve, suction

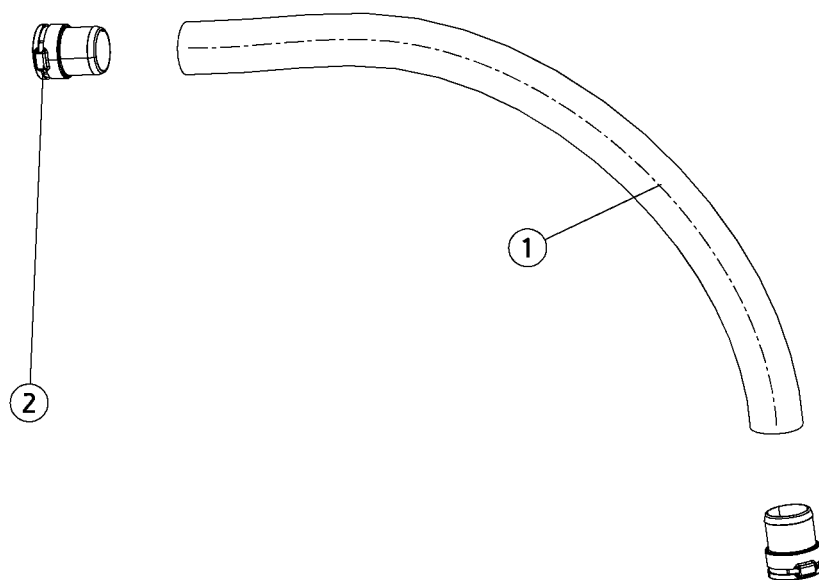

Position	Part NO.	Assembly	Description
1	MS-356		Hose Ø24, black
2	UG-725		Dürr connect, Ø25, female

AN-334

Amalgam separator,combi

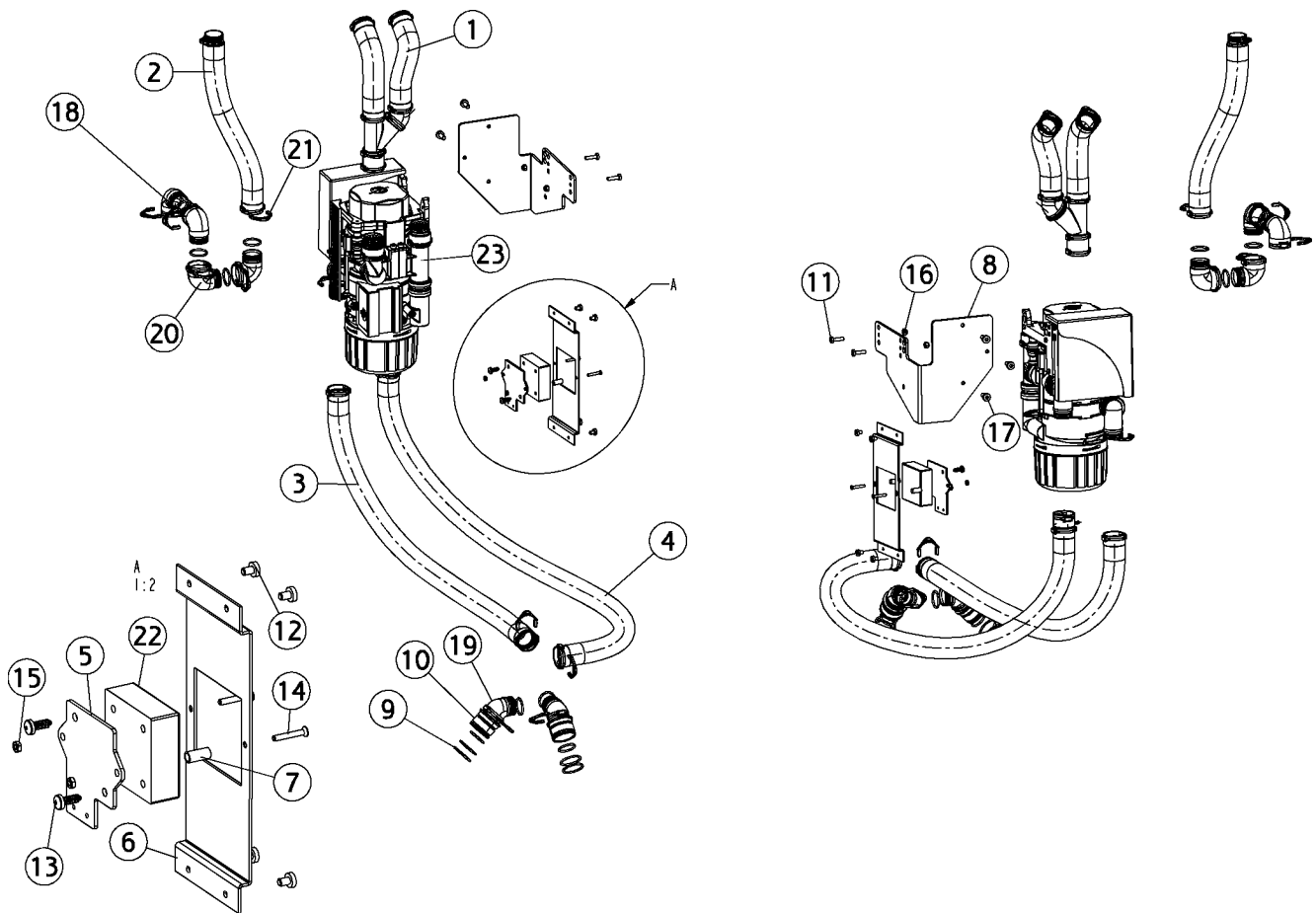


AN-334

Amalgam separator,combi

Position	Part NO.	Assembly	Description
1	AN-176	<i>Page 70</i>	Tube,suction valve,combi
2	AN-177	<i>Page 79</i>	Tube, cuspidor, combi
3	AN-178	<i>Page 80</i>	Tube, drain, combi
4	AN-179	<i>Page 81</i>	Tube,suction supply,combi
5	MC-900		Mount.fitting,display,Dürr
6	MG-133		Not a spare part
7	MG-485		Bush, control, cuspidor valve
8	MG-618		Mounting bracket, cas1
9	MR-212		Not a spare part
10	UC-827		Tube connector, Ø20-Ø32
11	UG-103		Screw, thread making, M4x16
12	UG-111		Screw, thread making, M4x6
13	UG-148		Plastic screw, m04x12
14	UG-149		Screw, M3x20
15	UG-207		Nut, M3
16	UG-209		Not a spare part
17	UG-222		Not a spare part
18	UG-320		O-ring, 20x2
19	UG-721		Dürr connect, joint link-45
20	UG-724		Dürr connect, angle-90
21	UG-730		Dürr connect, clips, blue
22	UG-919		Controlpanel, Dürr cas1
23	UG-920		Combi separator,cas1,Dürr,XO 4

AN-177


Tube, cuspidor, combi

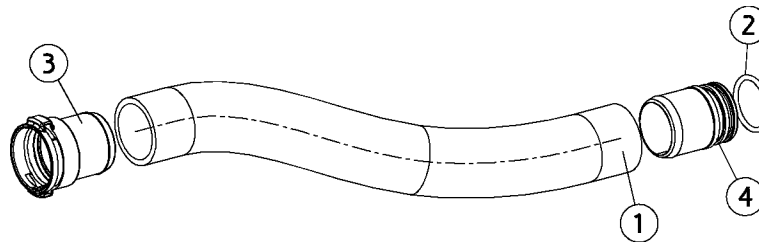

Position	Part NO.	Assembly	Description
1	MS-356		Hose Ø24, black
2	UG-320		O-ring, 20x2
3	UG-725		Dürr connect, Ø25, female
4	UG-726		Dürr connect, Ø25, male

AN-178


Tube, drain, combi

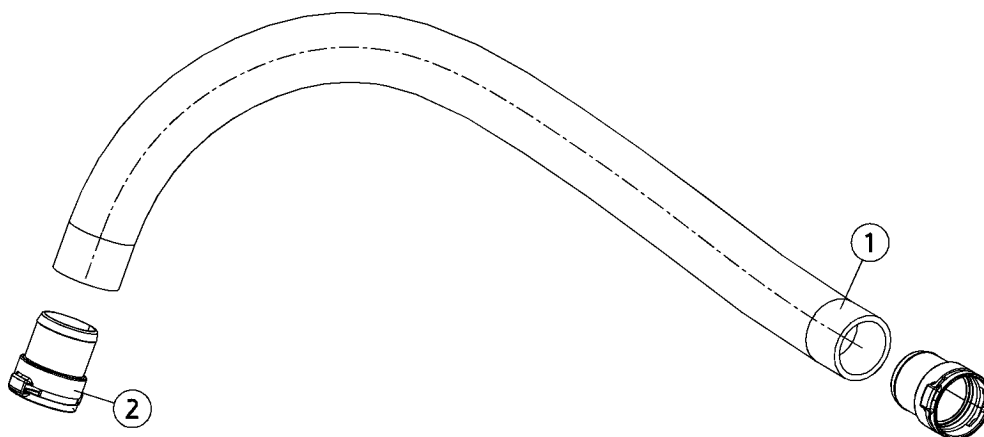

Position	Part NO.	Assembly	Description
1	MS-356		Hose Ø24, black
2	UG-725		Dürr connect, Ø25, female

AN-179

Tube,suction supply,combi

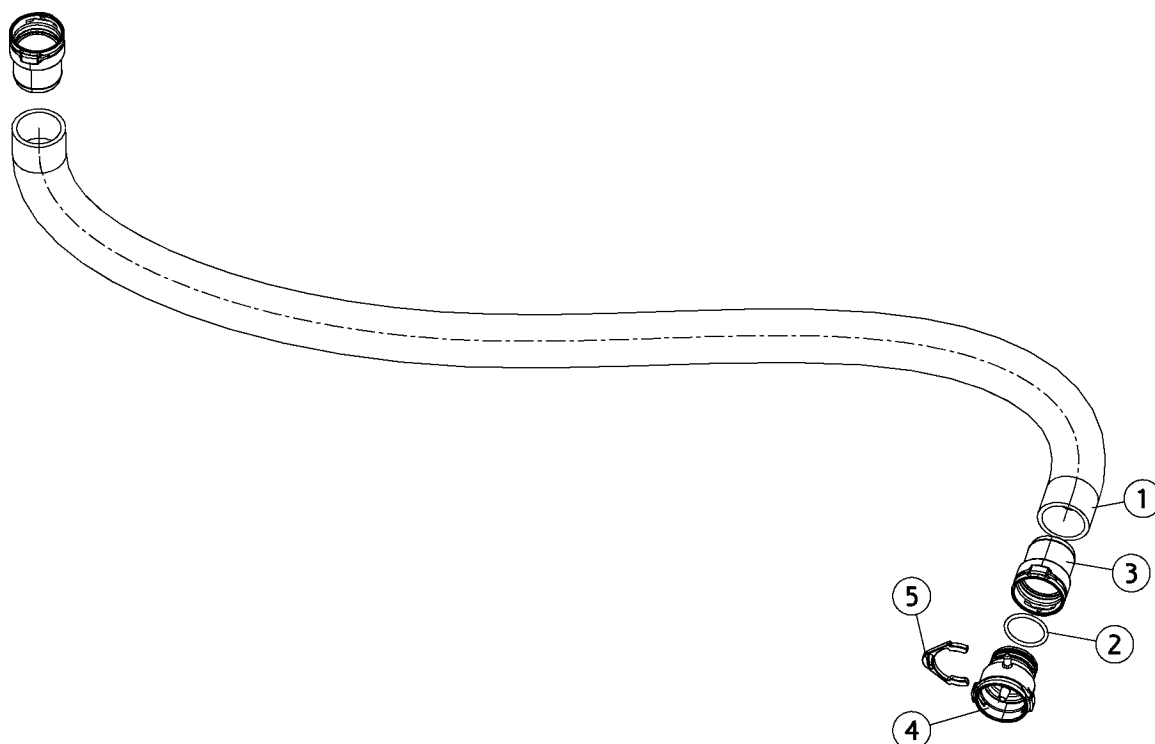

Position	Part NO.	Assembly	Description
1	MS-356		Hose Ø24, black
2	UG-320		O-ring, 20x2
3	UG-725		Dürr connect, Ø25, female
4	UG-729		Not a spare part
5	UG-730		Dürr connect, clips, blue


AN-179

Tube,suction supply,combi

XO 4 Unit stand, system for suction clean

AN-063


Suction clean

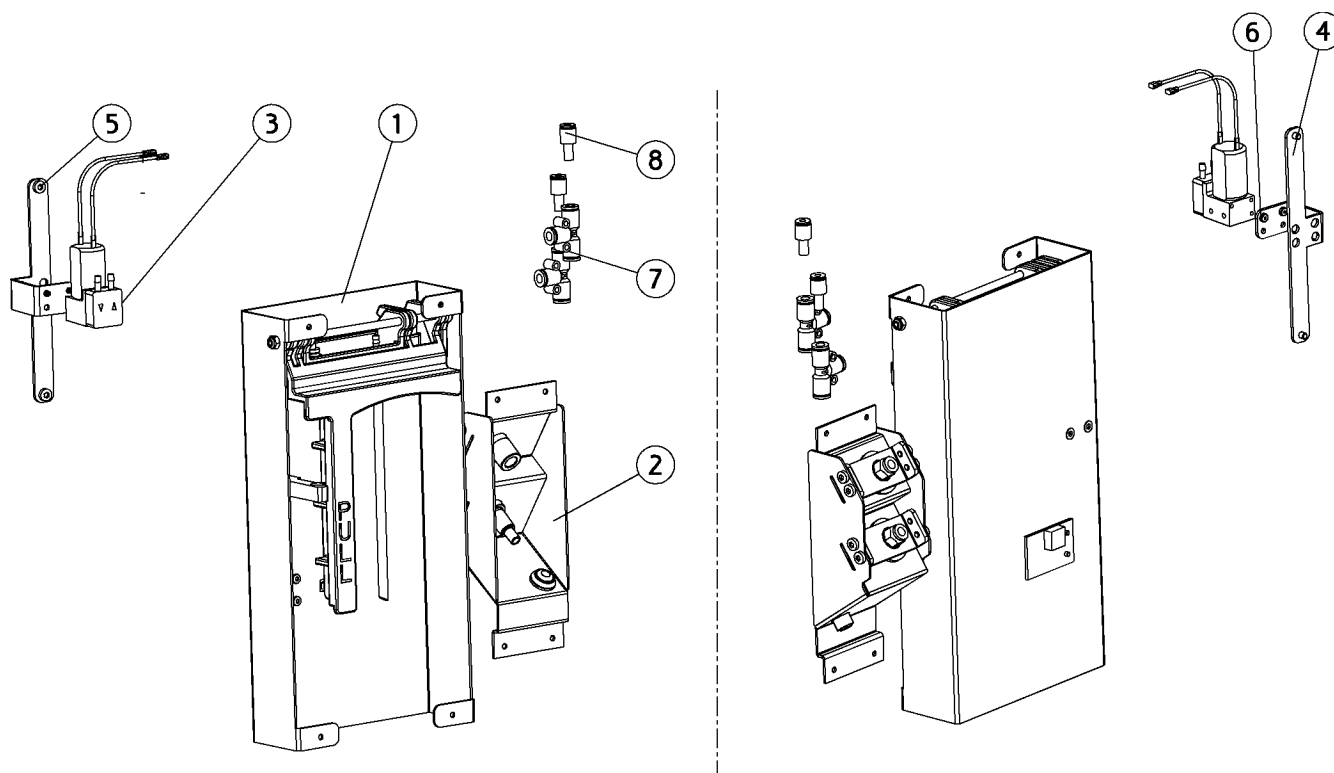


Position	Part NO.	Assembly	Description
1	AN-047	Page 85	Service module, suction clean
2	AN-164	Page 86	Service module, suction clean
3	AO-346		Pump, with connectors
4	MG-677		Bracket, pump
5	UG-111		Screw, thread making, M4x6
6	UG-224		Screw for plastic, M3x6
7	UG-708		Quick coupling, t-piece, 6/4
8	UG-717		Quick fitting, 6/4-4/2

AN-047

Service module, suction clean

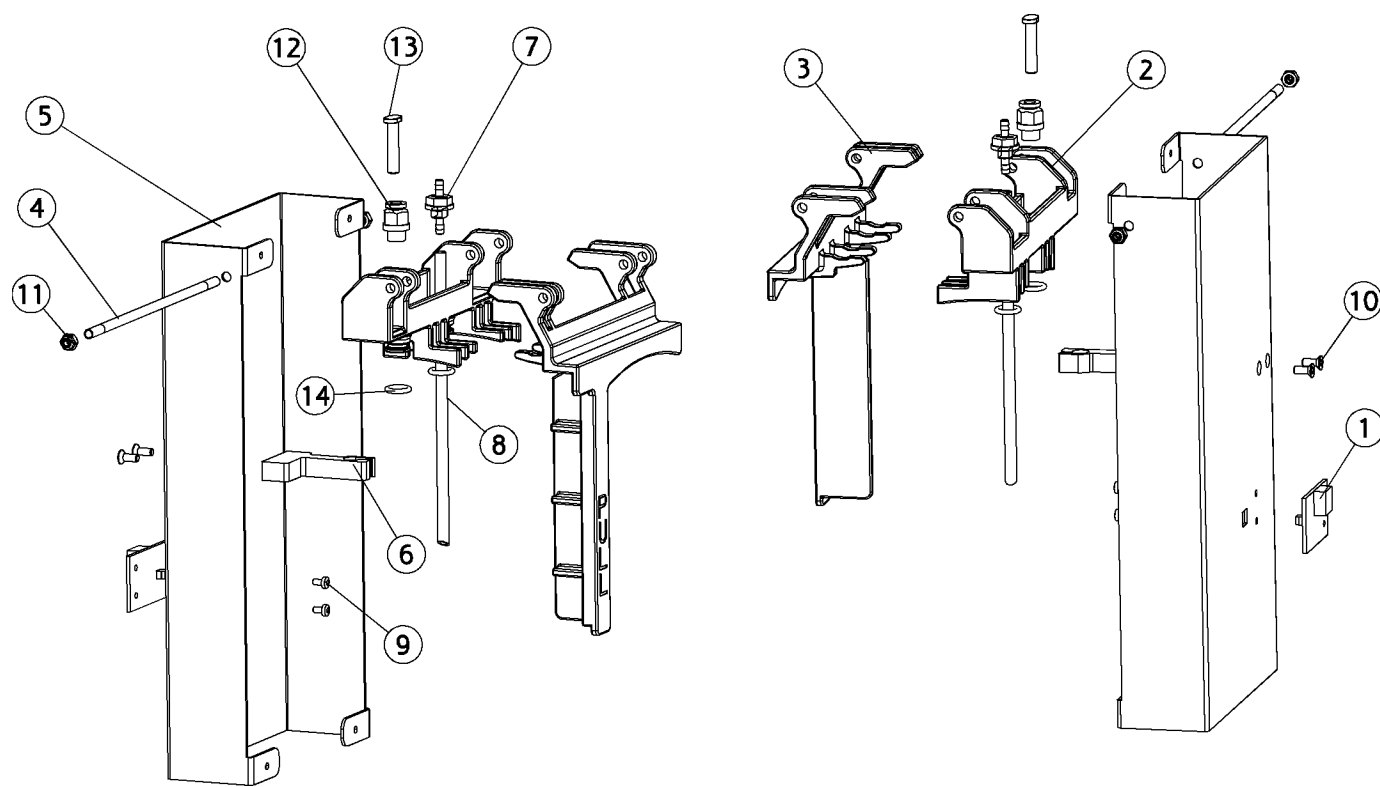

Position	Part NO.	Assembly	Description
1	AN-374		PCB, Level sensor, XO 4
2	MG-361		Receiver, XO suction disinfec.
3	MG-398		Holder for suc.dis. container
4	MG-436		Not a spare part
5	MG-497		Not a spare part
6	MG-596		Snap lock, clips
7	MG-914		Fitting, tubing 1/8
8	MG-917		Tubing, 4 X 6 X 159 yellow
9	UG-102		Screw, thread making, M3x006 DIN7500
10	UG-115		Screw, M4x010
11	UG-208		Locknut M5
12	UG-700		Quick connect, 6/4-r1/8
13	UG-718		Not a spare part
14	UH-046		O-ring, Ø9,30x2,4 EPDM black

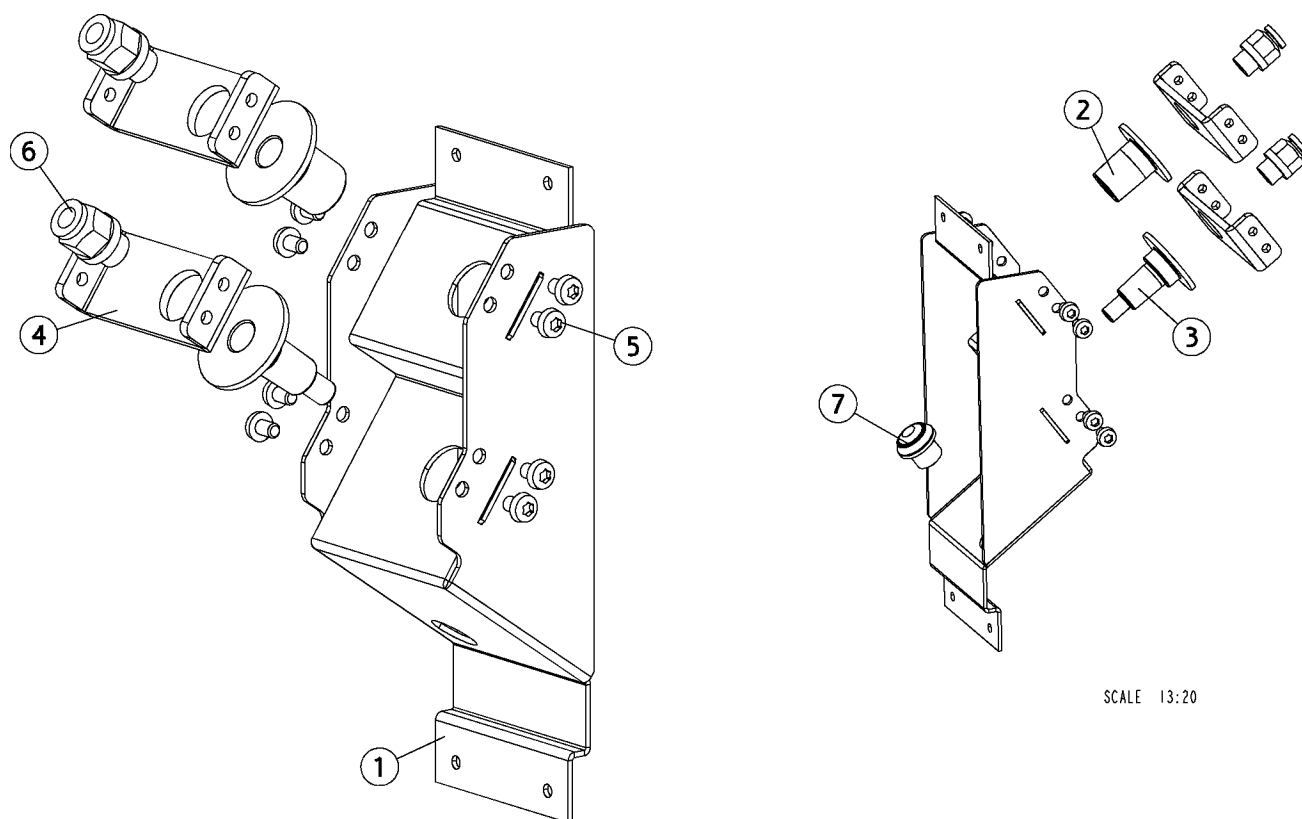


Position	Part NO.	Assembly	Description
1	MG-429		Insert, XO suction disinfection
2	MG-430		Connector suction disinfection
3	MG-431		Not a spare part
4	MG-507		Tightening fittings suc.clean
5	UG-111		Screw, thread making, M4x6
6	UG-700		Quick connect, 6/4-r1/8
7	UG-911		Not a spare part

XO 4 Unit stand, system for water clean

AN-166


Water clean

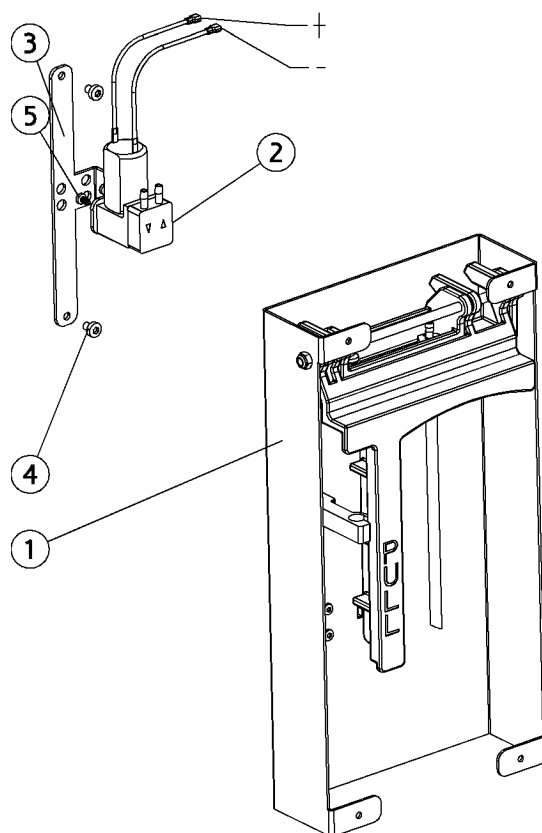


Position	Part NO.	Assembly	Description
1	AN-167	Page 89	Water clean, module
2	AO-346		Pump, with connectors
3	MG-677		Bracket, pump
4	UG-111		Screw, thread making, M4x6
5	UG-224		Screw for plastic, M3x6

AN-167

Water clean, module

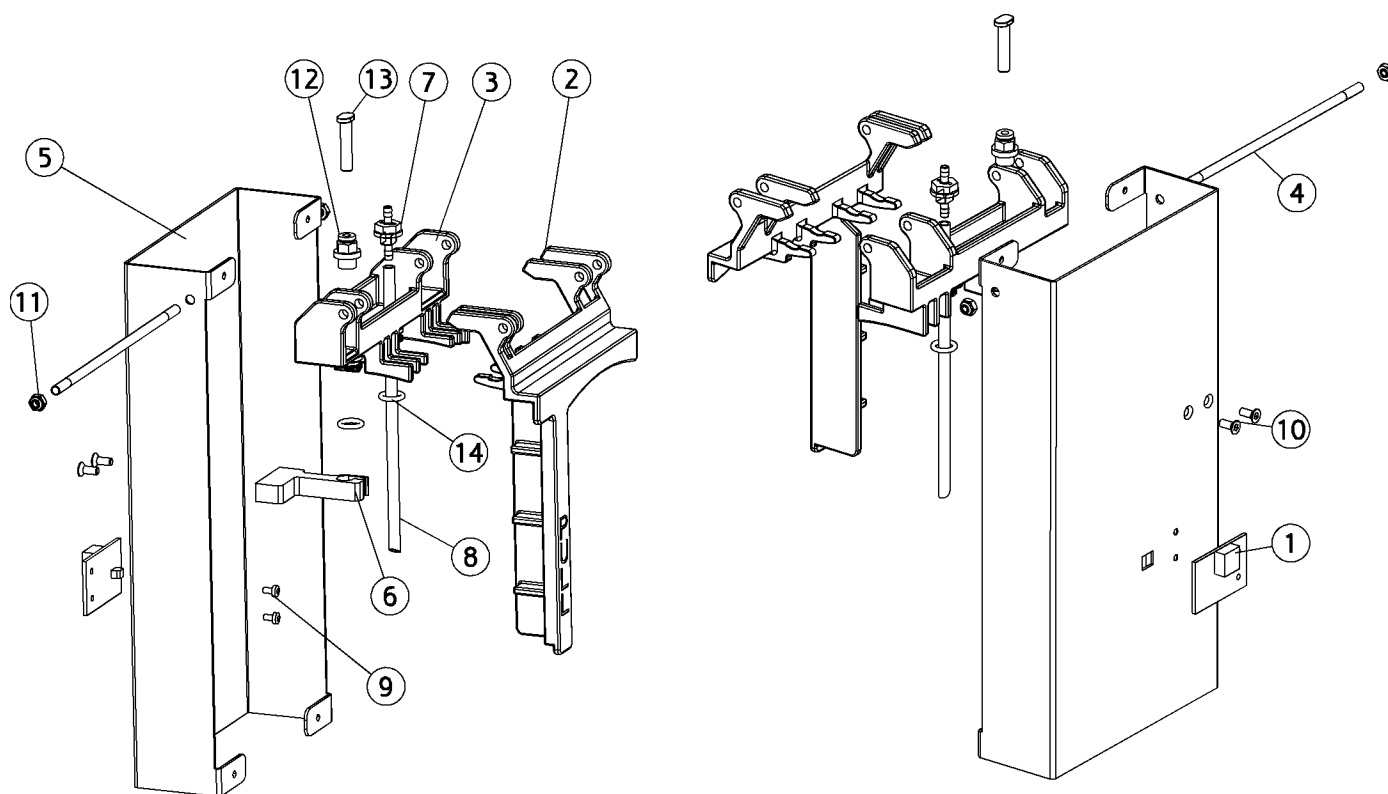


Position	Part NO.	Assembly	Description
1	AN-374		PCB, Level sensor, XO 4
2	MG-360		Holder for w.c. container
3	MG-396		Receiver, XO water clean
4	MG-436		Not a spare part
5	MG-497		Not a spare part
6	MG-596		Snap lock, clips
7	MG-914		Fitting, tubing 1/8
8	MG-916		Tubing, 4 X 6 X 159 nature
9	UG-102		Screw, thread making, M3x006 DIN7500
10	UG-115		Screw, M4x010
11	UG-208		Locknut M5
12	UG-702		Fitting 1/8 rg x 4 mm
13	UG-718		Not a spare part
14	UH-046		O-ring, Ø9,30x2,4 EPDM black

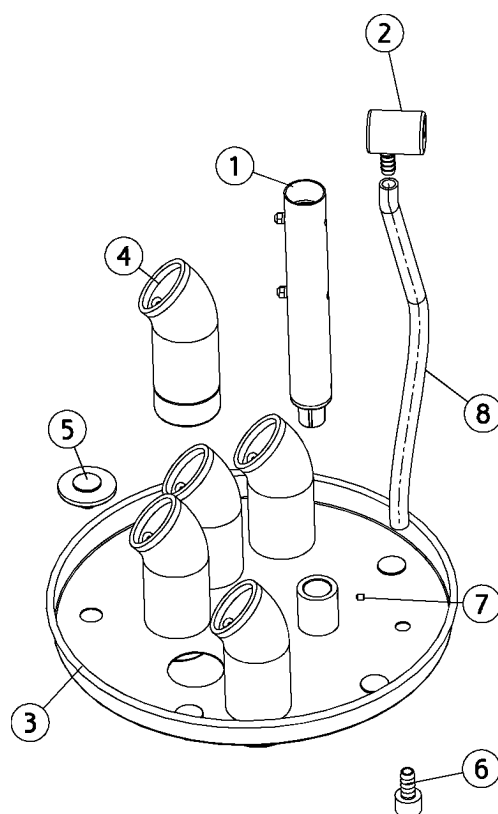

Position	Part NO.	Assembly	Description
1	AE-743		Adaptor, hygiflex, syringe
2	AO-344	Page 91	Water conn., water clean plate
3	MG-658		Not a spare part
4	MG-659		Instrumentholder, water clean
5	MG-661		Rubber foot, water clean
6	MG-662		Nipple, water clean
7	UA-707		Screw M3 x 4
8	UG-935		Tube, silicone

AO-344

Water conn., water clean plate

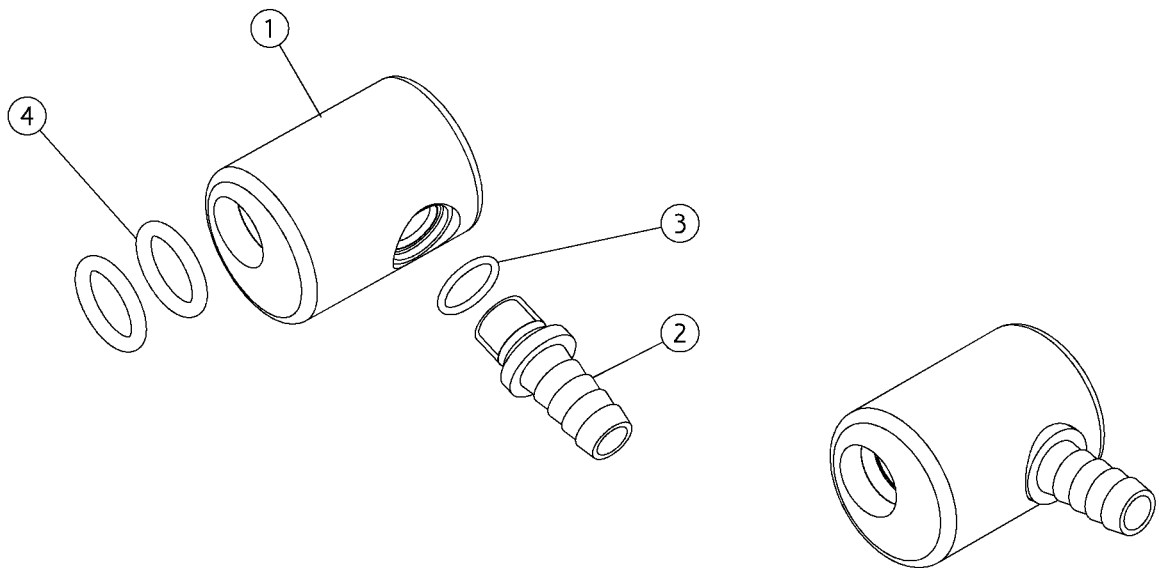

Position	Part NO.	Assembly	Description
1	MG-660		Nipple, flushtube, water clean
2	MG-893		Not a spare part
3	UC-089		Not a spare part
4	UH-073		O-ring Ø10,0x2,0 SIL

AO-344

Water conn., water clean plate

XO 4 Valve System and Instrument Tubes


TX-002

Valve modules

Position	Part NO.	Assembly	Description
1	AN-080	<i>Page 95</i>	Valve module, syringe
2	AN-081	<i>Page 98</i>	Valve module, motor/turbine
3	AN-082	<i>Page 99</i>	Valve module, scaler
4	AN-083	<i>Page 100</i>	Valve module, Lysta/Osseo
5	AN-169	<i>Page 103</i>	Valve module, DÛRR cam
6	AN-202	<i>Page 101</i>	Valve module, EMS w/light
7	AO-320	<i>Page 104</i>	Valve module, MX motor
8	AO-327	<i>Page 105</i>	Valve module, motor/turbine
9	AO-400	<i>Page 107</i>	Valve module, EMS NO PAiN
10	AO-995	<i>Page 111</i>	Valve module, DÛRR vistacam ix. usb
11	AO-999	<i>Page 113</i>	Valve module, DÛRR vistacam ix-hd. usb
12	AP-829	<i>Page 109</i>	Valve module, XO ODONTOCURE

AN-080

Valve module, syringe

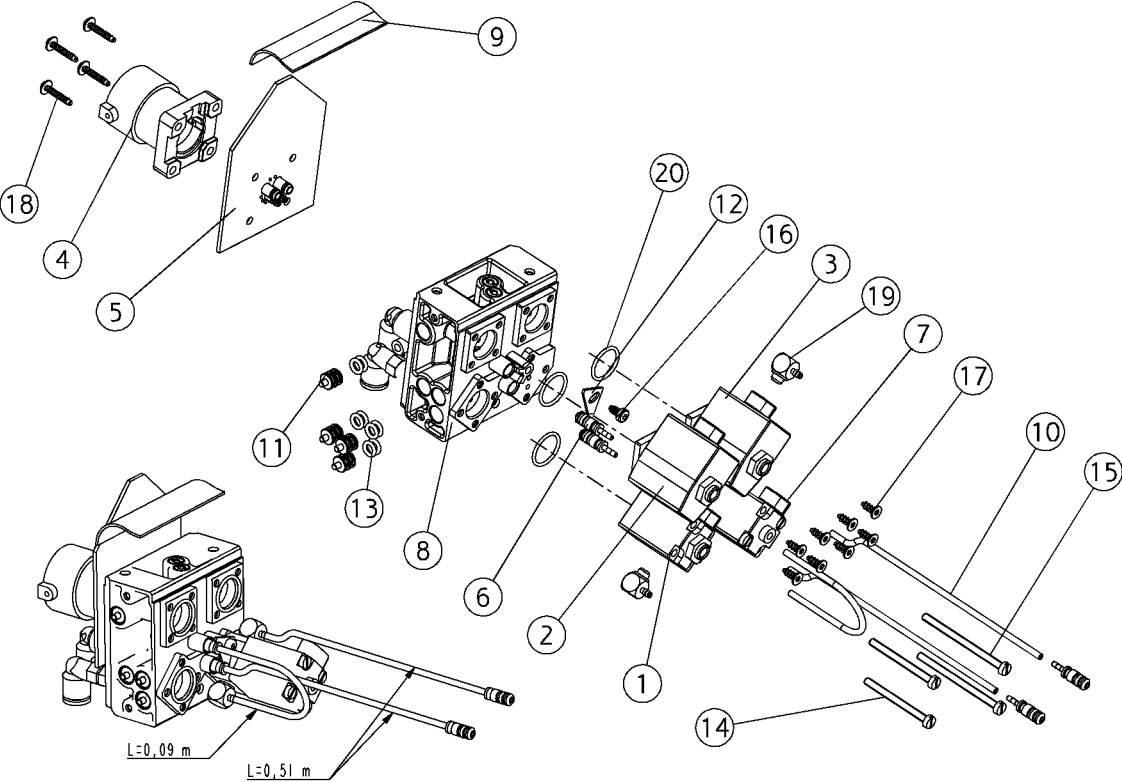


AN-080

Valve module, syringe

Position	Part NO.	Assembly	Description
1	AN-503		Proportional valve spray air + conct. XO 4
2	AN-504		Valve syringe, air, -block
3	AN-506		Valve water, 2/2, -block
4	AN-509	<i>Page 173</i>	Connector box with pins
5	AN-510	<i>Page 169</i>	PCB, Suspension, syringe/lc/osseo
6	AN-513	<i>Page 168</i>	Hose connection with o-ring
7	AN-903	<i>Page 97</i>	Proportional valve, water + conn.
8	AP-040	<i>Page 116</i>	Start block with fittings
9	CA-100		Flat cable start block
10	FS-078		Tube, green
11	MG-230		Sealing plug, valve block
12	MG-609		Locking plate for hose conn.
13	MR-202		O-ring, 4 x 1,5
14	UG-132		Not a spare part
15	UG-133		Not a spare part
16	UG-224		Screw for plastic, M3x6
17	UG-226		Bottom swaged screw 3x8
18	UG-240		Screw for plastic M3x14
19	UG-686		Angle fittings, M5xØ3 f. prop. valve
20	UG-848		O-ring Ø13,00x1,30 fpm

AN-903


Proportional valve, water + conn.

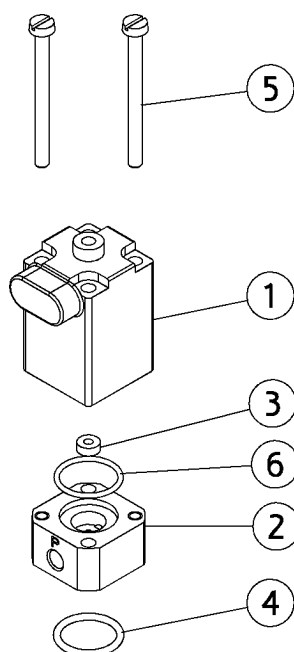

Position	Part NO.	Assembly	Description
1	AN-505		Prop valve, water + conct
2	MG-750		Not a spare part
3	MR-150		Gasket, seat, valve
4	SD-307		O-ring, 12 x 1,5 Viton
5	UG-132		Not a spare part
6	UH-076		O-ring 12,5x1,1 fkm

AN-081


Valve module, motor/turbine

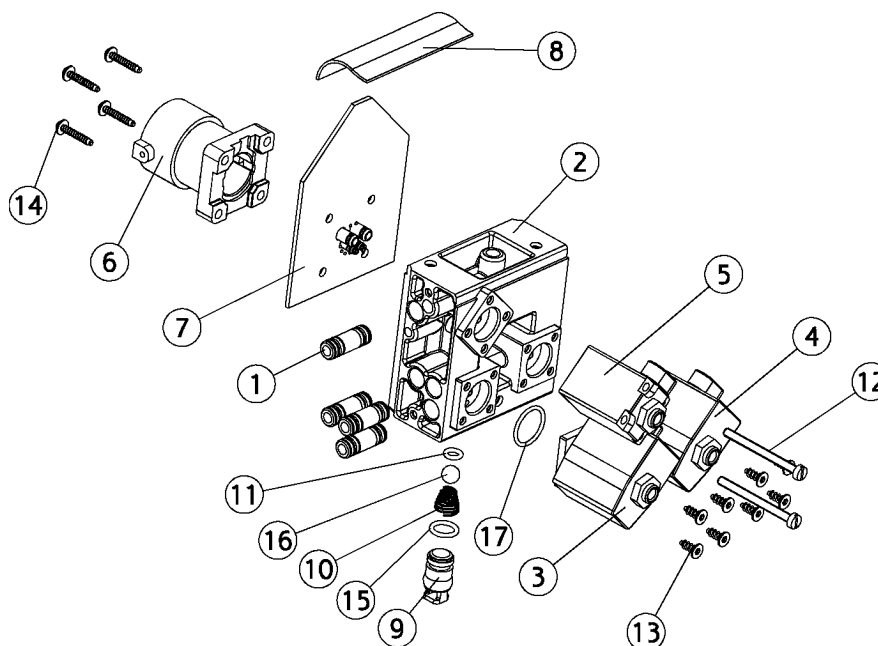


Position	Part NO.	Assembly	Description
1	AN-091	Page 119	Feed-through nozzle w. o-rings
2	AN-501	Page 118	Instrument block with balls
3	AN-506		Valve water, 2/2, -block
4	AN-507		Valve airspray, 3/2, -block
5	AN-508		Proportional valve drive air, -block
6	AN-509	Page 173	Connector box with pins
7	AN-510	Page 169	PCB, Suspension, syringe/lc/osseo
8	CA-101		Flat cable block valve
9	MG-498		Not a spare part
10	MG-969		Not a spare part
11	MR-202		O-ring, 4 x 1,5
12	UG-132		Not a spare part
13	UG-226		Bottom swaged screw 3x8
14	UG-240		Screw for plastic M3x14
15	UG-310		Not a spare part
16	UG-423		Not a spare part
17	UG-848		O-ring Ø13,00x1,30 fpm

AN-082

Valve module, scaler

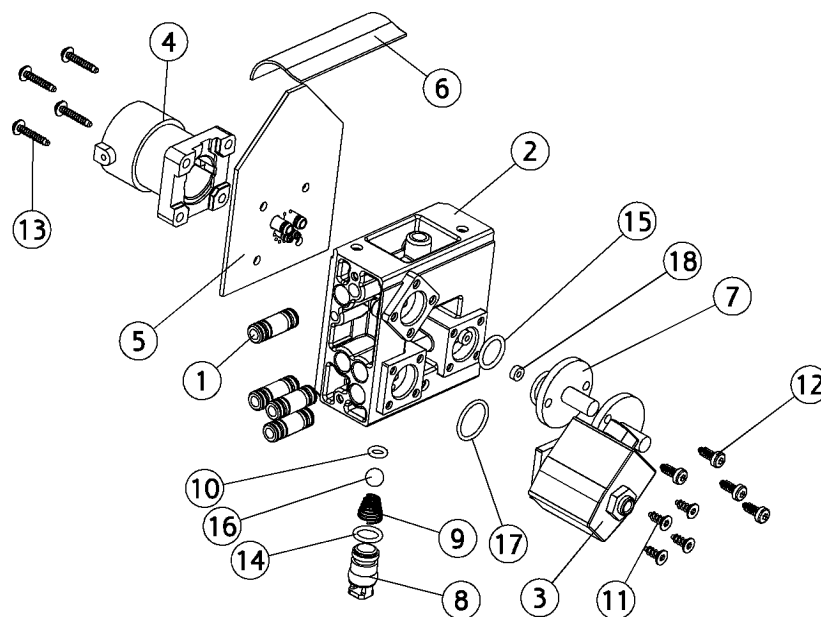


Position	Part NO.	Assembly	Description
1	AN-091	Page 119	Feed-through nozzle w. o-rings
2	AN-501	Page 118	Instrument block with balls
3	AN-506		Valve water, 2/2, -block
4	AN-509	Page 173	Connector box with pins
5	AN-510	Page 169	PCB, Suspension, syringe/lc/osseo
6	CA-101		Flat cable block valve
7	MG-234		Plug for valve block
8	MG-498		Not a spare part
9	MG-969		Not a spare part
10	MR-202		O-ring, 4 x 1,5
11	UG-226		Bottom swaged screw 3x8
12	UG-239		Screw, for plastic M3x8
13	UG-240		Screw for plastic M3x14
14	UG-310		Not a spare part
15	UG-316		O-ring Ø10x1,5
16	UG-423		Not a spare part
17	UG-848		O-ring Ø13,00x1,30 fpm
18	UH-075		Not a spare part

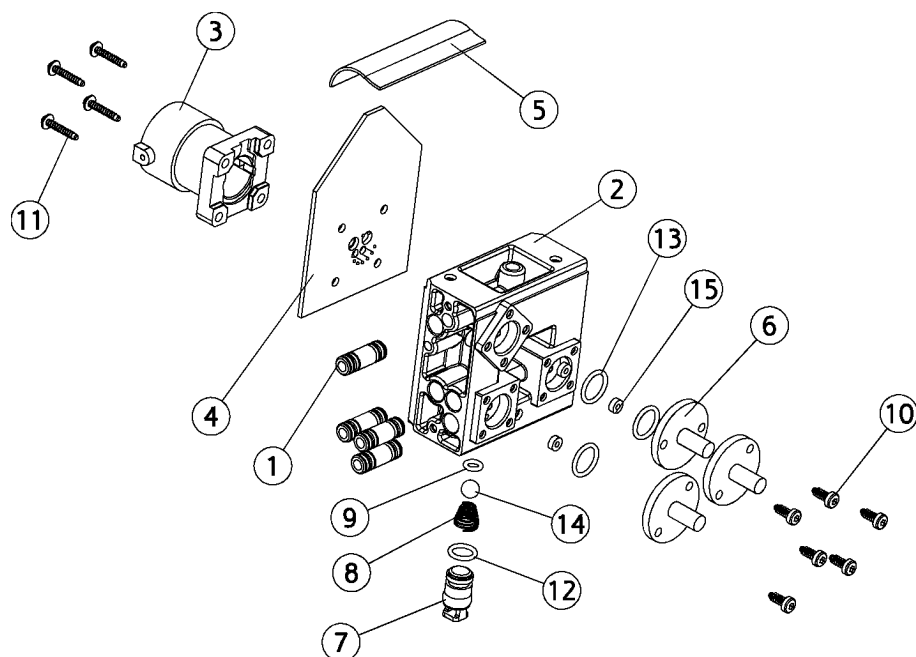

Position	Part NO.	Assembly	Description
1	AN-091	Page 119	Feed-through nozzle w. o-rings
2	AN-501	Page 118	Instrument block with balls
3	AN-509	Page 173	Connector box with pins
4	AN-511	Page 170	PCB, Suspension, LC LAMP
5	CA-101		Flat cable block valve
6	MG-234		Plug for valve block
7	MG-498		Not a spare part
8	MG-969		Not a spare part
9	MR-202		O-ring, 4 x 1,5
10	UG-239		Screw, for plastic M3x8
11	UG-240		Screw for plastic M3x14
12	UG-310		Not a spare part
13	UG-316		O-ring Ø10x1,5
14	UG-423		Not a spare part
15	UH-075		Not a spare part

AN-202

Valve module, EMS w/light

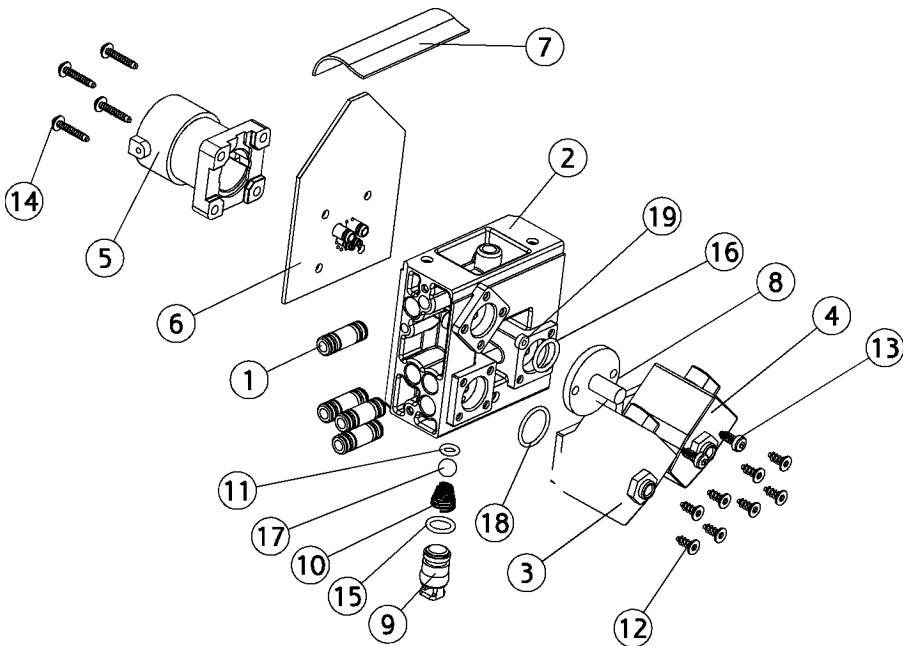


AN-202

Valve module, EMS w/light

Position	Part NO.	Assembly	Description
1	AN-091	<i>Page 119</i>	Feed-through nozzle w. o-rings
2	AN-501	<i>Page 118</i>	Instrument block with balls
3	AN-506		Valve water, 2/2, -block
4	AN-507		Valve airspray, 3/2, -block
5	AN-509	<i>Page 173</i>	Connector box with pins
6	AN-510	<i>Page 169</i>	PCB, Suspension, syringe/lc/osseo
7	CA-101		Flat cable block valve
8	MG-234		Plug for valve block
9	MG-498		Not a spare part
10	MG-969		Not a spare part
11	MR-202		O-ring, 4 x 1,5
12	UG-226		Bottom swaged screw 3x8
13	UG-239		Screw, for plastic M3x8
14	UG-240		Screw for plastic M3x14
15	UG-310		Not a spare part
16	UG-316		O-ring Ø10x1,5
17	UG-423		Not a spare part
18	UG-848		O-ring Ø13,00x1,30 fpm
19	UH-075		Not a spare part

AN-169


Valve module, DÜRR cam

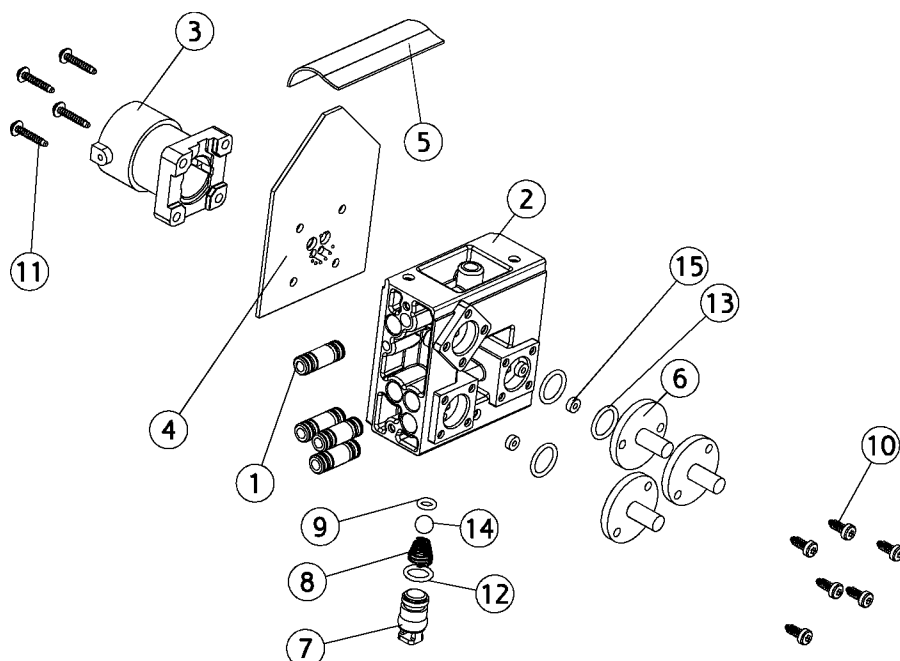

Position	Part NO.	Assembly	Description
1	AN-091	Page 119	Feed-through nozzle w. o-rings
2	AN-501	Page 118	Instrument block with balls
3	AN-509	Page 173	Connector box with pins
4	AN-512	Page 171	PCB hose, +stick -tube, camera
5	CA-101		Flat cable block valve
6	MG-234		Plug for valve block
7	MG-498		Not a spare part
8	MG-969		Not a spare part
9	MR-202		O-ring, 4 x 1,5
10	UG-239		Screw, for plastic M3x8
11	UG-240		Screw for plastic M3x14
12	UG-310		Not a spare part
13	UG-316		O-ring Ø10x1,5
14	UG-423		Not a spare part
15	UH-075		Not a spare part

AO-320


Valve module, MX motor

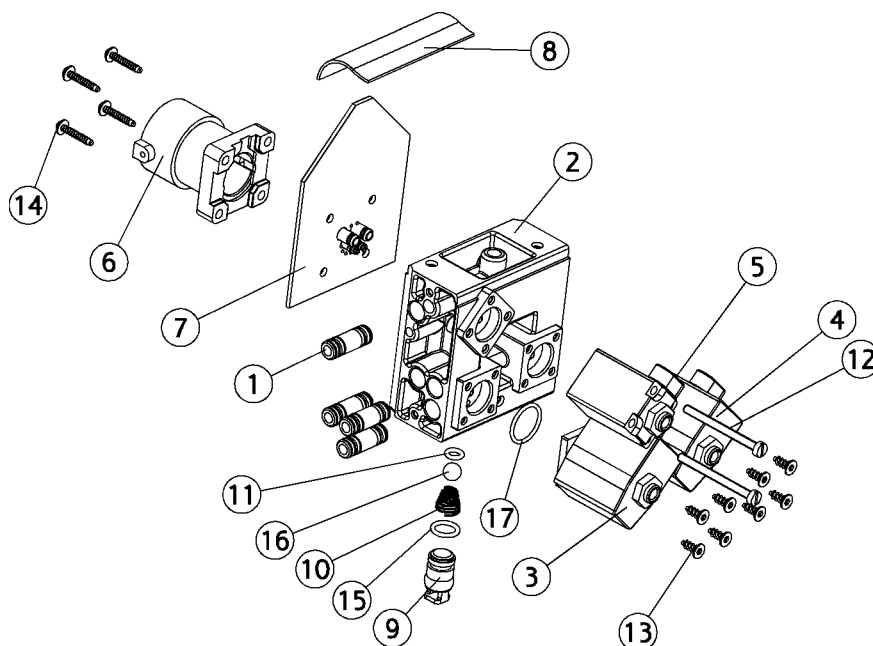

Position	Part NO.	Assembly	Description
1	AN-091	Page 119	Feed-through nozzle w. o-rings
2	AN-501	Page 118	Instrument block with balls
3	AN-506		Valve water, 2/2, -block
4	AN-507		Valve airspray, 3/2, -block
5	AN-508		Proportional valve drive air, -block
6	AN-509	Page 173	Connector box with pins
7	AO-318	Page 172	PCB, Suspension, MX/OSS/MC3/tur/lc/sca
8	CA-101		Flat cable block valve
9	MG-498		Not a spare part
10	MG-969		Not a spare part
11	MR-202		O-ring, 4 x 1,5
12	UG-132		Not a spare part
13	UG-226		Bottom swaged screw 3x8
14	UG-240		Screw for plastic M3x14
15	UG-310		Not a spare part
16	UG-423		Not a spare part
17	UG-848		O-ring Ø13,00x1,30 fpm

AO-327

Valve module, motor/turbine

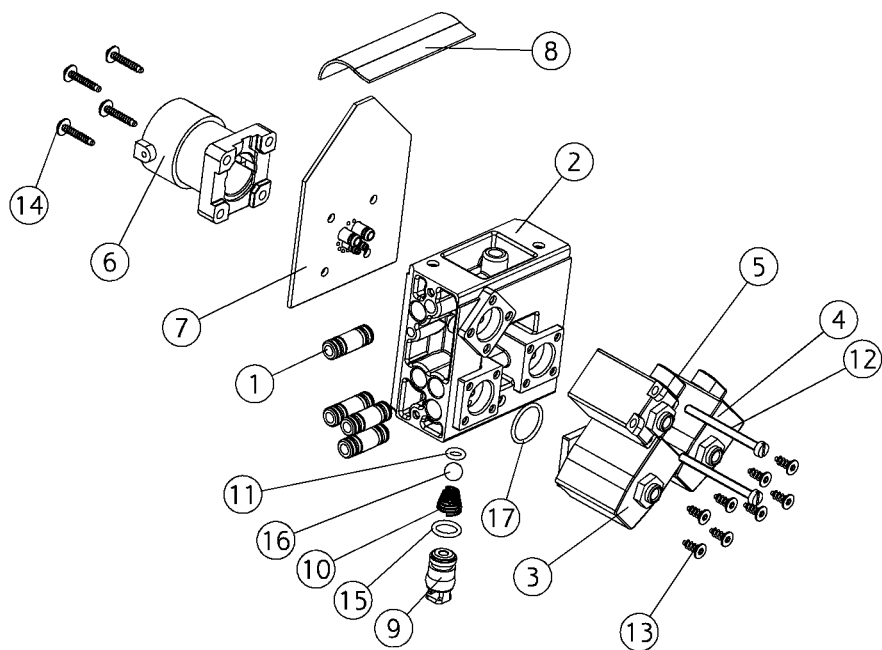


AO-327

Valve module, motor/turbine

Position	Part NO.	Assembly	Description
1	AN-091	<i>Page 119</i>	Feed-through nozzle w. o-rings
2	AN-501	<i>Page 118</i>	Instrument block with balls
3	AN-506		Valve water, 2/2, -block
4	AN-507		Valve airspray, 3/2, -block
5	AN-508		Proportional valve drive air, -block
6	AN-509	<i>Page 173</i>	Connector box with pins
7	AO-328		PCB, Suspension, MX2 motor/turbine
8	CA-101		Flat cable block valve
9	MG-498		Not a spare part
10	MG-969		Not a spare part
11	MR-202		O-ring, 4 x 1,5
12	UG-132		Not a spare part
13	UG-226		Bottom swaged screw 3x8
14	UG-240		Screw for plastic M3x14
15	UG-310		Not a spare part
16	UG-423		Not a spare part
17	UG-848		O-ring Ø13,00x1,30 fpm

AO-400

Valve module, EMS NO PAiN

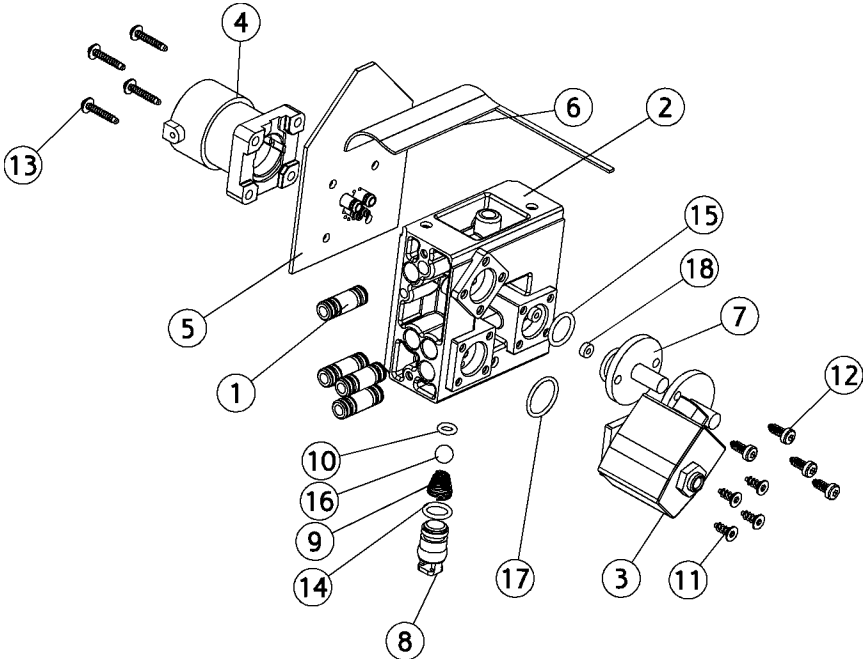


AO-400

Valve module, EMS NO PAiN

Position	Part NO.	Assembly	Description
1	AN-091	<i>Page 119</i>	Feed-through nozzle w. o-rings
2	AN-501	<i>Page 118</i>	Instrument block with balls
3	AN-506		Valve water, 2/2, -block
4	AN-509	<i>Page 173</i>	Connector box with pins
5	AN-510	<i>Page 169</i>	PCB, Suspension, syringe/lc/osseo
6	CA-109		Flat cable block, ems
7	MG-234		Plug for valve block
8	MG-498		Not a spare part
9	MG-969		Not a spare part
10	MR-202		O-ring, 4 x 1,5
11	UG-226		Bottom swaged screw 3x8
12	UG-239		Screw, for plastic M3x8
13	UG-240		Screw for plastic M3x14
14	UG-310		Not a spare part
15	UG-316		O-ring Ø10x1,5
16	UG-423		Not a spare part
17	UG-848		O-ring Ø13,00x1,30 fpm
18	UH-075		Not a spare part

AP-829

Valve module, XO ODONTOCURE

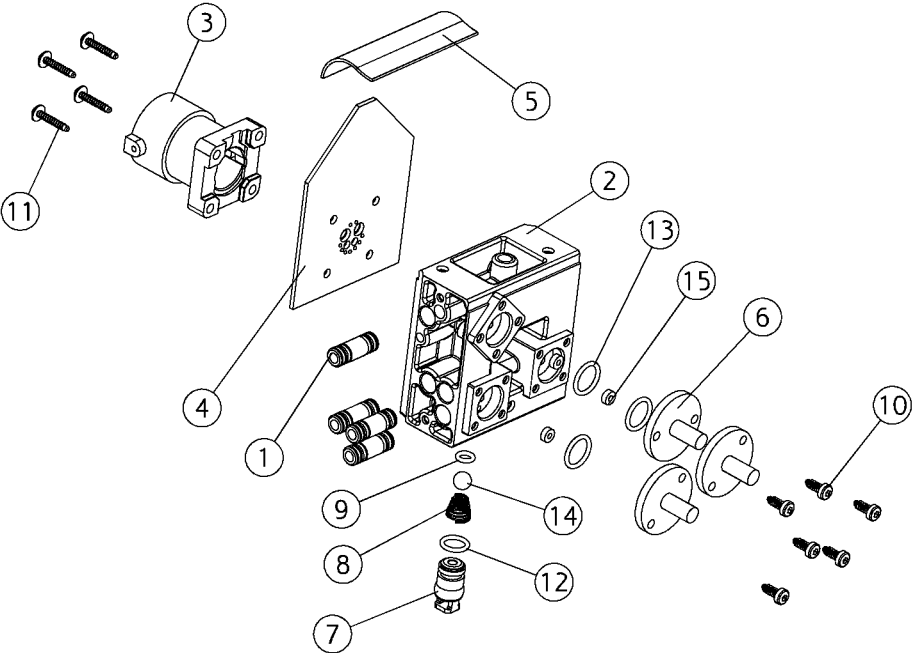


AP-829

Valve module, XO ODONTOCURE

Position	Part NO.	Assembly	Description
1	AN-091	<i>Page 119</i>	Feed-through nozzle w. o-rings
2	AN-501	<i>Page 118</i>	Instrument block with balls
3	AN-509	<i>Page 173</i>	Connector box with pins
4	AP-866		PCB, suspension, XO Odontocure
5	CA-101		Flat cable block valve
6	MG-234		Plug for valve block
7	MG-498		Not a spare part
8	MG-969		Not a spare part
9	MR-202		O-ring, 4 x 1,5
10	UG-239		Screw, for plastic M3x8
11	UG-240		Screw for plastic M3x14
12	UG-310		Not a spare part
13	UG-316		O-ring Ø10x1,5
14	UG-423		Not a spare part
15	UH-075		Not a spare part

AO-995 Valve module, DÜRR vistacam ix. usb

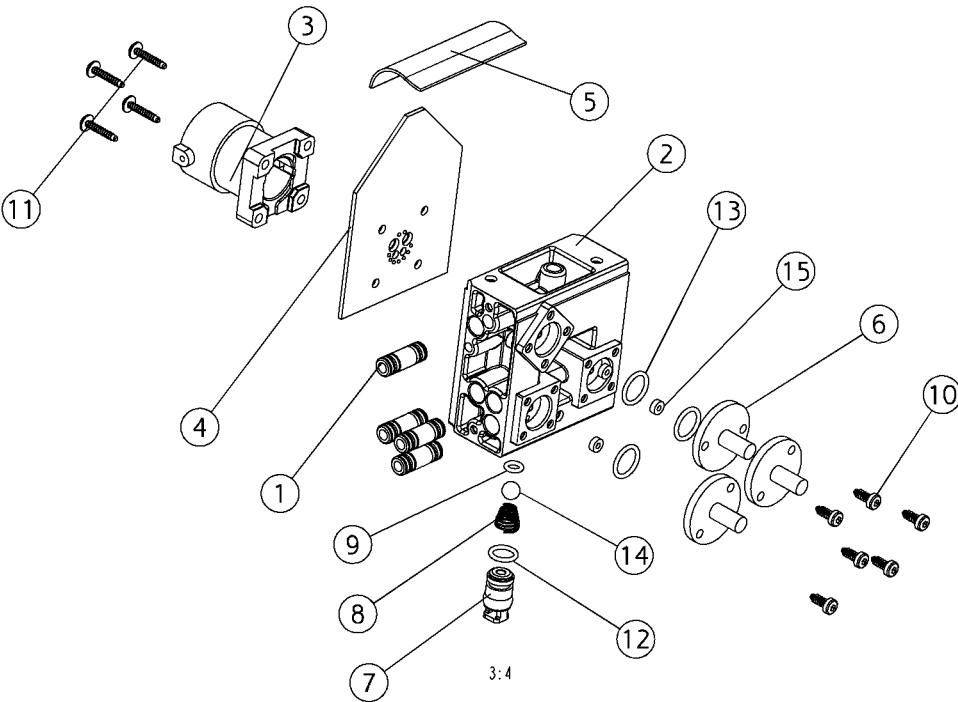


AO-995

Valve module, DÜRR vistacam ix. usb

Position	Part NO.	Assembly	Description
1	AN-091	<i>Page 119</i>	Feed-through nozzle w. o-rings
2	AN-501		Instrument block with balls
3	AN-509	<i>Page 173</i>	Connector box with pins
4	AP-866		PCB, suspension, XO Odontocure
5	CA-101		Flat cable block valve
6	MG-234		Plug for valve block
7	MG-498		Not a spare part
8	MG-969		Not a spare part
9	MR-202		O-ring, 4 x 1,5
10	UG-239		Screw, for plastic M3x8
11	UG-240		Screw for plastic M3x14
12	UG-310		Not a spare part
13	UG-316		O-ring Ø10x1,5
14	UG-423		Not a spare part
15	UH-075		Not a spare part

AO-999

Valve module, DÜRR vistacam ix-hd. usb

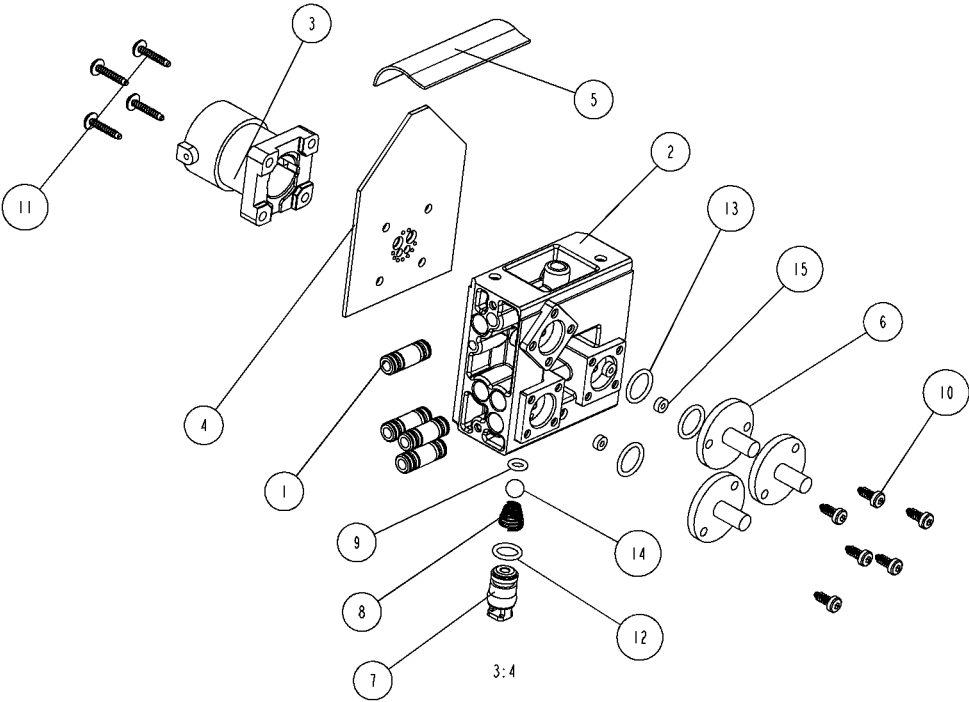


AO-999

Valve module, DÜRR vistacam ix-hd. usb

Position	Part NO.	Assembly	Description
1	AN-091	<i>Page 119</i>	Feed-through nozzle w. o-rings
2	AN-501	<i>Page 118</i>	Instrument block with balls
3	AN-509	<i>Page 173</i>	Connector box with pins
4	AO-347		PCB, Suspension, DÜRR vistacam IX-HD
5	CA-101		Flat cable block valve
6	MG-234		Plug for valve block
7	MG-498		Not a spare part
8	MG-969		Not a spare part
9	MR-202		O-ring, 4 x 1,5
10	UG-239		Screw, for plastic M3x8
11	UG-240		Screw for plastic M3x14
12	UG-310		Not a spare part
13	UG-316		O-ring Ø10x1,5
14	UG-423		Not a spare part
15	UH-075		Not a spare part

AN-270


Valve block w/o/instrument

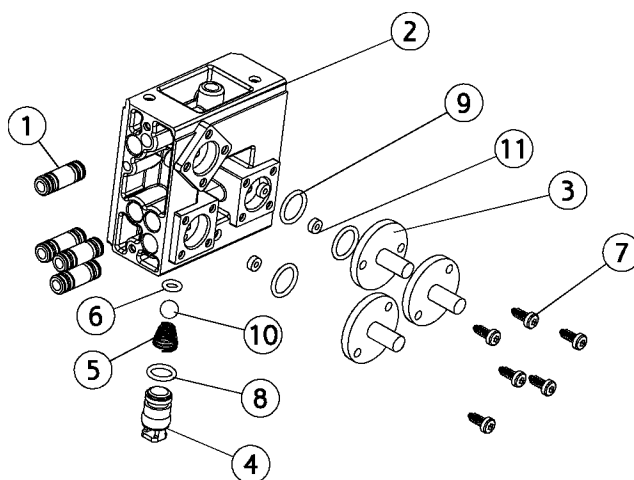

Position	Part NO.	Assembly	Description
1	AN-091	<i>Page 119</i>	Feed-through nozzle w. o-rings
2	AN-501	<i>Page 118</i>	Instrument block with balls
3	MG-234		Plug for valve block
4	MG-498		Not a spare part
5	MG-969		Not a spare part
6	MR-202		O-ring, 4 x 1,5
7	UG-239		Screw, for plastic M3x8
8	UG-310		Not a spare part
9	UG-316		O-ring Ø10x1,5
10	UG-423		Not a spare part
11	UH-075		Not a spare part


AP-040

Start block with fittings

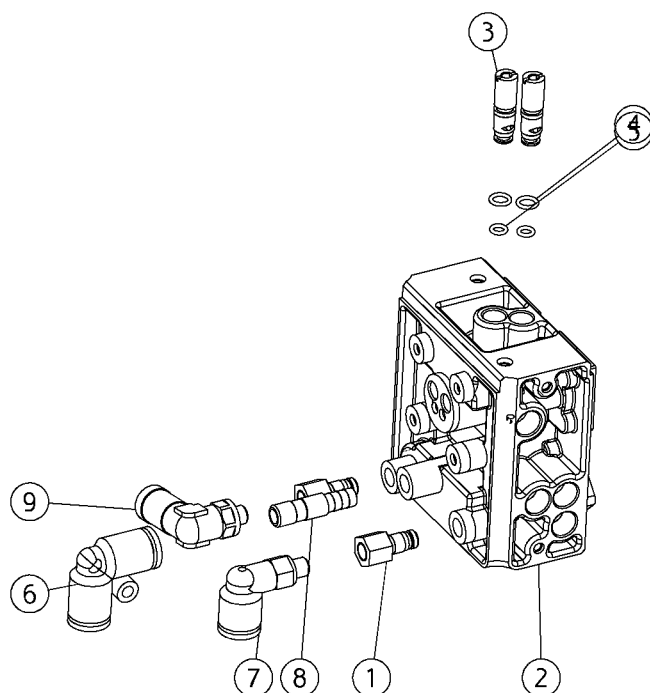

Position	Part NO.	Assembly	Description
1	AC-742	Page 117	Connection nipple with o-ring
2	AN-500		Connection block with balls
3	MG-236		Needle valve,start block,XO 4
4	MR-226		O-ring Ø4 x 1
5	MR-227		O-ring Ø3 x 1
6	UG-694		Quick coupling, angle, 4/6
7	UG-706		Quick connect, 6/4-M5-90
8	UH-008		Nipple tube, quick conn.,XO 4
9	UH-241		Quick connect, 4/2-M5-90 NSF



AC-742 Connection nipple with o-ring

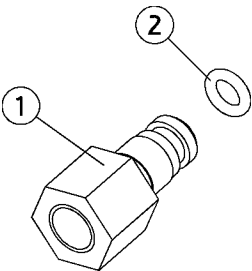

Position	Part NO.	Assembly	Description
1	MC-607		Not a spare part
2	MR-227		O-ring Ø3 x 1

AN-501


Instrument block with balls

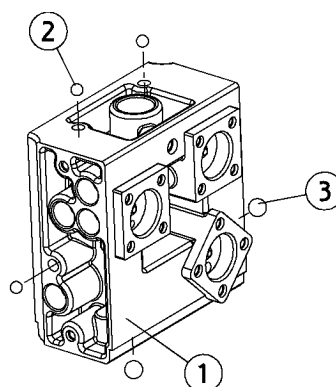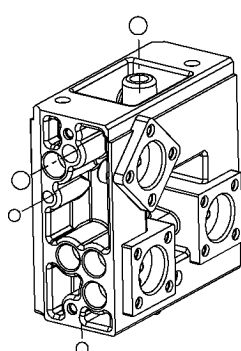

Position	Part NO.	Assembly	Description
1	MG-229		Not a spare part
2	UG-465		Not a spare part
3	UG-466		Not a spare part

AN-091


Feed-through nozzle w. o-rings

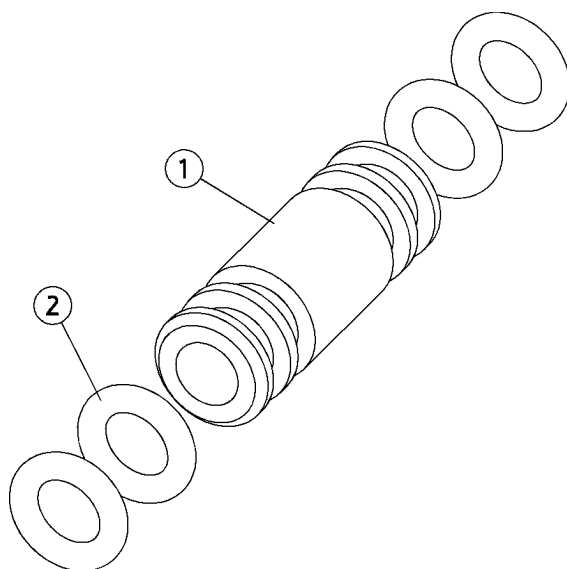

Position	Part NO.	Assembly	Description
1	MG-228		Feed-through nozzle
2	MR-202		O-ring, 4 x 1,5

AN-116


Conn.piece w.o-rings,block

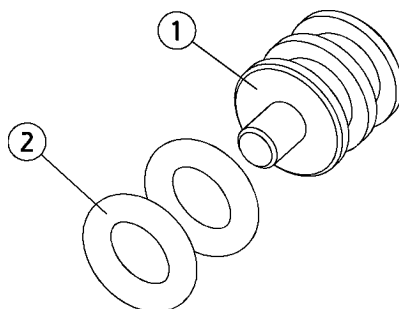

Position	Part NO.	Assembly	Description
1	MG-230		Sealing plug, valve block
2	MR-202		O-ring, 4 x 1,5

AN-682

Instrument tubings, all

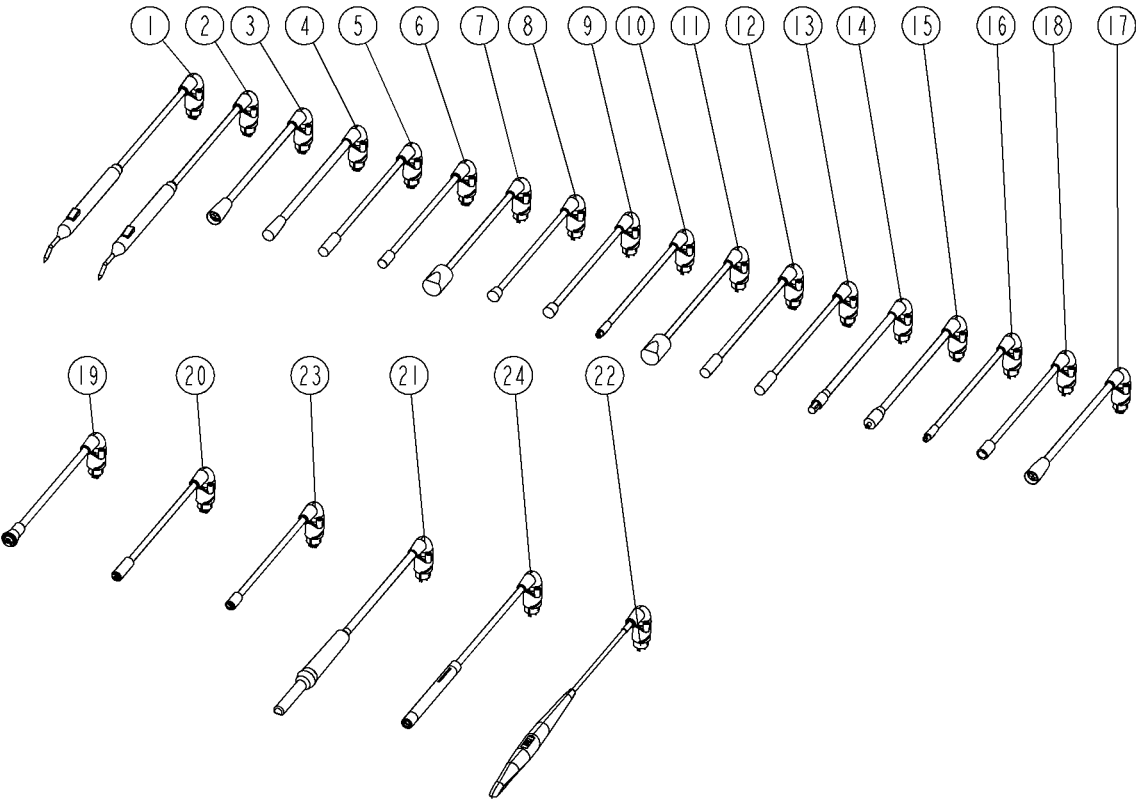


AN-682

Instrument tubings, all

Position	Part NO.	Assembly	Description
1	AN-382		3f syringe
2	AN-383		6f syringe
3	AN-384		Hose, MC3 /MC3 LED motor
4	AN-385		Hose, 4+2 air turbine
5	AN-386		Hose, EMS scaler
6	AN-800		Hose, XO scaler
7	AN-801		Hose, Dürr VistaCam (composite)
8	AN-802		Hose, Lysta LC 6100F, 9V, 5 pole
9	AN-387		Hose, Lysta LC 6100F,12v, 7 pole
10	AN-803		Hose, Lysta LCD 88
11	AN-804		Hose, Dürr VistaCam (s-video)
12	AN-845		Hose, EMS scaler w/light
13	AO-056		Hose, Satelec scaler
14	AO-057		Hose, Dürr VistaCam Easy
15	AO-217		Hose, MX motor
16	AO-218		Hose, Lysta LCD 89
17	AO-248		Hose, NSK motor M40 LED
18	AO-627		Hose, XO OSU, black
19	AO-446		Hose, MX2 motor
20	AO-448		Hose, EMS scaler No Pain
21	AO-996		Hose, Dürr VistaCam iX (USB)
22	AP-888		Hose, Dürr VistaCam iX HD (USB)
23	AP-889		Hose, XO Odontoson 360
24	AP-887		Hose, XO ODONTOCURE

AN-120

Suspension connector XO 4

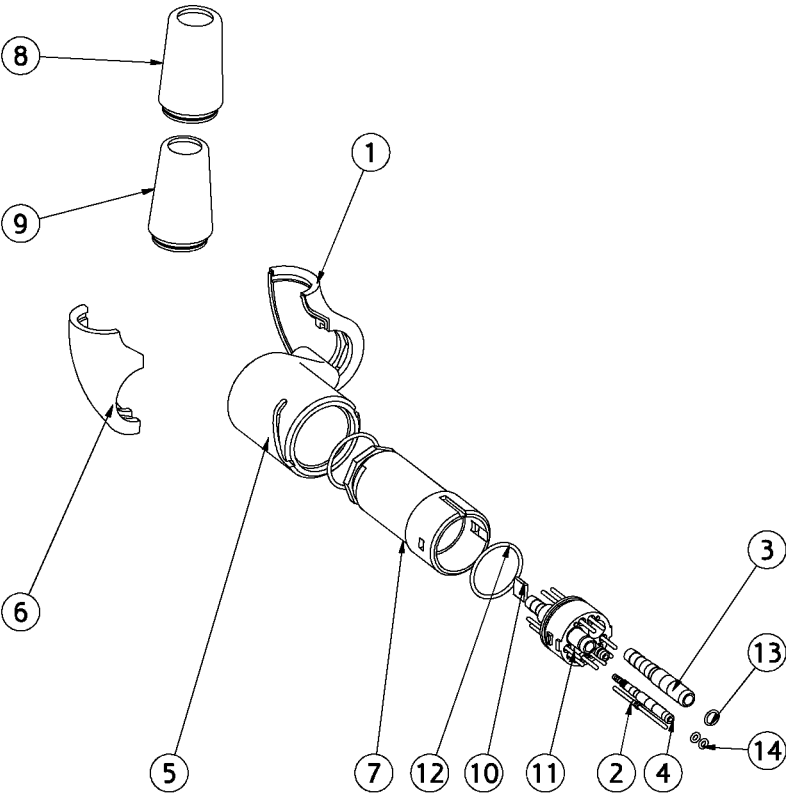


AN-120

Suspension connector XO 4

Position	Part NO.	Assembly	Description
1	MG-259		Not a spare part
2	MG-260		Not a spare part
3	MG-261		Not a spare part
4	MG-262		Not a spare part
5	MG-264		Not a spare part
6	MG-267		Not a spare part
7	MG-425		Not a spare part
8	MG-426		Not a spare part
9	MG-427		Not a spare part
10	MH-980		Not a spare part
11	MN-252		Not a spare part
12	UG-307		Not a spare part
13	UG-315		O-ring Ø3,3 x 0,6
14	UH-149		O-ring Ø1,5x0,7 viton

AN-190


Plug with pipes, instr.block

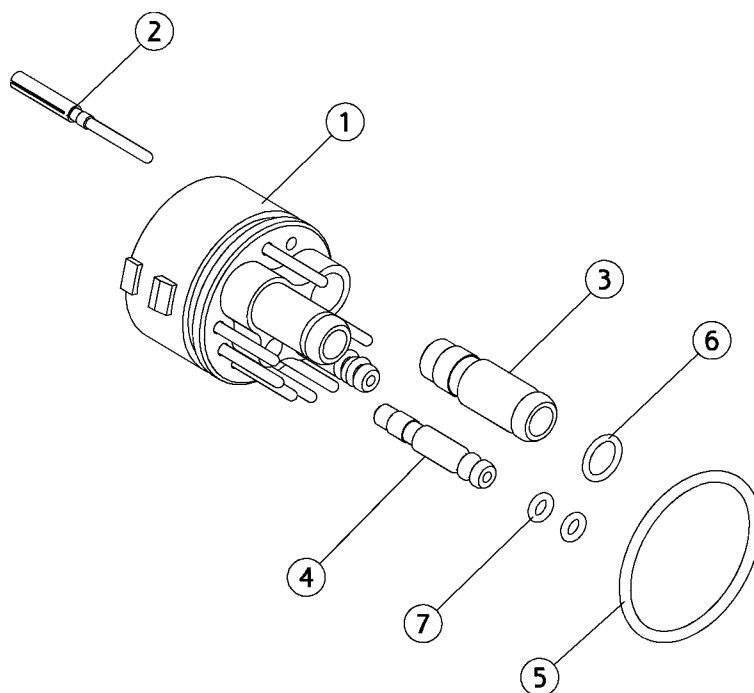

Position	Part NO.	Assembly	Description
1	MG-252		Not a spare part
2	MG-253		Not a spare part
3	MG-254		Not a spare part
4	MG-255		Not a spare part
5	UG-307		Not a spare part
6	UG-315		O-ring Ø3,3 x 0,6
7	UG-331		O-ring, Ø1,5x0,70

AN-192


Instr.tube connector,excl.tube

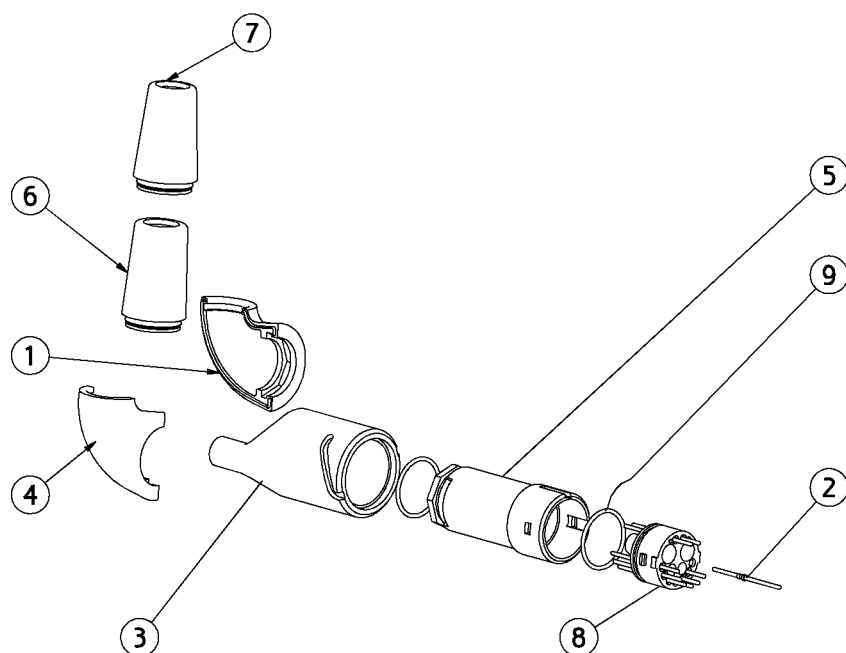

Position	Part NO.	Assembly	Description
1	MG-259		Not a spare part
2	MG-260		Not a spare part
3	MG-264		Not a spare part
4	MG-267		Not a spare part
5	MG-425		Not a spare part
6	MG-426		Not a spare part
7	MG-427		Not a spare part
8	MN-252		Not a spare part
9	UG-307		Not a spare part

AN-191


Plug w/o pipes,instr.block

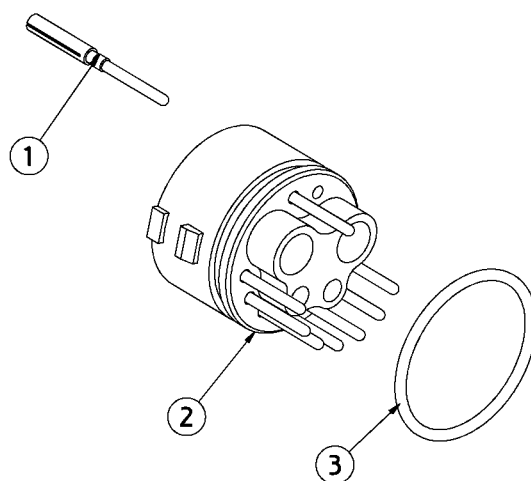

Position	Part NO.	Assembly	Description
1	MG-253		Not a spare part
2	MN-252		Not a spare part
3	UG-307		Not a spare part


AP-190

Connector w. tubings, inst block

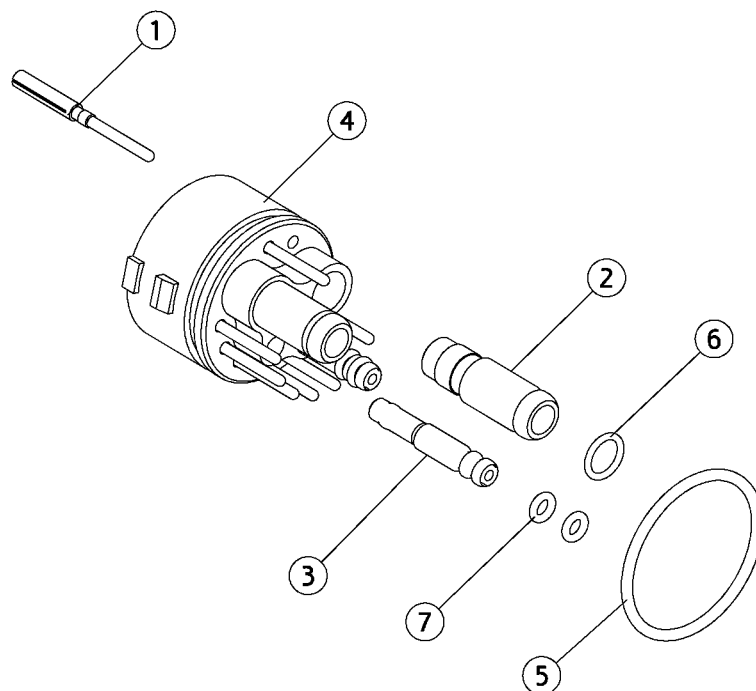

Position	Part NO.	Assembly	Description
1	MG-253		Not a spare part
2	MG-254		Not a spare part
3	MG-255		Not a spare part
4	MN-252		Not a spare part
5	UG-307		Not a spare part
6	UG-315		O-ring Ø3,3 x 0,6
7	UH-149		O-ring Ø1,5x0,7 viton

XO 4 Instrument Bridge

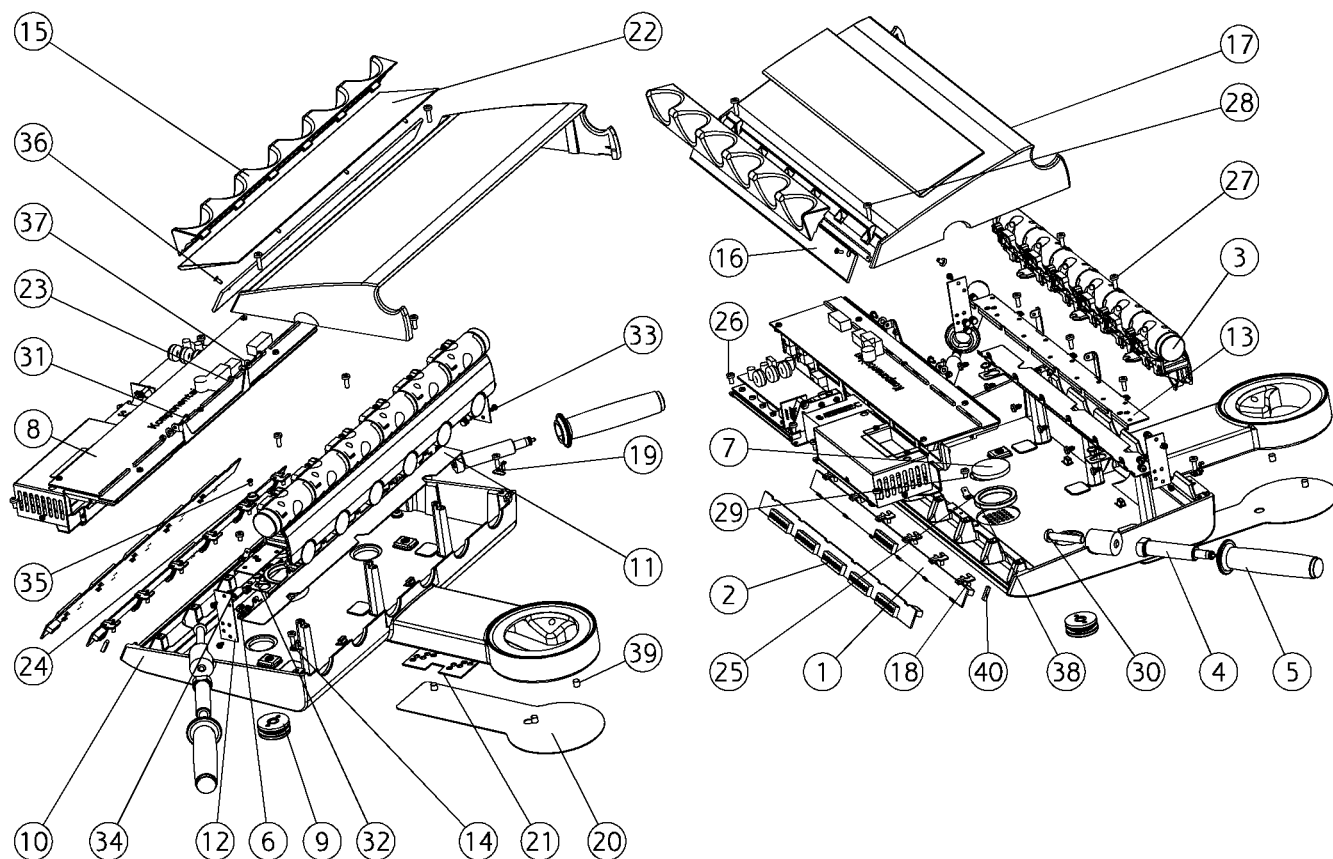

AN-085

Instrument bridge, junction

Position	Part NO.	Assembly	Description
1	AN-370		PCB, Display, XO 4-2 & 4-1
2	AN-381		PCB, Display, XO 4-6
3	AN-010	<i>Page 155</i>	Inst.suspension base, 6 rolls
4	AN-084	<i>Page 158</i>	Receiver, handles
5	AN-110		Handle XO 4
6	AN-116	<i>Page 120</i>	Conn.piece w.o-rings,block
7	AN-515		Loudspeaker with cable
8	AN-519	<i>Page 163</i>	Rocker piece, bridge,assy
9	AN-781	<i>Page 138</i>	Bearing, arm for tray
10	AN-442		Lower part bridge, (XO-900 XO Grey, RAL 7047-80)
10	AN-935		Lower part bridge, (XO-905 XO White,(RAL7047-30))
10	AN-724		Lower part bridge, (XO-903 Silver Metallic)
10	AO-270		Lower part bridge, (XO-906 XO Titanium)
10	AP-645		Lower part bridge, (CTBS)
10	AN-443		Lower part bridge, (XO-901 Sea Blue Metallic)
11	MG-231		Bottom rail, valves
12	MG-232		End plate
13	MG-233		Top rail, valves
14	MG-238		Not a spare part
15	MG-239		Instrument pad
16	MG-269		Display front cover
17	AO-288		Bridge, top cover, (XO-906 XO Titanium)
17	AN-462		Bridge, top cover, (XO-900 XO Grey, RAL 7047-80)
17	AN-734		Bridge, top cover, (XO-903 Silver Metallic)
17	AP-646		Bridge, top cover, (CTBS)
17	AN-463		Bridge, top cover, (XO-901 Sea Blue Metallic)
17	AN-950		Bridge, top cover, (XO-905 XO White,(RAL7047-30))
18	MG-443		Clips, display PCB
19	MG-447		Not a spare part
20	AN-954		Cover plate bridge, (XO-905 XO White,(RAL7047-30))
20	AN-471		Cover plate bridge, (XO-901 Sea Blue Metallic)
20	AN-738		Cover plate bridge, (XO-903 Silver Metallic)


AN-085

Instrument bridge, junction



Position	Part NO.	Assembly	Description
20	AO-289		Cover plate bridge, (XO-906 XO Titanium)
20	AN-470		Cover plate bridge, (XO-900 XO Grey, RAL 7047-80)
20	AP-435		Cover plate bridge, (CTBS)
21	MG-819		Not a spare part
22	MG-897		Instrument protect.,bridge,XO 4
23	UA-858		Not a spare part
24	UC-072		Not a spare part
25	UC-337		Not a spare part
26	UG-067		Screw M4x8 torx
27	UG-068		Not a spare part
28	UG-069		Screw M4x16 torx
29	UG-111		Screw, thread making, M4x6
30	UG-138		Not a spare part
31	UG-139		Screw. M3x8
32	UG-220		M4x6 torx kær
33	UG-224		Screw for plastic, M3x6
34	UG-227		Screw for plastic M4x8
35	UG-232		Screw M3x6 us
36	UG-236		Screw M3x8, uh, thread forming
37	UG-254		Disc, Ø04.3xØ09.0x0.8
38	UG-329		Not a spare part
39	UG-414		Magnet Ø6,35x6,35 neodymium
40	UG-422		Not a spare part

AN-841

Arm,inst.table 2005, w/ 1 tray

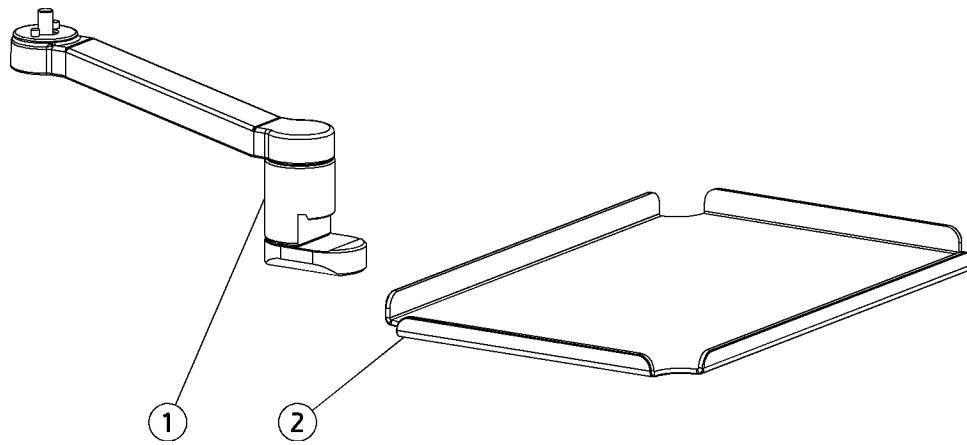

Position	Part NO.	Assembly	Description
1	AN-842	Page 134	Arm, handinstrument table, 2005
2	FA-070		Hand instrument table, 1 norm tray

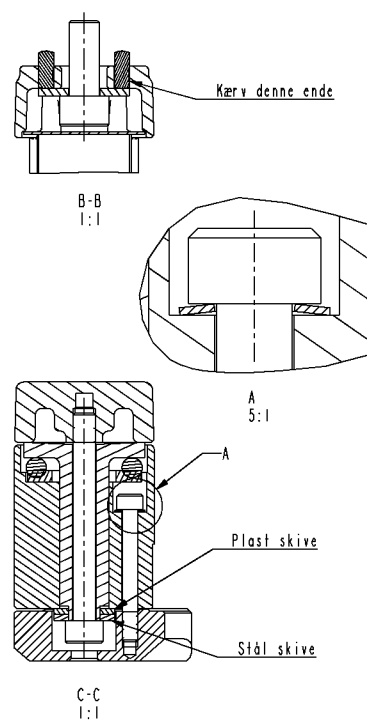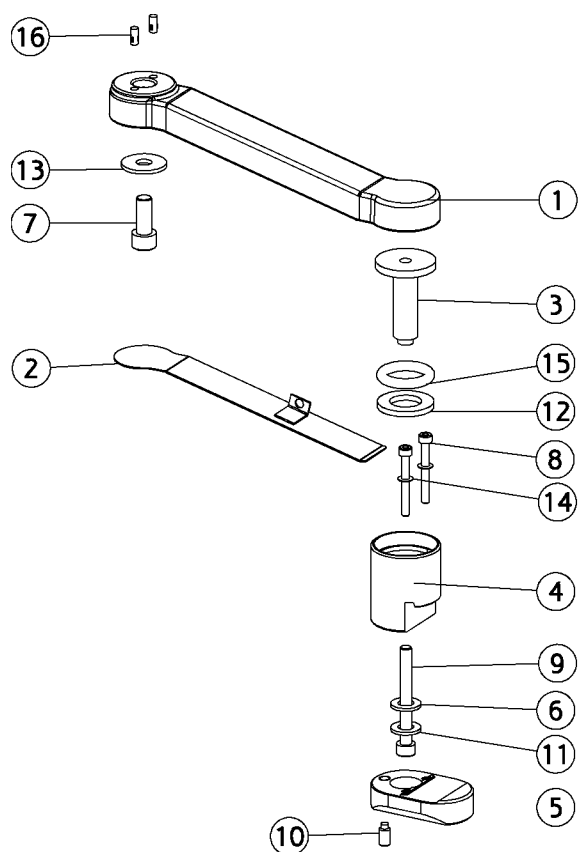


AN-842

Arm, handinstrument table, 2005

Position	Part NO.	Assembly	Description
1	AN-783	<i>Page 140</i>	Arm tray w/ magnets,
2	AN-472		Plate arm tray,
3	MG-807		Not a spare part
4	MG-808		Not a spare part
5	MG-809		Not a spare part
6	UC-462		Not a spare part
7	UG-001		Not a spare part
8	UG-013		Not a spare part
9	UG-065		Not a spare part
10	UG-130		Pointed screw ,M6x12
11	UG-252		Not a spare part
12	UG-293		Not a spare part
13	UG-294		Not a spare part
14	UG-295		Not a spare part
15	UG-335		O-ring Ø19,99xØ5,33
16	UG-891		Not a spare part

AN-844


Arm,instr.table, double tray

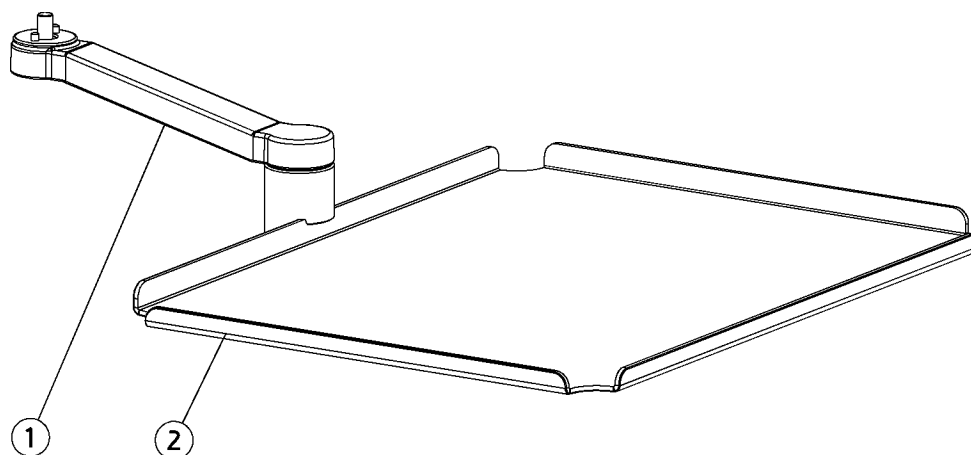

Position	Part NO.	Assembly	Description
1	AN-842	<i>Page 134</i>	Arm, handinstrument table, 2005
2	FA-069		Hand instrument table, 2 norm trays

AN-115


Tray with arm on bridge

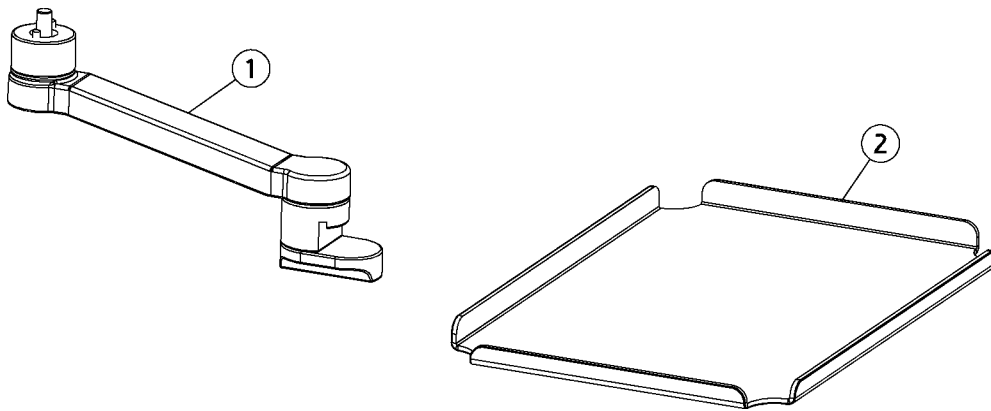

Position	Part NO.	Assembly	Description
1	AN-782	Page 139	Arm for handinstrument table
2	FA-070		Hand instrument table, 1 norm tray

AN-781


Bearing, arm for tray

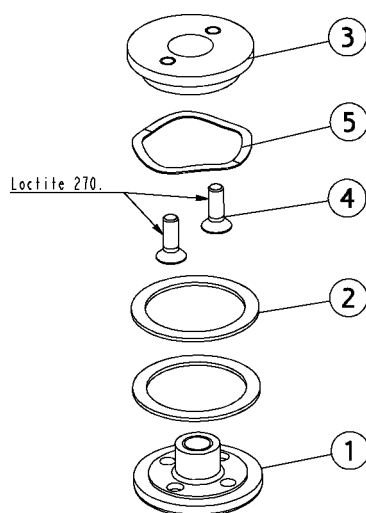

Position	Part NO.	Assembly	Description
1	MG-357		Disc, arm, tray
2	MG-595		Washer, arm for tray
3	MG-607		Washer, arm for tray
4	UG-040		Allen screw, ui6kt M4x12mm, din 7991
5	UG-277		Corrugated spring Ø26.5x31,7x0.4x2.1


AN-782

Arm for handinstrument table

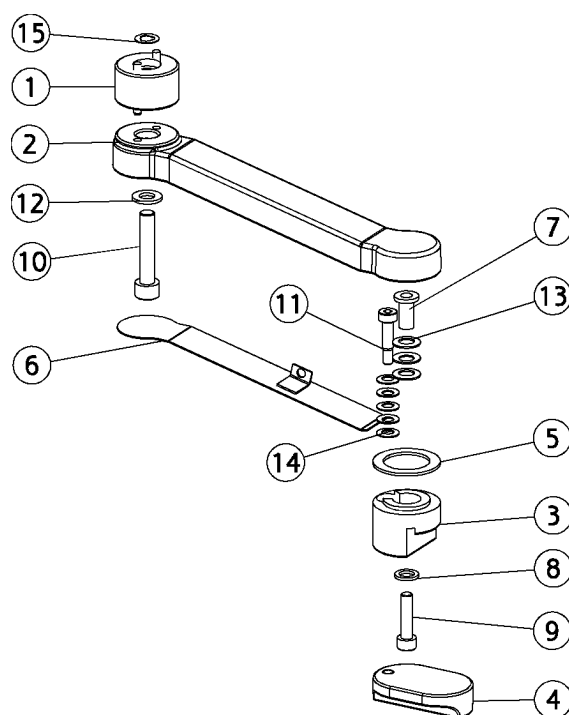

Position	Part NO.	Assembly	Description
1	AN-172	Page 141	Distance ring, tray holder, arm
2	AN-783	Page 140	Arm tray w/ magnets,
3	MG-245		Not a spare part
4	MG-246		Not a spare part
5	MG-288		Not a spare part
6	AN-472		Plate arm tray,
7	MG-612		Not a spare part
8	MG-613		Not a spare part
9	UA-588		Screw, M6 x 25
10	UG-017		Screw, allen M8x40 din 912
11	UG-041		Not a spare part
12	UG-252		Not a spare part
13	UG-271		Not a spare part
14	UG-272		Not a spare part
15	UG-275		Locking washer M8


AN-783 Arm tray w/ magnets,

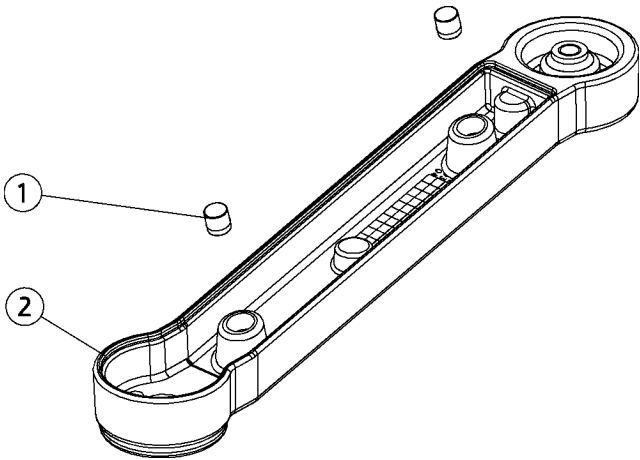


Position	Part NO.	Assembly	Description
1	AN-127	Page 198	Magnet Ø6,35 with spring ring
2	AN-468		Arm tray,

AN-172

Distance ring, tray holder, arm

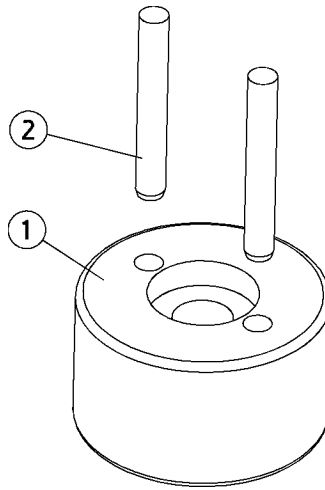

Position	Part NO.	Assembly	Description
1	MG-606		Not a spare part
2	UG-434		Not a spare part

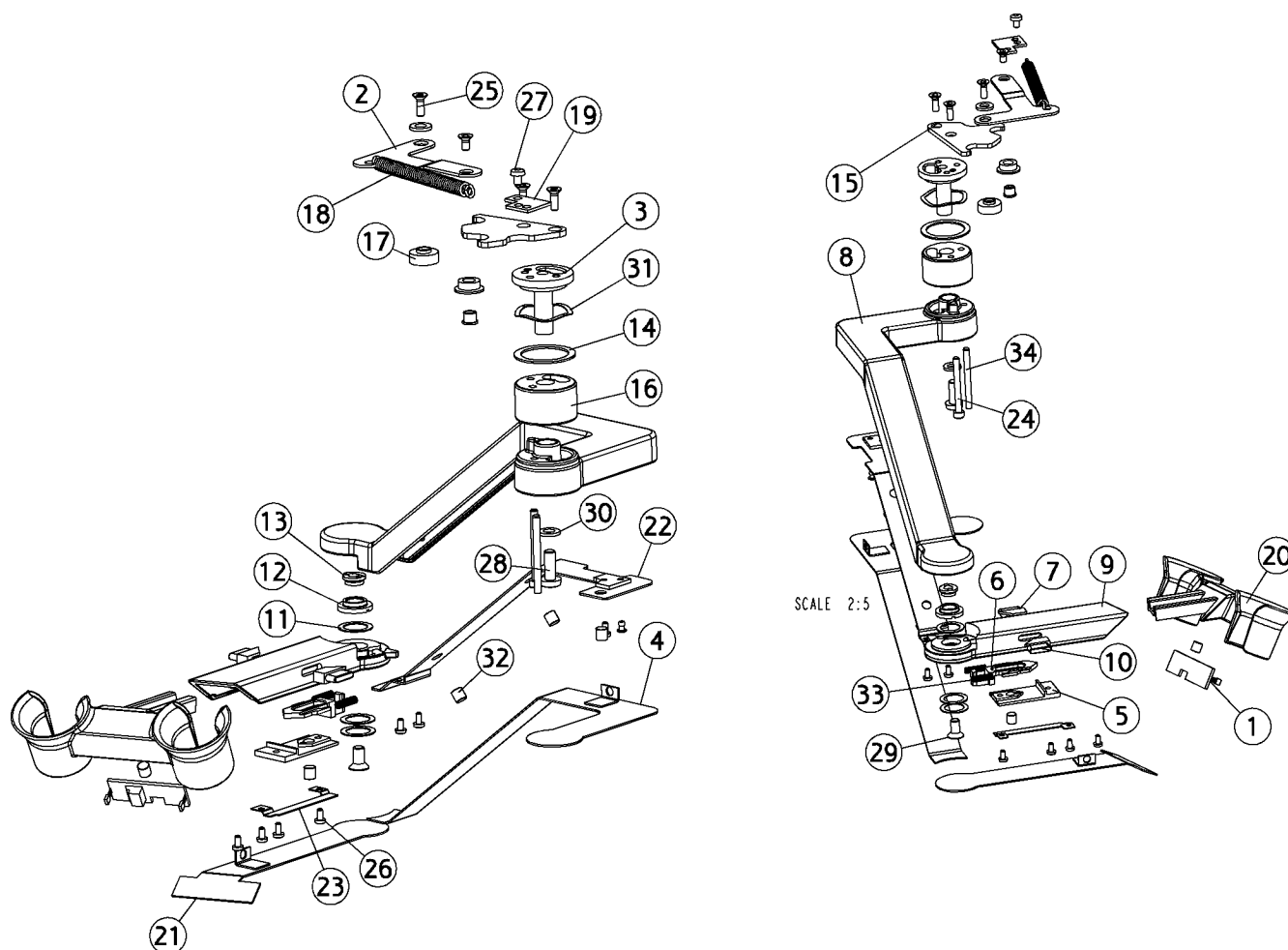

AN-009

Holder, suction hoses, ty. 2004

Position	Part NO.	Assembly	Description
1	AN-376		PCB, Suction holder, XO 4
2	AN-171		Arm, snap lock
3	MG-243		Not a spare part
4	MG-249		Not a spare part
5	MG-354		Not a spare part
6	MG-355		Not a spare part
7	MG-356		Not a spare part
8	AN-465		Arm suc.tube holder large, (XO-901 Sea Blue Metallic)
8	AN-464		Arm suc.tube holder large, (XO-909 Iron Grey (RAL 7011-30))
9	AN-466		Arm suc.tube holder,small, (XO-909 Iron Grey (RAL 7011-30))
9	AN-467		Arm suc.tube holder small, (XO-901 Sea Blue Metallic)
10	MG-502		Not a spare part
11	MG-586		Not a spare part
12	MG-587		Not a spare part
13	MG-588		Not a spare part
14	MG-595		Washer, arm for tray
15	MG-598		Plate, snap lock
16	MG-599		Not a spare part
17	MG-600		Bushing, arm, suction holder
18	MG-604		Spring,snap lock,suction arm
19	MG-605		Mounting bracket, spring
20	MG-616		Suction hoses holder,bridg.XO 4
21	AN-448		Cover, arm link2, (XO-909 Iron Grey (RAL 7011-30))
22	MG-641		Not a spare part
23	MG-653		Not a spare part
24	UG-038		Screw, allen M4x40
25	UG-040		Allen screw, ui6kt M4x12mm, din 7991
26	UG-102		Screw, thread making, M3x006 DIN7500
27	UG-111		Screw, thread making, M4x6
28	UG-113		Screw, thread making, M6x16
29	UG-152		Not a spare part
30	UG-256		Disc, Ø06.4xØ12.0x1.6


AN-009

Holder, suction hoses, ty. 2004

Position	Part NO.	Assembly	Description
31	UG-277		Corrugated spring Ø26.5x31,7x0.4x2.1
32	UG-414		Magnet Ø6,35x6,35 neodymium
33	UG-429		Not a spare part
34	UG-446		Not a spare part


AN-756

Holder, suction hoses,ty. 2005

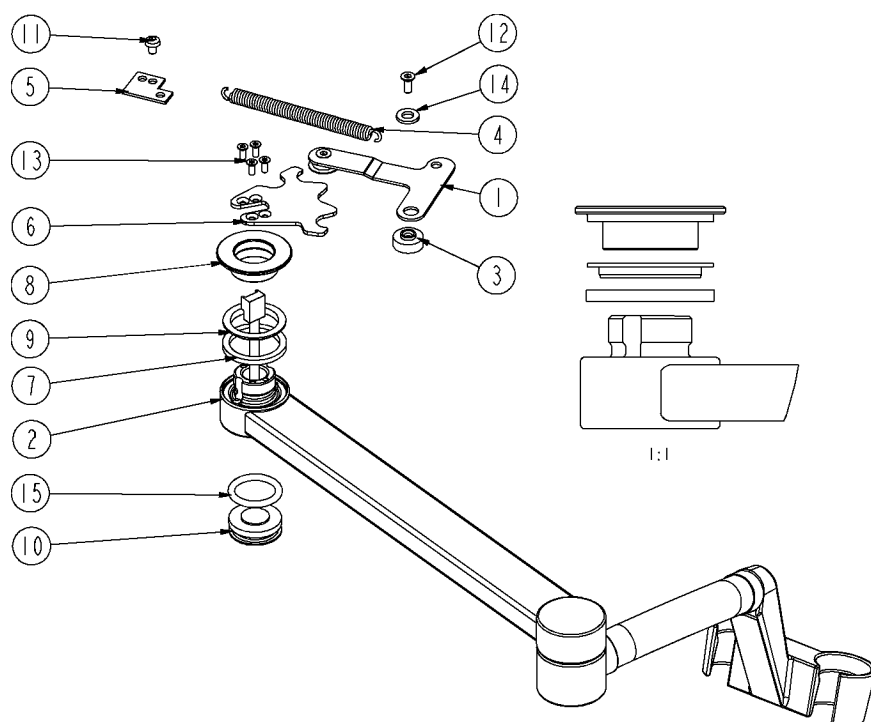

Position	Part NO.	Assembly	Description
1	AN-171	Page 146	Arm, snap lock
2	AN-757	Page 147	Arm suction hose holder bridge
3	MG-600		Bushing, arm, suction holder
4	MG-604		Spring, snap lock, suction arm
5	MG-605		Mounting bracket, spring
6	MG-751		Plate, position lock
7	MG-756		Sliding disc
8	MG-757		Not a spare part
9	MG-758		Adaptateur
10	MG-763		Plug, suctionho., bridge
11	UG-111		Screw, thread making, M4x6
12	UG-115		Screw, M4x010
13	UG-236		Screw M3x8, uh, thread forming
14	UG-256		Disc, Ø06.4xØ12.0x1.6
15	UG-304		O-ring, suction holder


AN-171 Arm, snap lock

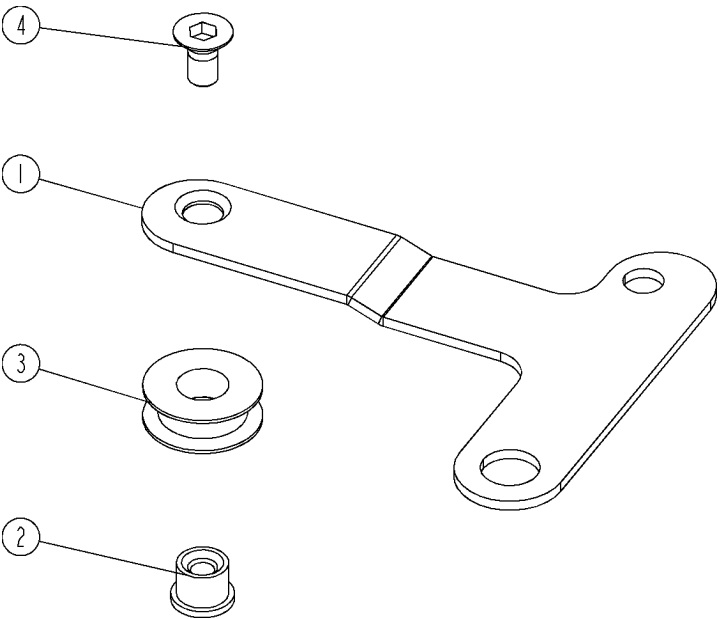

Position	Part NO.	Assembly	Description
1	MG-601		Arm, click lock f. suc. holder
2	MG-602		Not a spare part
3	MG-603		Roller f. click lock
4	UG-039		Not a spare part

AN-757


Arm suction hose holder bridge

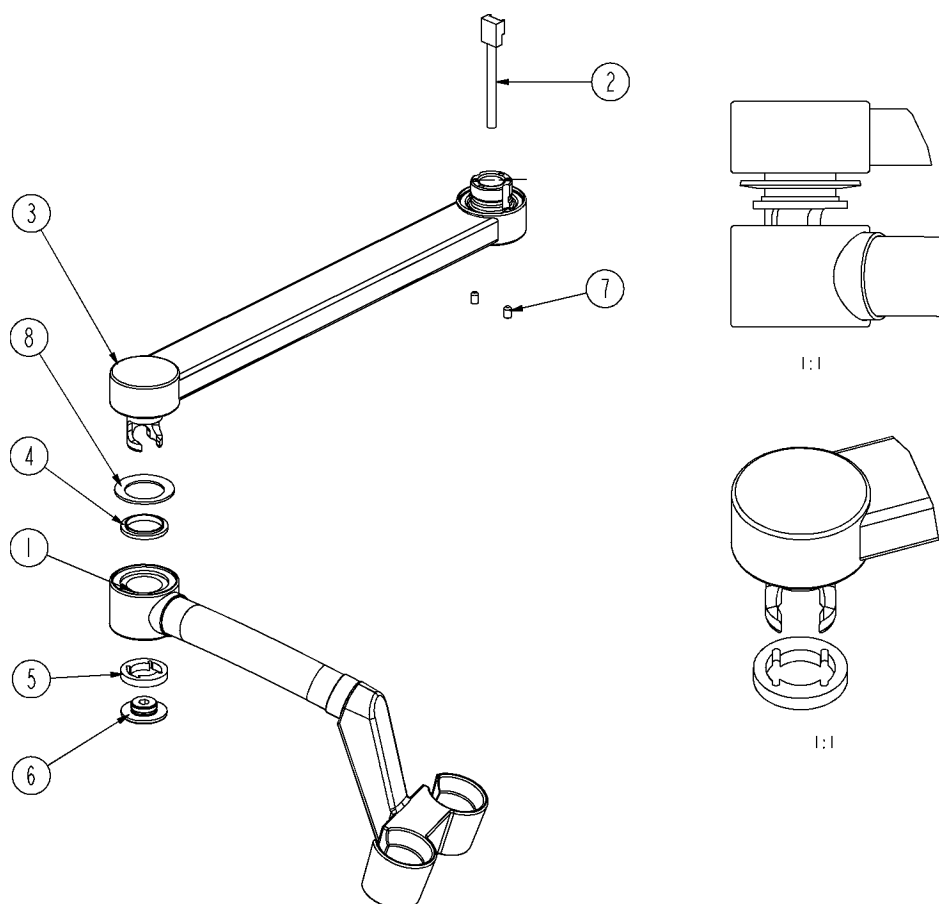

Position	Part NO.	Assembly	Description
1	AN-759	Page 148	Arm2 suc. hose holder with PCB
2	CA-045		Cable, succ. holder-bridge, v2
3	AN-761		Arm1 shh,
4	MG-753		Not a spare part
5	MG-754		Not a spare part
6	MG-755		Not a spare part
7	UG-116		Pointed screw, M4x6
8	UG-285		Not a spare part

AN-759


Arm2 suc. hose holder with PCB

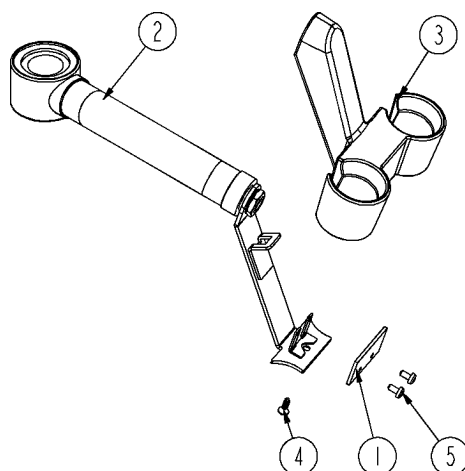


Position	Part NO.	Assembly	Description
1	AN-299		PCB, suction holder type 2005
2	AN-760	<i>Page 149</i>	Arm2 suction hose holder
3	MG-762		Suction tube holder
4	UC-086		Screw M3x10
5	UG-102		Screw, thread making, M3x006 DIN7500

AN-760

Arm2 suction hose holder

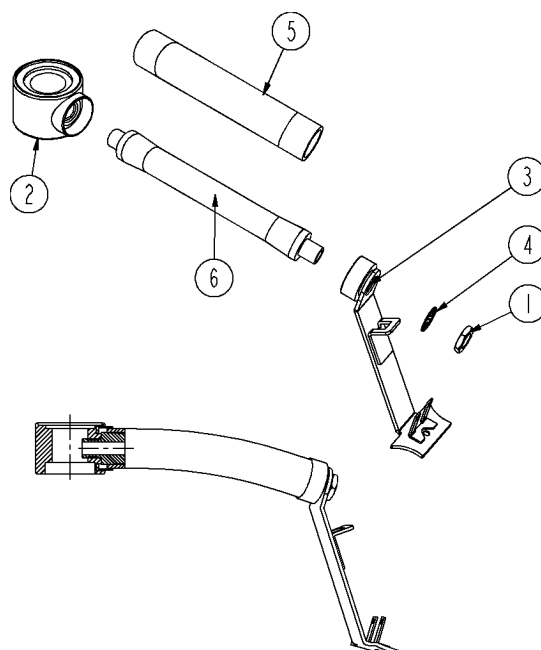


Position	Part NO.	Assembly	Description
1	ME-017		Nut, succ.hose holder, M10x1
2	AN-762		Link arm2 shh,
3	AN-758		Fitting shh,
4	UC-170		Not a spare part
5	UG-947		Silicone tube Ø21 x 16,8 RAL 7011
6	UG-949		Spring suc. holder, bridge

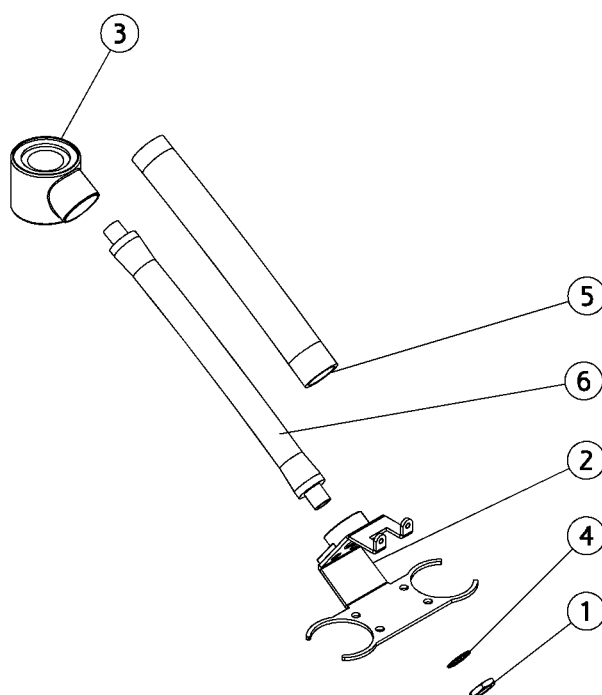


Position	Part NO.	Assembly	Description
1	ME-017		Nut, suct.hose holder, M10x1
2	AP-540		Not a spare part
3	AP-539		Not a spare part
4	UC-170		Not a spare part
5	UG-947		Silicone tube Ø21 x 16,8 RAL 7011
6	UH-141		Not a spare part

AP-225

Arm2 suction holder with PCB

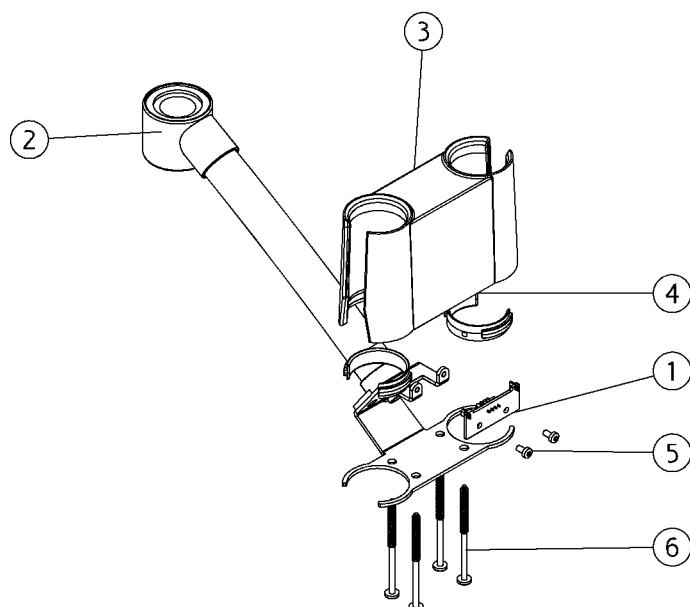


Position	Part NO.	Assembly	Description
1	AO-987		PCB, suction holder auto return
2	AP-224	<i>Page 150</i>	Arm2, suction holder, 2013
3	MG-986		Holder, auto return, bridge,
4	MN-043		Not a spare part
5	UH-088		Not a spare part
6	UH-155		Screw, panhead, plastite, K40X60/30

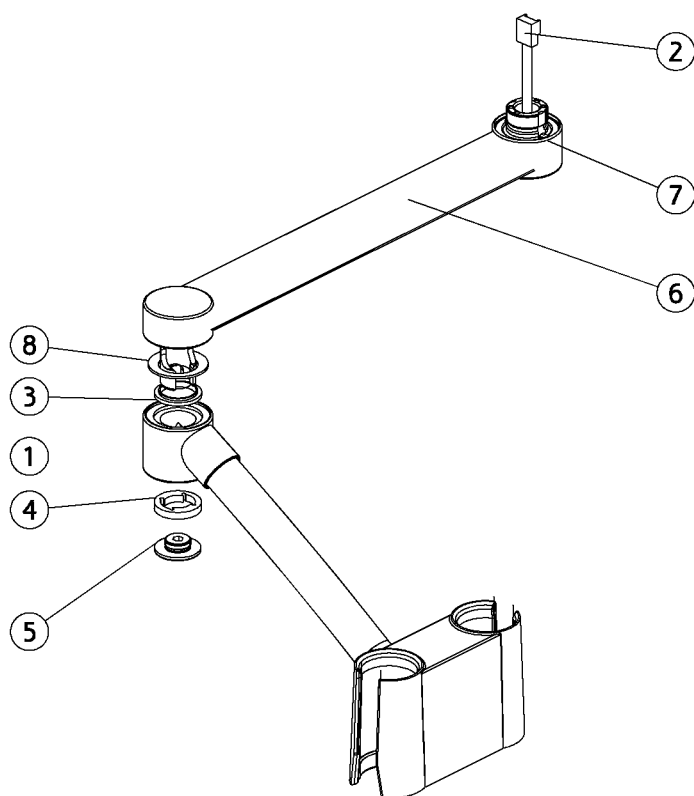

Position	Part NO.	Assembly	Description
1	AP-225	Page 151	Arm2 suction holder with PCB
2	CA-089		Not a spare part
3	MG-753		Not a spare part
4	MG-754		Not a spare part
5	MG-755		Not a spare part
6	AP-538		Not a spare part
7	UG-116		Pointed screw, M4x6
8	UG-285		Not a spare part

AP-533

Suction tube holder, bridge, type 2013,

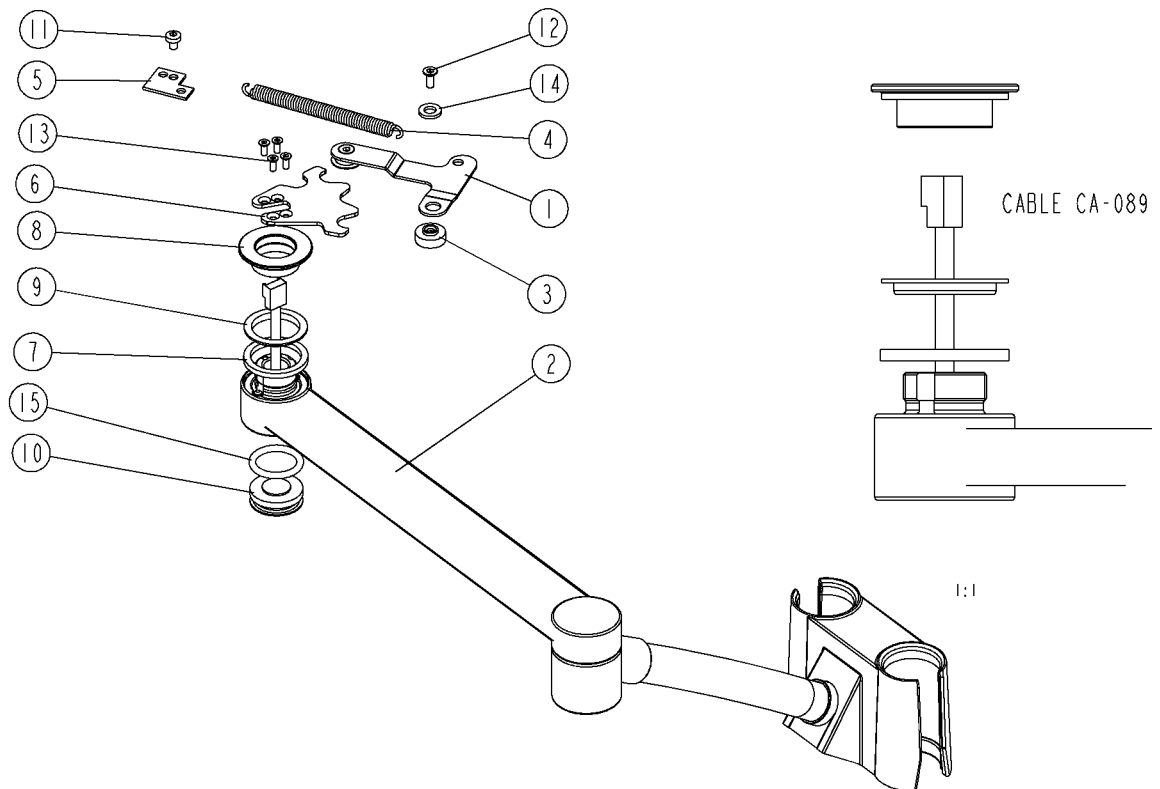


AP-533

Suction tube holder, bridge, type 2013,

Position	Part NO.	Assembly	Description
1	AN-171	<i>Page 146</i>	Arm, snap lock
2	AP-226	<i>Page 152</i>	Suction tube holder, bridge, 2013
3	MG-600		Bushing, arm, suction holder
4	MG-604		Spring, snap lock, suction arm
5	MG-605		Mounting bracket, spring
6	MG-751		Plate, position lock
7	MG-756		Sliding disc
8	MG-757		Not a spare part
9	MG-758		Adaptateur
10	MG-763		Plug, suctionho., bridge
11	UG-111		Screw, thread making, M4x6
12	UG-115		Screw, M4x010
13	UG-236		Screw M3x8, uh, thread forming
14	UG-256		Disc, Ø06.4xØ12.0x1.6
15	UG-304		O-ring, suction holder

AN-010


Inst.suspension base, 6 rolls

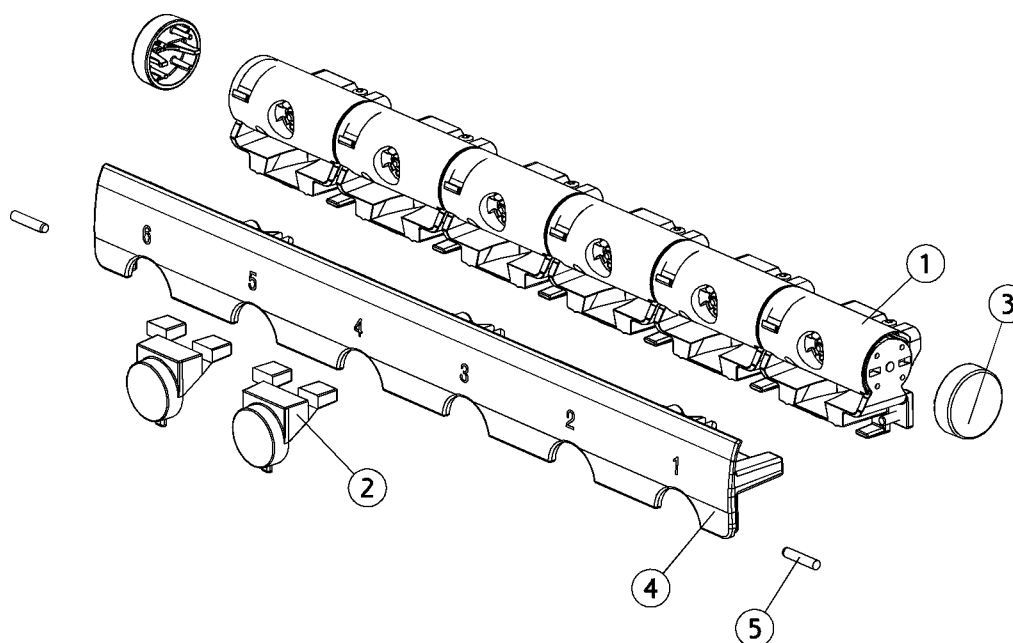

Position	Part NO.	Assembly	Description
1	AN-011		Balanced suspension
2	MG-172		Cover plug, instr.suspension
3	MG-175		End cover
4	AN-949		Rear plate, bridge, (XO-905 XO White,(RAL7047-30))
4	AN-733		Rear plate, bridge, (XO-903 Silver Metallic)
4	AN-460		Rear plate, bridge, (XO-900 XO Grey, RAL 7047-80)
4	AP-433		Rear plate, bridge, (CTBS)
4	AO-287		Rear plate, bridge, (XO-906 XO Titanium)
4	AN-461		Rear plate, bridge, (XO-901 Sea Blue Metallic)
5	UG-419		Not a spare part

AN-011


Balanced suspension

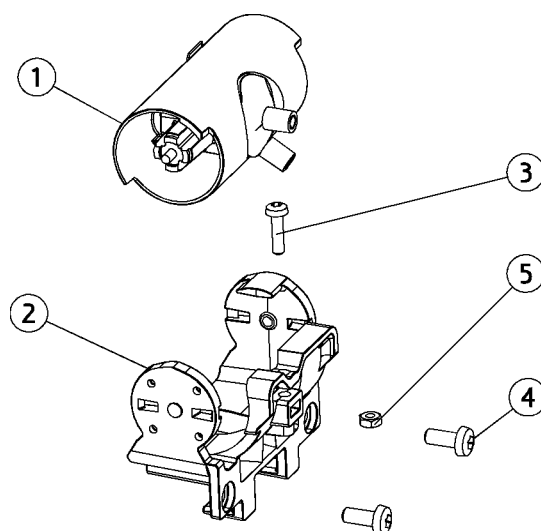

Position	Part NO.	Assembly	Description
1	AN-012	<i>Page 157</i>	Balanced suspension roll
2	AN-217	<i>Page 160</i>	Bearing pedestal,instr.susp.
3	UG-109		Screw, thread making, M3x12
4	UG-110		Screw, thread making, M4x10
5	UG-207		Nut, M3

AN-012


Balanced suspension roll

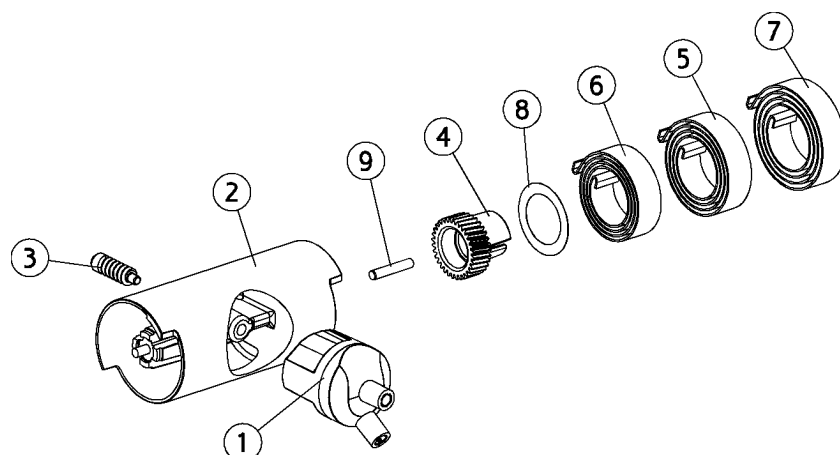

Position	Part NO.	Assembly	Description
1	AN-014	<i>Page 159</i>	Balanced suspension side stop
2	MG-159		Not a spare part
3	MG-164		Not a spare part
4	MG-166		Not a spare part
5	MG-169		Not a spare part
6	MG-439		Not a spare part
7	MG-440		Not a spare part
8	UG-264		Not a spare part
9	UG-445		Not a spare part

AN-084


Receiver, handles

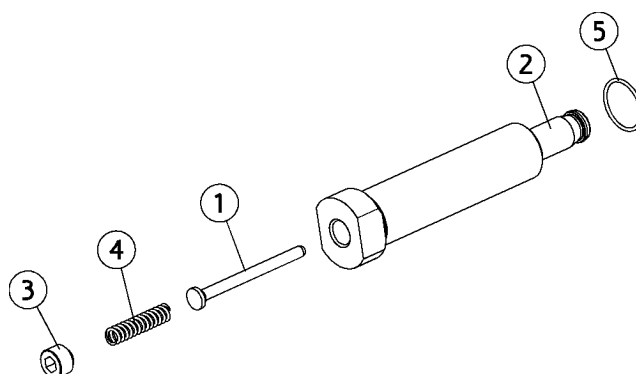

Position	Part NO.	Assembly	Description
1	MG-456		Pressure bar
2	MG-468		Not a spare part
3	UG-121		Not a spare part
4	UG-426		Not a spare part
5	UH-072		O-ring Ø13x0,9 viton

AN-014


Balanced suspension side stop

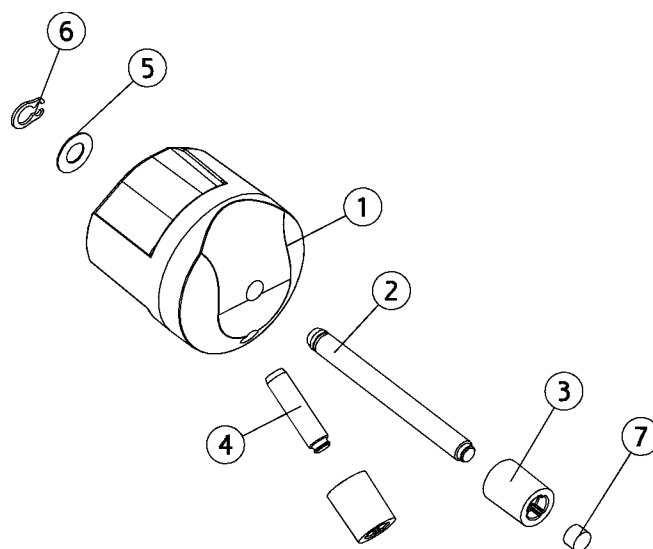

Position	Part NO.	Assembly	Description
1	MG-160		Not a spare part
2	MG-161		Not a spare part
3	MG-162		Tracking roller f. AN-011
4	MG-176		Not a spare part
5	UG-297		Not a spare part
6	UG-354		Not a spare part
7	UG-415		Magnet Ø3x2

AN-217


Bearing pedestal,instr.susp.

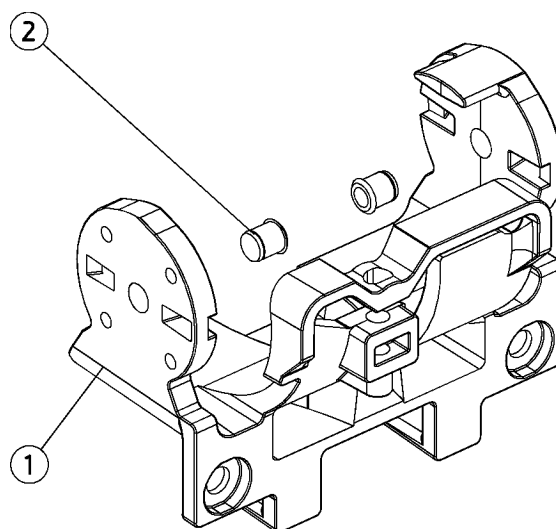

Position	Part NO.	Assembly	Description
1	MG-158		Not a spare part
2	MG-627		Not a spare part

AN-013


Suspension, arm XO 4

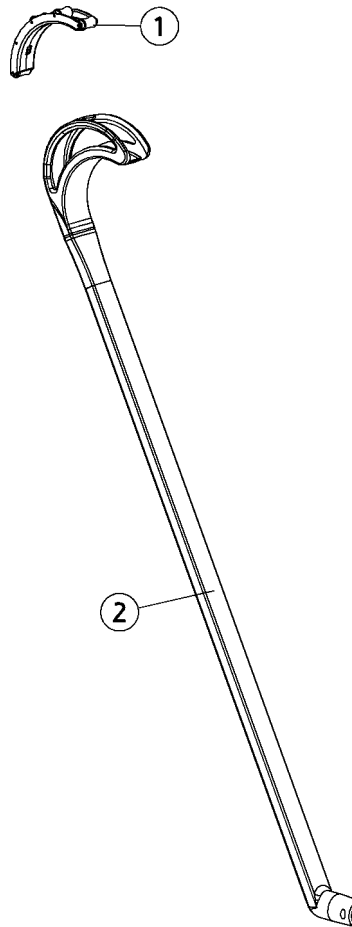

Position	Part NO.	Assembly	Description
1	AN-015	Page 162	Roll guide inst suspension XO 4
2	MG-167		Not a spare part

AN-015


Roll guide inst suspension XO 4

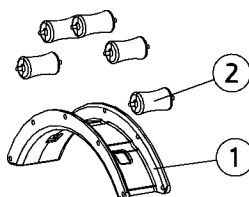

Position	Part NO.	Assembly	Description
1	MG-173		Not a spare part
2	MG-174		Not a spare part

AN-519

Rocker piece, bridge,assy

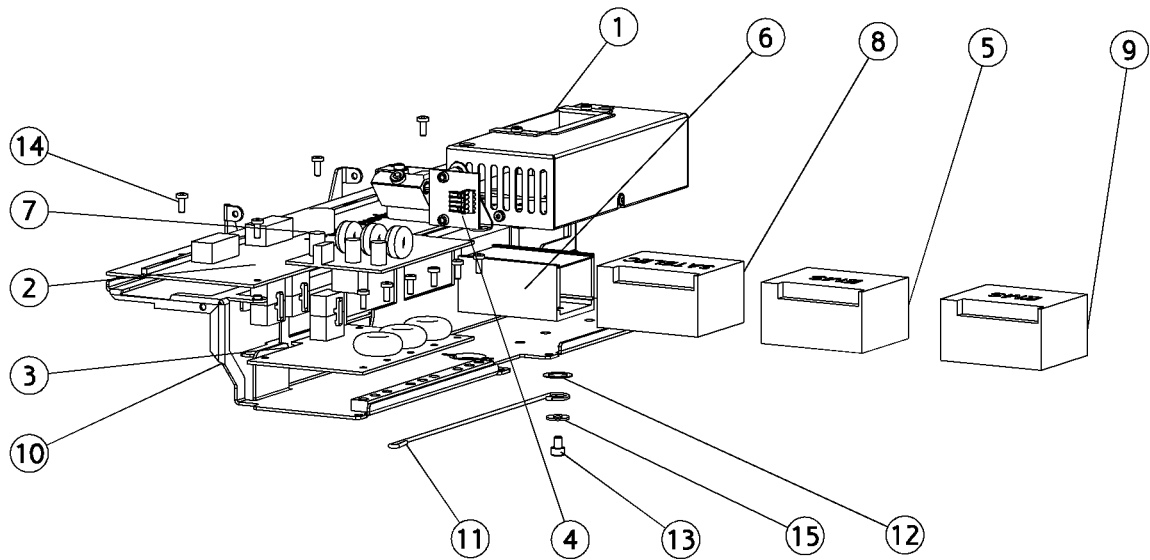

AN-519

Rocker piece, bridge,assy


Position	Part NO.	Assembly	Description
1	100111		Power module, osu
2	AN-368		PCB, Bridge, XO 4
3	AO-137		PCB, Powerdriver type 2007
4	AP-921		PCB, flow meter
5	AO-246		Control box & cable,EMS
6	AN-396		Control box, XO scaler
7	AO-134		PCB, Circuit board, DMX
8	AO-245		Control box+cable,satelec
9	AO-892		Elec module, piezon no pain EMS
10	MG-614		Not a spare part
11	MG-817		Support pin, bridge
12	UC-786		Adapter disc
13	UG-014		Not a spare part
14	UG-139		Screw. M3x8
15	UG-279		Washer, din9021 Ø4,3xØ12x1

AO-631 Electronic mount, bridge assembly

Position	Part NO.	Assembly	Description
1	AN-368		PCB, Bridge, XO 4
2	AO-137		PCB, Powerdriver type 2007
3	AD-660		Not a spare part
4	AP-921		PCB, flow meter
5	MG-614		Not a spare part
6	MG-817		Support pin, bridge
7	UC-786		Adapter disc
8	UG-014		Not a spare part
9	UG-139		Screw. M3x8
10	UG-279		Washer, din9021 Ø4,3xØ12x1
11	UG-427		Not a spare part

AP-921

PCB, flow meter

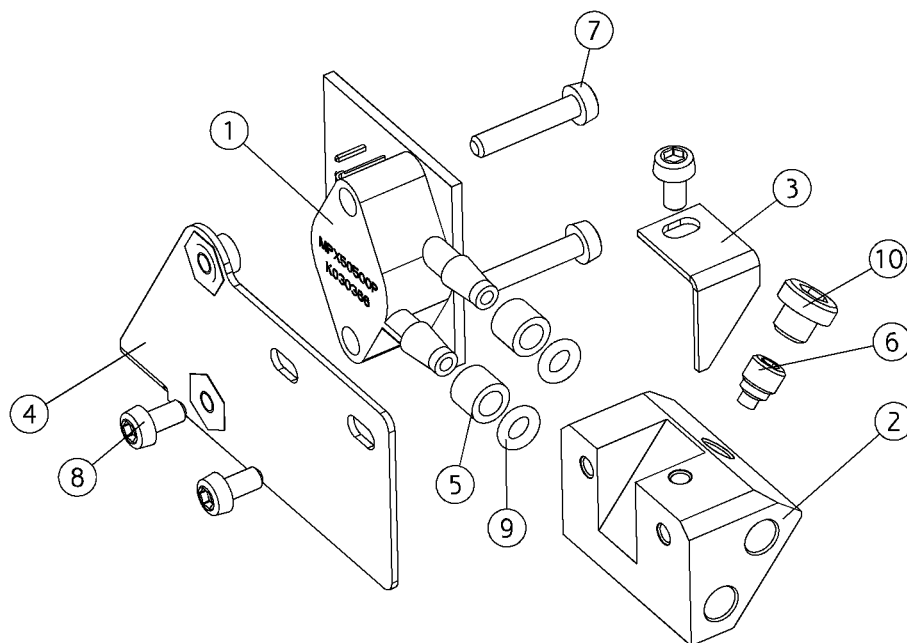


AP-921

PCB, flow meter

Position	Part NO.	Assembly	Description
1	AD-609		Not a spare part
2	MG-510		Block, flowmeter
3	MG-511		Not a spare part
4	MG-512		Not a spare part
5	MG-515		Not a spare part
6	MG-608		Nozzle, flowmeter
7	UG-141		Screw, M3 x16
8	UG-142		Not a spare part
9	UG-323		Not a spare part
10	UH-242		Not a spare part

AN-513


Hose connection with o-ring

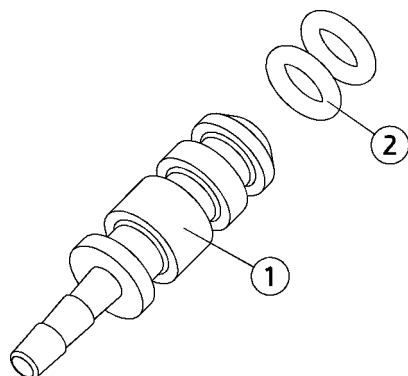

Position	Part NO.	Assembly	Description
1	ME-269		Not a spare part
2	MR-227		O-ring Ø3 x 1


AN-510

PCB, Suspension, syringe/lc/osseo

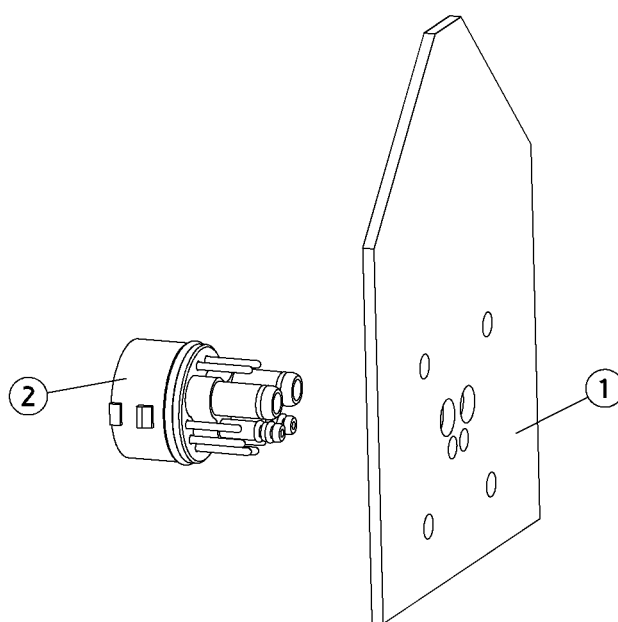

Position	Part NO.	Assembly	Description
1	AD-605		PCB, Balanced suspension XO 4
2	AP-190	Page 128	Connector w. tubings, inst block


AN-511 PCB, Suspension, LC LAMP

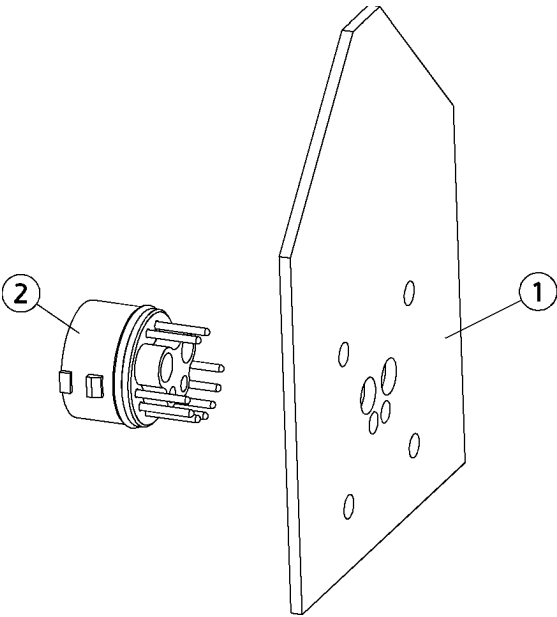

Position	Part NO.	Assembly	Description
1	AD-605		PCB, Balanced suspension XO 4
2	AN-191		Plug w/o pipes,instr.block


AN-512

PCB hose, +stick -tube, camera

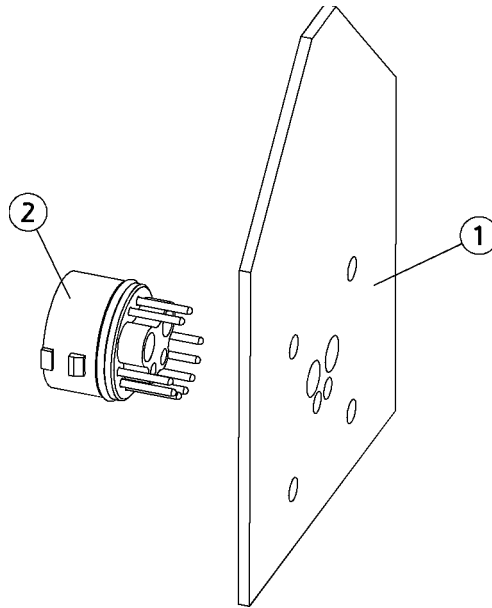

Position	Part NO.	Assembly	Description
1	AD-651		Not a spare part
2	AN-191		Plug w/o pipes,instr.block


AO-318 PCB, Suspension, MX/OSS/MC3/tur/lc/sca

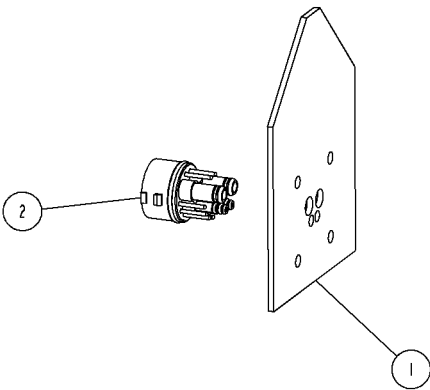

Position	Part NO.	Assembly	Description
1	AD-654		Not a spare part
2	AP-190	Page 128	Connector w. tubings, inst block

AN-509

Connector box with pins

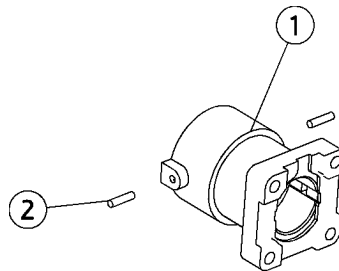

Position	Part NO.	Assembly	Description
1	MG-251		Not a spare part
2	UG-443		Not a spare part


AN-509

Connector box with pins

XO 4 Arm system for Instrument Bridge

AN-016


Arm,instr. bridge,basic

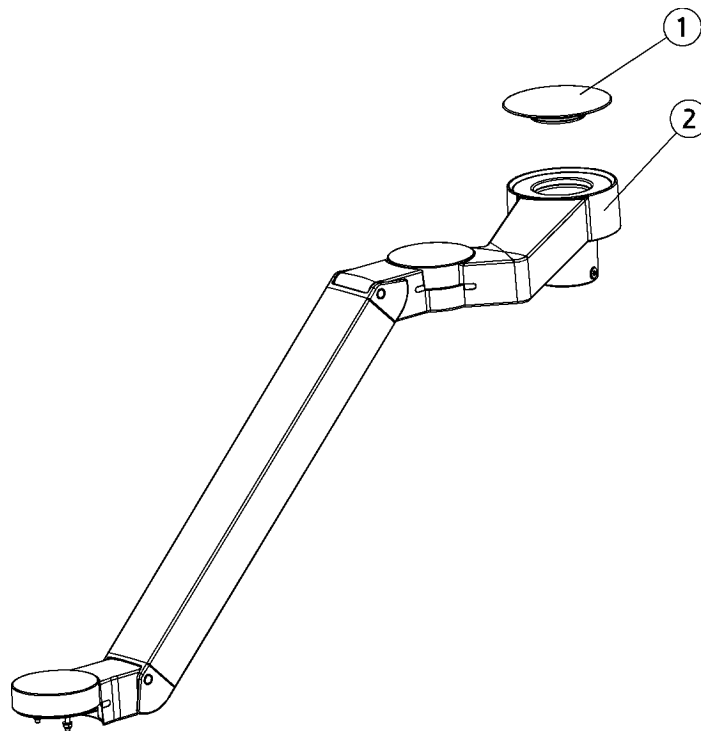

Position	Part NO.	Assembly	Description
1	AN-139	Page 177	Cover rear arm, w/o lamp
2	AN-142	Page 178	Arm,bridge excl cover

AN-139


Cover rear arm, w/o lamp

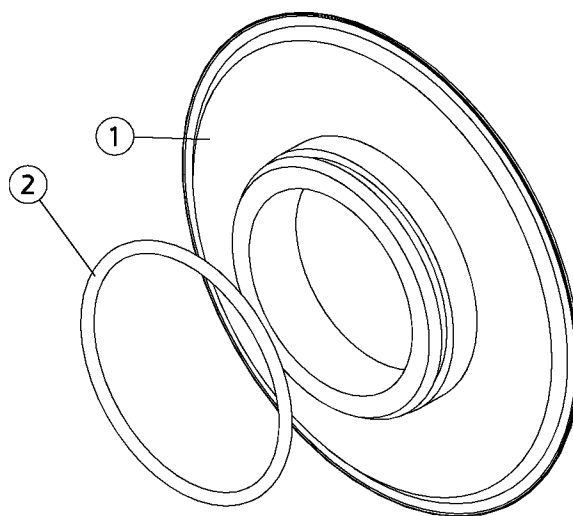

Position	Part NO.	Assembly	Description
1	AP-430		Cover plug,bridge arm Ø100, (CTBS)
1	AO-284		Cover plug,bridge arm Ø100, (XO-906 XO Titanium)
1	AO-728		Cover plug,bridge arm Ø100, (XO-903 Silver Metallic)
1	AN-946		Cover plug,bridge arm Ø100, (XO-905 XO White,(RAL7047-30))
1	AO-450		Cover plug,bridge arm Ø100, (XO-900 XO Grey, RAL 7047-80)
1	AO-451		Cover plug,bridge arm Ø100, (XO-901 Sea Blue Metallic)
2	UG-325		O-ring 3,53x69,44 NBR 70

AN-142


Arm,bridge excl cover

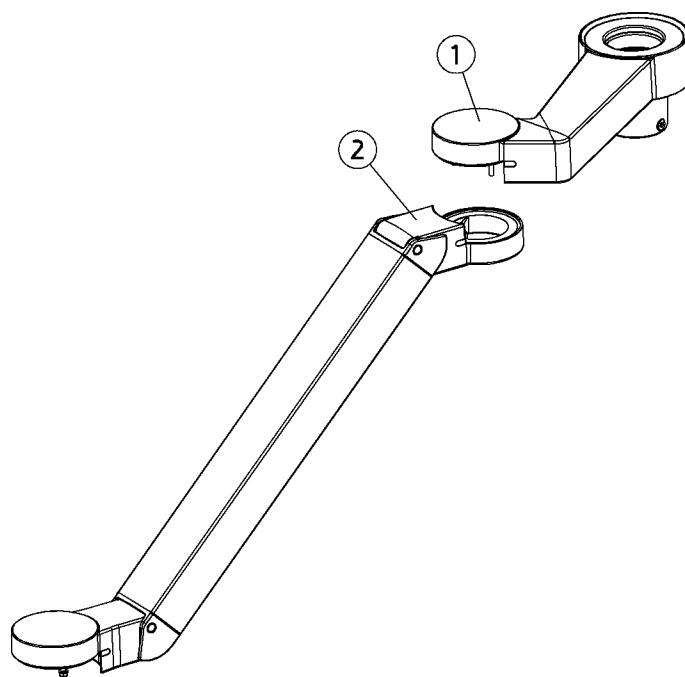

Position	Part NO.	Assembly	Description
1	AN-017	Page 179	Rear arm, instrument bridge
2	AN-023	Page 184	Forearm, instrument bridge

AN-017

Rear arm, instrument bridge

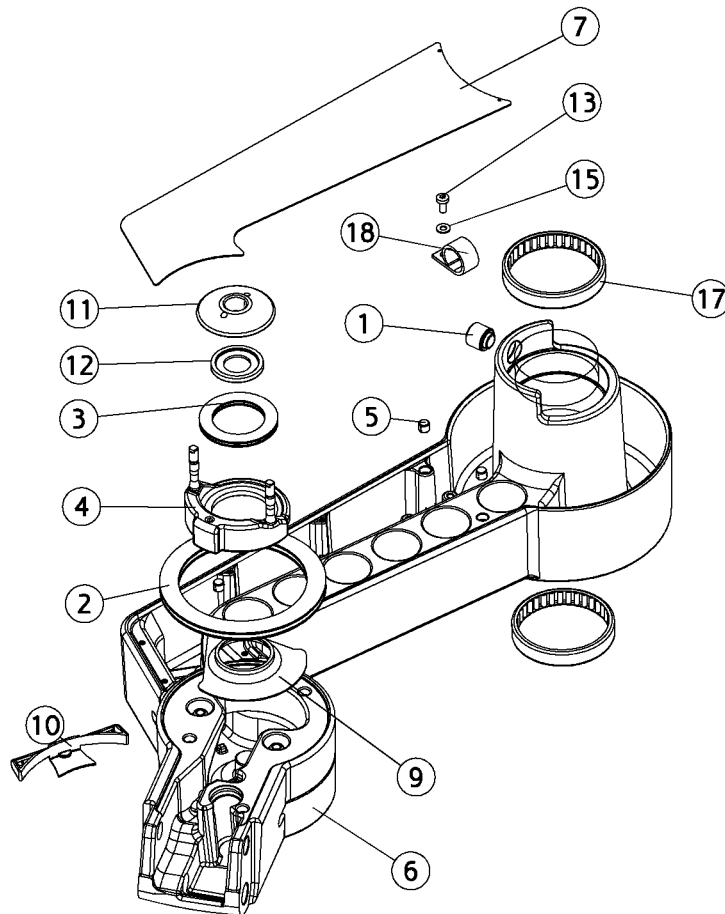

AN-017

Rear arm, instrument bridge

Position	Part NO.	Assembly	Description
1	AN-018	<i>Page 182</i>	Friction brake, arms
2	AN-020	<i>Page 204</i>	Axle bearing 85x110x6, bridge
3	AN-021	<i>Page 202</i>	Axle bearing 35x52x4, bridge
4	AN-022	<i>Page 199</i>	Suspension link, bridge
5	AN-127	<i>Page 198</i>	Magnet Ø6,35 with spring ring
6	AP-408		Rear arm, bridge, (CTBS)
6	AN-415		Rear arm, bridge, (XO-901 Sea Blue Metallic)
6	AN-414		Rear arm, bridge, (XO-900 XO Grey, RAL 7047-80)
6	AN-711		Rear arm, bridge, (XO-903 Silver Metallic)
6	AO-257		Rear arm, bridge, (XO-906 XO Titanium)
6	AN-922		Rear arm, bridge, (XO-905 XO White,(RAL7047-30))
7	AO-259		Cover, rear arm, bridge, (XO-906 XO Titanium)
7	AN-419		Cover, rear arm, bridge, (XO-901 Sea Blue Metallic)
7	AN-418		Cover, rear arm, bridge, (XO-900 XO Grey, RAL 7047-80)
7	AP-410		Cover, rear arm, bridge, (CTBS)
7	AN-924		Cover, rear arm, bridge, (XO-905 XO White,(RAL7047-30))
7	AN-713		Cover, rear arm, bridge, (XO-903 Silver Metallic)
8	AN-925		Link1, forearm bridge, (XO-905 XO White,(RAL7047-30))
8	AN-420		Link1, forearm bridge, (XO-900 XO Grey, RAL 7047-80)
8	AO-260		Link1, forearm bridge, (XO-906 XO Titanium)
8	AP-411		Link1, forearm bridge, (CTBS)
8	AN-714		Link1, forearm bridge, (XO-903 Silver Metallic)
8	AN-421		Link1, forearm bridge, (XO-901 Sea Blue Metallic)
9	MG-070		Not a spare part
10	MG-103		Not a spare part
11	MG-896		Not a spare part
12	MG-902		Not a spare part
13	UG-110		Screw, thread making, M4x10
14	UG-206		Not a spare part
15	UG-254		Disc, Ø04.3xØ09.0x0.8
16	UG-256		Disc, Ø06.4xØ12.0x1.6
17	UG-469		Not a spare part


AN-017

Rear arm, instrument bridge

Position	Part NO.	Assembly	Description
18	UG-528		Not a spare part


AN-018

Friction brake, arms

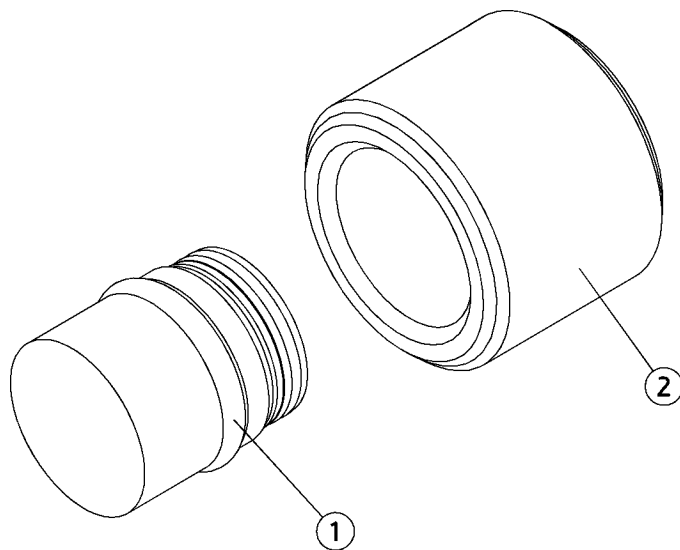


Position	Part NO.	Assembly	Description
1	AN-019	<i>Page 183</i>	Brake block, instrument bridge
2	MG-016		Not a spare part

AN-019 Brake block, instrument bridge

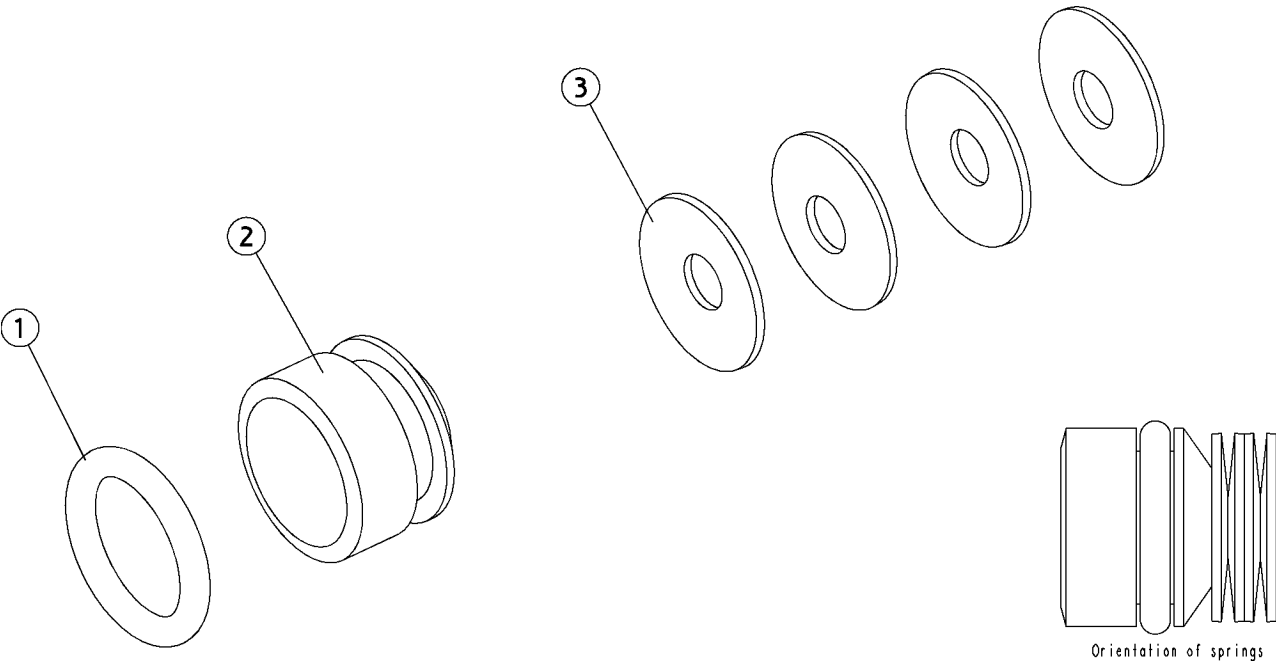

Position	Part NO.	Assembly	Description
1	000536		Not a spare part
2	MG-099		Not a spare part
3	UG-263		Not a spare part

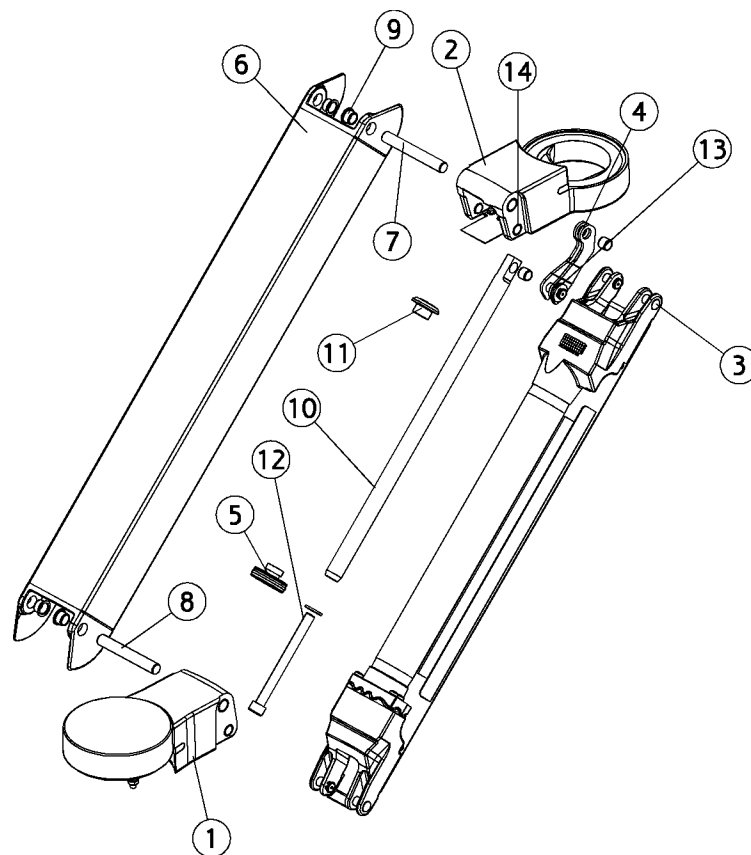


AN-023

Forearm, instrument bridge

Position	Part NO.	Assembly	Description
1	AN-024	<i>Page 186</i>	Forearm link2, instrument bridge
2	AN-025	<i>Page 189</i>	Forearm link1, instrument bridge
3	AN-028	<i>Page 191</i>	Forearm bottom, instrument bridge
4	AN-032	<i>Page 195</i>	Draw bar, instrument bridge
5	AN-117	<i>Page 196</i>	Control socket spring brdg.arm
6	AN-712		Top forearm, bridge, (XO-903 Silver Metallic)
6	AN-417		Top forearm, bridge, (XO-901 Sea Blue Metallic)
6	AO-258		Top forearm, bridge, (XO-906 XO Titanium)
6	AN-416		Top forearm, bridge, (XO-900 XO Grey, RAL 7047-80)
6	AN-923		Top forearm, bridge, (XO-905 XO White,(RAL7047-30))
6	AP-409		Top forearm, bridge, (CTBS)
7	MG-077		Axle, rear arm, bridge,XO 4
8	MG-078		Axle, front arm, bridge,XO 4
9	MG-079		Bushing, 12x14 with click
10	MG-084		Not a spare part
11	MG-449		Not a spare part
12	UG-011		Not a spare part
13	UG-259		Washer Ø10.5xØ20.0x2.0
14	UG-461		Not a spare part

AN-024

Forearm link2, instrument bridge

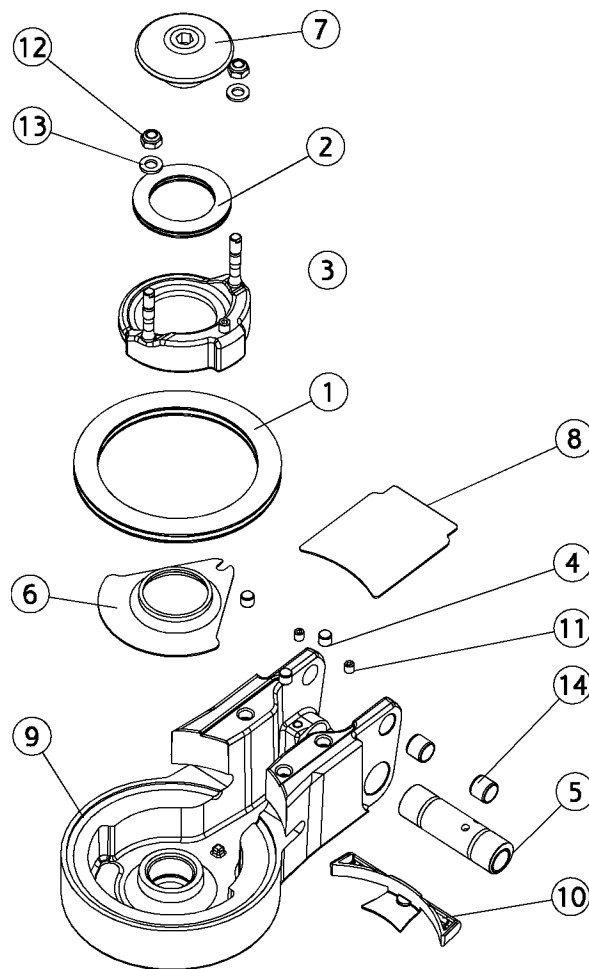


AN-024

Forearm link2, instrument bridge

Position	Part NO.	Assembly	Description
1	AN-020	<i>Page 204</i>	Axle bearing 85x110x6, bridge
2	AN-021	<i>Page 202</i>	Axle bearing 35x52x4, bridge
3	AN-022	<i>Page 199</i>	Suspension link, bridge
4	AN-127	<i>Page 198</i>	Magnet Ø6,35 with spring ring
5	AN-644	<i>Page 188</i>	Excentric link2 bridge arm
6	MG-070		Not a spare part
7	MG-071		Not a spare part
8	AP-413		Cover plate, link2, bridge, (CTBS)
8	AN-927		Cover plate, link2, bridge, (XO-905 XO White,(RAL7047-30))
8	AO-262		Cover plate, link2, bridge, (XO-906 XO Titanium)
8	AN-716		Cover plate, link2, bridge, (XO-903 Silver Metallic)
8	AN-426		Cover plate, link2, bridge, (XO-900 XO Grey, RAL 7047-80)
8	AN-427		Cover plate, link2, bridge, (XO-901 Sea Blue Metallic)
9	AN-434		Link2, forearm, bridge, (XO-900 XO Grey, RAL 7047-80)
9	AP-415		Link2, forearm, bridge, (CTBS)
9	AN-720		Link2, forearm, bridge, (XO-903 Silver Metallic)
9	AN-435		Link2, forearm, bridge, (XO-901 Sea Blue Metallic)
9	AN-931		Link2, forearm, bridge, (XO-905 XO White,(RAL7047-30))
9	AO-266		Link2, forearm, bridge, (XO-906 XO Titanium)
10	MG-103		Not a spare part
11	UA-563		Not a spare part
12	UG-206		Not a spare part
13	UG-256		Disc, Ø06.4xØ12.0x1.6
14	UG-461		Not a spare part


AN-644 Excentric link2 bridge arm

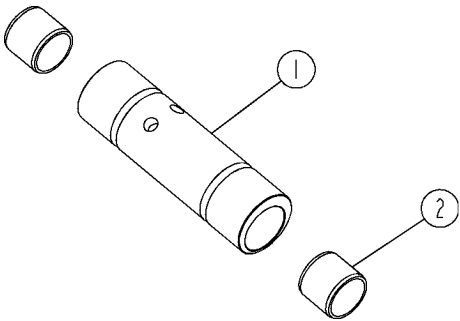

Position	Part NO.	Assembly	Description
1	MG-670		Not a spare part
2	UG-462		Not a spare part

AN-025

Forearm link1, instrument bridge

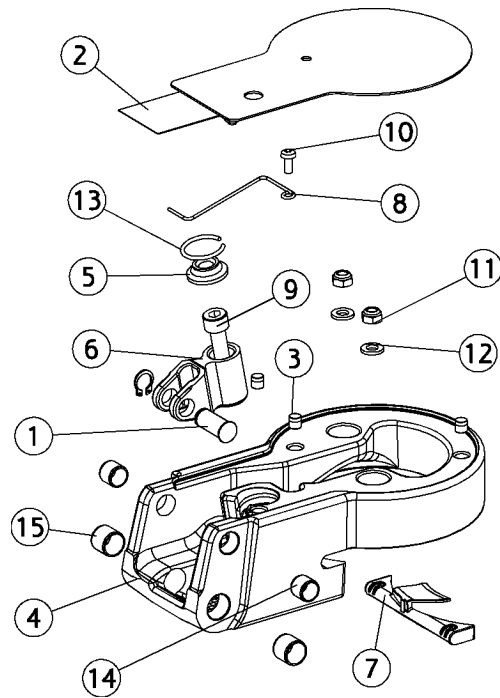


AN-025

Forearm link1, instrument bridge

Position	Part NO.	Assembly	Description
1	AN-026	<i>Page 193</i>	Axle 10x22, instrument bridge
2	AN-027	<i>Page 194</i>	Cover plate link1,instr.bridge
3	AN-127	<i>Page 198</i>	Magnet Ø6,35 with spring ring
4	AN-925		Link1, forearm bridge, (XO-905 XO White,(RAL7047-30))
4	AN-420		Link1, forearm bridge, (XO-900 XO Grey, RAL 7047-80)
4	AO-260		Link1, forearm bridge, (XO-906 XO Titanium)
4	AP-411		Link1, forearm bridge, (CTBS)
4	AN-714		Link1, forearm bridge, (XO-903 Silver Metallic)
4	AN-421		Link1, forearm bridge, (XO-901 Sea Blue Metallic)
5	MG-076		Not a spare part
6	MG-101		Not a spare part
7	MG-102		Not a spare part
8	MG-337		Not a spare part
9	MG-524		Not a spare part
10	UG-110		Screw, thread making, M4x10
11	UG-206		Not a spare part
12	UG-256		Disc, Ø06.4xØ12.0x1.6
13	UG-359		Not a spare part
14	UG-461		Not a spare part
15	UG-462		Not a spare part

AN-028

Forearm bottom, instrument bridge

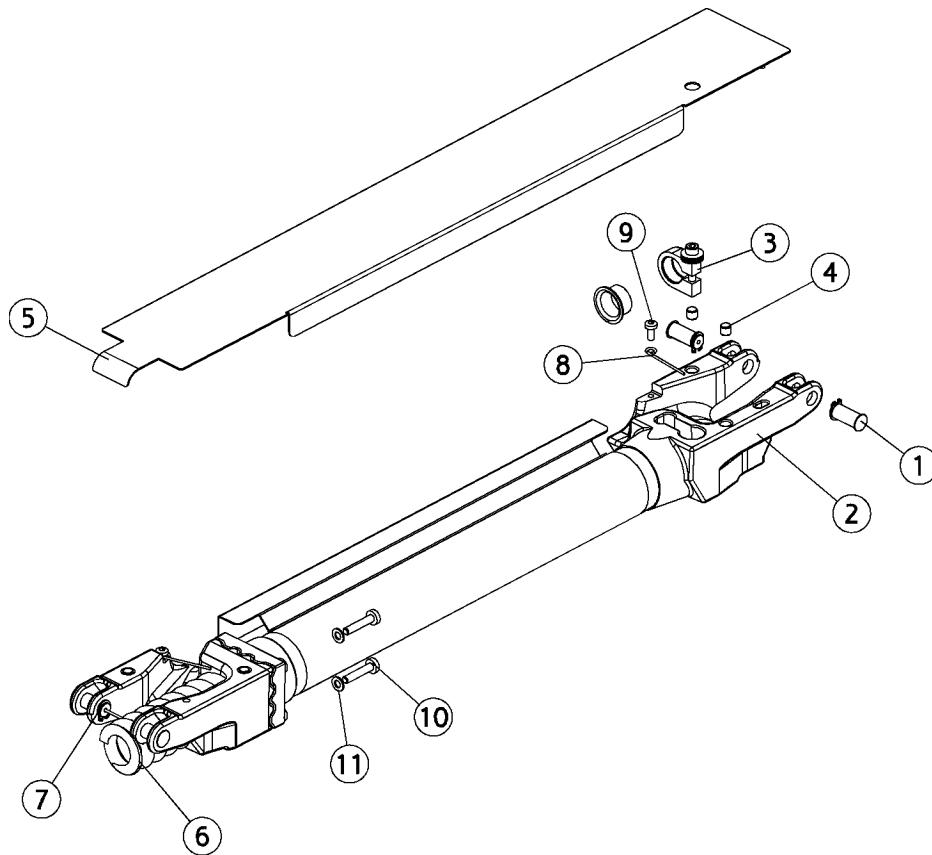


AN-028

Forearm bottom, instrument bridge

Position	Part NO.	Assembly	Description
1	AN-026	<i>Page 193</i>	Axle 10x22, instrument bridge
2	AN-901		Bottom bridge arm glued, (XO-903 Silver Metallic)
2	AN-525		Bottom bridge arm glued, (XO-907) (XO-901 Sea Blue Metallic)
2	AN-998		Bottom bridge arm glued, (XO-905 XO White,(RAL7047-30))
2	AP-476		Bottom bridge arm glued, (CTBS)
2	AN-524		Bottom bridge arm glued, (XO-900 XO Grey, RAL 7047-80)
2	AO-315		Bottom bridge arm glued, (XO-906 XO Titanium)
3	AN-031	<i>Page 197</i>	Forearm brake, instrument bridge
4	AN-127	<i>Page 198</i>	Magnet Ø6,35 with spring ring
5	AN-428		Cover, forearm, bridge, (XO-900 XO Grey, RAL 7047-80)
5	AP-414		Cover, forearm, bridge, (CTBS)
5	AN-717		Cover, forearm, bridge, (XO-903 Silver Metallic)
5	AN-928		Cover, forearm, bridge, (XO-905 XO White,(RAL7047-30))
5	AO-263		Cover, forearm, bridge, (XO-906 XO Titanium)
5	AN-429		Cover, forearm, bridge, (XO-901 Sea Blue Metallic)
6	MG-999		Not a spare part
7	MG-306		Not a spare part
8	MG-320		Not a spare part
9	UG-110		Screw, thread making, M4x10
10	UG-112		Not a spare part
11	UG-260		Not a spare part


AN-026 Axle 10x22, instrument bridge

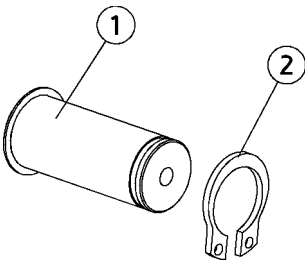

Position	Part NO.	Assembly	Description
1	MG-097		Not a spare part
2	UG-355		Not a spare part

AN-027


Cover plate link1,instr.bridge

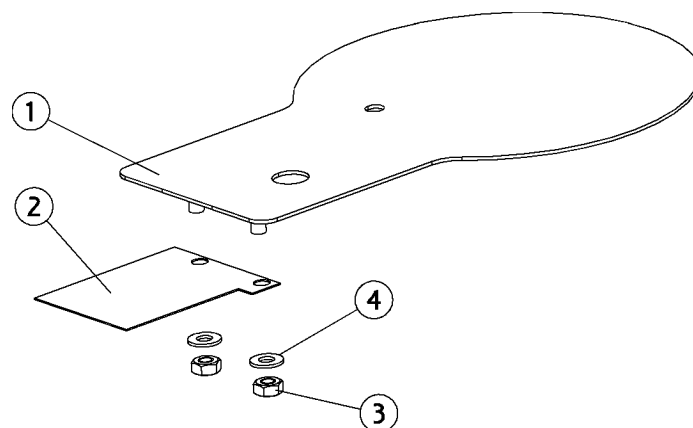

Position	Part NO.	Assembly	Description
1	AN-422		Cover plate. link1, bridge, (XO-900 XO Grey, RAL 7047-80)
1	AP-412		Cover plate. link1, bridge, (CTBS)
1	AN-423		Cover plate. link1, bridge, (XO-901 Sea Blue Metallic)
1	AO-261		Cover plate. link1, bridge, (XO-906 XO Titanium)
1	AN-926		Cover plate. link1, bridge, (XO-905 XO White,(RAL7047-30))
1	AN-715		Cover plate. link1, bridge, (XO-903 Silver Metallic)
2	MG-075		Cover plate, front arm,bridge
3	UG-205		Nut, M4, din 934
4	UG-254		Disc, Ø04.3xØ09.0x0.8

AN-032


Draw bar, instrument bridge

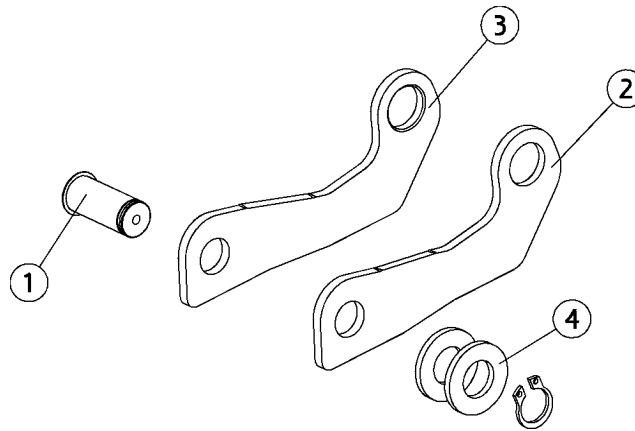

Position	Part NO.	Assembly	Description
1	AN-026	<i>Page 193</i>	Axle 10x22, instrument bridge
2	MG-082		Not a spare part
3	MG-083		Not a spare part
4	UG-259		Washer Ø10.5xØ20.0x2.0

AN-117


Control socket spring brdg.arm

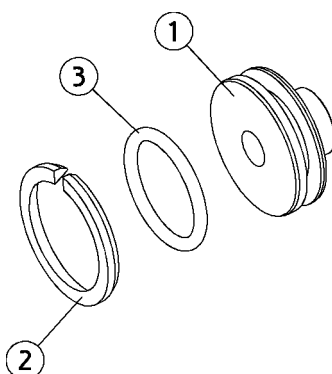

Position	Part NO.	Assembly	Description
1	MG-451		Support washer, spring, bridge
2	MG-508		Not a spare part
3	UG-321		Not a spare part

AN-031


Forearm brake, instrument bridge

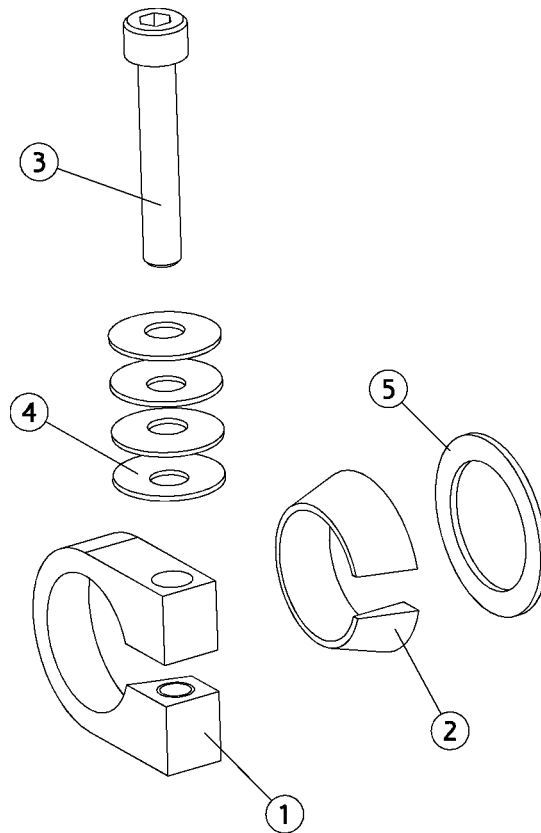

Position	Part NO.	Assembly	Description
1	MG-332		Not a spare part
2	MG-333		Not a spare part
3	UG-015		Not a spare part
4	UG-262		Not a spare part
5	UG-266		Not a spare part

AN-127


Magnet Ø6,35 with spring ring

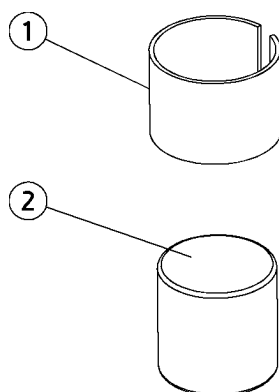

Position	Part NO.	Assembly	Description
1	MG-784		Locking spring, magnet
2	UG-414		Magnet Ø6,35x6,35 neodymium


AN-022

Suspension link, bridge

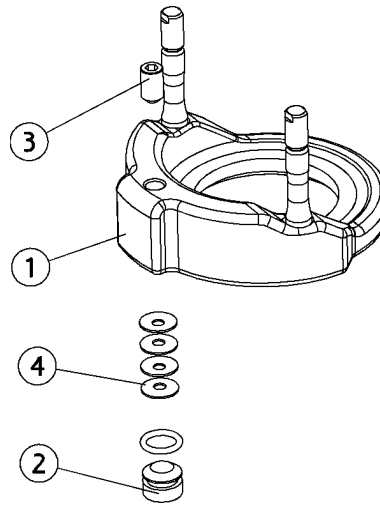


Position	Part NO.	Assembly	Description
1	AN-007		Suspension link bridge XO 4
2	AN-019	<i>Page 183</i>	Brake block, instrument bridge
3	UG-105		Pointed screw, M6x10
4	UG-263		Not a spare part

AN-103 Axle 8x17

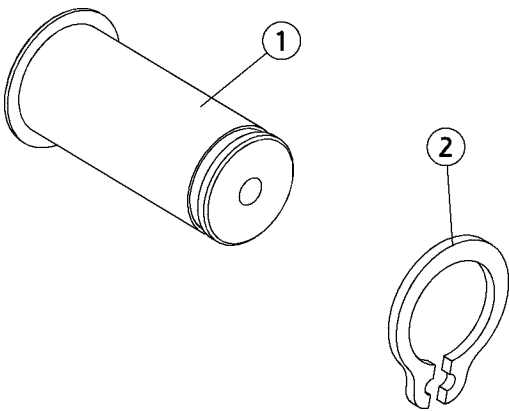


Position	Part NO.	Assembly	Description
1	MG-309		Not a spare part
2	UG-357		Not a spare part

AN-095 Axle bearing 20x35x4

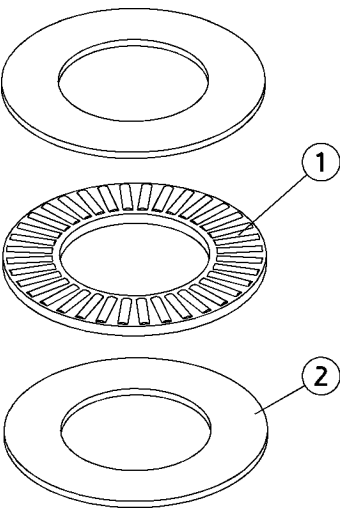


Position	Part NO.	Assembly	Description
1	UG-450		Not a spare part
2	UG-454		Not a spare part

AN-021 Axle bearing 35x52x4, bridge

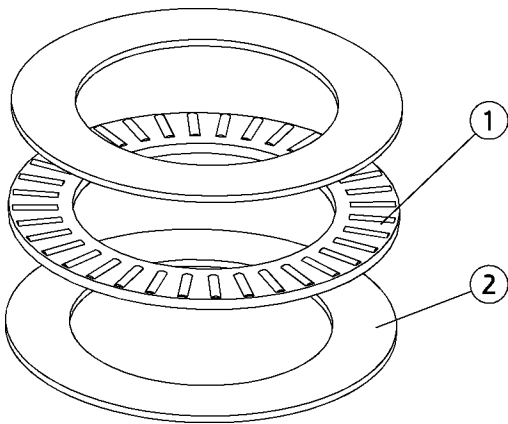


Position	Part NO.	Assembly	Description
1	UG-451		Not a spare part
2	UG-455		Not a spare part

AN-094 **Axle bearing 60x85x5**

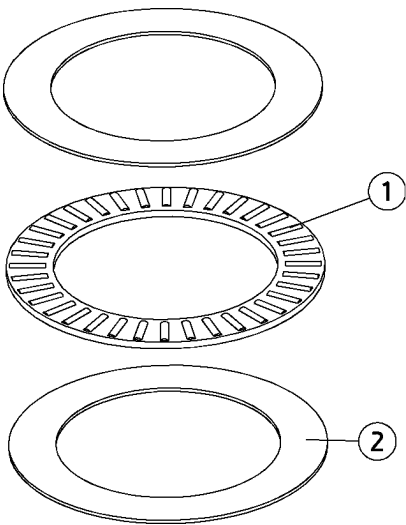

Position	Part NO.	Assembly	Description
1	UG-452		Not a spare part
2	UG-456		Not a spare part

AN-020 Axle bearing 85x110x6, bridge

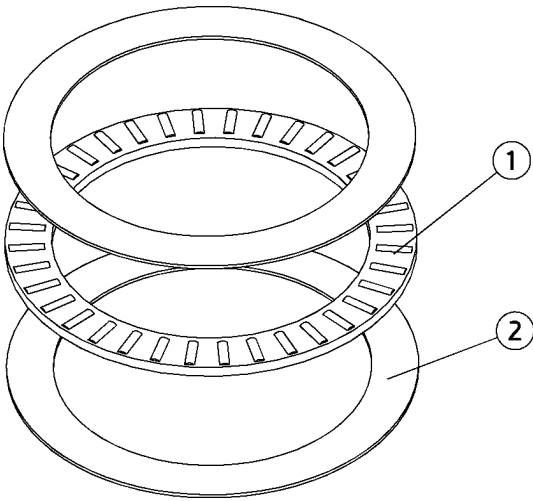


Position	Part NO.	Assembly	Description
1	UG-453		Not a spare part
2	UG-457		Not a spare part

XO 4 Peristaltic Pump

AO-159


XO pump, panel + misc.

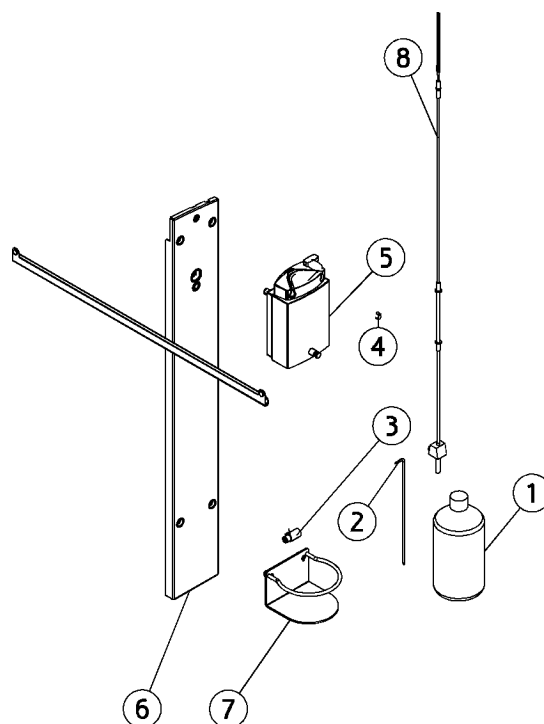

Position	Part NO.	Assembly	Description
1	207105		Bottle, 1 l
2	209190		Spear for odontoson-m
3	300042		Not a spare part
4	360195		Not a spare part
5	AO-160	Page 207	Pump, tubing, complete
6	AO-237	Page 212	Panel, rear right, f.pump, (XO-905 XO White,(RAL7047-30))
6	AP-492	Page 212	Panel, rear right, f.pump, (CTBS)
7	AO-196	Page 214	Shelf for bottle
8	XO-055		Disposable irrigation kit - XO Peristaltic pump - 50 pcs

AO-160


Pump, tubing, complete

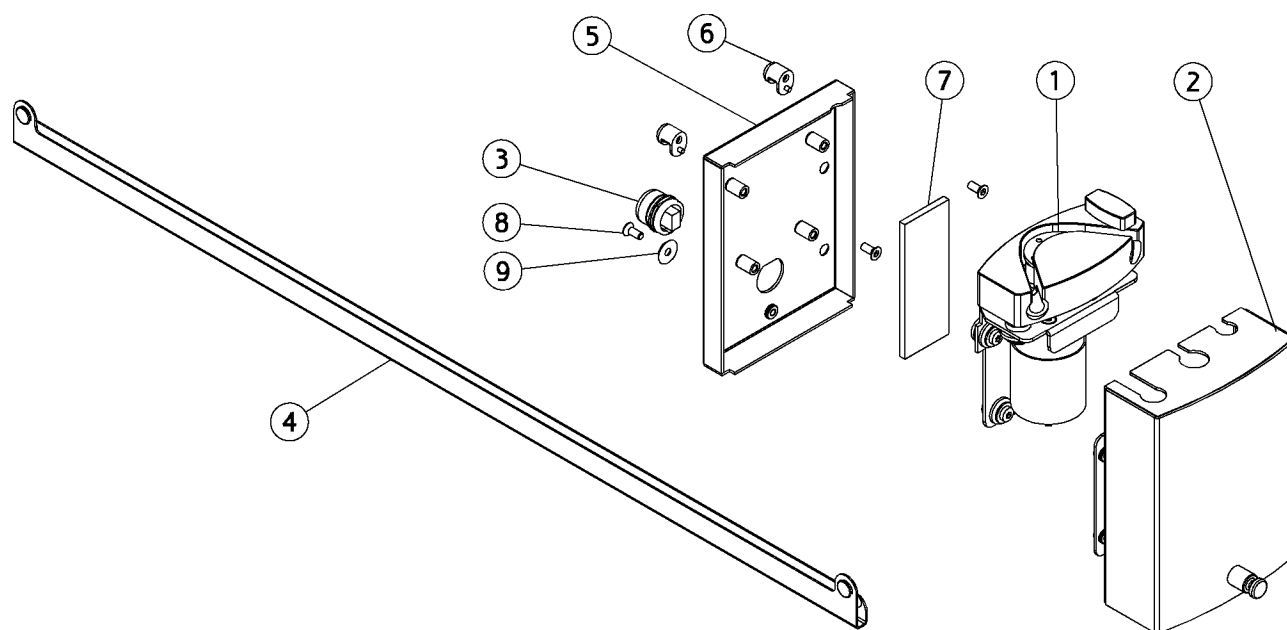

Position	Part NO.	Assembly	Description
1	AO-161	Page 208	Pump with motor
2	AO-163	Page 210	Cover f. pump (Unknown colour)
3	AO-234	Page 215	Connector pump, complete
4	AO-236	Page 217	Tubing guide pump - bridgearm
5	AO-693		Not a spare part (XO-908 Iron Grey (RAL 7011-5))
6	MG-859		Hang-up bracket f. pump box
7	MG-876		Not a spare part
8	UG-115		Screw, M4x010
9	UG-298		Not a spare part

AO-161

Pump with motor

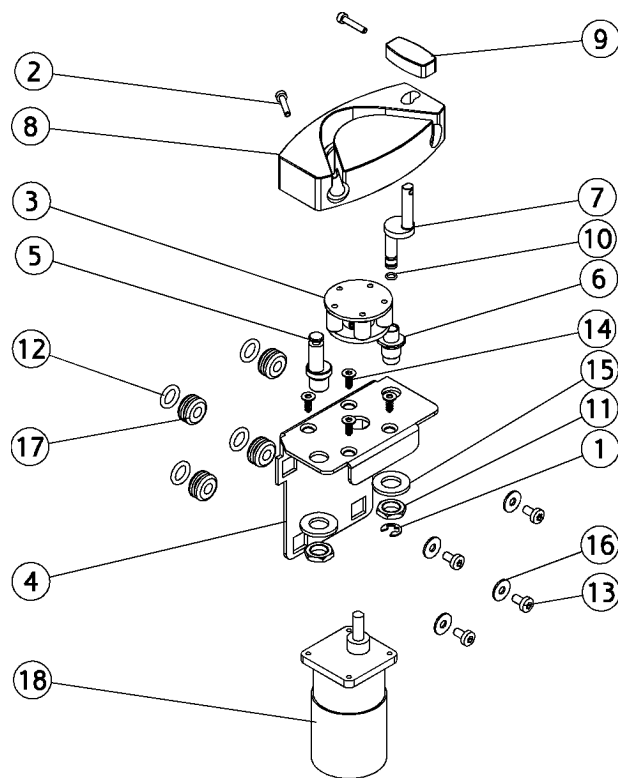


AO-161

Pump with motor

Position	Part NO.	Assembly	Description
1	001005		Not a spare part
2	003716		Screw M3 x 16
3	AO-162	<i>Page 211</i>	Pump wheel
4	MG-860		Not a spare part
5	MG-861		Axle, XO pump
6	MG-862		Not a spare part
7	MG-863		Not a spare part
8	MG-866		Not a spare part
9	MG-867		Not a spare part
10	MR-226		O-ring Ø4 x 1
11	UC-171		Nut, M10 x 1
12	UC-833		O-ring flextray holder
13	UG-067		Screw M4x8 torx
14	UG-160		Not a spare part
15	UG-259		Washer Ø10.5xØ20.0x2.0
16	UG-279		Washer, din9021 Ø4,3xØ12x1
17	UG-424		Not a spare part
18	UG-602		Not a spare part


AO-163

Cover f. pump

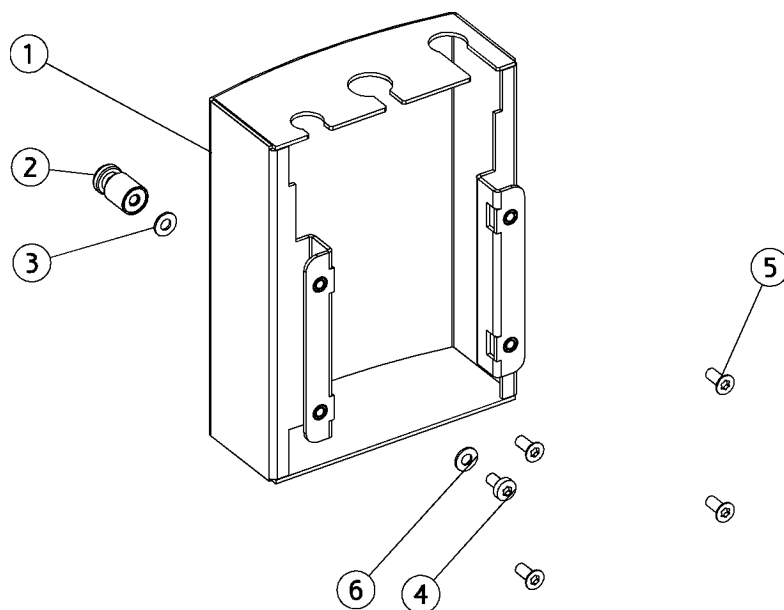

Position	Part NO.	Assembly	Description
1	MG-858		Not a spare part
2	MG-868		Not a spare part
3	MR-045		Not a spare part
4	UG-067		Screw M4x8 torx
5	UG-115		Screw, M4x010
6	UG-254		Disc, Ø04.3xØ09.0x0.8


AO-162 Pump wheel

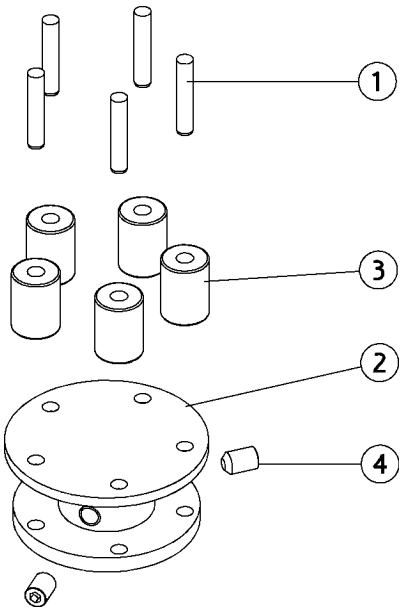

Position	Part NO.	Assembly	Description
1	001316		Not a spare part
2	MG-864		Not a spare part
3	MG-865		Not a spare part
4	UG-116		Pointed screw, M4x6


AO-164

Panel, rear, left for pump

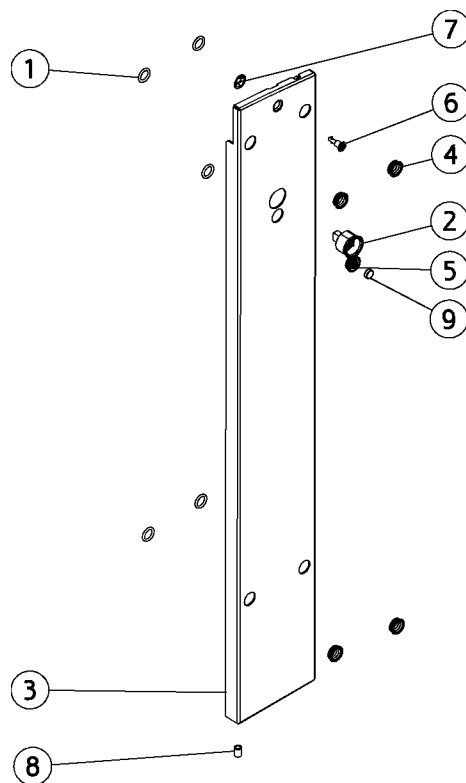

Position	Part NO.	Assembly	Description
1	000544		O-ring 11.60 x 2.40
2	AO-235	<i>Page 216</i>	Connector f. panel,pump
3	AO-186		Rear panel, left for pump, (XO-906 XO Titanium)
3	AO-191		Rear panel, left for pump, (XO-903 Silver Metallic)
3	AO-193		Rear panel, left for pump, (XO-900 XO Grey, RAL 7047-80)
3	AP-615		Rear panel, left for pump, (CTBS)
3	AO-190		Rear panel, left for pump, (XO-905 XO White,(RAL7047-30))
3	AO-192		Rear panel, left for pump, (XO-901 Sea Blue Metallic)
4	MG-870		Bushing, guide, XO pump
5	MG-886		Not a spare part
6	UG-412		Lock pin, panel
7	UG-413		Lock washer, panel
8	UG-912		Hose pvc transp. Ø5xØ9
9	UG-937		Not a spare part


AP-492 Panel, rear right, f.pump,

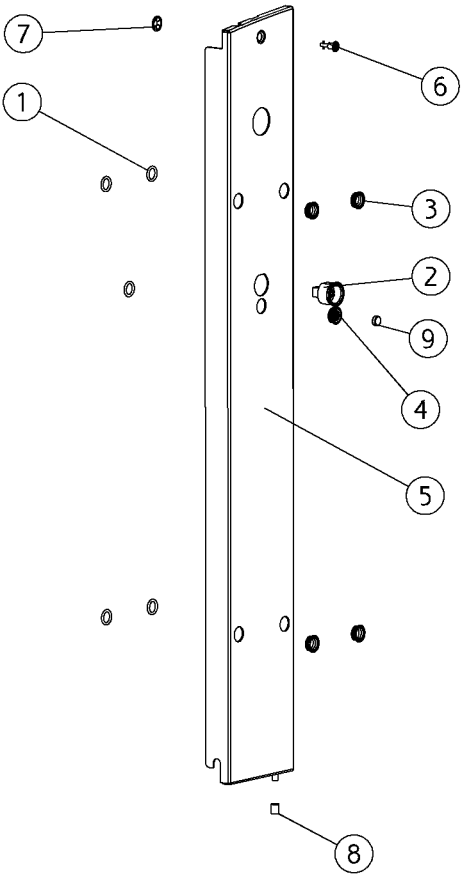

Position	Part NO.	Assembly	Description
1	000544		O-ring 11.60 x 2.40
2	AO-235	Page 216	Connector f. panel,pump
3	MG-870		Bushing, guide, XO pump
4	MG-886		Not a spare part
5	AP-459		Rear panel, right for pump, (CTBS)
6	UG-412		Lock pin, panel
7	UG-413		Lock washer, panel
8	UG-912		Hose pvc transp. Ø5xØ9
9	UG-937		Not a spare part


AO-196

Shelf for bottle

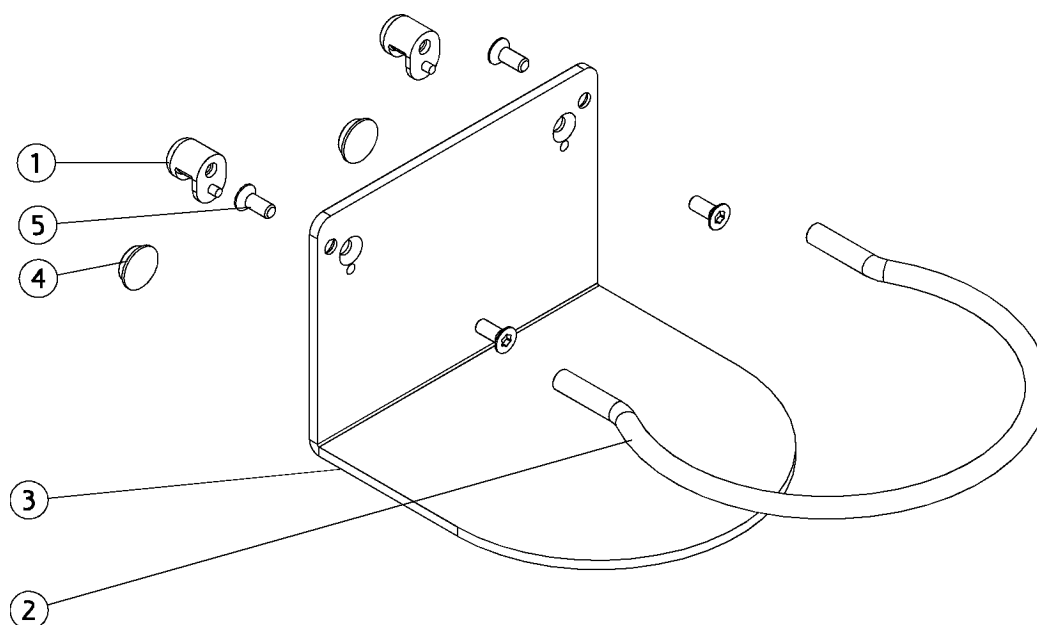

Position	Part NO.	Assembly	Description
1	MG-859		Hang-up bracket f. pump box
2	MG-871		Not a spare part
3	MG-872		Not a spare part
4	UA-093		Rubber foot, footcontrol ii
5	UG-115		Screw, M4x010


AO-234 Connector pump, complete

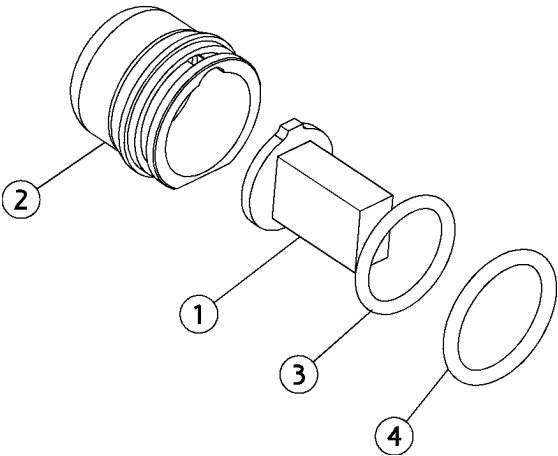

Position	Part NO.	Assembly	Description
1	AD-628		PCB, connector pump
2	MG-884		Not a spare part
3	MR-225		O-ring Ø 1,78 x 14
4	MR-228		O-ring Ø16x2


AO-235

Connector f. panel,pump

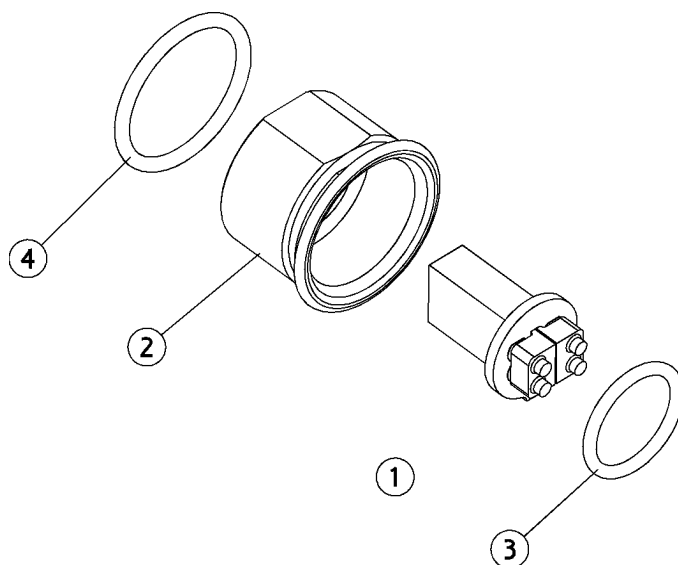

Position	Part NO.	Assembly	Description
1	AD-627		Not a spare part
2	MG-885		Not a spare part
3	MR-225		O-ring Ø 1,78 x 14
4	UG-320		O-ring, 20x2

AO-236 Tubing guide pump - bridgearm


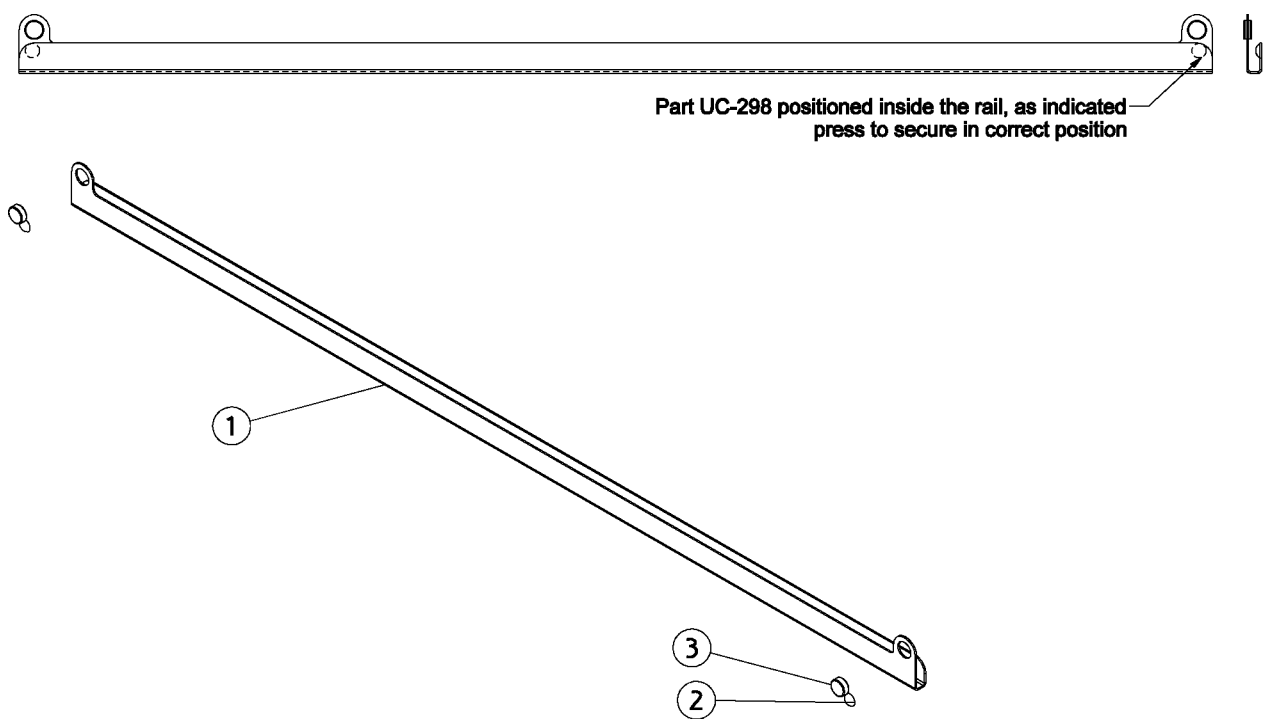


Position	Part NO.	Assembly	Description
1	MG-879		Not a spare part
2	UC-298		Product foot, round Ø8 x 1.6
3	UG-937		Not a spare part

AO-236

Tubing guide pump - bridgearm

XO 4 Foot Control

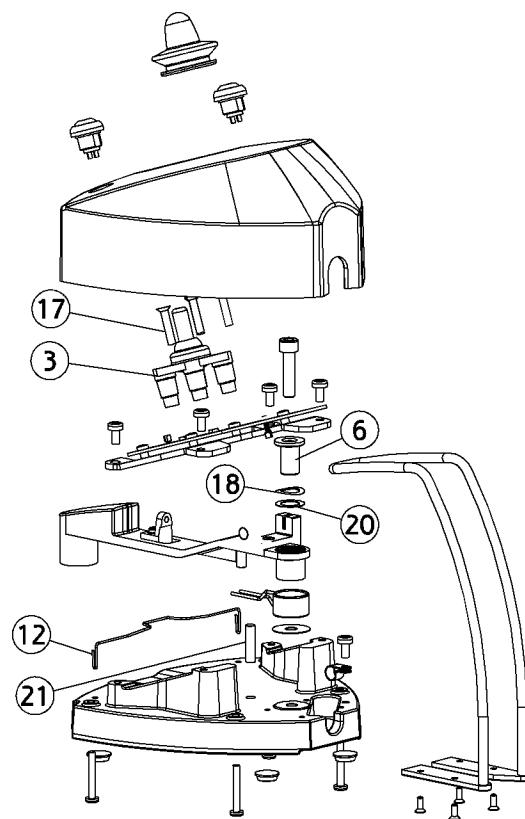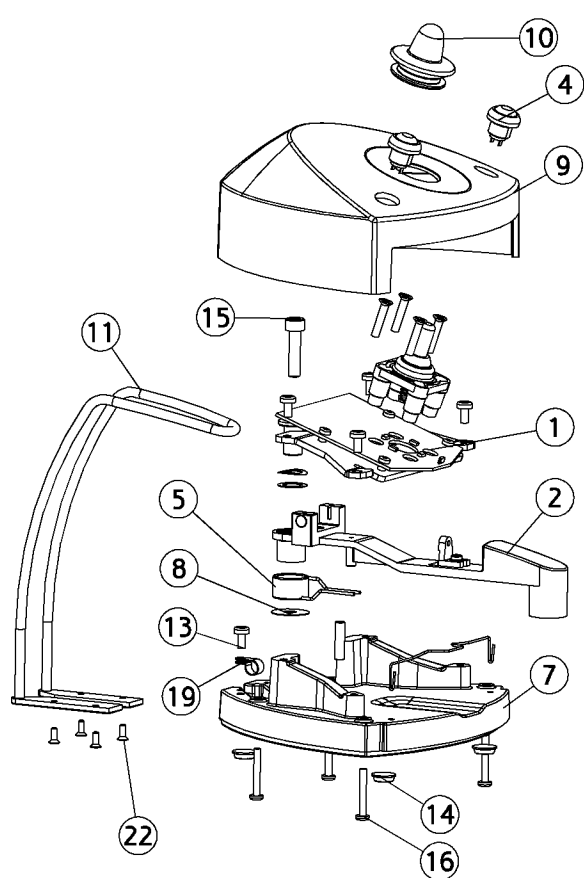


AN-200

Foot control with hoop

Position	Part NO.	Assembly	Description
1	AN-373	<i>Page 222</i>	PCB, Foot control, XO 4
2	AN-328	<i>Page 223</i>	Foot control arm, w/magnets
3	AN-645	<i>Page 224</i>	Joystick, foot control, short
4	CA-038		Switch +cable, footcontrol x04
5	ME-161		Spring,crank, footcontrol ii
6	ME-162		Bush, crank, footcontrol ii
7	ME-164		Not a spare part
8	ME-252		Not a spare part
9	MG-484		Cover,footcontrol,
10	MG-743		Rubber cap, joystick
11	MG-794		Hoop, foot control, short,XO 4
12	MG-806		Spring, pedal,foot control
13	UA-082		Not a spare part
14	UA-093		Rubber foot, footcontrol ii
15	UA-588		Screw, M6 x 25
16	UA-859		Screw, eco-syn, 3,5 x 25.
17	UA-860		Screw, M4 x 20
18	UB-057		Corrugated spring15.8x10.5x0.3
19	UC-505		Cable clips rs1. 7/9 w1
20	UC-786		Adapter disc
21	UC-846		Not a spare part
22	UG-236		Screw M3x8, uh, thread forming

AD-610


PCB, Foot control XO 4

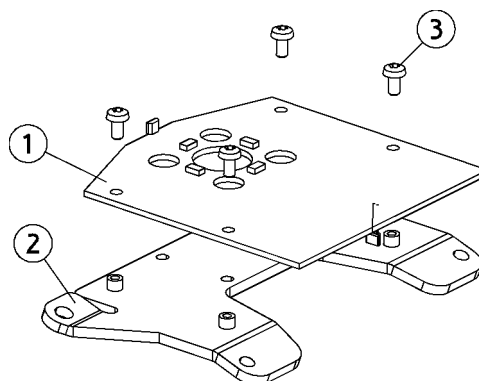

Position	Part NO.	Assembly	Description
1	AJ-610		Not a spare part
2	MG-619		Not a spare part
3	UG-102		Screw, thread making, M3x006 DIN7500

AN-328


Foot control arm, w/magnets

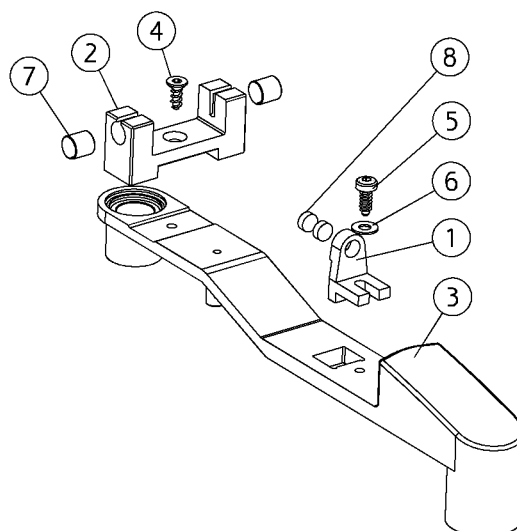

Position	Part NO.	Assembly	Description
1	MG-620		Not a spare part
2	MG-621		Not a spare part
3	MG-638		Not a spare part
4	UG-226		Bottom swaged screw 3x8
5	UG-239		Screw, for plastic M3x8
6	UG-268		Not a spare part
7	UG-414		Magnet Ø6,35x6,35 neodymium
8	UH-228		Not a spare part

AN-645


Joystick, foot control, short

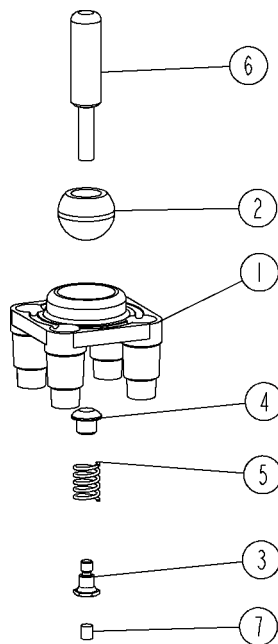

Position	Part NO.	Assembly	Description
1	ME-165		Not a spare part
2	ME-166		Not a spare part
3	ME-168		Not a spare part
4	ME-169		Not a spare part
5	ME-170		Not a spare part
6	MG-748		Not a spare part
7	UC-848		Magnet Ø3X4

AE-974



Joystick, footcontrol (long)

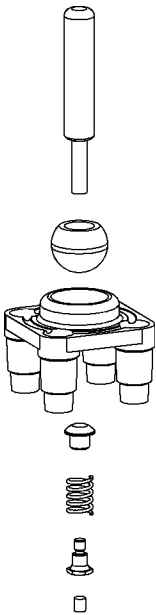

Position	Part NO.	Assembly	Description
1	ME-165		Not a spare part
2	ME-166		Not a spare part
3	ME-167		Not a spare part
4	ME-168		Not a spare part
5	ME-169		Not a spare part
6	ME-170		Not a spare part
7	UC-848		Magnet Ø3X4

AO-058

Foot control, w/o hoop

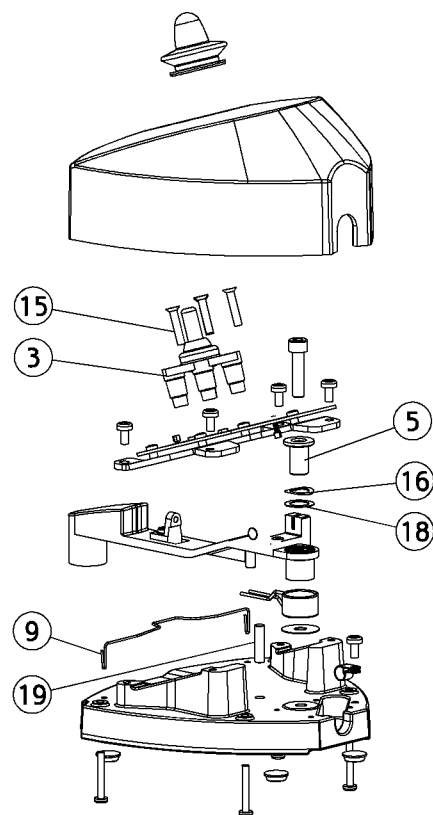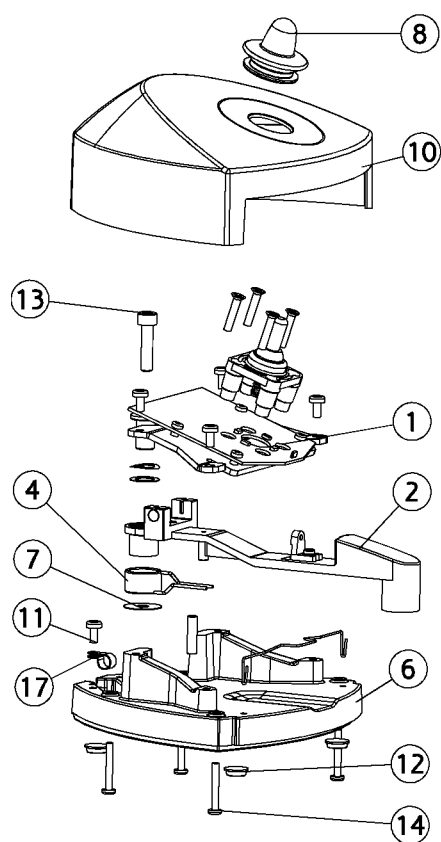

AO-058

Foot control, w/o hoop

Position	Part NO.	Assembly	Description
1	AN-373	<i>Page 222</i>	PCB, Foot control, XO 4
2	AN-328	<i>Page 223</i>	Foot control arm, w/magnets
3	AN-645	<i>Page 224</i>	Joystick, foot control, short
4	ME-161		Spring,crank, footcontrol ii
5	ME-162		Bush, crank, footcontrol ii
6	ME-164		Not a spare part
7	ME-252		Not a spare part
8	MG-743		Rubber cap, joystick
9	MG-806		Spring, pedal,foot control
10	MG-844		Not a spare part
11	UA-082		Not a spare part
12	UA-093		Rubber foot, footcontrol ii
13	UA-588		Screw, M6 x 25
14	UA-859		Screw, eco-syn, 3,5 x 25.
15	UA-860		Screw, M4 x 20
16	UB-057		Corrugated spring15.8x10.5x0.3
17	UC-505		Cable clips rs1. 7/9 w1
18	UC-786		Adapter disc
19	UC-846		Not a spare part

AO-058

Foot control, w/o hoop

XO 4 Patient Chair and Lifting System


TX-003

Upholstery

Position	Part NO.	Assembly	Description
1	AP-947	<i>Page 267</i>	Upholstery, short back, complete set, skai
2	AP-948	<i>Page 269</i>	Upholstery, short back, complete set, scuba
3	AP-949	<i>Page 271</i>	Upholstery, short back, complete set, XO Comfort
4	AO-633	<i>Page 260</i>	Upholstery, patient chair, short back, separate, skai
5	AP-939	<i>Page 263</i>	Upholstery, patient chair, short back, separate, scuba
6	AP-946	<i>Page 266</i>	Upholstery, patient chair, short back, separate, XO Comfort

AN-001


Chair guide XO 4

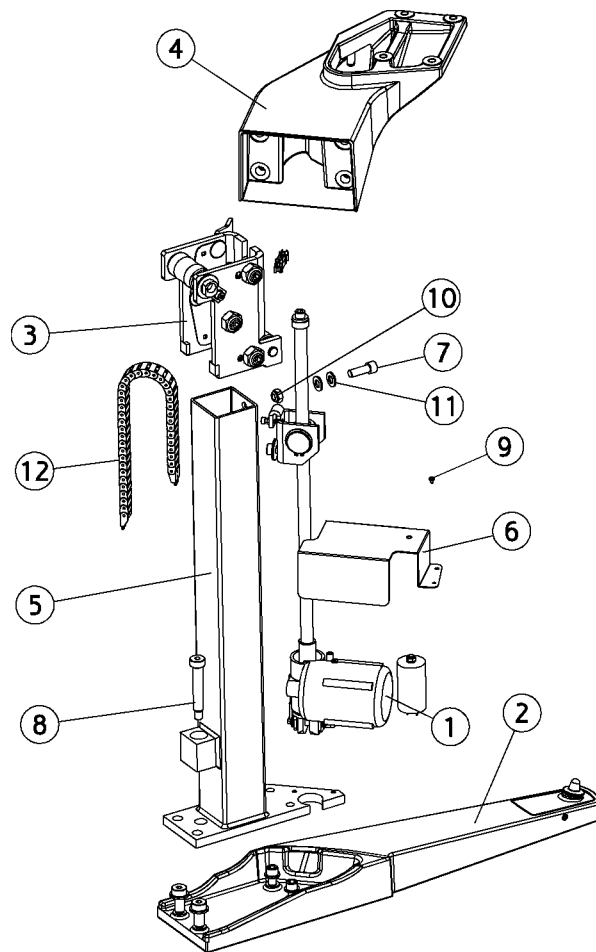

Position	Part NO.	Assembly	Description
1	AN-002	Page 232	Lifting motor, chair XO 4
2	AN-003	Page 233	Supporting leg XO 4
3	AN-004	Page 234	Carriage,guide
4	AN-035	Page 235	Carrier, arm-seat
5	MG-037		Tube, guide, stand, chair
6	MG-504		Coverplate,lifting motor,XO 4
7	UG-009		Not a spare part
8	UG-010		Not a spare part
9	UG-111		Screw, thread making, M4x6
10	UG-212		Not a spare part
11	UG-253		Disc Ø13.0xØ24.0x2.5
12	UG-905		Chain 0.6m + fitting

AN-002


Lifting motor, chair XO 4

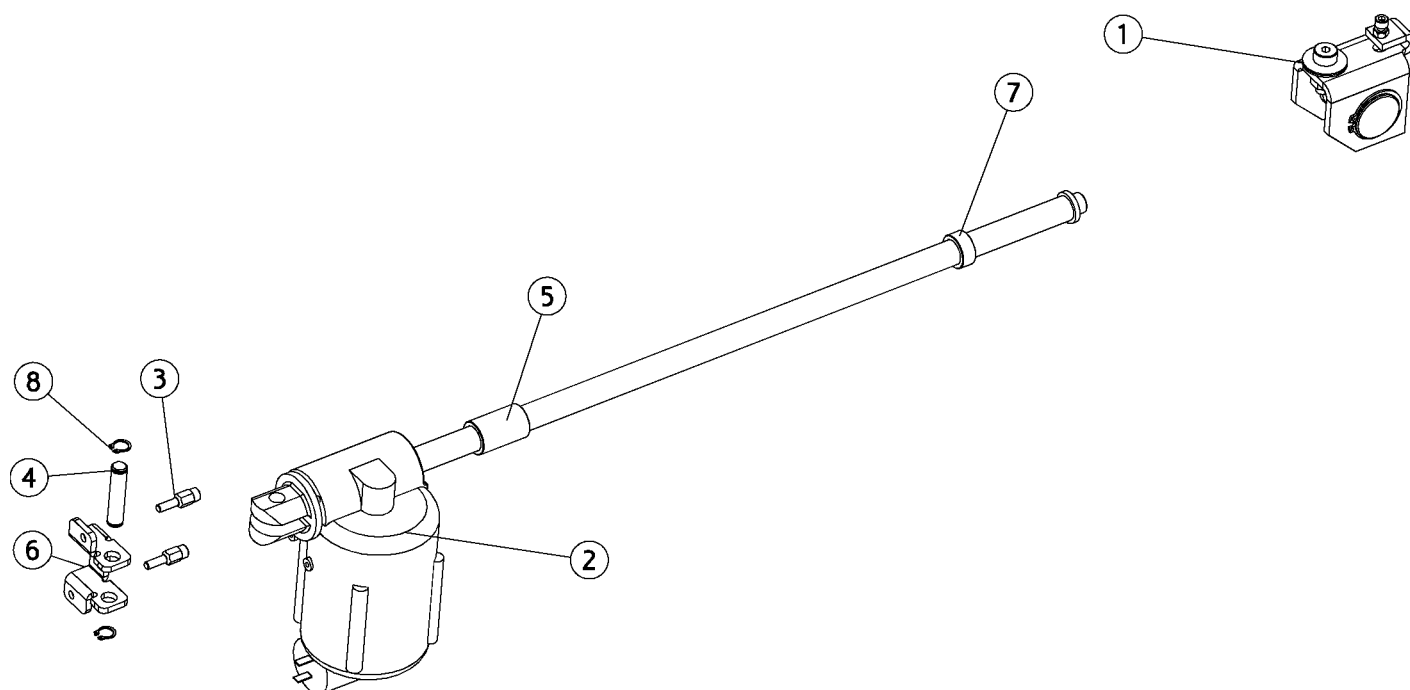

Position	Part NO.	Assembly	Description
1	AN-033	Page 237	Base, spindle nut, chair guide
2	AN-151	Page 241	Motor chair lift
3	AO-626		Bolt, motor bracket
4	MG-001		Rod-pin, lifting motor
5	MG-008		Bushing, endstop, lower XO 4
6	MG-391		Suspension for lifting motor
7	MG-745		Bush, motorguide, top
8	UG-353		Seeger-a-ring, Ø12

AN-003


Supporting leg XO 4

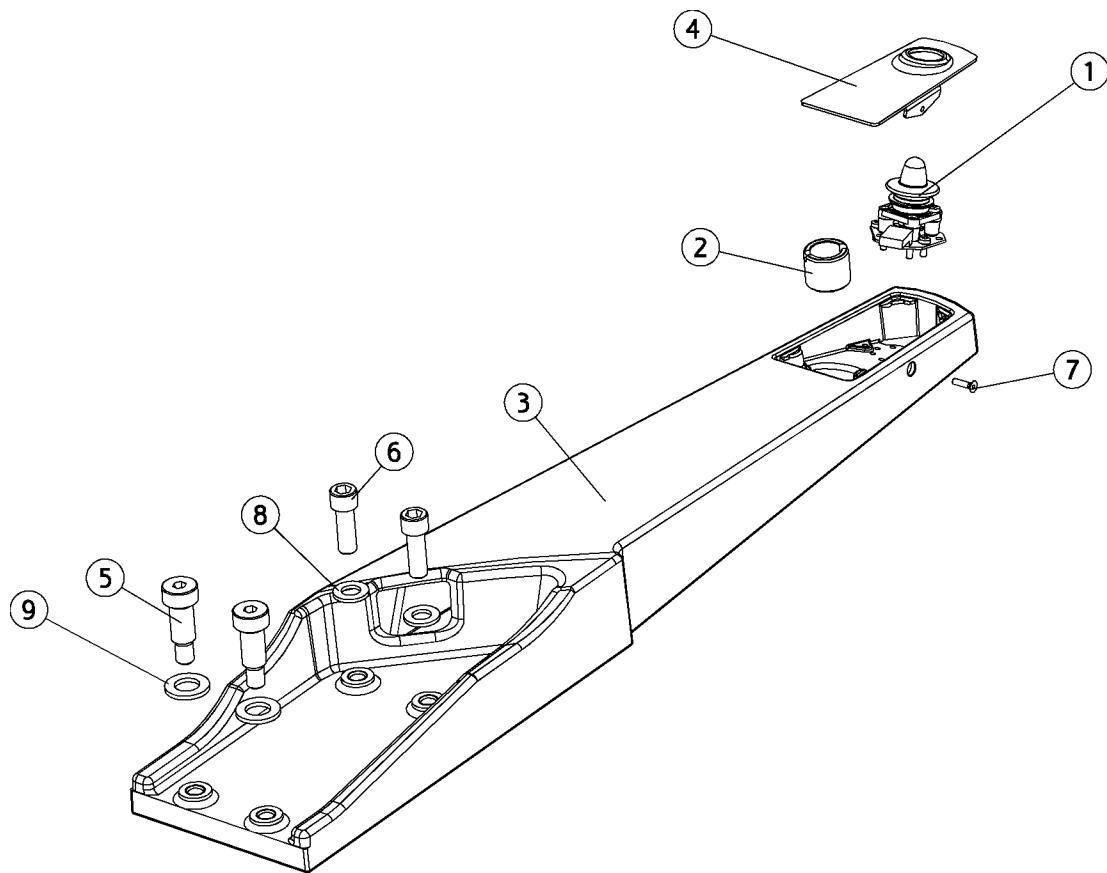

Position	Part NO.	Assembly	Description
1	AN-034	Page 238	Joystick,supporting leg,guide
2	MG-009		Bushing, adjustment of supporting leg
3	AP-678		Supporting leg, (CTBS)
3	AN-483		Supporting leg, (XO-909 Iron Grey (RAL 7011-30))
3	AO-718		Supporting leg, (XO-908 Iron Grey (RAL 7011-5))
4	AP-679		Cover supporting leg, (CTBS)
4	AO-719		Cover supporting leg, (XO-908 Iron Grey (RAL 7011-5))
4	AN-484		Cover supporting leg, (XO-909 Iron Grey (RAL 7011-30))
5	UG-008		Not a spare part
6	UG-009		Not a spare part
7	UG-034		Screw ui6kt din7991 M4x16
8	UG-253		Disc Ø13.0xØ24.0x2.5
9	UG-257		Disc Ø17.0xØ30.0x3.0

AN-004


Carriage,guide

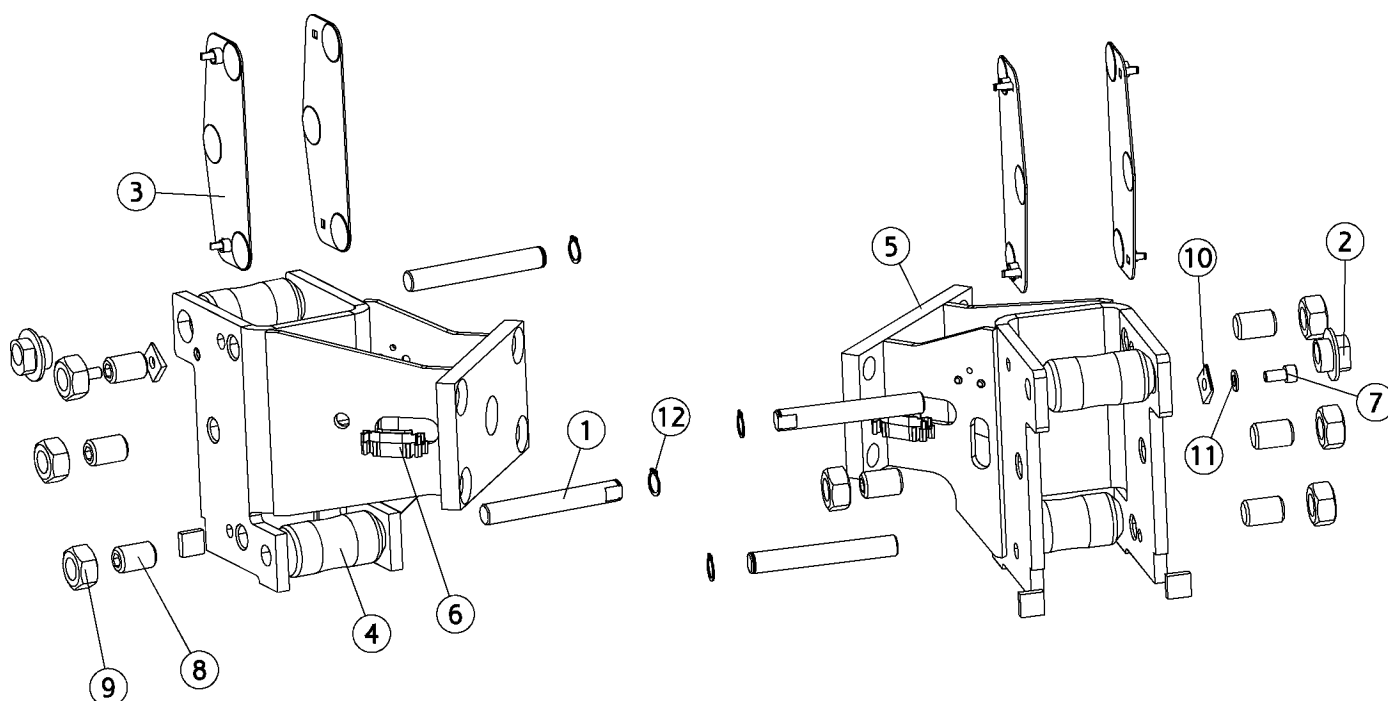

Position	Part NO.	Assembly	Description
1	MG-002		Pivot, roller, guide
2	MG-013		Not a spare part
3	MG-014		Sliding plate, guide
4	MG-038		Roll, guide
5	MG-051		Not a spare part
6	MG-394		Cable guide, chair
7	UG-003		Bolt inside hex, M08x016 din912
8	UG-101		Not a spare part
9	UG-202		Not a spare part
10	UG-251		Not a spare part
11	UG-252		Not a spare part
12	UG-351		Not a spare part

AN-035

Carrier, arm-seat

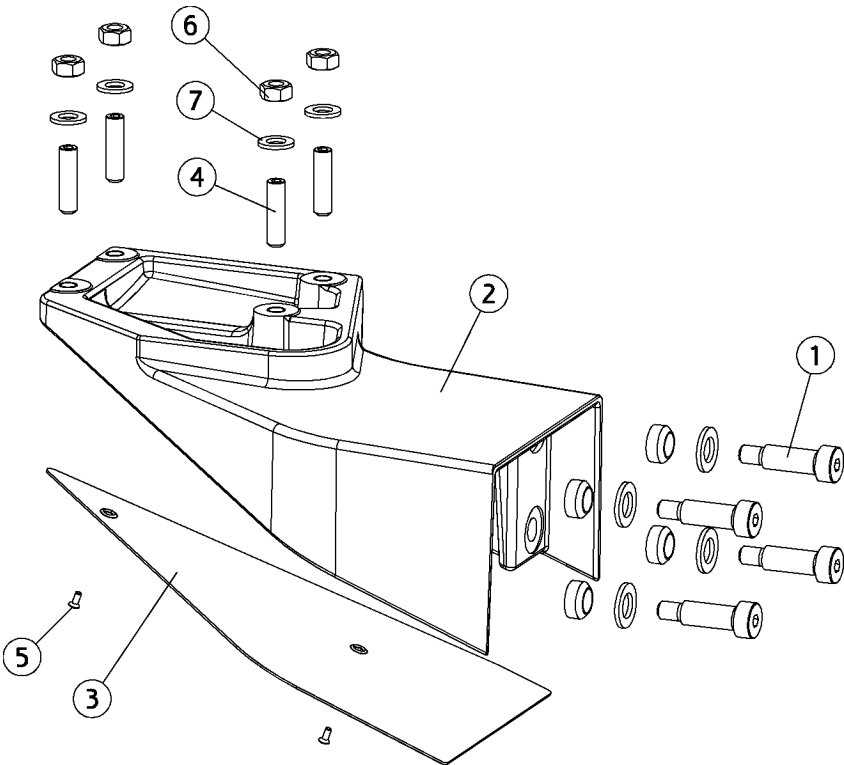


AN-035

Carrier, arm-seat

Position	Part NO.	Assembly	Description
1	AN-036	<i>Page 240</i>	Guide, standard pieces
2	AN-439		Arm, guide, chair, (XO-901 Sea Blue Metallic)
2	AN-438		Arm, guide, chair, (XO-900 XO Grey, RAL 7047-80)
2	AN-722		Arm, guide, chair, (XO-903 Silver Metallic)
2	AP-417		Arm, guide, chair, (CTBS)
2	AO-268		Arm, guide, chair, (XO-906 XO Titanium)
2	AN-933		Arm, guide, chair, (XO-905 XO White,(RAL7047-30))
3	AN-934		Cover plate, arm guide, chair, (XO-905 XO White,(RAL7047-30))
3	AN-723		Cover plate, arm guide, chair, (XO-903 Silver Metallic)
3	AN-441		Cover plate, arm guide, chair, (XO-901 Sea Blue Metallic)
3	AO-269		Cover plate, arm guide, chair, (XO-906 XO Titanium)
3	AN-440		Cover plate, arm guide, chair, (XO-900 XO Grey, RAL 7047-80)
3	AP-418		Cover plate, arm guide, chair, (CTBS)
4	UG-104		Not a spare part
5	UG-115		Screw, M4x010
6	UG-203		Nut, M12
7	UG-253		Disc Ø13.0xØ24.0x2.5

AN-033


Base, spindle nut, chair guide

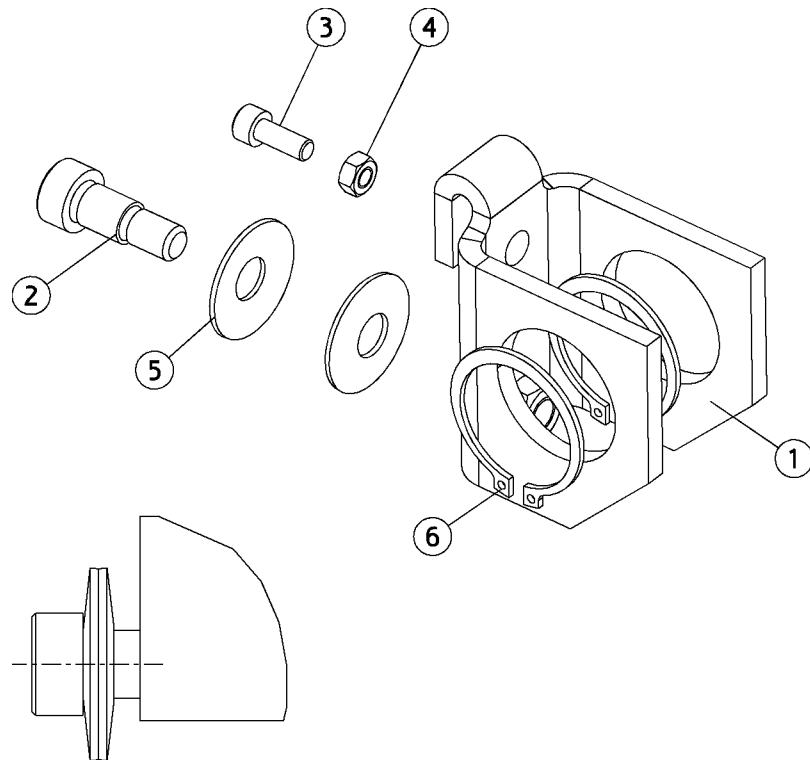

Position	Part NO.	Assembly	Description
1	MG-110		Not a spare part
2	UG-060		Adapter bolt Ø12M10x16
3	UG-063		Bolt i6kt din912 M6x16
4	UG-210		Not a spare part
5	UG-283		Disc spring, Ø34xØ12.3x1.5x2.7
6	UG-352		Not a spare part


AN-034

Joystick,supporting leg,guide

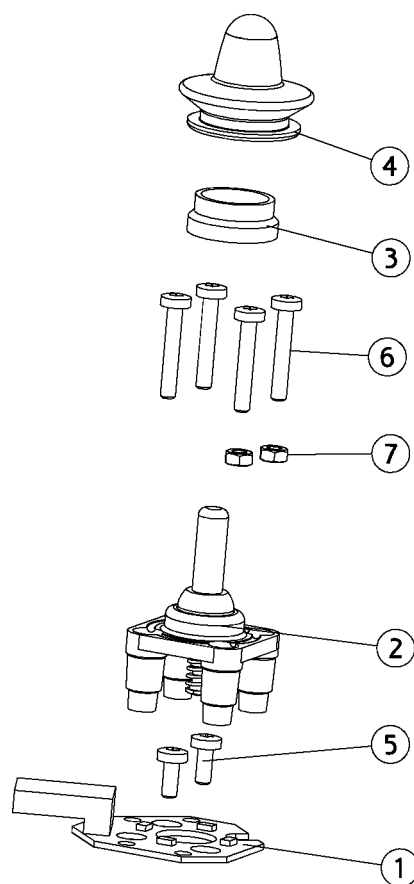

Position	Part NO.	Assembly	Description
1	AN-375		PCB, Joystick (pat. chair), XO 4
2	AN-776	Page 239	Joystick, guide
3	MG-571		Inner ring, rubber, foot control
4	MG-743		Rubber cap, joystick
5	UG-110		Screw, thread making, M4x10
6	UG-137		Not a spare part
7	UG-205		Nut, M4, din 934


AN-776 Joystick, guide

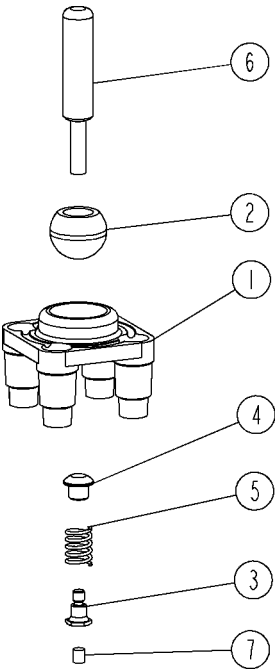

Position	Part NO.	Assembly	Description
1	ME-165		Not a spare part
2	ME-166		Not a spare part
3	ME-168		Not a spare part
4	ME-169		Not a spare part
5	ME-170		Not a spare part
6	MG-742		Not a spare part
7	UC-848		Magnet Ø3X4


AN-036

Guide, standard pieces

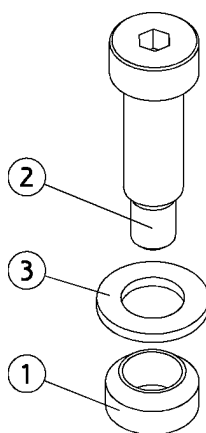

Position	Part NO.	Assembly	Description
1	MG-177		Bushing, arm
2	UG-021		Adapter bolt i6k Ø16M12x040
3	UG-257		Disc Ø17.0xØ30.0x3.0


AN-151 Motor chair lift

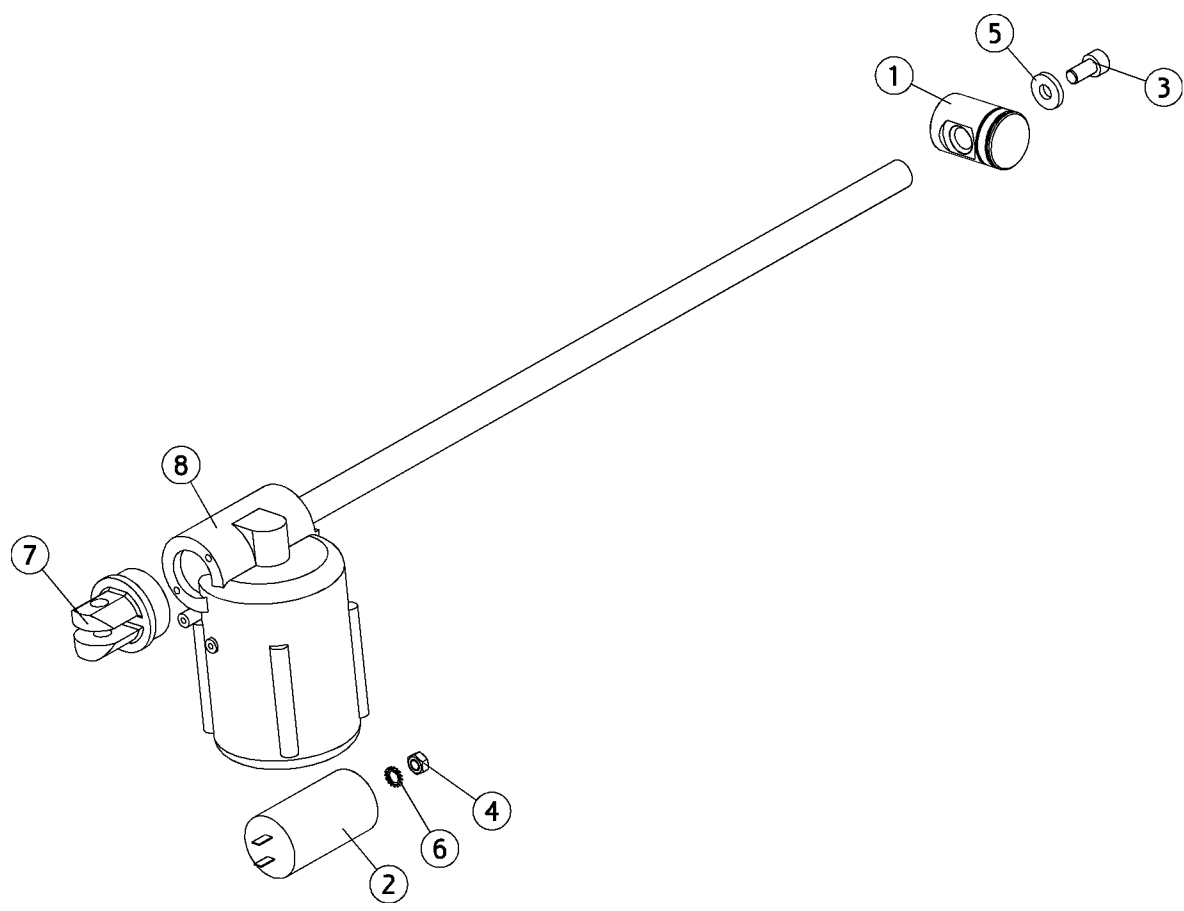

Position	Part NO.	Assembly	Description
1	MG-040		Spindle nut, motorguide
2	MH-659		Capacitor, lifting motor, 40uf
3	UG-023		Bolt, lifting motor, M10x20 left
4	UG-204		Not a spare part
5	UG-269		Not a spare part
6	UG-278		Not a spare part
7	UG-406		Not a spare part
8	UG-600		Not a spare part

AN-631

Chair base XO 4

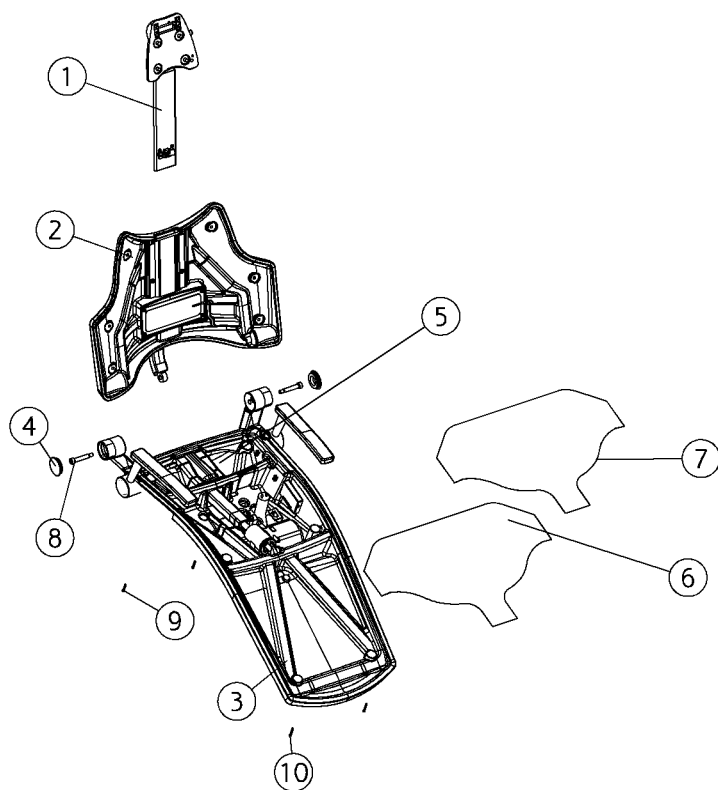


AN-631

Chair base XO 4

Position	Part NO.	Assembly	Description
1	AN-086	<i>Page 244</i>	Neck rest
2	AN-087	<i>Page 247</i>	Back base chair
3	AN-088	<i>Page 249</i>	Chair seat
4	MG-735		Not a spare part
5	MG-737		Not a spare part
6	MG-746		Plastic foot protection, chair
7	MG-980		Plastic foot protection,chair,scuba
8	UG-056		Pass bolt, d12xM10x55 iso7379
9	UG-057		Screw ux M4x30
10	UG-058		Screw torq M4x35

AN-086

Neck rest

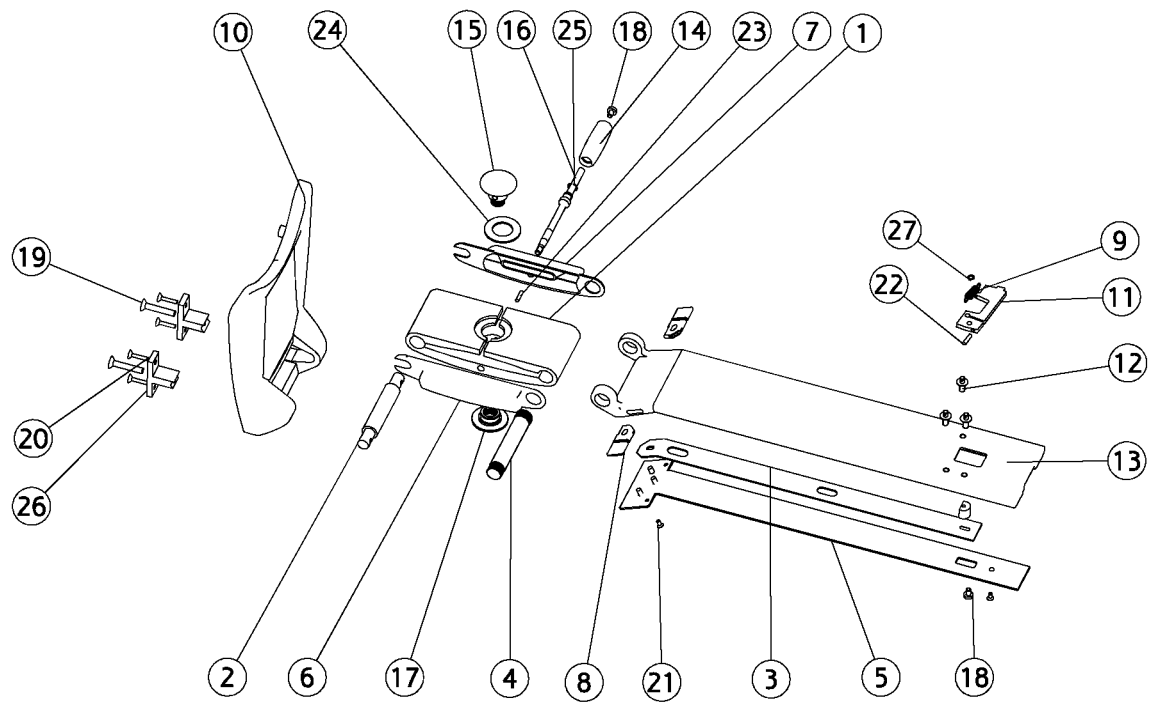


AN-086

Neck rest

Position	Part NO.	Assembly	Description
1	MG-205		Toggle bracket
2	MG-206		Not a spare part
3	MG-211		Sliding bar, neck rest
4	MG-214		TD544600
5	MG-215		Not a spare part
6	MG-217		Not a spare part
7	MG-218		Cover, handle, neck rest
8	MG-220		Not a spare part
9	MG-222		Spring for brake
10	AO-367		Base neck rest, (XO-906 XO Titanium)
10	AN-630		Base neck rest, (XO-901 Sea Blue Metallic)
10	AN-629		Base neck rest, (XO-900 XO Grey, RAL 7047-80)
10	AP-505		Base neck rest, (CTBS)
10	AO-366		Base neck rest, (XO-905 XO White,(RAL7047-30))
10	AN-770		Base neck rest, (XO-903 Silver Metallic)
11	MG-406		Brake (plate), neck rest
12	MG-407		Not a spare part
13	MG-414		Not a spare part
14	MG-799		Knob for handle XO 4 neck rest
15	MG-824		Adjustment bushing, neck rest, female
16	MG-825		Not a spare part
17	MG-826		Adjustment bushing, neck rest, male
18	UG-111		Screw, thread making, M4x6
19	UG-230		Not a spare part
20	UG-231		Not a spare part
21	UG-232		Screw M3x6 us
22	UG-233		Pointed screw M4, neck rest, break
23	UG-234		Pointed screw M4, neck rest
24	UG-288		Sliding disc Ø32
25	UG-380		Seeger, wire ring
26	UG-428		Not a spare part
27	UH-029		Seeger-a-ring, din 471 Ø4

AN-805


Kit, fric.brake, neck rest

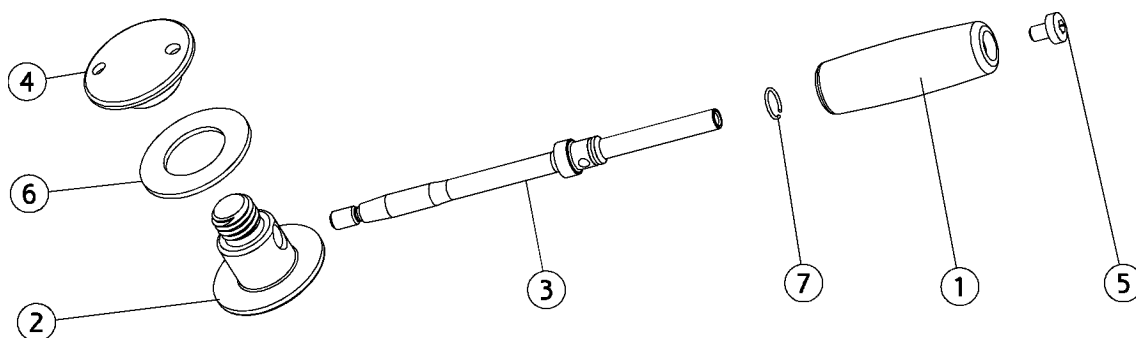

Position	Part NO.	Assembly	Description
1	MG-799		Knob for handle XO 4 neck rest
2	MG-824		Adjustment bushing, neck rest, female
3	MG-825		Not a spare part
4	MG-826		Adjustment bushing, neck rest, male
5	UG-111		Screw, thread making, M4x6
6	UG-288		Sliding disc Ø32
7	UG-380		Seeger, wire ring

AN-087

Back base chair

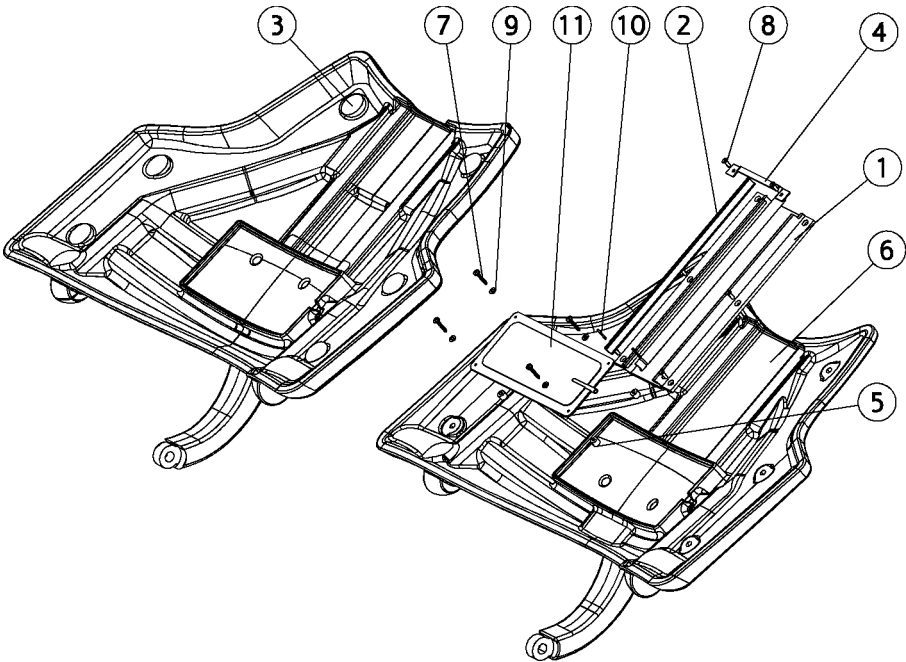


AN-087

Back base chair

Position	Part NO.	Assembly	Description
1	MG-213		Alu guide, neck rest
2	MG-219		U-guide, teflon, neck rest
3	AN-827		Base back rest - long, patient chair, (XO-903 Silver Metallic)
3	AN-693		Base back rest - long, patient chair, (XO-900 XO Grey, RAL 7047-80)
3	AN-628		Base back rest - long, patient chair, (XO-901 Sea Blue Metallic)
4	MG-611		Not a spare part
5	MG-740		Not a spare part
6	AP-504		Base, chair back, short, (CTBS)
6	AO-365		Base, chair back, short, (XO-905 XO White,(RAL7047-30))
6	AO-380		Base, chair back, short, (XO-901 Sea Blue Metallic)
6	AO-379		Base, chair back, short, (XO-900 XO Grey, RAL 7047-80)
6	AO-335		Base, chair back, short, (XO-906 XO Titanium)
6	AO-403		Base, chair back, short, (XO-903 Silver Metallic)
7	UG-059		Screw, ph 3,5x25
8	UG-161		Screw M4x12 us, pa6.6
9	UG-254		Disc, Ø04.3xØ09.0x0.8
10	UG-467		Not a spare part
11	UG-814		Balloon, chair

AN-088

Chair seat

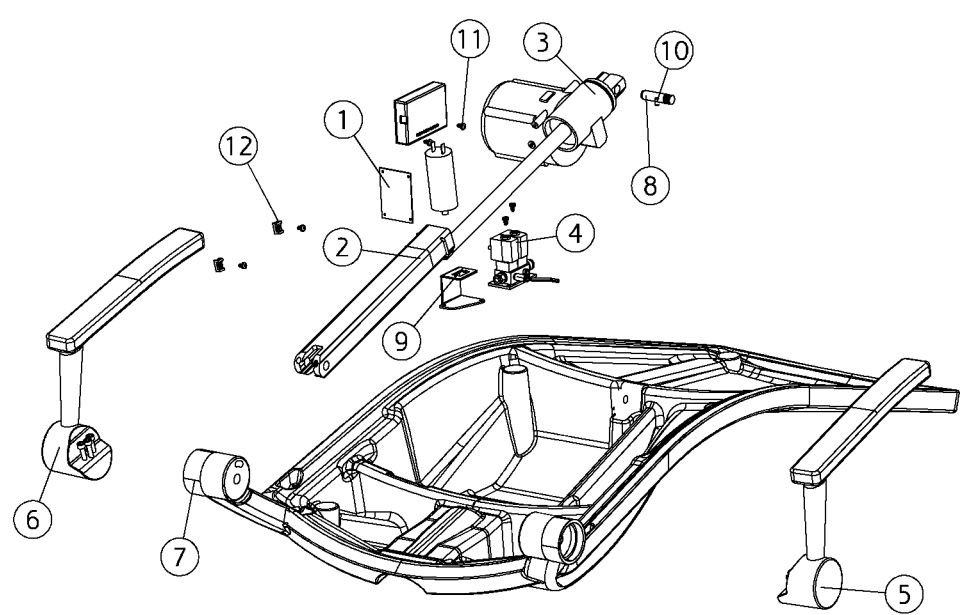


AN-088

Chair seat

Position	Part NO.	Assembly	Description
1	AN-960		PCB, chair
2	AP-502	<i>Page 251</i>	Push pull, chair, (CTBS)
2	AN-825	<i>Page 251</i>	Push pull, chair, (XO-903 Silver Metallic)
2	AN-823	<i>Page 251</i>	Push pull, chair, (XO-900 XO Grey, RAL 7047-80)
2	AO-402	<i>Page 251</i>	Push pull, chair, (XO-905 XO White,(RAL7047-30))
2	AO-401	<i>Page 251</i>	Push pull, chair, (XO-906 XO Titanium)
2	AN-824	<i>Page 251</i>	Push pull, chair, (XO-901 Sea Blue Metallic)
3	AN-156	<i>Page 252</i>	Motor chair back
4	AN-326	<i>Page 253</i>	Lumbar support
5	AO-792		Arm rest chair, right,
6	AO-793		Arm rest chair, left,
7	AN-626		Base, chair seat, patient chair, (XO-901 Sea Blue Metallic)
7	AN-726		Base, chair seat, patient chair, (XO-903 Silver Metallic)
7	AN-625		Base, chair seat, patient chair, (XO-900) (7047 7047)
7	AO-198		Base, chair seat, patient chair, (XO-905 XO White,(RAL7047-30))
7	AO-317		Base, chair seat, patient chair, (XO-906 XO Titanium)
7	AP-503		Base, chair seat, patient chair, (CTBS)
8	MG-738		Not a spare part
9	MN-471		Not a spare part
10	UG-127		Not a spare part
11	UG-227		Screw for plastic M4x8
12	UG-449		Not a spare part


AN-146 Push pull, chair

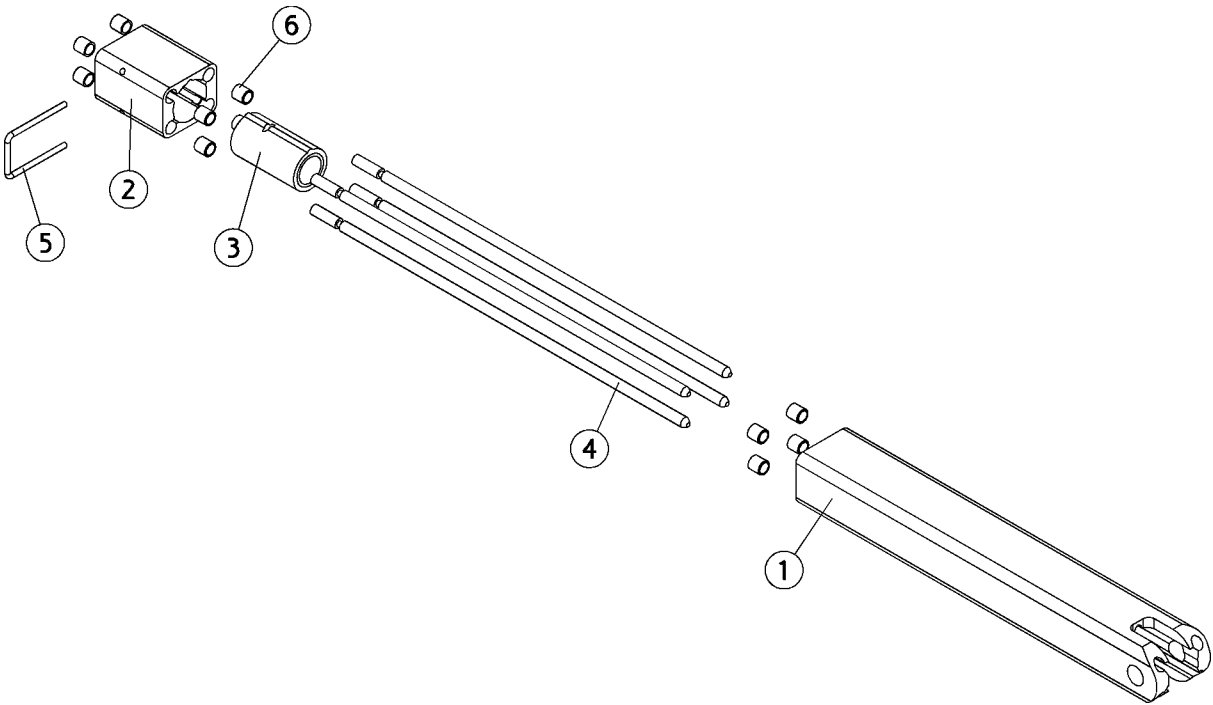

Position	Part NO.	Assembly	Description
1	MG-198		Not a spare part
2	MG-199		Not a spare part
3	MG-200		Bush
4	MG-201		Locking rod
5	MG-202		Not a spare part
6	UG-444		Not a spare part

AN-156


Motor chair back

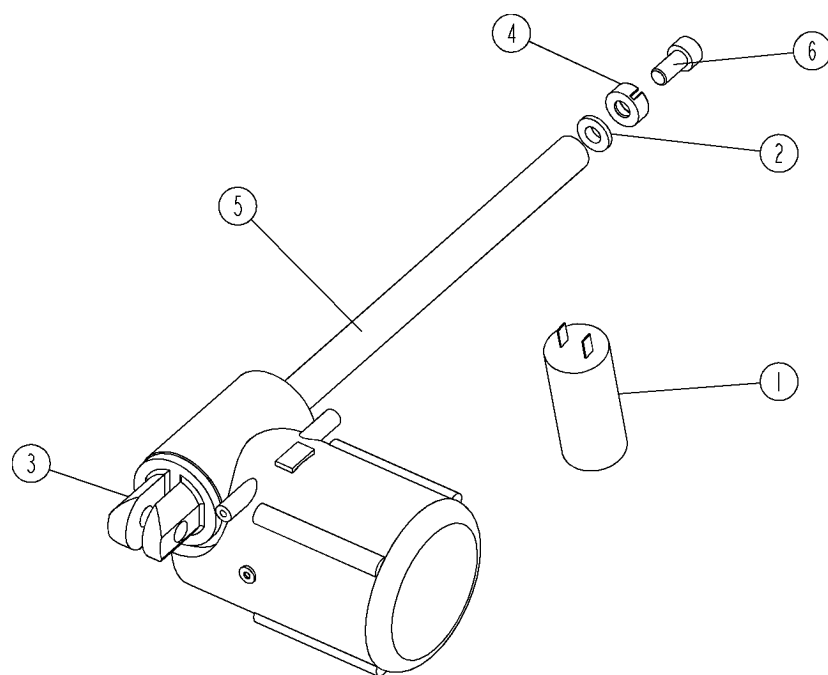

side 1 of 2

Position	Part NO.	Assembly	Description
1	MH-660		Capacitor, backrestmotor, 25u
2	UG-273		Disc, Ø20.5xØ10x3
3	UG-406		Not a spare part
4	UG-436		Bush, nylon
5	UG-601		Not a spare part
6	UH-011		Bolt, back rest, motor, M10x20,right


AN-326 Lumbar support

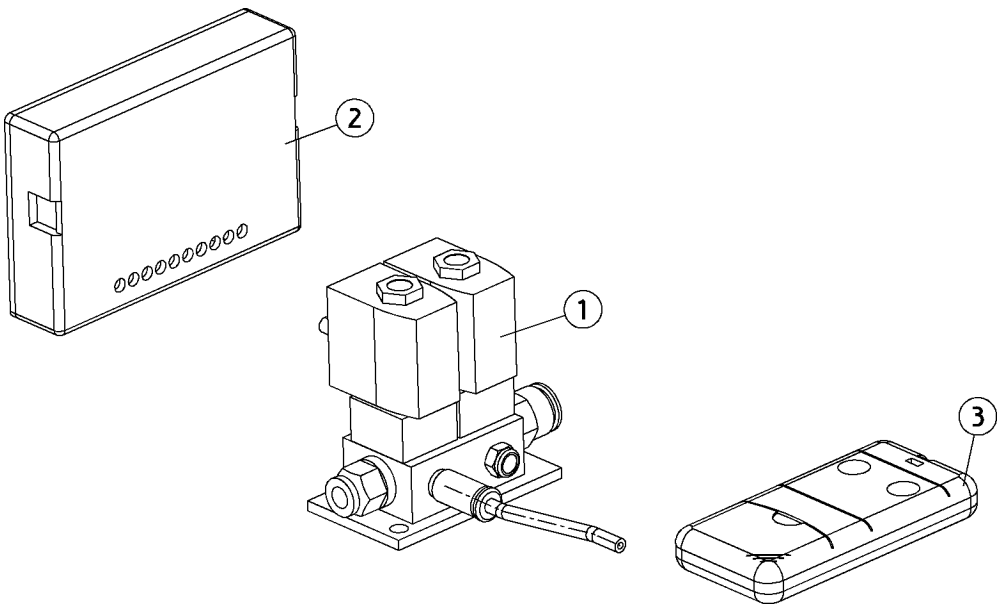

Position	Part NO.	Assembly	Description
1	AN-329	Page 254	Valve system lumbar support
2	ML-572		Receiver, lumbar support
3	ML-573		Remote control lumbar support


AN-329

Valve system lumbar support

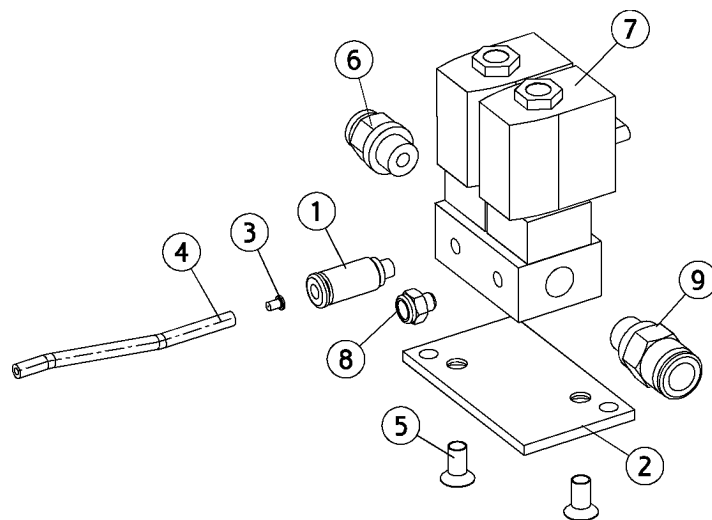

Position	Part NO.	Assembly	Description
1	AS-009		Fitting 4/2 M5
2	MG-622		Not a spare part
3	MG-623		Reducer, lumbar support XO 4
4	UC-830		Tubing 4/2 blue polyurethan
5	UG-154		Not a spare part
6	UG-700		Quick connect, 6/4-r1/8
7	UG-807		Lumbar support valves
8	UG-810		Not a spare part
9	UG-816		Exhaust valve, lumbar


AO-792 Arm rest chair, right,

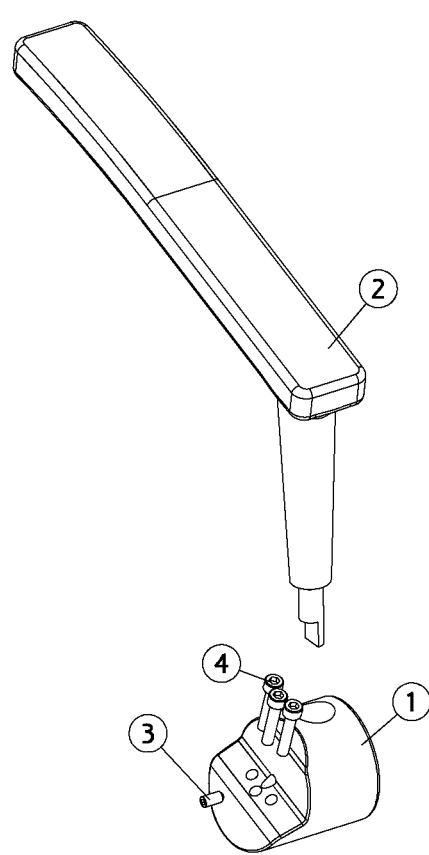

Position	Part NO.	Assembly	Description
1	AO-970		Konsol armrest right (XO-908)
2	MG-705		Right armrest, (adjustable)
3	UA-127		Not a spare part
4	UG-006		Not a spare part

AO-793


Arm rest chair, left,

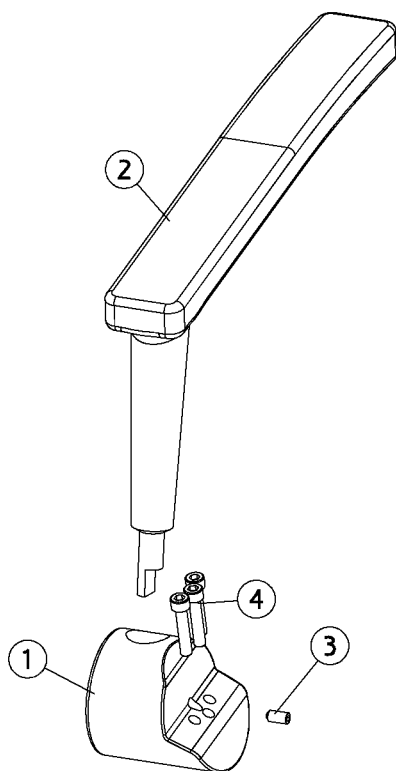

Position	Part NO.	Assembly	Description
1	AO-971		Konsol armrest left (XO-908)
2	MG-707		Left armrest
3	UA-127		Not a spare part
4	UG-006		Not a spare part

AN-632

Upholstery, patient chair, long back, separate

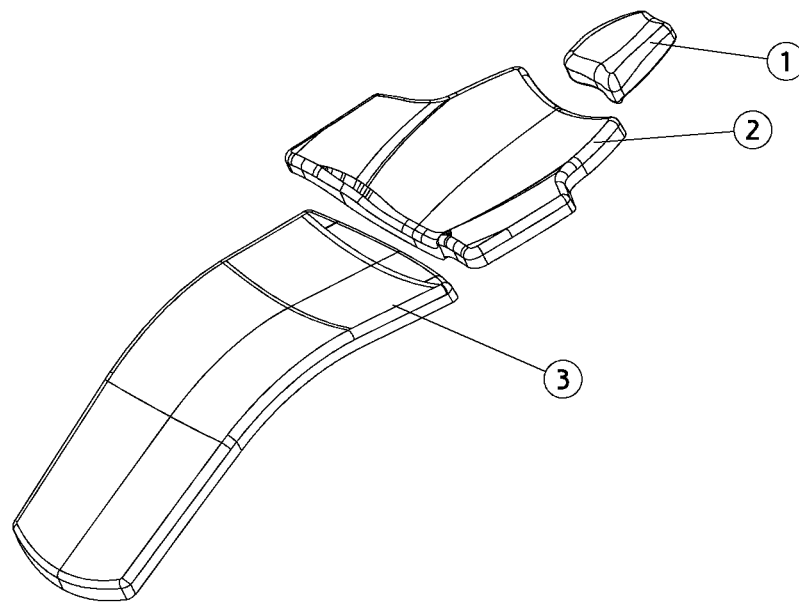

AN-632

Upholstery, patient chair, long back, separate


Position	Part NO.	Assembly	Description
1	AN-623		Upholstery, patient chair, neck rest, (BA-243 Safran Yellow)
1	AO-021		Upholstery, patient chair, neck rest, (BA-374 Orange)
1	AO-358		Upholstery, patient chair, neck rest, (XO-960 Chalk White)
1	AO-364		Upholstery, patient chair, neck rest, (XO-958 Golden Honey)
1	AN-618		Upholstery, patient chair, neck rest, (WB-230 Grey)
1	AO-374		Upholstery, patient chair, neck rest, (XO-959 Silk Grey)
1	AO-371		Upholstery, patient chair, neck rest, (XO-957 Sap Green)
1	AN-619		Upholstery, patient chair, neck rest, (BA-407 Crystal White)
1	AN-621		Upholstery, patient chair, neck rest, (BA-209 Sea Blue)
1	AO-022		Upholstery, patient chair, neck rest, (XO-954 Beige)
1	AO-023		Upholstery, patient chair, neck rest, (XO-955 Bordeaux)
1	AN-622		Upholstery, patient chair, neck rest, (XO-950 Light Blue)
1	AN-624		Upholstery, patient chair, neck rest, (XO-951 Red)
1	AO-355		Upholstery, patient chair, neck rest, (XO-961 Scarlet Red)
1	AO-024		Upholstery, patient chair, neck rest, (XO-956 Light Green)
1	AN-617		Upholstery, patient chair, neck rest, (BA-211 Dark Grey)
2	AN-611		Upholstery, patient chair back rest, long, (BA-407 Crystal White)
2	AN-612		Upholstery, patient chair back rest, long, (WB-234 Black)
2	AN-609		Upholstery, patient chair back rest, long, (BA-211 Dark Grey)
2	AO-017		Upholstery, patient chair back rest, long, (BA-374 Orange)
2	AN-613		Upholstery, patient chair back rest, long, (BA-209 Sea Blue)
2	AN-615		Upholstery, patient chair back rest, long, (BA-243 Safran Yellow)
2	AN-610		Upholstery, patient chair back rest, long, (WB-230 Grey)
3	AO-013		Upholstery, patient chair, seat, (BA-374 Orange)
3	AN-606		Upholstery, patient chair, seat, (XO-950 Light Blue)
3	AN-602		Upholstery, patient chair, seat, (WB-230 Grey)
3	AO-373		Upholstery, patient chair, seat, (XO-959 Silk Grey)
3	AO-360		Upholstery, patient chair, seat, (XO-960 Chalk White)
3	AO-370		Upholstery, patient chair, seat, (XO-957 Sap Green)
3	AO-015		Upholstery, patient chair, seat, (XO-955 Bordeaux)
3	AN-603		Upholstery, patient chair, seat, (BA-407 Crystal White)
3	AO-363		Upholstery, patient chair, seat, (XO-958 Golden Honey)

AN-632**Upholstery, patient chair, long back, separate**

Position	Part NO.	Assembly	Description
3	AO-016		Upholstery, patient chair, seat, (XO-956 Light Green)
3	AN-607		Upholstery, patient chair, seat, (BA-243 Safran Yellow)
3	AN-608		Upholstery, patient chair, seat, (XO-951 Red)
3	AO-491		Upholstery, patient chair, seat, (XO-961 Scarlet Red)
3	AN-605		Upholstery, patient chair, seat, (BA-209 Sea Blue)
3	AN-601		Upholstery, patient chair, seat, (BA-211 Dark Grey)
3	AO-014		Upholstery ,patient chair, seat, (XO-954 Beige)
3	AN-604		Upholstery, patient chair, seat, (WB-234 Black)

AO-633

Upholstery, patient chair, short back, separate, skai

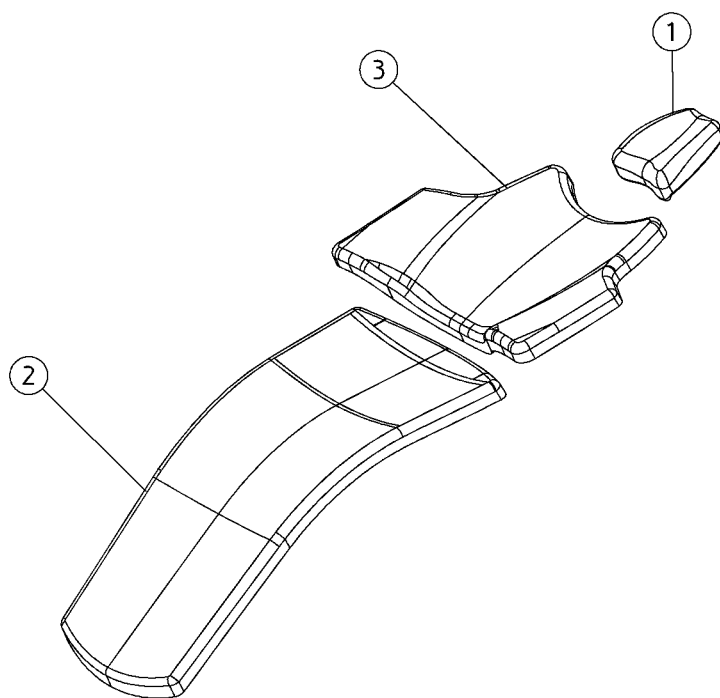

AO-633

Upholstery, patient chair, short back, separate, skai

Position	Part NO.	Assembly	Description
1	AN-623		Upholstery, patient chair, neck rest, (BA-243 Safran Yellow)
1	AO-021		Upholstery, patient chair, neck rest, (BA-374 Orange)
1	AO-358		Upholstery, patient chair, neck rest, (XO-960 Chalk White)
1	AO-364		Upholstery, patient chair, neck rest, (XO-958 Golden Honey)
1	AN-618		Upholstery, patient chair, neck rest, (WB-230 Grey)
1	AO-374		Upholstery, patient chair, neck rest, (XO-959 Silk Grey)
1	AO-371		Upholstery, patient chair, neck rest, (XO-957 Sap Green)
1	AN-619		Upholstery, patient chair, neck rest, (BA-407 Crystal White)
1	AN-621		Upholstery, patient chair, neck rest, (BA-209 Sea Blue)
1	AO-022		Upholstery, patient chair, neck rest, (XO-954 Beige)
1	AO-023		Upholstery, patient chair, neck rest, (XO-955 Bordeaux)
1	AN-622		Upholstery, patient chair, neck rest, (XO-950 Light Blue)
1	AN-624		Upholstery, patient chair, neck rest, (XO-951 Red)
1	AO-355		Upholstery, patient chair, neck rest, (XO-961 Scarlet Red)
1	AO-024		Upholstery, patient chair, neck rest, (XO-956 Light Green)
1	AN-617		Upholstery, patient chair, neck rest, (BA-211 Dark Grey)
2	AO-013		Upholstery, patient chair, seat, (BA-374 Orange)
2	AN-606		Upholstery, patient chair, seat, (XO-950 Light Blue)
2	AN-602		Upholstery, patient chair, seat, (WB-230 Grey)
2	AO-373		Upholstery, patient chair, seat, (XO-959 Silk Grey)
2	AO-360		Upholstery, patient chair, seat, (XO-960 Chalk White)
2	AO-370		Upholstery, patient chair, seat, (XO-957 Sap Green)
2	AO-015		Upholstery, patient chair, seat, (XO-955 Bordeaux)
2	AN-603		Upholstery, patient chair, seat, (BA-407 Crystal White)
2	AO-363		Upholstery, patient chair, seat, (XO-958 Golden Honey)
2	AO-016		Upholstery, patient chair, seat, (XO-956 Light Green)
2	AN-607		Upholstery, patient chair, seat, (BA-243 Safran Yellow)
2	AN-608		Upholstery, patient chair, seat, (XO-951 Red)
2	AO-491		Upholstery, patient chair, seat, (XO-961 Scarlet Red)
2	AN-605		Upholstery, patient chair, seat, (BA-209 Sea Blue)
2	AN-601		Upholstery, patient chair, seat, (BA-211 Dark Grey)
2	AO-014		Upholstery ,patient chair, seat, (XO-954 Beige)


AO-633

Upholstery, patient chair, short back, separate, skai

Position	Part NO.	Assembly	Description
2	AN-604		Upholstery, patient chair, seat, (WB-234 Black)
3	AO-378		Upholstery, patient chair, back rest, short, (XO-956 Light Green)
3	AO-369		Upholstery, patient chair, back rest, short, (XO-957 Sap Green)
3	AO-362		Upholstery, patient chair, back rest, short, (XO-950 Light Blue)
3	AO-018		Upholstery, patient chair, back rest, short, (XO-954 Beige)
3	AO-353		Upholstery, patient chair, back rest, short, (BA-209 Sea Blue)
3	AO-351		Upholstery, patient chair, back rest, short, (WB-234 Black)
3	AN-616		Upholstery, patient chair, back rest, short, (XO-951 Red)
3	AO-490		Upholstery, patient chair, back rest, short, (XO-961 Scarlet Red)
3	AO-356		Upholstery, patient chair, back rest, short, (XO-954 Beige)
3	AO-020		Upholstery, patient chair, back rest, short, (XO-956 Light Green)
3	AN-614		Upholstery, patient chair, back rest, short, (XO-950 Light Blue)
3	AO-372		Upholstery, patient chair, back rest, short, (XO-959 Silk Grey)
3	AO-352		Upholstery, patient chair, back rest, short, (WB-230 Grey)
3	AO-377		Upholstery, patient chair, back rest, short, (BA-243 Safran Yellow)
3	AO-357		Upholstery, patient chair, back rest, short, (BA-211 Dark Grey)
3	AO-375		Upholstery, patient chair, back rest, short, (XO-955 Bordeaux)
3	AO-368		Upholstery, patient chair, back rest, short, (BA-374 Orange)
3	AO-376		Upholstery, patient chair, back rest, short, (XO-951 Red)
3	AO-361		Upholstery, patient chair, back rest, short, (XO-958 Golden Honey)
3	AO-354		Upholstery, patient chair, back rest, short, (XO-961 Scarlet Red)
3	AO-359		Upholstery, patient chair, back rest, short, (XO-960 Chalk White)

AP-939

Upholstery, patient chair, short back, separate, scuba

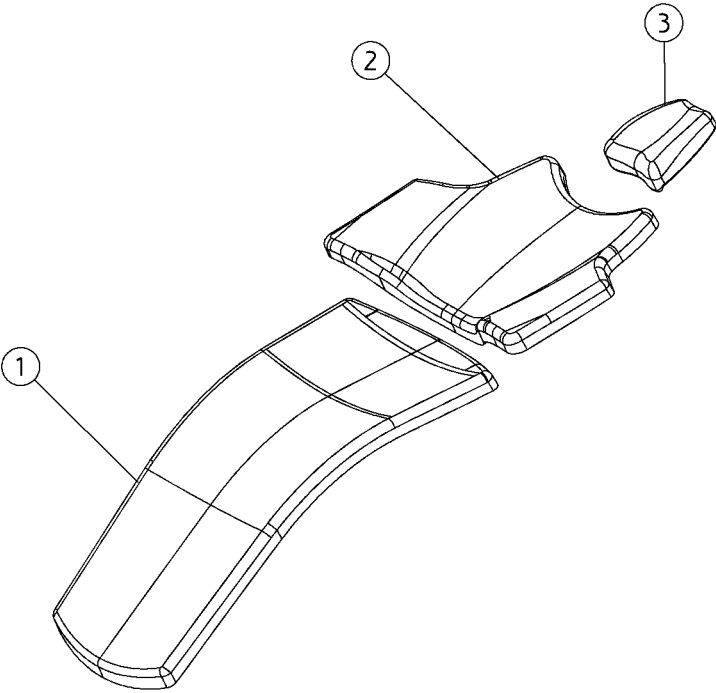

AP-939

Upholstery, patient chair, short back, separate, scuba

Position	Part NO.	Assembly	Description
1	AO-865		Upholstery, patient chair, seat, scuba, (XO-915 Sage)
1	AO-866		Upholstery, patient chair, seat, scuba, (XO-916 Tourmaline)
1	AO-860		Upholstery, patient chair, seat, scuba, (XO-910 Cocoon)
1	AP-587		Upholstery, patient chair, seat, scuba, (XO-921 Truffle (025))
1	AO-868		Upholstery, patient chair, seat, scuba, (XO-918 Cinnamon)
1	AP-588		Upholstery, patient chair, seat, scuba, (XO-923 Mahogany (006))
1	AP-586		Upholstery, patient chair, seat, scuba, (XO-922 Brick (021))
1	AO-861		Upholstery, patient chair, seat, scuba, (XO-911 Dune)
1	AO-863		Upholstery, patient chair, seat, scuba, (XO-913 Jet)
1	AO-867		Upholstery, patient chair, seat, scuba, (XO-917 Autumn)
1	AO-869		Upholstery, patient chair, seat, scuba, (XO-919 Charcoal)
1	AO-864		Upholstery, patient chair, seat, scuba, (XO-914 Dolphin)
1	AP-589		Upholstery, patient chair, seat, scuba, (XO-924 Sapphire (016))
1	AO-862		Upholstery, patient chair, seat, scuba, (XO-912 Walnut)
1	AP-585		Upholstery, patient chair, seat, scuba, (XO-920 Coral (022))
2	AO-872		Upholstery, patient chair, back rest, short, scuba, (XO-912 Walnut)
2	AO-879		Upholstery, patient chair, back rest, short, scuba, (XO-919 Charcoal)
2	AP-594		Upholstery, patient chair, back rest, short, scuba, (XO-924 Sapphire (016))
2	AO-875		Upholstery, patient chair, back rest, short, scuba, (XO-915 Sage)
2	AO-874		Upholstery, patient chair, back rest, short, scuba, (XO-914 Dolphin)
2	AO-870		Upholstery, patient chair, back rest, short, scuba, (XO-910 Cocoon)
2	AO-877		Upholstery, patient chair, back rest, short, scuba, (XO-917 Autumn)
2	AP-593		Upholstery, patient chair, back rest, short, scuba, (XO-923 Mahogany (006))
2	AP-590		Upholstery, patient chair, back rest, short, scuba, (XO-920 Coral (022))
2	AO-876		Upholstery, patient chair, back rest, short, scuba, (XO-916 Tourmaline)
2	AP-591		Upholstery, patient chair, back rest, short, scuba, (XO-922 Brick (021))
2	AO-873		Upholstery, patient chair, back rest, short, scuba, (XO-913 Jet)
2	AO-871		Upholstery, patient chair, back rest, short, scuba, (XO-911 Dune)


AP-939

Upholstery, patient chair, short back, separate, scuba

Position	Part NO.	Assembly	Description
2	AO-878		Upholstery, patient chair, back rest, short, scuba, (XO-918 Cinnamon)
2	AP-592		Upholstery, patient chair, back rest, short, scuba, (XO-921 Truffle (025))
3	AO-855		Upholstery, patient chair, neck rest, scuba, (XO-915 Sage)
3	AO-854		Upholstery, patient chair, neck rest, scuba, (XO-914 Dolphin)
3	AP-582		Upholstery, patient chair, neck rest, scuba, (XO-921 Truffle (025))
3	AO-856		Upholstery, patient chair, neck rest, scuba, (XO-916 Tourmaline)
3	AO-850		Upholstery, patient chair, neck rest, scuba, (XO-910 Cocoon)
3	AP-583		Upholstery, patient chair, neck rest, scuba, (XO-923 Mahogany (006))
3	AP-580		Upholstery, patient chair, neck rest, scuba, (XO-920 Coral (022))
3	AO-858		Upholstery, patient chair, neck rest, scuba, (XO-918 Cinnamon)
3	AO-859		Upholstery, patient chair, neck rest, scuba, (XO-919 Charcoal)
3	AO-857		Upholstery, patient chair, neck rest, scuba, (XO-917 Autumn)
3	AO-853		Upholstery, patient chair, neck rest, scuba, (XO-913 Jet)
3	AO-851		Upholstery, patient chair, neck rest, scuba, (XO-911 Dune)
3	AO-852		Upholstery, patient chair, neck rest, scuba, (XO-912 Walnut)
3	AP-584		Upholstery, patient chair, neck rest, scuba, (XO-924 Sapphire (016))
3	AP-581		Upholstery, patient chair, neck rest, scuba, (XO-922 Brick (021))

AP-946


Upholstery, patient chair, short back, separate, XO Comfort

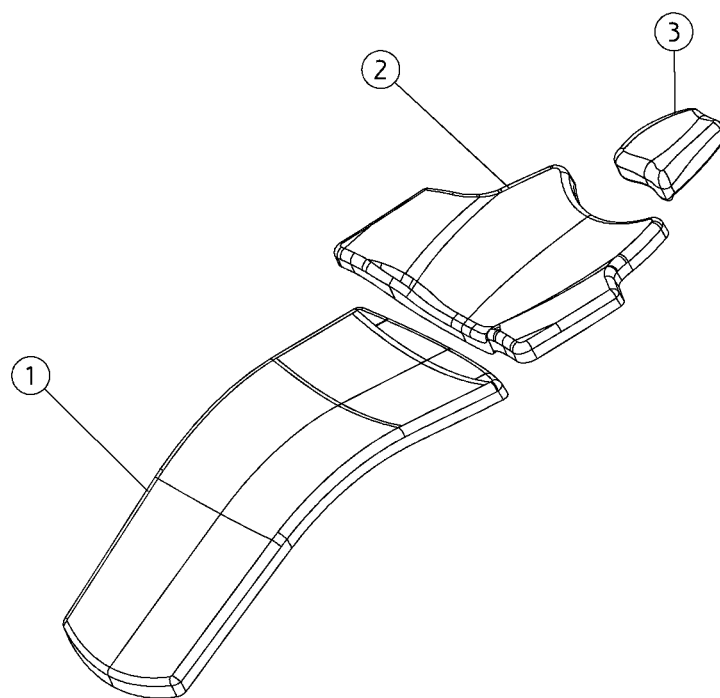

Position	Part NO.	Assembly	Description
1	AP-800		Upholstery, patient chair, seat, soft cushion, (CTBS)
2	AP-801		Upholstery, patient chair, neck rest, soft cushion, (CTBS)
3	AP-802		Upholstery, patient chair, back rest, short, soft cushion, (CTBS)
4	AP-797		Upholstery, patient chair, seat, standard cushion, (CTBS)
5	AP-798		Upholstery, patient chair, neck rest, standard cushion, (CTBS)
6	AP-799		Upholstery, patient chair, back rest, short, standard cushion, (CTBS)

AP-947

Upholstery, short back, complete set, skai

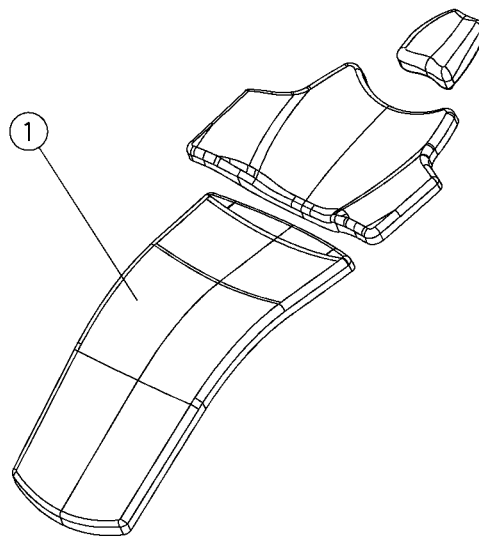


AP-947

Upholstery, short back, complete set, skai

Position	Part NO.	Assembly	Description
1	AN-317	<i>Page 267</i>	XO patient chair upholstery with skai, (XO-951 Red)
1	AN-071	<i>Page 267</i>	XO patient chair upholstery with skai, (XO-956 Light Green)
1	AO-334	<i>Page 267</i>	XO patient chair upholstery with skai, (XO-961 Scarlet Red)
1	AN-108	<i>Page 267</i>	XO patient chair upholstery with skai, (BA-374 Orange)
1	AO-333	<i>Page 267</i>	XO patient chair upholstery with skai, (XO-960 Chalk White)
1	AO-331	<i>Page 267</i>	XO patient chair upholstery with skai, (XO-958 Golden Honey)
1	AN-316	<i>Page 267</i>	XO patient chair upholstery with skai, (BA-243 Safran Yellow)
1	AN-314	<i>Page 267</i>	XO patient chair upholstery with skai, (BA-209 Sea Blue)
1	AN-310	<i>Page 267</i>	XO patient chair upholstery with skai, (BA-211 Dark Grey)
1	AN-311	<i>Page 267</i>	XO patient chair upholstery with skai, (WB-230 Grey)
1	AN-312	<i>Page 267</i>	XO patient chair upholstery with skai, (BA-407 Crystal White)
1	AN-315	<i>Page 267</i>	XO patient chair upholstery with skai, (XO-950 Light Blue)
1	AO-330	<i>Page 267</i>	XO patient chair upholstery with skai, (XO-957 Sap Green)
1	AO-102	<i>Page 267</i>	XO patient chair upholstery with skai, (BA-376 Pastel Blue)
1	AN-313	<i>Page 267</i>	XO patient chair upholstery with skai, (WB-234 Black)
1	AO-332	<i>Page 267</i>	XO patient chair upholstery with skai, (XO-959 Silk Grey)
1	AN-053	<i>Page 267</i>	XO patient chair upholstery with skai, (XO-954 Beige)
1	AN-055	<i>Page 267</i>	XO patient chair upholstery with skai, (XO-955 Bordeaux)

AP-948

Upholstery, short back, complete set, scuba

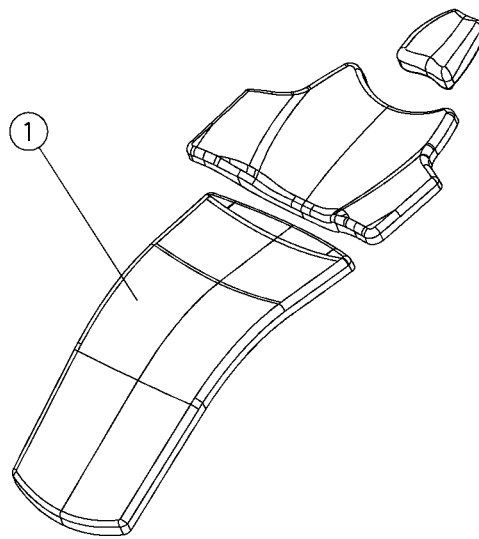


AP-948

Upholstery, short back, complete set, scuba


Position	Part NO.	Assembly	Description
1	AO-425	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-969 Vanilla)
1	AP-572	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-921 Truffle (025))
1	AP-573	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-923 Mahogany (006))
1	AO-430	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-974 Dolphin)
1	AO-426	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-910 Cocoon)
1	AO-433	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-917 Autumn)
1	AP-571	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-922 Brick (021))
1	AO-432	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-916 Tourmaline)
1	AO-434	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-918 Cinnamon)
1	AO-429	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-913 Jet)
1	AP-574	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-924 Sapphire (016))
1	AO-428	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-912 Walnut)
1	AP-570	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-920 Coral (022))
1	AO-431	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-915 Sage)
1	AO-427	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-911 Dune)
1	AO-435	<i>Page 269</i>	XO patient chair upholstery with XO scuba, (XO-919 Charcoal)

AP-949 Upholstery, short back, complete set, XO Comfort

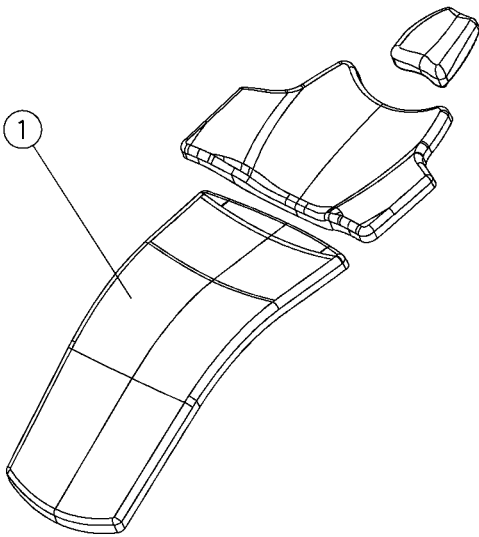

Position	Part NO.	Assembly	Description
1	AP-789		Standard cushions for patient chair, (CTBS)
2	AP-790		Soft cushions for patient chair, (CTBS)

AN-685 Child's cushion w knee support

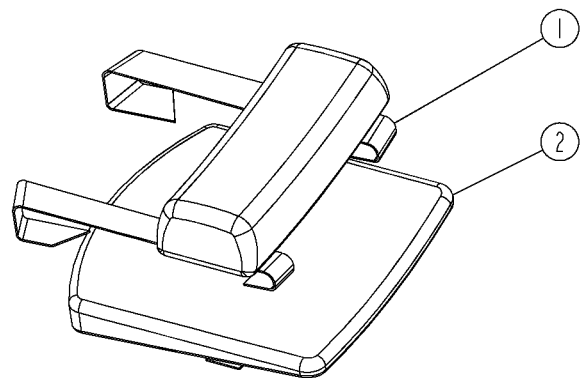


Child patient cushion with support for knee

Position	Part NO.	Assembly	Description
1	AN-686		Knee support with straps (Unknown colour)
2	MG-802		Not a spare part

AN-256


Suction h.h. chair left-handed

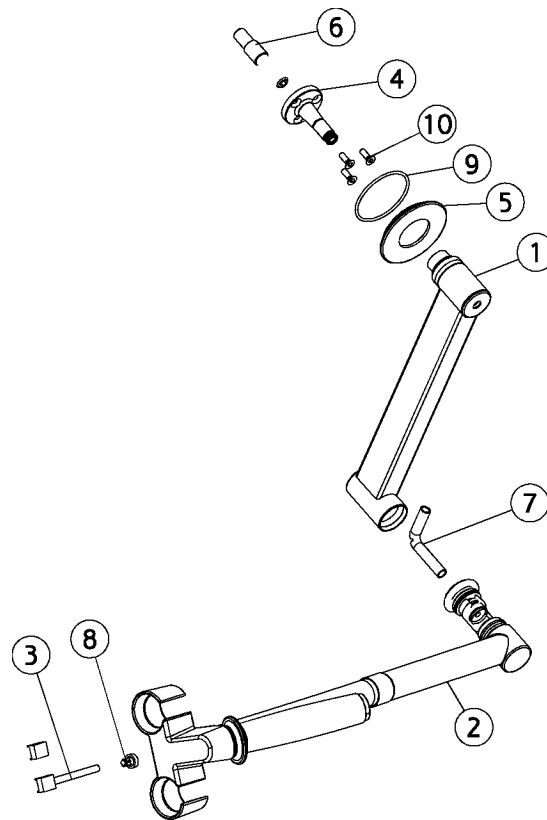

Position	Part NO.	Assembly	Description
1	AN-258	Page 275	Arm1 shh chair w components
2	AN-261	Page 277	Arm2 shh left-handed w handle
3	CA-046		Cable, suction holder, chair
4	MG-773		Axle chair-arm1
5	MG-776		Cover plug joint seat-back
6	MG-783		Threaded bush M10, Ø12x15
7	MG-910		Not a spare part
8	ML-281		Cable binder 150 mm
9	MR-210		O-ring 45 x 2
10	UG-040		Allen screw, ui6kt M4x12mm, din 7991

AN-257


Suction h.h chair right-handed

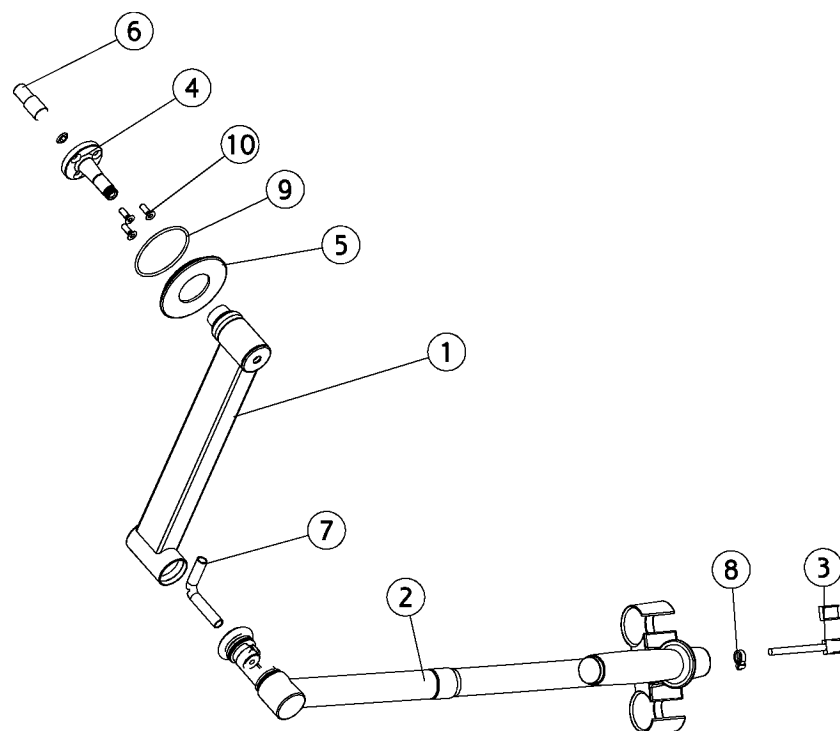

Position	Part NO.	Assembly	Description
1	AN-258	Page 275	Arm1 shh chair w components
2	AN-262	Page 278	Arm2 shh right-handed w handle
3	CA-046		Cable, suction holder, chair
4	MG-773		Axle chair-arm1
5	MG-776		Cover plug joint seat-back
6	MG-783		Threaded bush M10, Ø12x15
7	MG-910		Not a spare part
8	ML-281		Cable binder 150 mm
9	MR-210		O-ring 45 x 2
10	UG-040		Allen screw, ui6kt M4x12mm, din 7991

AN-258


Arm1 shh chair w components

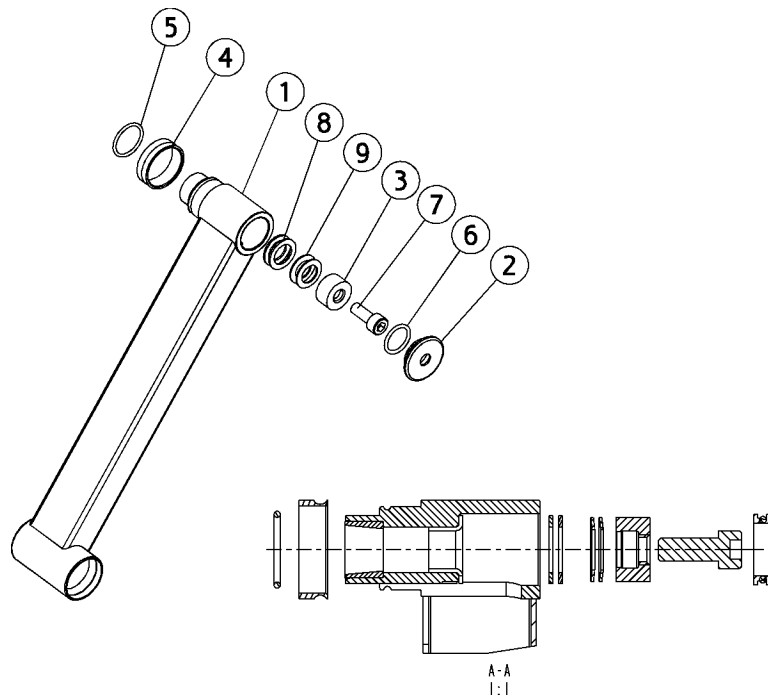

Position	Part NO.	Assembly	Description
1	AN-268	Page 276	Arm1 suction hose holder chair
2	MG-777		Cover plug, arm1
3	MG-778		Thrust collar, arm1
4	MG-800		Bearing, tube holder
5	MR-209		O-ring 17.1 x 1.6
6	MR-225		O-ring Ø 1,78 x 14
7	UG-063		Bolt i6kt din912 M6x16
8	UG-265		Adaptor disc Ø12xØ18x1.0 din 988
9	UG-274		Disc spring, Ø17xØ10,5x0,7x1,1

AN-268


Arm1 suction hose holder chair

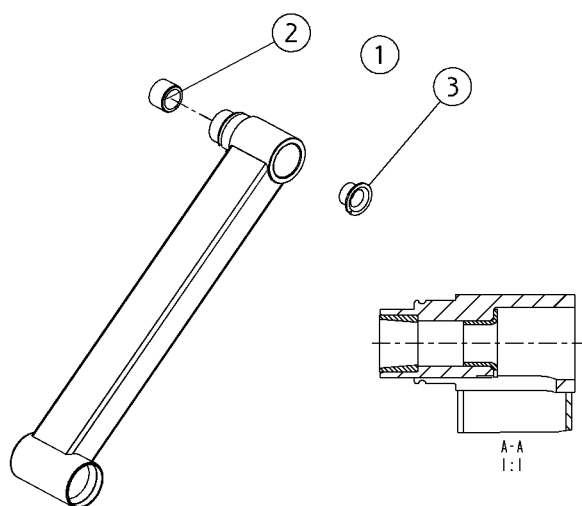

Position	Part NO.	Assembly	Description
1	AO-786		Arm1 shh chair,
2	MG-774		Not a spare part
3	UG-471		Not a spare part

AN-261


Arm2 shh left-handed w handle

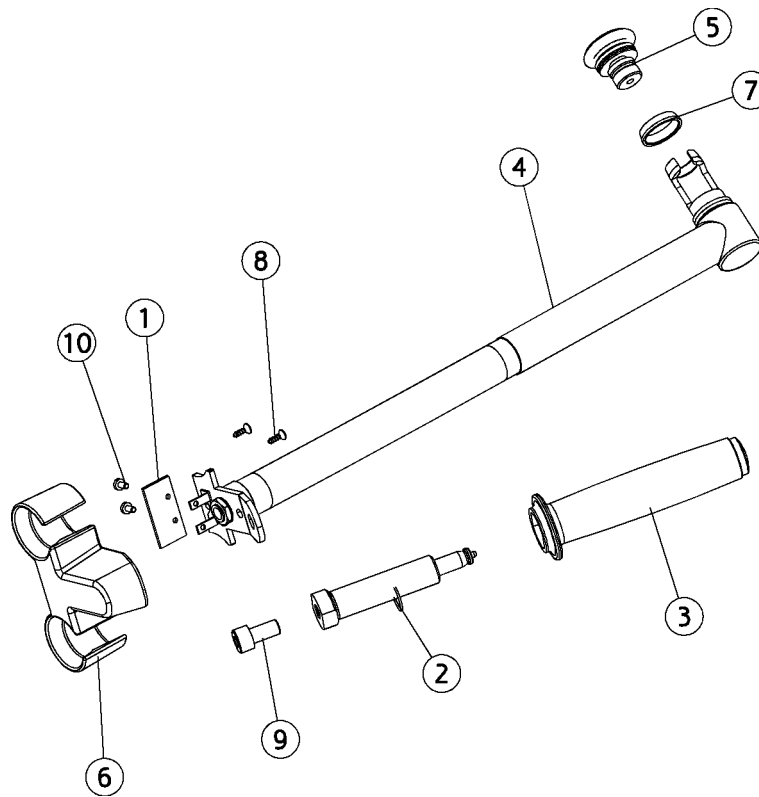

Position	Part NO.	Assembly	Description
1	AN-299		PCB, suction holder type 2005
2	AN-084	Page 158	Receiver, handles
3	AN-110		Handle XO 4
4	AN-259	Page 279	Arm2 suc.hose hldr left-handed
5	AN-263	Page 291	Brake, bolt arm2
6	MG-764		Holder, suction, chair,
7	MG-781		Not a spare part
8	UC-086		Screw M3x10
9	UG-003		Bolt inside hex, M08x016 din912
10	UG-102		Screw, thread making, M3x006 DIN7500

AN-262


Arm2 shh right-handed w handle

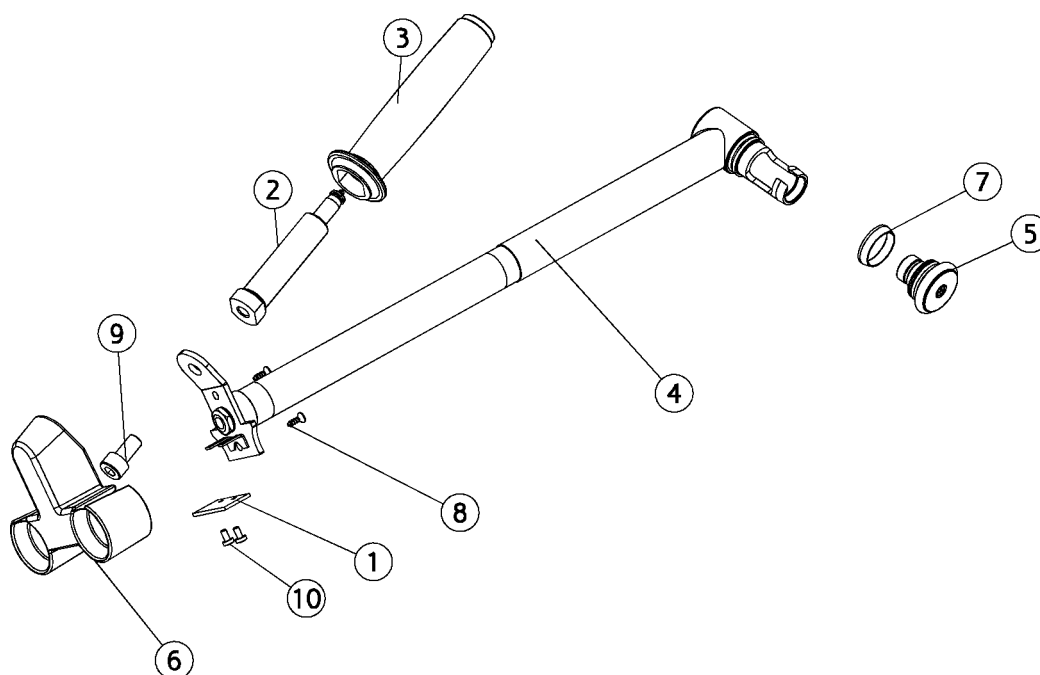

Position	Part NO.	Assembly	Description
1	AN-299		PCB, suction holder type 2005
2	AN-084	Page 158	Receiver, handles
3	AN-110		Handle XO 4
4	AN-260	Page 280	Arm2 suc.hose h. right-handed
5	AN-263	Page 291	Brake, bolt arm2
6	MG-764		Holder, suction, chair,
7	MG-781		Not a spare part
8	UC-086		Screw M3x10
9	UG-003		Bolt inside hex, M08x016 din912
10	UG-102		Screw, thread making, M3x006 DIN7500

AN-259


Arm2 suc.hose hldr left-handed

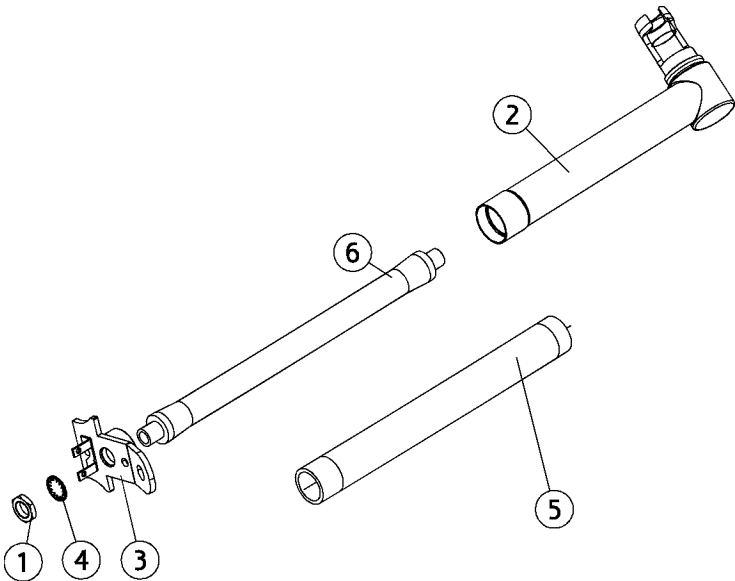

Position	Part NO.	Assembly	Description
1	ME-017		Nut, suct.hose holder, M10x1
2	AN-265		Arm2 shh left handed,
3	AN-269		Fittings shh chair,
4	UC-170		Not a spare part
5	UG-947		Silicone tube Ø21 x 16,8 RAL 7011
6	UG-978		Spring sh l200 2xM10x1 strong

AN-260


Arm2 suc.hose h. right-handed

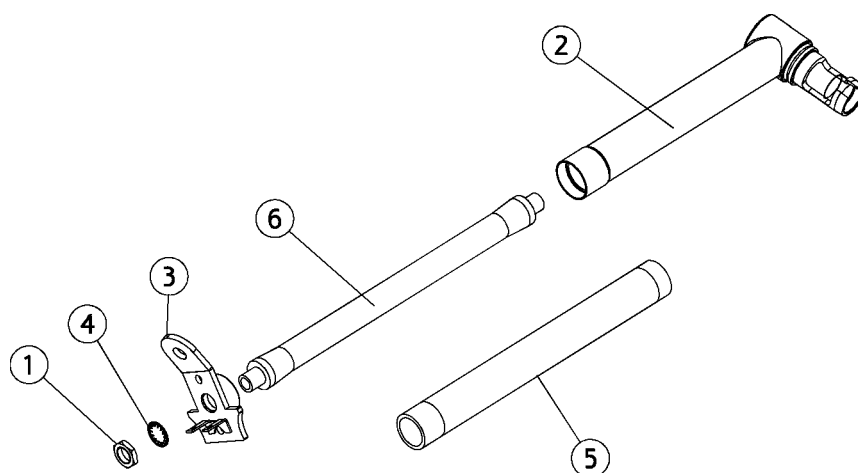

Position	Part NO.	Assembly	Description
1	ME-017		Nut, suct.hose holder, M10x1
2	AN-266		Arm2 shh right handed,
3	AN-269		Fittings shh chair,
4	UC-170		Not a spare part
5	UG-947		Silicone tube Ø21 x 16,8 RAL 7011
6	UG-978		Spring sh l200 2xM10x1 strong

AP-541

Suction h.h. auto return chair, left side,

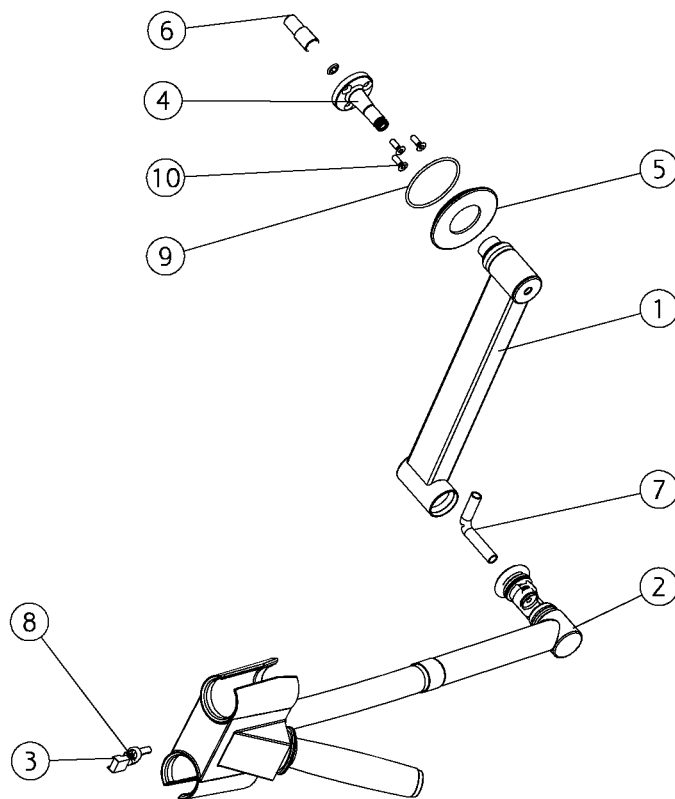


AP-541

Suction h.h. auto return chair, left side,

Position	Part NO.	Assembly	Description
1	AN-258	<i>Page 275</i>	Arm1 shh chair w components
2	AP-545	<i>Page 288</i>	Suction h.h. arm2, right side w/handle,
3	CA-046		Cable, suction holder, chair
4	MG-773		Axle chair-arm1
5	MG-776		Cover plug joint seat-back
6	MG-783		Threaded bush M10, Ø12x15
7	MG-910		Not a spare part
8	ML-281		Cable binder 150 mm
9	MR-210		O-ring 45 x 2
10	UG-040		Allen screw, ui6kt M4x12mm, din 7991

AP-543 Suction h.h. arm2, left side w/handle,

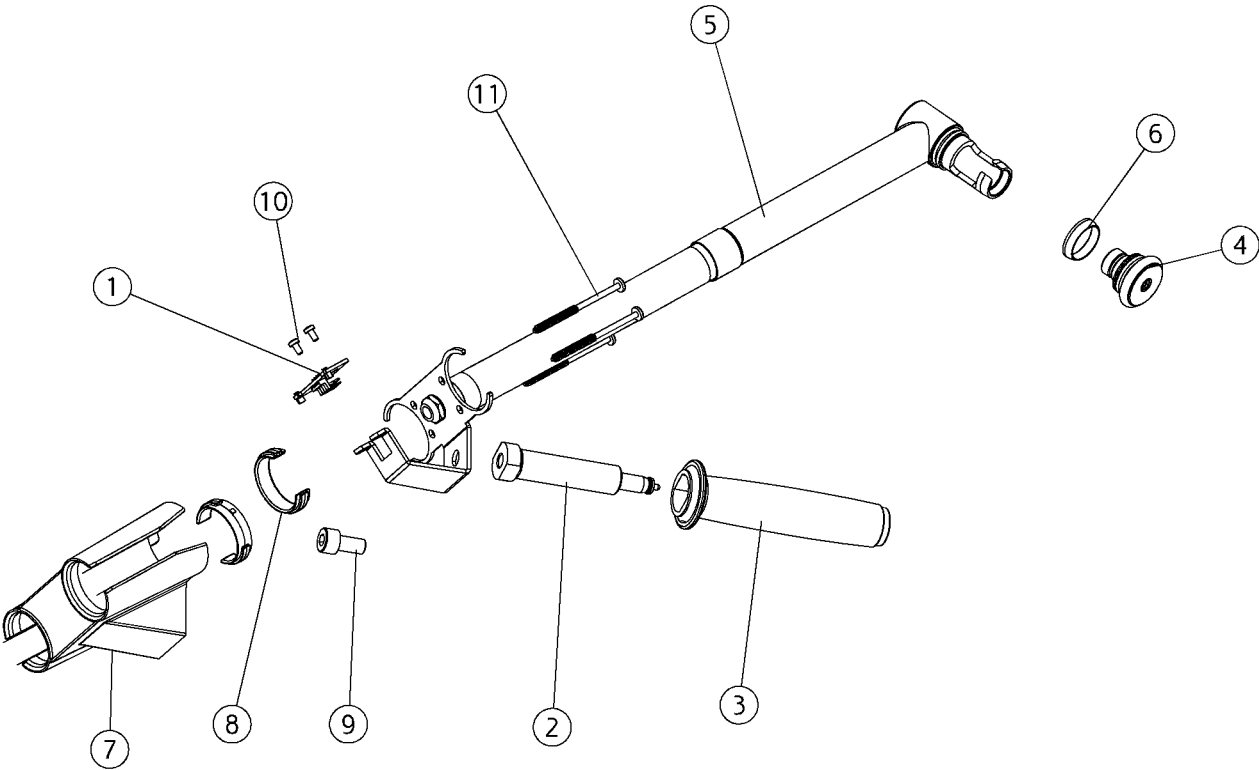


AP-543

Suction h.h. arm2, left side w/handle,

Position	Part NO.	Assembly	Description
1	AO-987		PCB, suction holder auto return
2	AN-084	<i>Page 158</i>	Receiver, handles
3	AN-110		Handle XO 4
4	AN-263	<i>Page 291</i>	Brake, bolt arm2
5	AP-544	<i>Page 285</i>	Suction h.h. arm2, left side,
6	MG-781		Not a spare part
7	MG-989		Holder, auto return, chair,
8	MN-043		Not a spare part
9	UG-003		Bolt inside hex, M08x016 din912
10	UH-088		Not a spare part
11	UH-155		Screw, panhead, plastite, K40X60/30

AP-544


Suction h.h. arm2, left side,

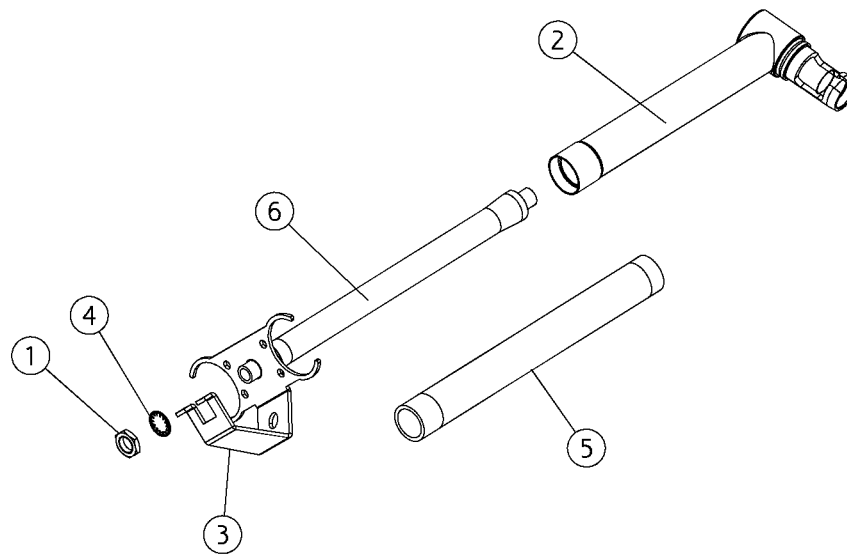

Position	Part NO.	Assembly	Description
1	ME-017		Nut, suct.hose holder, M10x1
2	AO-788		Arm2 shh right handed, (XO-908 Iron Grey (RAL 7011-5))
2	AN-266		Arm2 shh right handed, (XO-909 Iron Grey (RAL 7011-30))
2	AP-698		Arm2 shh right handed, (CTBS)
3	AP-537		Bracket for suction hose holder
4	UC-170		Not a spare part
5	UG-947		Silicone tube Ø21 x 16,8 RAL 7011
6	UG-978		Spring sh l200 2xM10x1 strong

AP-542

Suction h.h. auto return chair, right side,

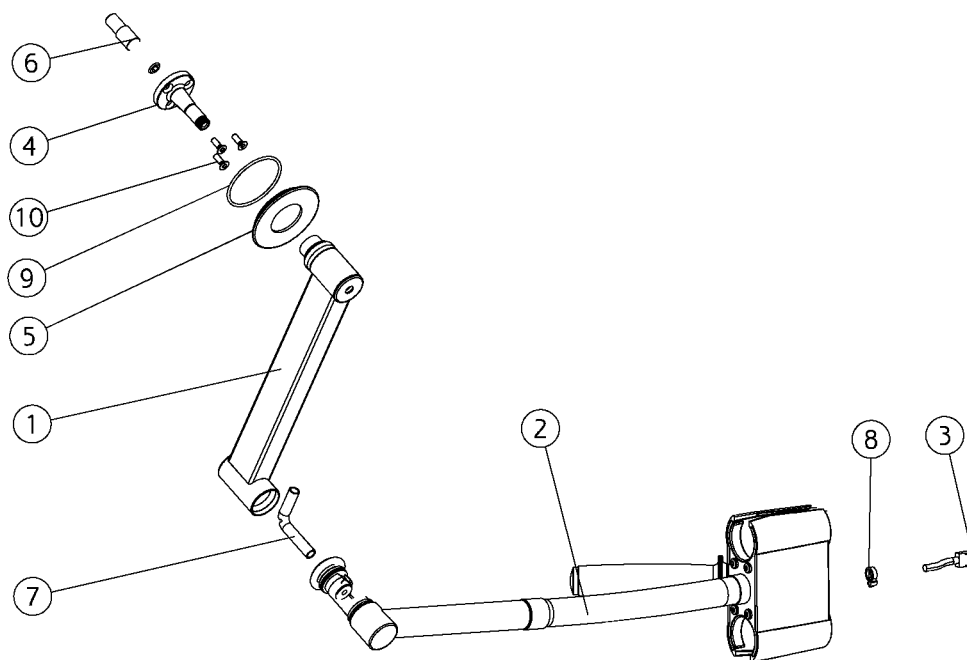


AP-542

Suction h.h. auto return chair, right side,

Position	Part NO.	Assembly	Description
1	AN-258	<i>Page 275</i>	Arm1 shh chair w components
2	AP-543	<i>Page 283</i>	Suction h.h. arm2, left side w/handle,
3	CA-046		Cable, suction holder, chair
4	MG-773		Axle chair-arm1
5	MG-776		Cover plug joint seat-back
6	MG-783		Threaded bush M10, Ø12x15
7	MG-910		Not a spare part
8	ML-281		Cable binder 150 mm
9	MR-210		O-ring 45 x 2
10	UG-040		Allen screw, ui6kt M4x12mm, din 7991

AP-545

Suction h.h. arm2, right side w/handle,

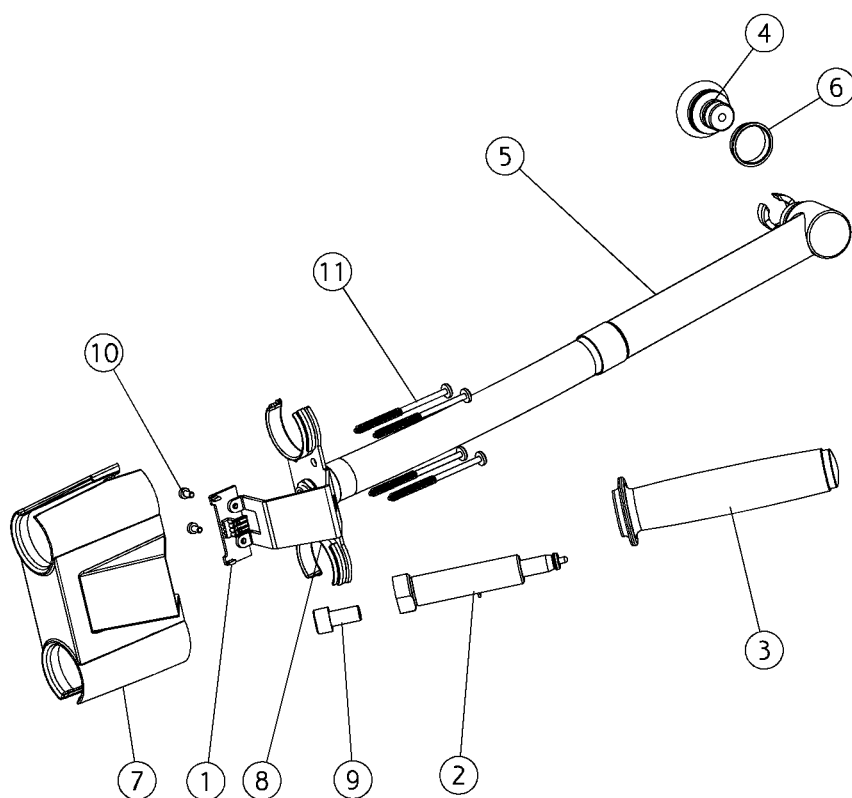


AP-545

Suction h.h. arm2, right side w/handle,

Position	Part NO.	Assembly	Description
1	AD-905		Not a spare part
2	AN-084	<i>Page 158</i>	Receiver, handles
3	AN-110		Handle XO 4
4	AN-263	<i>Page 291</i>	Brake, bolt arm2
5	AP-546	<i>Page 290</i>	Suction h.h. arm2, right side,
6	MG-781		Not a spare part
7	MG-989		Holder, auto return, chair,
8	MN-043		Not a spare part
9	UG-003		Bolt inside hex, M08x016 din912
10	UH-088		Not a spare part
11	UH-155		Screw, panhead, plastite, K40X60/30

AP-546


Suction h.h. arm2, right side,

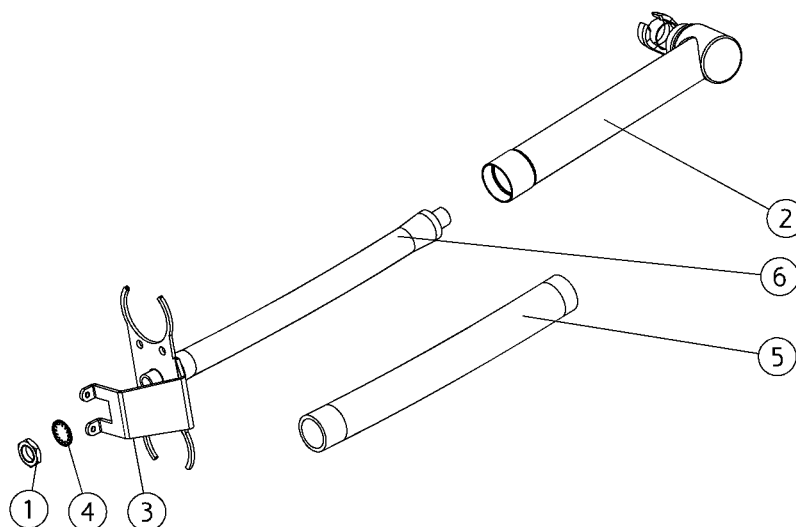

Position	Part NO.	Assembly	Description
1	ME-017		Nut, suct.hose holder, M10x1
2	AN-265		Arm2 shh left handed, (XO-909 Iron Grey (RAL 7011-30))
2	AP-700		Arm2 shh left handed, (CTBS)
2	AO-787		Arm2 shh left handed, (XO-908 Iron Grey (RAL 7011-5))
3	AP-537		Bracket for suction hose holder
4	UC-170		Not a spare part
5	UG-947		Silicone tube Ø21 x 16,8 RAL 7011
6	UG-978		Spring sh l200 2xM10x1 strong


AN-263

Brake, bolt arm2

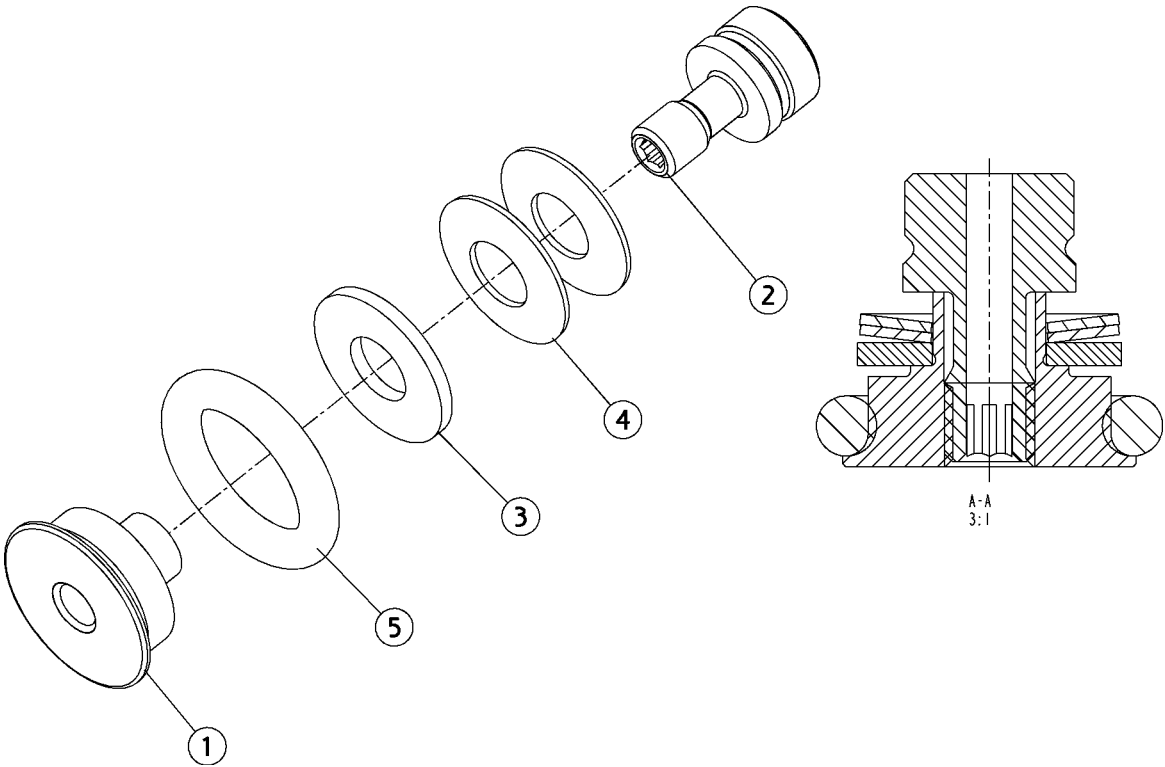


Position	Part NO.	Assembly	Description
1	MG-779		Not a spare part
2	MG-780		Not a spare part
3	MG-785		Not a spare part
4	UG-290		Not a spare part
5	UG-335		O-ring Ø19,99xØ5,33

AN-267 Suc.hose hldr bridge, cover

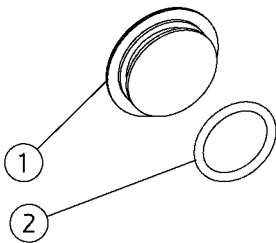

Position	Part NO.	Assembly	Description
1	MG-789		Cover, suction holder, chair
2	SA-890		O-ring 19.3 x 2.4


AN-831 Suction hose Ø16 l=1710mm

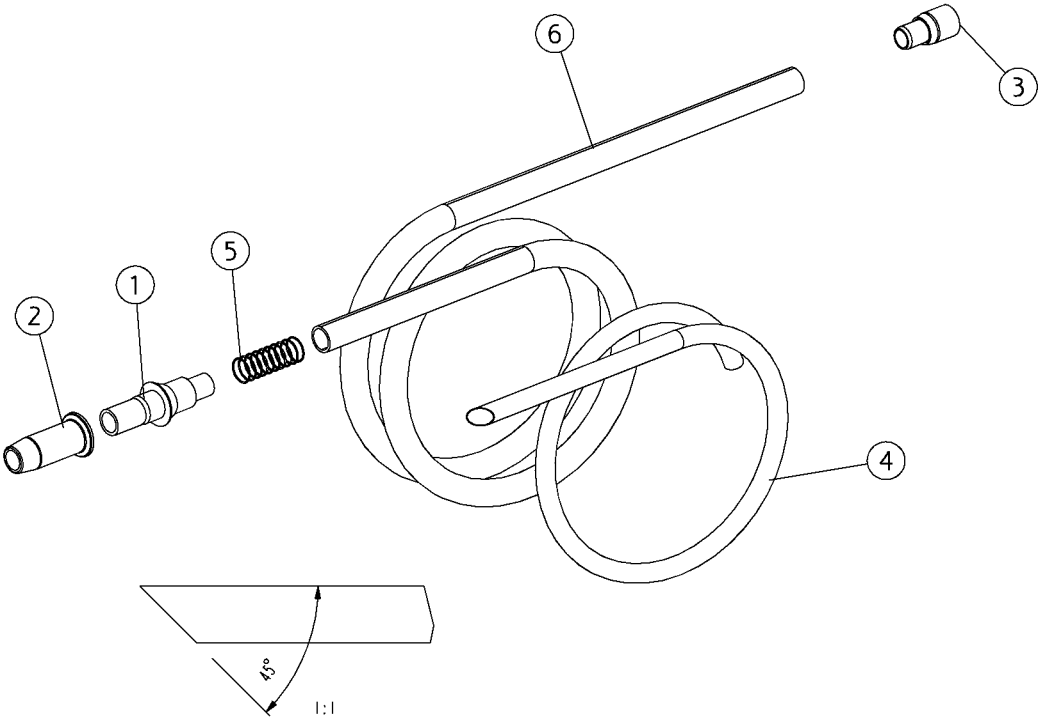

Position	Part NO.	Assembly	Description
1	MG-475		Suction nipple,Ø16,
2	MG-480		Jacket,suction tube nipple Ø16
3	MG-482		Nipple, suction hose Ø16,
4	MS-360		Hose 15mm x 14 mm alcryn
5	UC-852		O-ring Ø18 x 1 black
6	UH-193		Hose, suction, Ø17, grey (RAL 7011)


AN-832

Suction hose Ø11 l=1710mm

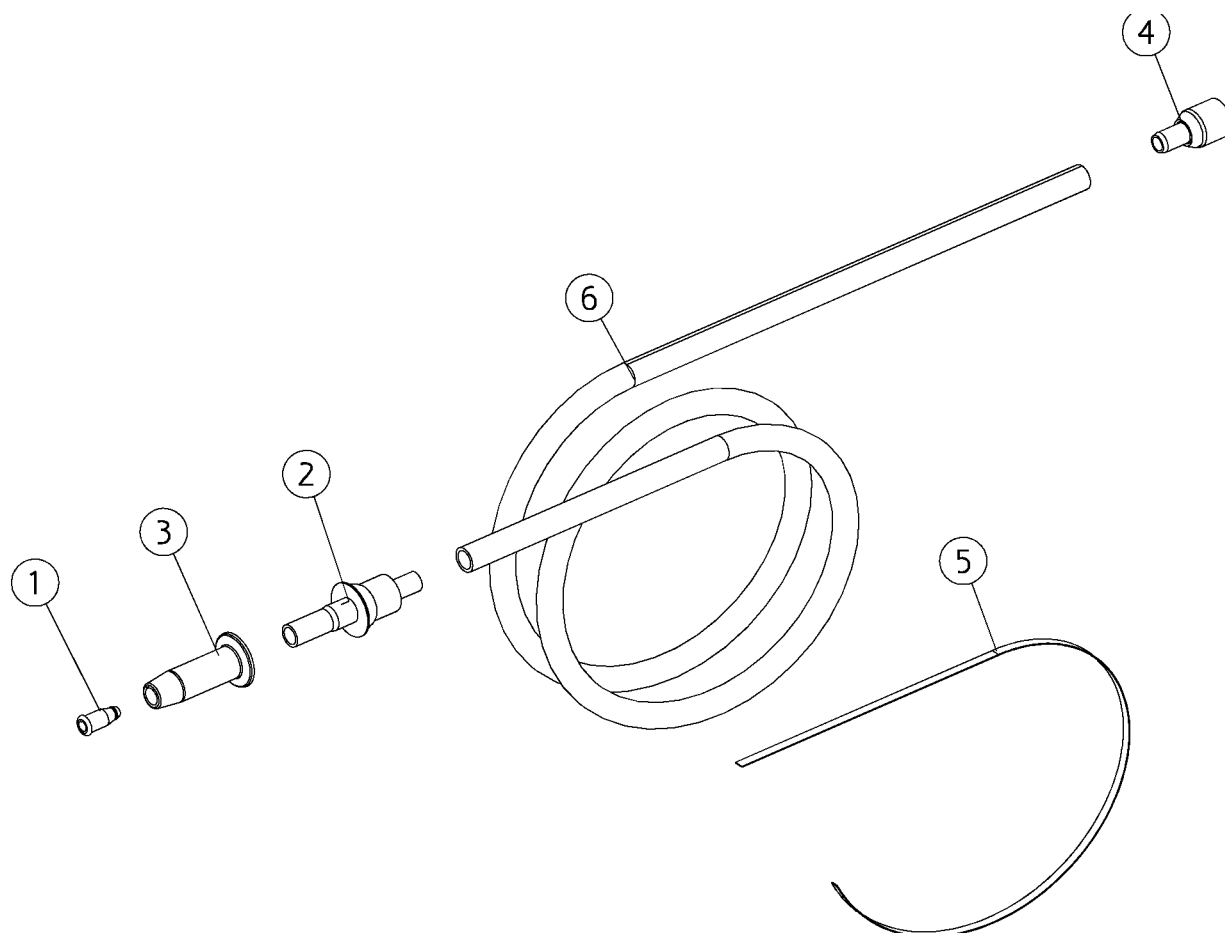

Position	Part NO.	Assembly	Description
1	MC-263		Reduction, suction Ø11 - 7
2	MG-476		Suction nipple, Ø11,
3	MG-481		Jacket, suction tube nipple Ø11
4	MG-483		Nipple, suction hose Ø11,
5	UC-692		Strapex ribbon 6 mm, white
6	UH-194		Hose, suction, Ø12, grey, (RAL 7011)


AN-837 Suction hose Ø16 l=2090mm

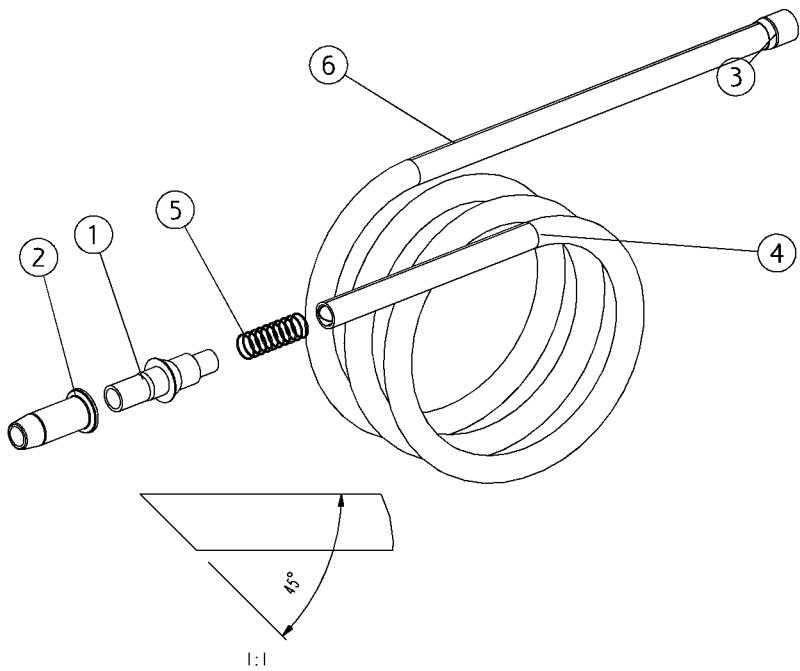

Position	Part NO.	Assembly	Description
1	MG-475		Suction nipple,Ø16,
2	MG-480		Jacket,suction tube nipple Ø16
3	MG-482		Nipple, suction hose Ø16,
4	MS-360		Hose 15mm x 14 mm alcryn
5	UC-852		O-ring Ø18 x 1 black
6	UH-193		Hose, suction, Ø17, grey (RAL 7011)

AN-838

Suction hose Ø11 l=2090mm

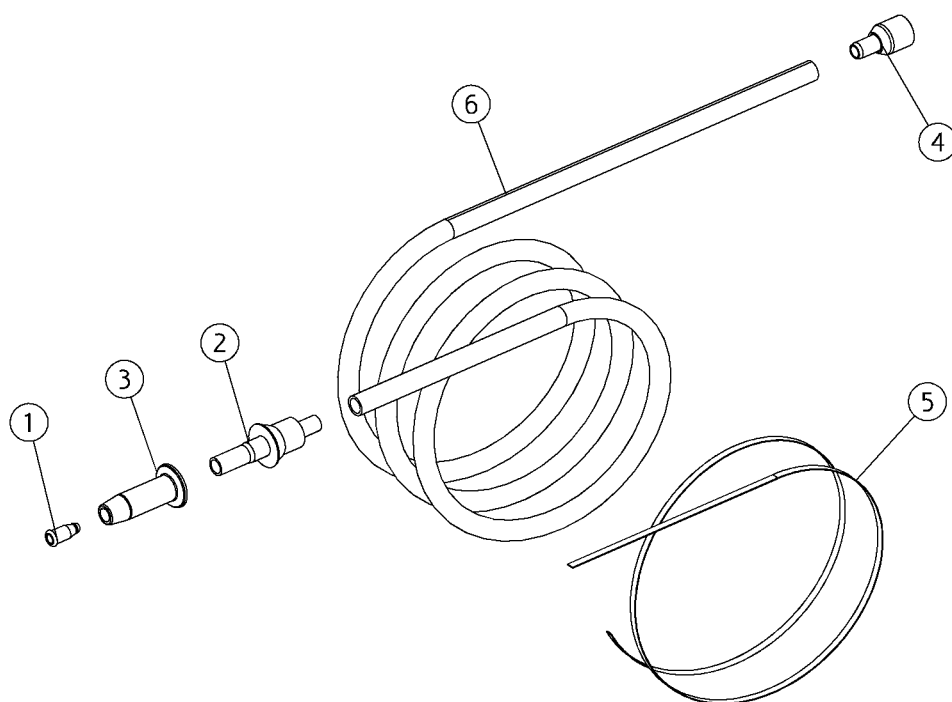


Position	Part NO.	Assembly	Description
1	MC-263		Reduction, suction Ø11 - 7
2	MG-476		Suction nipple, Ø11,
3	MG-481		Jacket, suction tube nipple Ø11
4	MG-483		Nipple, suction hose Ø11,
5	UC-692		Strapex ribbon 6 mm, white
6	UH-194		Hose, suction, Ø12, grey, (RAL 7011)

XO 4 Lamp and Arm system

AN-092


Arm lamp

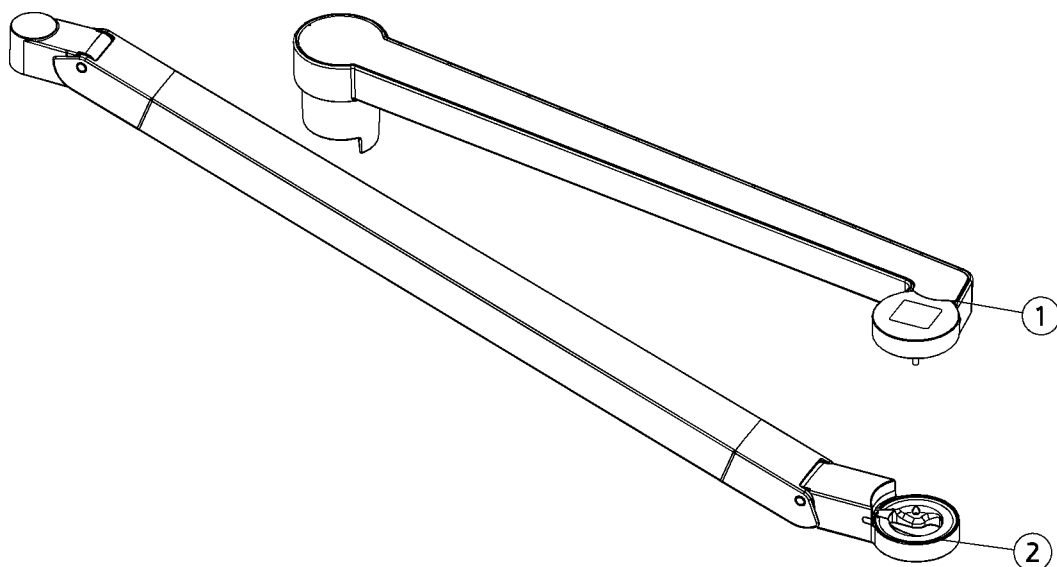

Position	Part NO.	Assembly	Description
1	AN-586	Page 299	Rear arm lamp assy., (XO-901 Sea Blue Metallic)
1	AN-791	Page 299	Rear arm lamp assy., (XO-903 Silver Metallic)
1	AN-585	Page 299	Rear arm lamp assy., (XO-900 XO Grey, RAL 7047-80)
2	AN-641	Page 301	Forearm lamp, (XO-900 XO Grey, RAL 7047-80)
2	AN-643	Page 301	Forearm lamp, (XO-901 Sea Blue Metallic)
2	AN-792	Page 301	Forearm lamp, (XO-903 Silver Metallic)

AN-093

Back arm, lamp

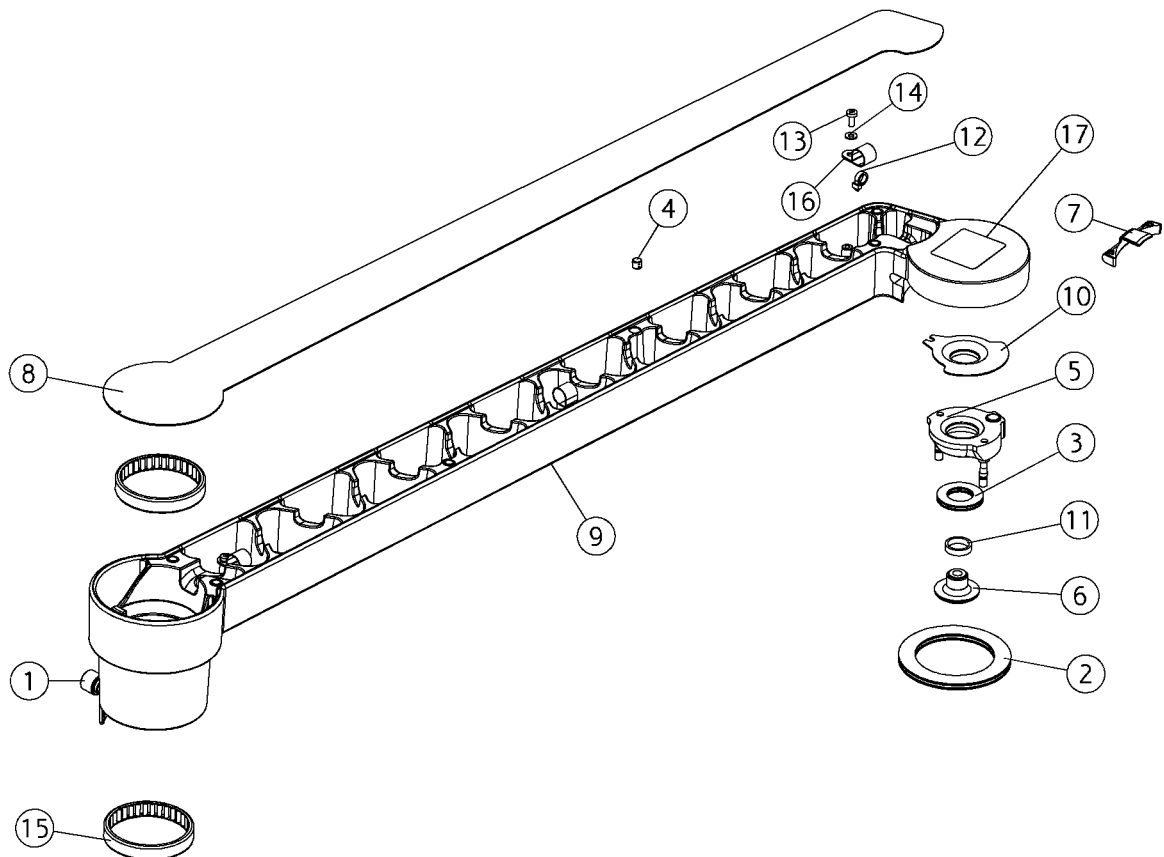


AN-093

Back arm, lamp

Position	Part NO.	Assembly	Description
1	AN-018	<i>Page 182</i>	Friction brake, arms
2	AN-094	<i>Page 203</i>	Axle bearing 60x85x5
3	AN-095	<i>Page 201</i>	Axle bearing 20x35x4
4	AN-127	<i>Page 198</i>	Magnet Ø6,35 with spring ring
5	AP-898	<i>Page 337</i>	Suspension, Joint, screen, lamp, complete
6	MG-290		Not a spare part
7	MG-291		Coverplug rear arm lamp/screen
8	AO-274		Cover plate, rear arm, lamp, (XO-906 XO Titanium)
8	AN-793		Cover plate, rear arm, lamp, (XO-903 Silver Metallic)
8	AN-552		Cover plate, rear arm, lamp, (XO-901 Sea Blue Metallic)
8	AP-420		Cover plate, rear arm, lamp, (CTBS)
8	AN-551		Cover plate, rear arm, lamp, (XO-900 XO Grey, RAL 7047-80)
8	AN-936		Cover plate, rear arm, lamp, (XO-905 XO White,(RAL7047-30))
9	AN-945		Rear arm lamp, (XO-905 XO White,(RAL7047-30))
9	AO-283		Rear arm lamp, (XO-906 XO Titanium)
9	AN-568		Rear arm lamp, (XO-901 Sea Blue Metallic)
9	AN-794		Rear arm lamp, (XO-903 Silver Metallic)
9	AP-429		Rear arm lamp, (CTBS)
9	AN-567		Rear arm lamp, (XO-900 XO Grey, RAL 7047-80)
10	MG-328		Not a spare part
11	MG-329		Not a spare part
12	ML-281		Cable binder 150 mm
13	UG-110		Screw, thread making, M4x10
14	UG-254		Disc, Ø04.3xØ09.0x0.8
15	UG-458		Not a spare part
16	UG-527		Not a spare part
17	UG-948		Not a spare part

AN-097

Forearm lamp

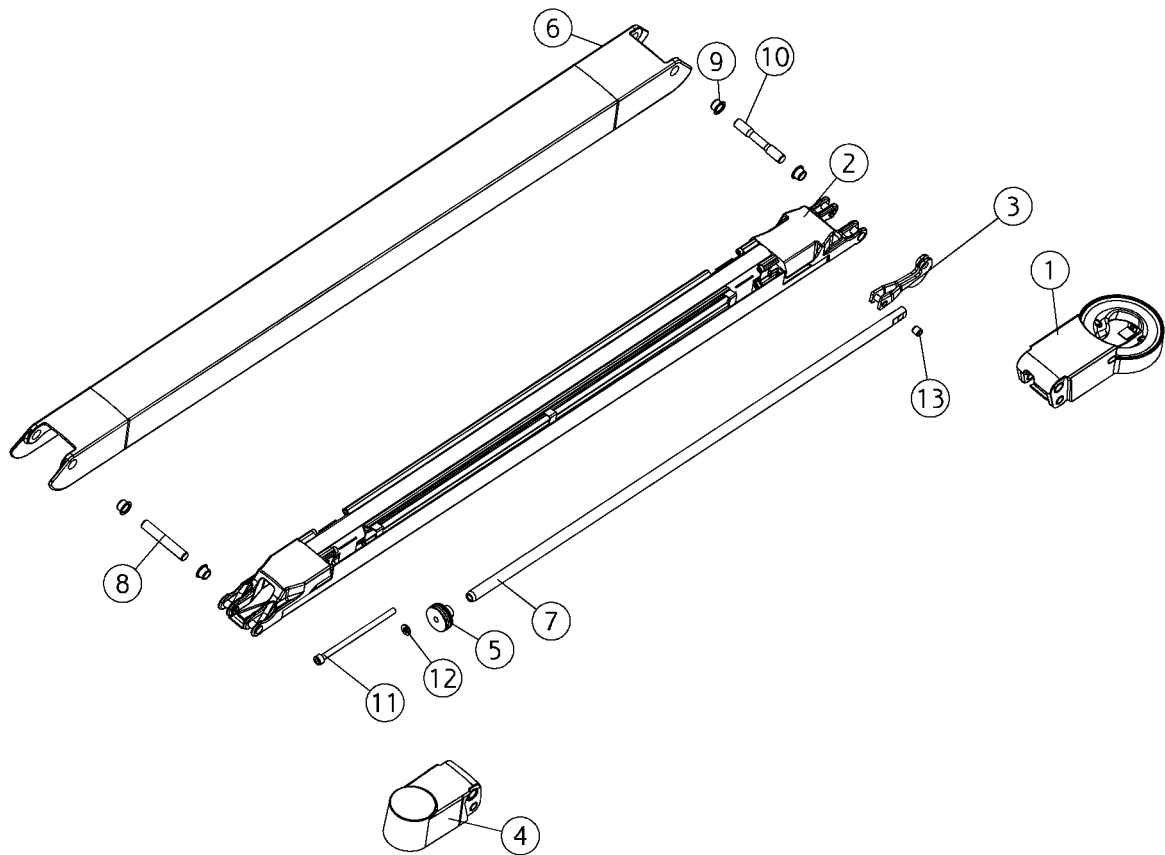


AN-097

Forearm lamp

Position	Part NO.	Assembly	Description
1	AN-098	<i>Page 303</i>	Link1, forearm lamp
2	AN-099	<i>Page 305</i>	Bottom forearm lamp
3	AN-104	<i>Page 312</i>	Lever lamp
4	AN-105	<i>Page 307</i>	Link2, forearm lamp
5	AN-118	<i>Page 313</i>	Control socket spring lamp arm
6	AP-421		Top, forearm, lamp, (CTBS)
6	AN-851		Top, forearm, lamp, (XO-903 Silver Metallic)
6	AO-275		Top, forearm, lamp, (XO-906 XO Titanium)
6	AN-937		Top, forearm, lamp, (XO-905 XO White,(RAL7047-30))
6	AN-553		Top, forearm, lamp, (XO-900 XO Grey, RAL 7047-80)
6	AN-554		Top, forearm, lamp, (XO-901 Sea Blue Metallic)
7	MG-296		Not a spare part
8	MG-298		Axle Ø10x65
9	MG-299		Bushing Ø10x8 for u-profile
10	MN-145		Not a spare part
11	UG-031		Not a spare part
12	UG-256		Disc, Ø06.4xØ12.0x1.6
13	UG-459		Not a spare part

AN-098

Link1, forearm lamp

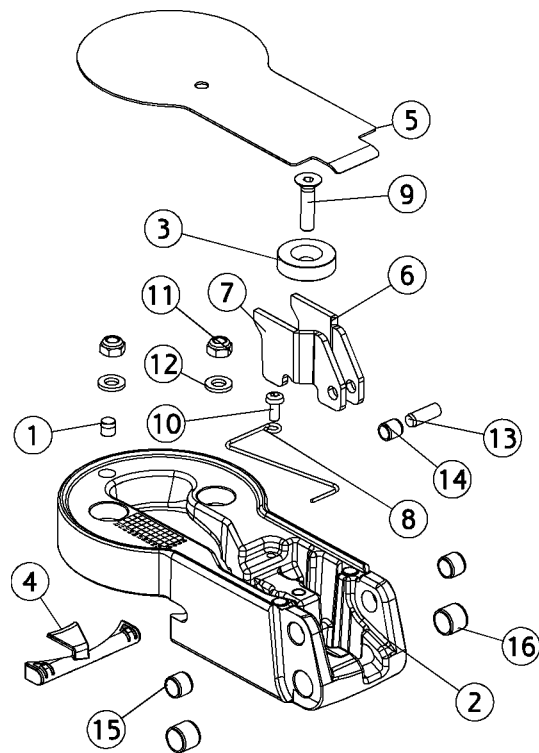


AN-098

Link1, forearm lamp

Position	Part NO.	Assembly	Description
1	AN-127	<i>Page 198</i>	Magnet Ø6,35 with spring ring
2	AO-276		Link1, forearm, lamp, (XO-906 XO Titanium)
2	AN-852		Link1, forearm, lamp, (XO-903 Silver Metallic)
2	AN-555		Link1, forearm, lamp, (XO-900 XO Grey, RAL 7047-80)
2	AP-422		Link1, forearm, lamp, (CTBS)
2	AN-938		Link1, forearm, lamp, (XO-905 XO White,(RAL7047-30))
2	AN-556		Link1, forearm, lamp, (XO-901 Sea Blue Metallic)
3	MG-304		Not a spare part
4	MG-305		Not a spare part
5	AN-939		Cover, rear, link1, arm, lamp, (XO-905 XO White,(RAL7047-30))
5	AN-558		Cover, rear, link1, arm, lamp, (XO-901 Sea Blue Metallic)
5	AN-853		Cover, rear, link1, arm, lamp, (XO-903 Silver Metallic)
5	AP-423		Cover, rear, link1, arm, lamp, (CTBS)
5	AO-277		Cover, rear, link1, arm, lamp, (XO-906 XO Titanium)
5	AN-557		Cover, rear, link1, arm, lamp, (XO-900 XO Grey, RAL 7047-80)
6	MG-314		Not a spare part
7	MG-315		Not a spare part
8	MG-337		Not a spare part
9	UG-032		Screw M6x25
10	UG-110		Screw, thread making, M4x10
11	UG-206		Not a spare part
12	UG-256		Disc, Ø06.4xØ12.0x1.6
13	UG-416		Not a spare part
14	UG-459		Not a spare part
15	UG-460		Not a spare part
16	UG-461		Not a spare part

AN-099

Bottom forearm lamp

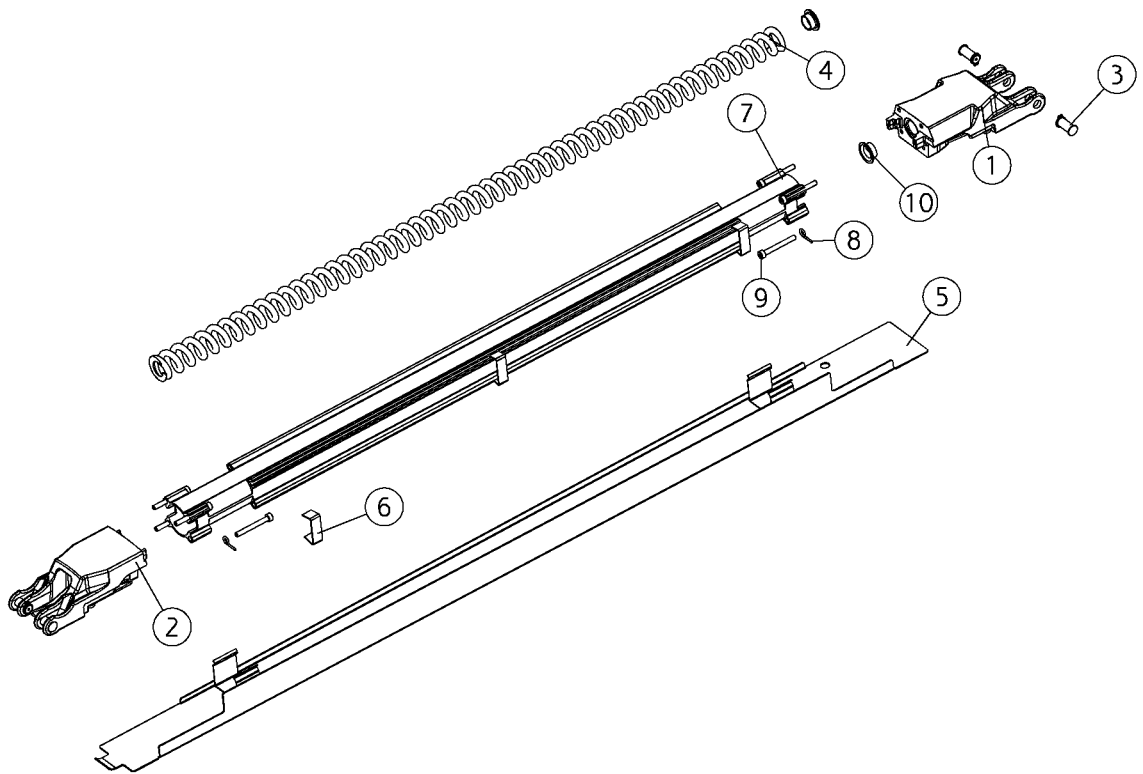


AN-099

Bottom forearm lamp

Position	Part NO.	Assembly	Description
1	AN-100	<i>Page 309</i>	Suspension1 forearm lamp
2	AN-102	<i>Page 310</i>	Suspension2 forearm lamp
3	AN-103	<i>Page 200</i>	Axle 8x17
4	MG-310		Not a spare part
5	AN-295		Cover forearm lamp, (XO-903) (XO-903 Silver Metallic)
5	AN-543		Cover forearm lamp, (XO-900 XO Grey, RAL 7047-80)
5	AN-544		Cover forearm lamp, (XO-901 Sea Blue Metallic)
5	AP-426		Cover forearm lamp, (CTBS)
5	AO-280		Cover forearm lamp, (XO-906 XO Titanium)
5	AN-942		Cover forearm lamp, (XO-905 XO White,(RAL7047-30))
6	MG-464		Not a spare part
7	MG-466		Not a spare part
8	MG-669		Not a spare part
9	UG-013		Not a spare part
10	UG-463		Not a spare part

AN-105

Link2, forearm lamp

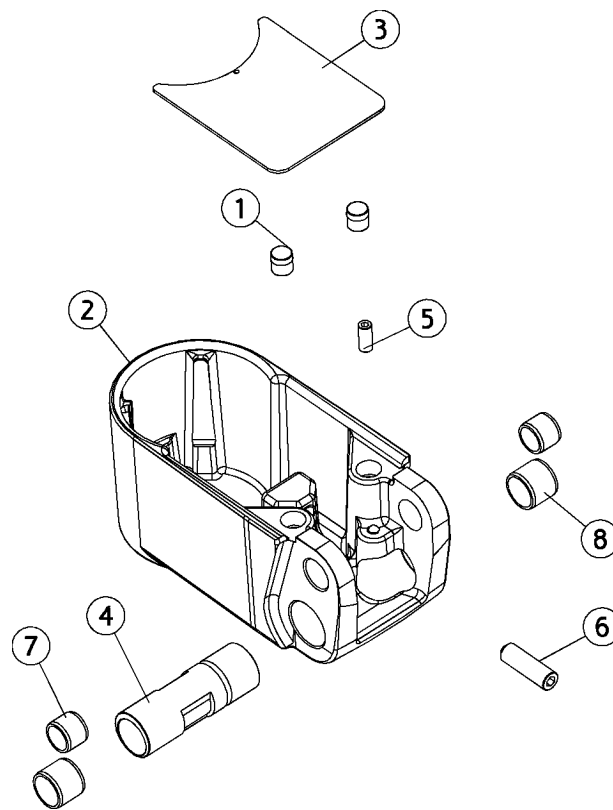


AN-105

Link2, forearm lamp

Position	Part NO.	Assembly	Description
1	AN-127	<i>Page 198</i>	Magnet Ø6,35 with spring ring
2	AN-943		Link2, arm lamp, (XO-905 XO White,(RAL7047-30))
2	AO-281		Link2, arm lamp, (XO-906 XO Titanium)
2	AN-563		Link2, arm lamp, (XO-900 XO Grey, RAL 7047-80)
2	AN-854		Link2, arm lamp, (XO-903 Silver Metallic)
2	AP-427		Link2, arm lamp, (CTBS)
2	AN-564		Link2, arm lamp, (XO-901 Sea Blue Metallic)
3	AN-944		Cover link2 lamp, (XO-905 XO White,(RAL7047-30))
3	AN-565		Cover link2 lamp, (XO-900 XO Grey, RAL 7047-80)
3	AP-428		Cover link2 lamp, (CTBS)
3	AN-855		Cover link2 lamp, (XO-903 Silver Metallic)
3	AN-566		Cover link2 lamp, (XO-901 Sea Blue Metallic)
3	AO-282		Cover link2 lamp, (XO-906 (XO-906 XO Titanium)
4	MG-465		Not a spare part
5	UG-118		Not a spare part
6	UG-119		Pointed screw, M6x20
7	UG-460		Not a spare part
8	UG-461		Not a spare part

AN-100


Suspension1 forearm lamp

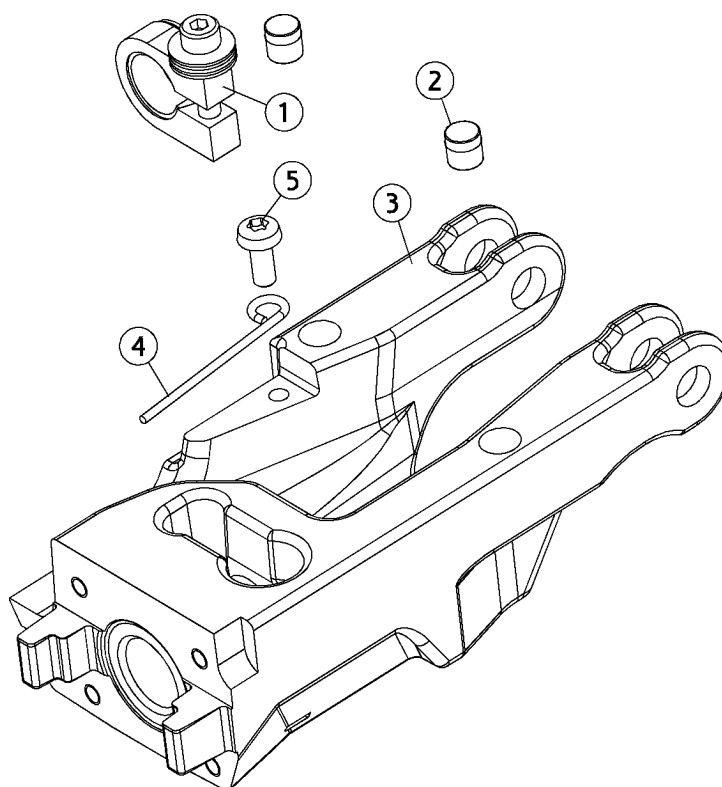

Position	Part NO.	Assembly	Description
1	AN-101	<i>Page 311</i>	Brake forearm lamp
2	AN-127	<i>Page 198</i>	Magnet Ø6,35 with spring ring
3	AO-278		Suspension1, forearm, lamp, (XO-906 XO Titanium)
3	AN-856		Suspension1, forearm, lamp, (XO-903 Silver Metallic)
3	AN-560		Suspension1, forearm, lamp, (XO-901 Sea Blue Metallic)
3	AN-940		Suspension1, forearm, lamp, (XO-905 XO White,(RAL7047-30))
3	AN-559		Suspension1, forearm, lamp, (XO-900 XO Grey, RAL 7047-80)
3	AP-424		Suspension1, forearm, lamp, (CTBS)
4	MG-320		Not a spare part
5	UG-110		Screw, thread making, M4x10

AN-102


Suspension2 forearm lamp

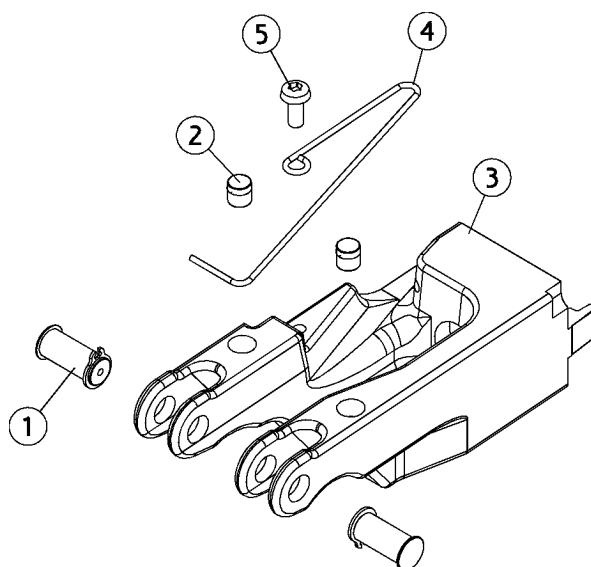

Position	Part NO.	Assembly	Description
1	AN-103	Page 200	Axle 8x17
2	AN-127	Page 198	Magnet Ø6,35 with spring ring
3	AN-941		Suspension2 lamp, (XO-905 XO White,(RAL7047-30))
3	AO-279		Suspension2 lamp, (XO-906 XO Titanium)
3	AP-425		Suspension2 lamp, (CTBS)
3	AN-561		Suspension2 lamp, (XO-900 XO Grey, RAL 7047-80)
3	AN-857		Suspension2 lamp, (XO-903 Silver Metallic)
3	AN-562		Suspension2 lamp, (XO-901 Sea Blue Metallic)
4	MG-513		Not a spare part
5	UG-110		Screw, thread making, M4x10

AN-101


Brake forearm lamp

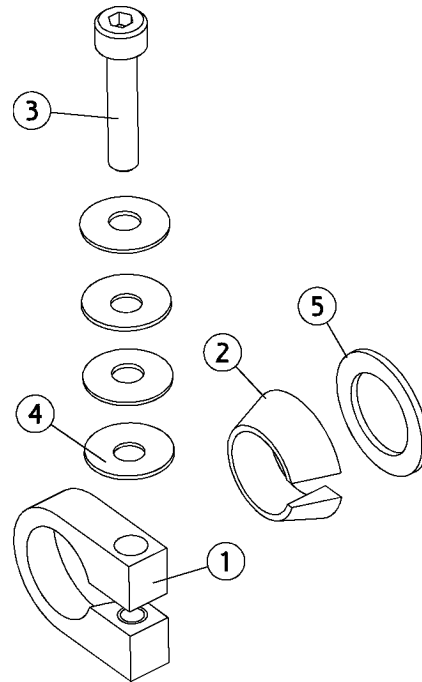

Position	Part NO.	Assembly	Description
1	MG-334		Not a spare part
2	MG-335		Not a spare part
3	UG-012		Allen screw, 6kt M4x20 din 912
4	UG-261		Disc spring, Ø04.2xØ12.0x0.60x1.00
5	UG-265		Adaptor disc Ø12xØ18x1.0 din 988

AN-104


Lever lamp

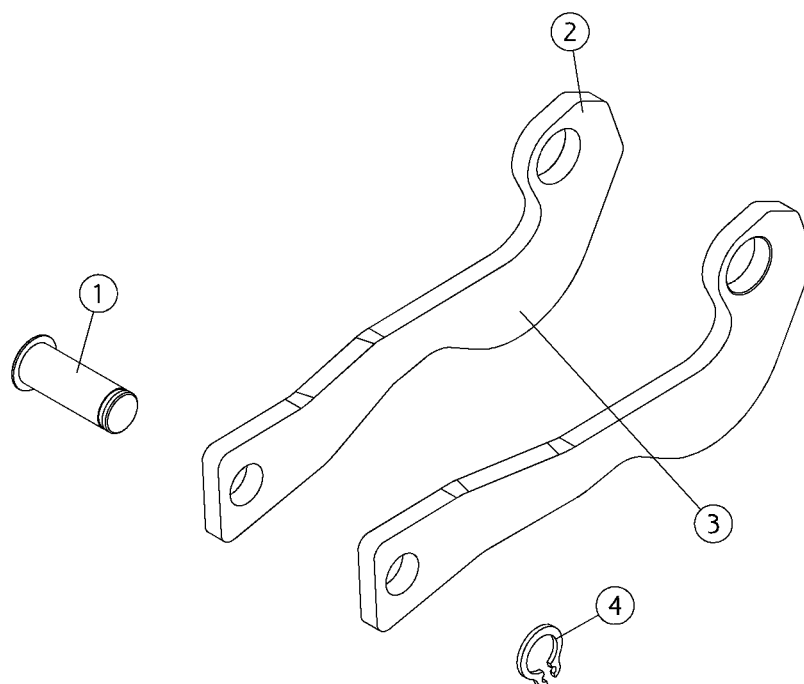

Position	Part NO.	Assembly	Description
1	MG-336		Not a spare part
2	MN-146		Not a spare part
3	MN-147		Not a spare part
4	UG-356		Not a spare part

AN-118


Control socket spring lamp arm

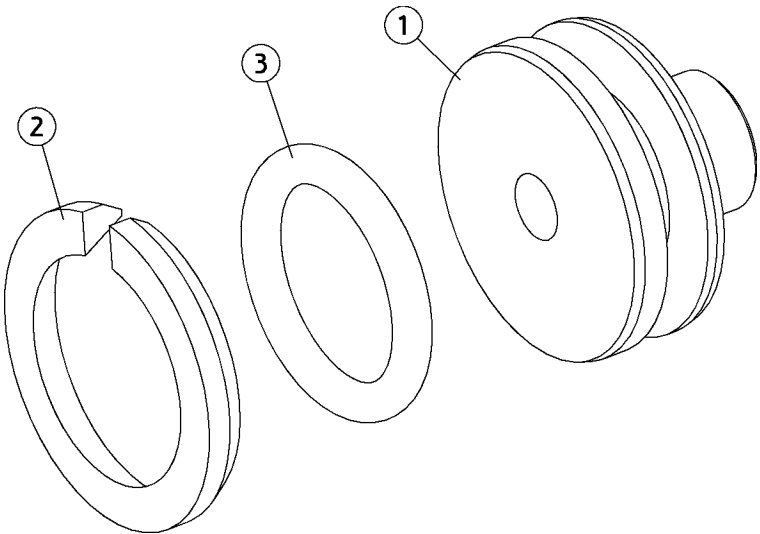

Position	Part NO.	Assembly	Description
1	MG-300		Not a spare part
2	MG-509		Not a spare part
3	UG-322		Not a spare part

AN-149


Lamp head

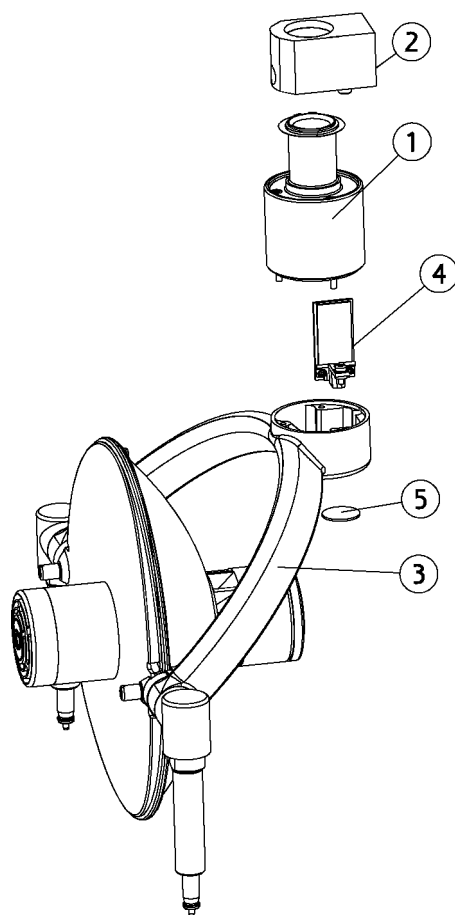

Position	Part NO.	Assembly	Description
1	AN-785	Page 316	Axle lamp, (XO-901 Sea Blue Metallic)
1	AN-784	Page 316	Axle lamp, (XO-900 XO Grey, RAL 7047-80)
1	AN-786	Page 316	Axle lamp, (XO-903 Silver Metallic)
2	AN-128	Page 317	Bearing block lamp
3	AO-216	Page 318	Lamp head,w/o reflector, (XO-906 XO Titanium)
3	AN-292	Page 318	Lamp head,w/o reflector, (XO-900 XO Grey, RAL 7047-80)
3	AN-975	Page 318	Lamp head,w/o reflector, (XO-905 XO White,(RAL7047-30))
3	AN-294	Page 318	Lamp head,w/o reflector, (XO-903 Silver Metallic)
3	AN-293	Page 318	Lamp head,w/o reflector, (XO-901 Sea Blue Metallic)
4	AN-599	Page 335	PCB, Lamp sensor
5	MG-490		Lense, sensor, lamp

AN-141


Suspension lamp, arm

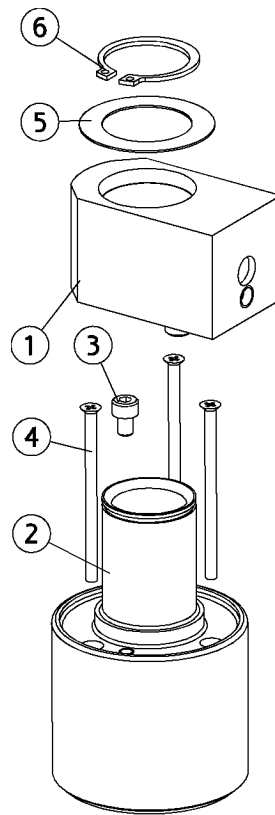

Position	Part NO.	Assembly	Description
1	AN-128	<i>Page 317</i>	Bearing block lamp
2	AN-569		Shaft lamp-link2, (XO-900 XO Grey, RAL 7047-80)
2	AN-858		Shaft lamp-link2, (XO-903 Silver Metallic)
2	AN-570		Shaft lamp-link2, (XO-901 Sea Blue Metallic)
3	UG-014		Not a spare part
4	UG-117		Screw ux M3x50 din965 bn3333
5	UG-270		Not a spare part
6	UG-360		Not a spare part

AN-107


Axle lamp

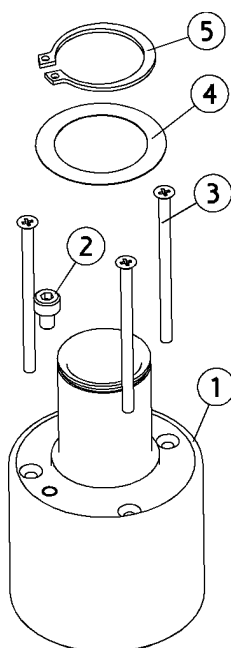


Position	Part NO.	Assembly	Description
1	AN-569		Shaft lamp-link2, (XO-900 XO Grey, RAL 7047-80)
1	AN-858		Shaft lamp-link2, (XO-903 Silver Metallic)
1	AN-570		Shaft lamp-link2, (XO-901 Sea Blue Metallic)
2	UG-014		Not a spare part
3	UG-117		Screw ux M3x50 din965 bn3333
4	UG-267		Not a spare part
5	UG-358		Not a spare part

AN-128

Bearing block lamp

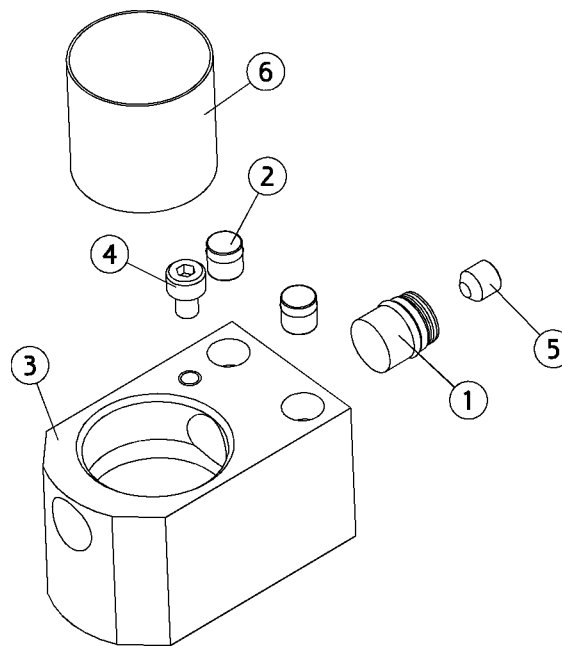

Position	Part NO.	Assembly	Description
1	AN-019	<i>Page 183</i>	Brake block, instrument bridge
2	AN-127	<i>Page 198</i>	Magnet Ø6,35 with spring ring
3	MG-303		Not a spare part
4	UG-014		Not a spare part
5	UG-108		Not a spare part
6	UG-468		Not a spare part

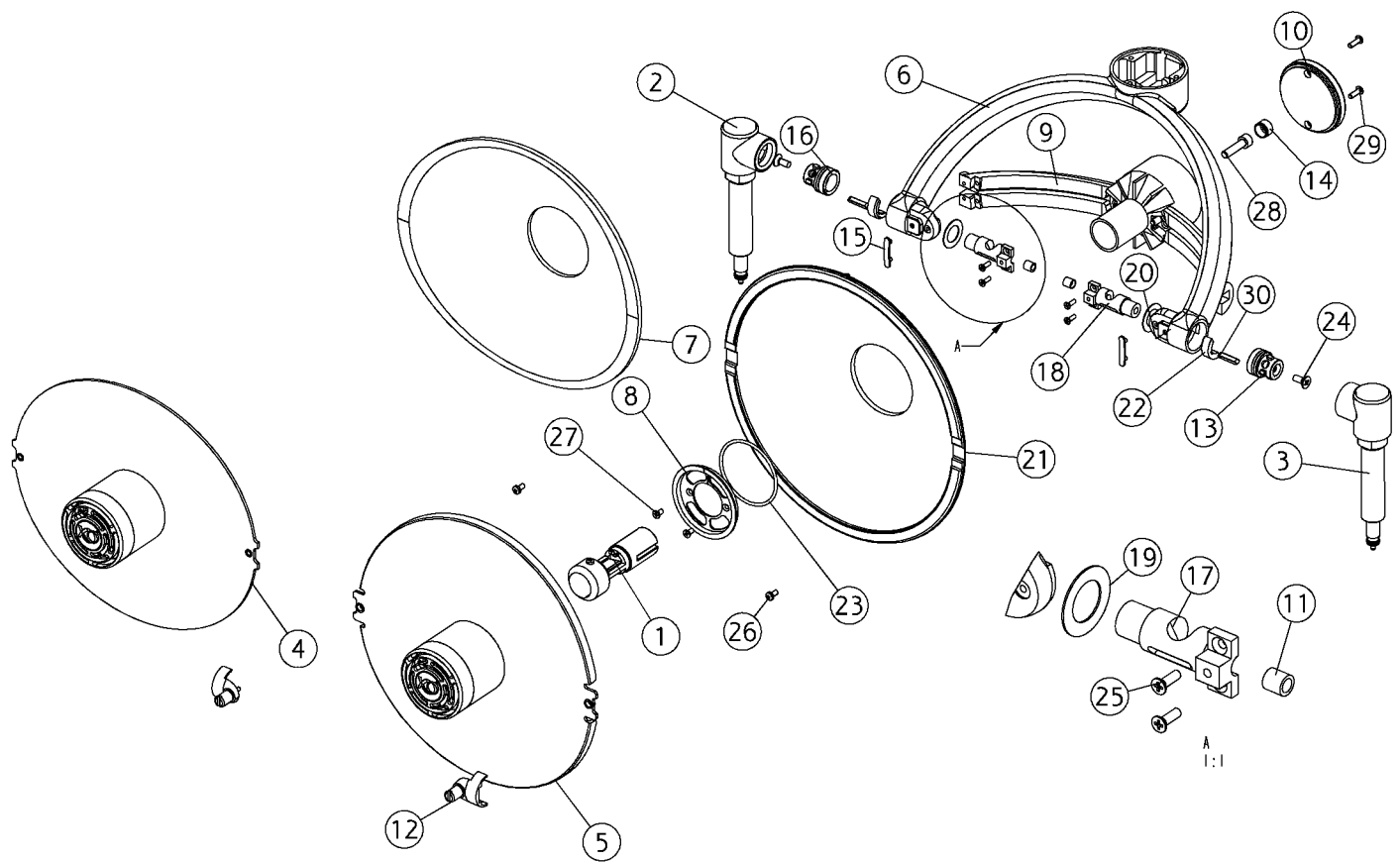

AN-152

Lamp head XO 4, parts

Position	Part NO.	Assembly	Description
1	AN-547	Page 321	Socket, including bulb
2	AO-343	Page 334	Handle, left, (XO-906 XO Titanium)
2	AO-341	Page 334	Handle, left, (XO-905 XO White,(RAL7047-30))
2	AN-859	Page 334	Handle, left, (XO-900 XO Grey, RAL 7047-80)
2	AP-495	Page 334	Handle, left, (CTBS)
2	AN-752	Page 334	Handle, left, (XO-903 Silver Metallic)
2	AN-691	Page 334	Handle, left, (XO-901 Sea Blue Metallic)
3	AO-340	Page 333	Handle, right, (XO-905 XO White,(RAL7047-30))
3	AN-692	Page 333	Handle, right, (XO-901 Sea Blue Metallic)
3	AN-753	Page 333	Handle, right, (XO-903 Silver Metallic)
3	AN-860	Page 333	Handle, right, (XO-900 XO Grey, RAL 7047-80)
3	AO-342	Page 333	Handle, right, (XO-906 XO Titanium)
3	AP-494	Page 333	Handle, right, (CTBS)
4	AN-550		Fan with splash screen, oval
5	AO-034		Splash screen,
6	AN-863		Hoop lamp, (XO-903 Silver Metallic)
6	AN-862		Hoop lamp, (XO-901 Sea Blue Metallic)
6	AN-861		Hoop lamp, (XO-900 XO Grey, RAL 7047-80)
7	MG-181		Reflector, lamp
8	MG-495		End ring
9	MG-496		Not a spare part
10	MG-679		Not a spare part
11	MG-680		Not a spare part
12	MG-681		Screw mount. f. splash screen, lamp
13	MG-682		Reciever, handle, right
14	MG-683		Not a spare part
15	MG-684		Not a spare part
16	MG-685		Reciever, handle, left
17	MG-686		Not a spare part
18	MG-687		Not a spare part
19	MG-703		Not a spare part
20	MG-718		Not a spare part


AN-152

Lamp head XO 4, parts

Position	Part NO.	Assembly	Description
21	AO-472		Kit, reflector led, complete
22	MG-847		Not a spare part
23	MR-210		O-ring 45 x 2
24	UG-046		Screw ux M4x12 din965
25	UG-047		Screw ux M2.5x8 din965
26	UG-048		Not a spare part
27	UG-049		Screw ux M3x6 din965
28	UG-050		Not a spare part
29	UG-051		Not a spare part
30	UG-470		Not a spare part

AN-547


Socket, including bulb

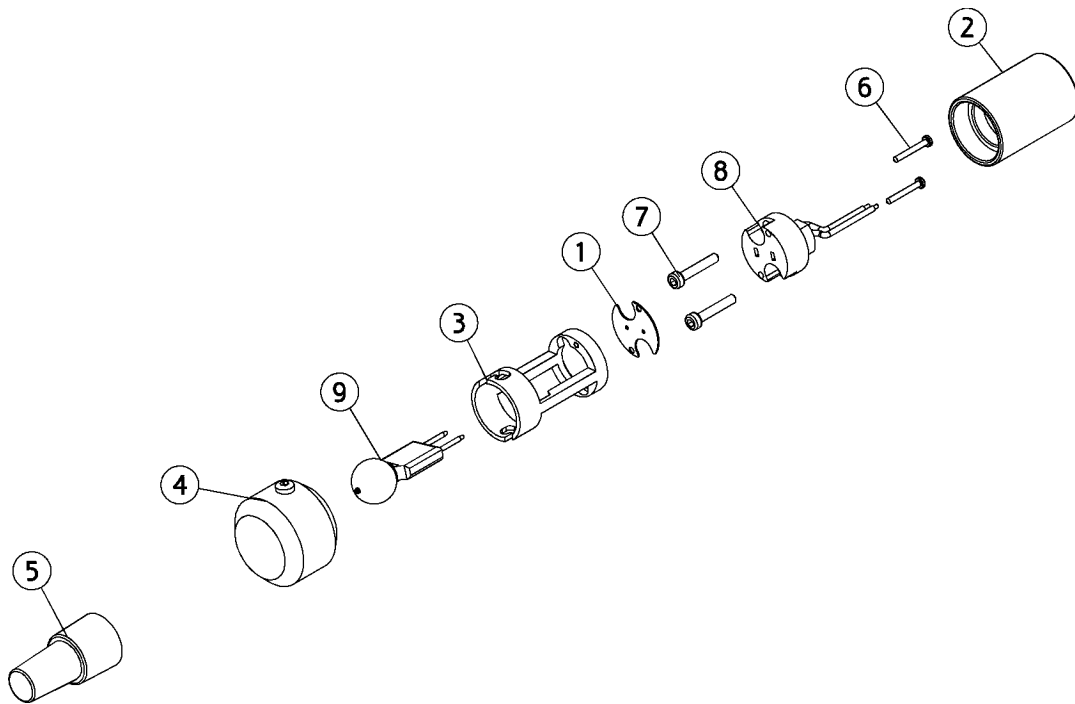

Position	Part NO.	Assembly	Description
1	MG-689		Not a spare part
2	MG-691		Not a spare part
3	MG-692		Not a spare part
4	MG-719		Covering, bulb, lamp
5	MG-721		Tool, bulb, lamp
6	UG-052		Not a spare part
7	UG-053		Not a spare part
8	UG-448		Not a spare part
9	WH-007		Bulb, operatinig lamp

AO-621


Lamp head, XO flex

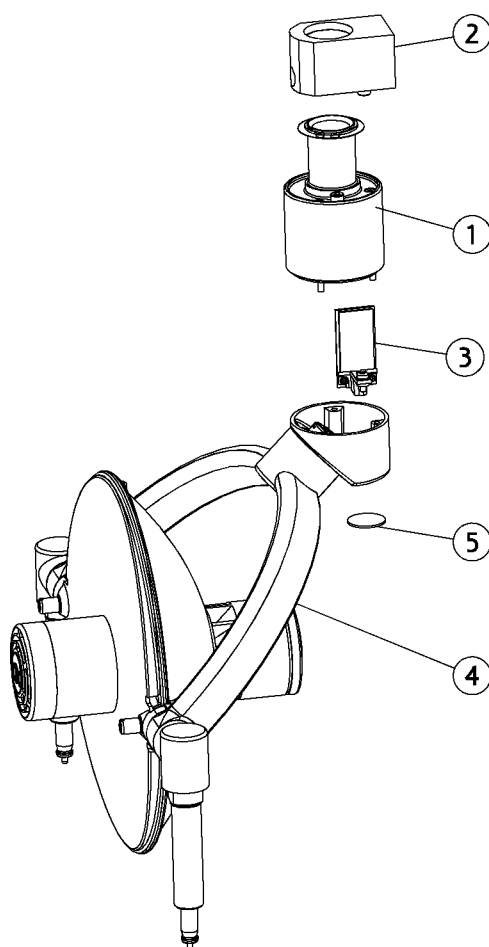

Position	Part NO.	Assembly	Description
1	AN-785	Page 316	Axle lamp, (XO-901 Sea Blue Metallic)
1	AN-784	Page 316	Axle lamp, (XO-900 XO Grey, RAL 7047-80)
1	AN-786	Page 316	Axle lamp, (XO-903 Silver Metallic)
2	AN-128	Page 317	Bearing block lamp
3	AN-599	Page 335	PCB, Lamp sensor
4	AO-622	Page 323	Lamp head, XO flex (Unknown colour)
5	MG-490		Lense, sensor, lamp

AO-622

Lamp head, XO flex

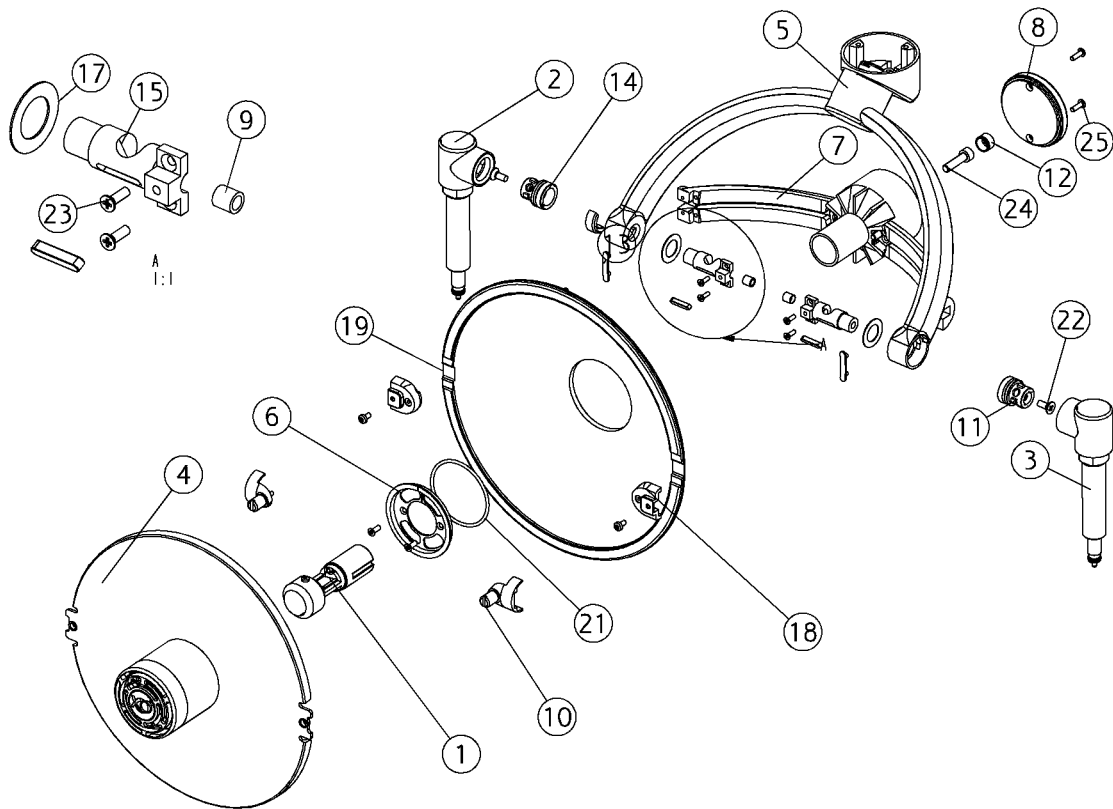

AO-622


Lamp head, XO flex

Position	Part NO.	Assembly	Description
1	AN-547	<i>Page 321</i>	Socket, including bulb
2	AO-343	<i>Page 334</i>	Handle, left, (XO-906 XO Titanium)
2	AO-341	<i>Page 334</i>	Handle, left, (XO-905 XO White,(RAL7047-30))
2	AN-859	<i>Page 334</i>	Handle, left, (XO-900 XO Grey, RAL 7047-80)
2	AP-495	<i>Page 334</i>	Handle, left, (CTBS)
2	AN-752	<i>Page 334</i>	Handle, left, (XO-903 Silver Metallic)
2	AN-691	<i>Page 334</i>	Handle, left, (XO-901 Sea Blue Metallic)
3	AO-340	<i>Page 333</i>	Handle, right, (XO-905 XO White,(RAL7047-30))
3	AN-692	<i>Page 333</i>	Handle, right, (XO-901 Sea Blue Metallic)
3	AN-753	<i>Page 333</i>	Handle, right, (XO-903 Silver Metallic)
3	AN-860	<i>Page 333</i>	Handle, right, (XO-900 XO Grey, RAL 7047-80)
3	AO-342	<i>Page 333</i>	Handle, right, (XO-906 XO Titanium)
3	AP-494	<i>Page 333</i>	Handle, right, (CTBS)
4	AO-034		Splash screen,
5	AO-623	<i>Page 326</i>	Swivel lamp 3 axis
6	MG-495		End ring
7	MG-496		Not a spare part
8	MG-679		Not a spare part
9	MG-680		Not a spare part
10	MG-681		Screw mount. f. splash screen, lamp
11	MG-682		Reciever, handle, right
12	MG-683		Not a spare part
13	MG-684		Not a spare part
14	MG-685		Reciever, handle, left
15	MG-686		Not a spare part
16	MG-687		Not a spare part
17	MG-703		Not a spare part
18	MG-718		Not a spare part
19	AO-472		Kit, reflector led, complete
20	MG-847		Not a spare part
21	MR-210		O-ring 45 x 2
22	UG-046		Screw ux M4x12 din965

AO-622

Lamp head, XO flex


Position	Part NO.	Assembly	Description
23	UG-047		Screw ux M2.5x8 din965
24	UG-050		Not a spare part
25	UG-051		Not a spare part
26	UG-470		Not a spare part
27	UH-081		Not a spare part
28	UH-083		Not a spare part

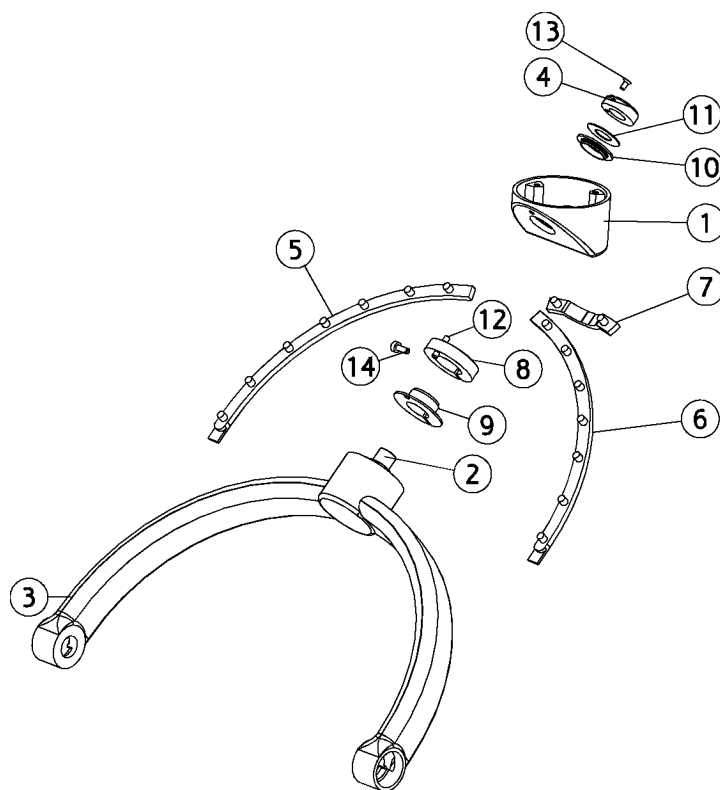


Position	Part NO.	Assembly	Description
1	MG-950		Not a spare part
2	MG-951		Not a spare part
3	MG-952		Not a spare part
4	MG-953		Not a spare part
5	MG-954		Not a spare part
6	MG-955		Not a spare part
7	MG-956		Not a spare part
8	MG-958		Not a spare part
9	MG-959		Bearing, op lamp
10	UH-058		Flange, bearing Ø14/Ø16x3,XO 4
11	UH-059		Washer din988 Ø10xØ20x0,5
12	UH-060		Not a spare part
13	UH-061		Not a spare part
14	UH-062		Not a spare part

AO-636

XO daylight led, lamp head

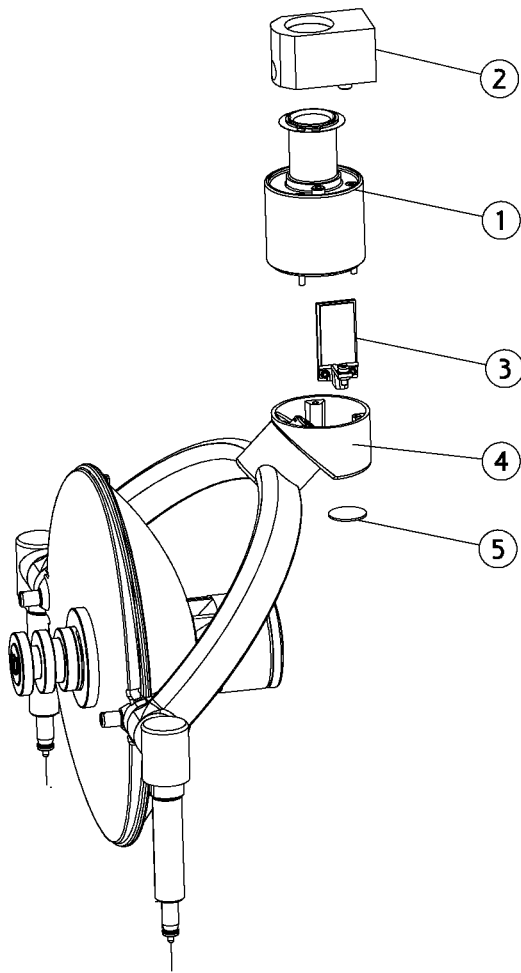

Position	Part NO.	Assembly	Description
1	AN-785	Page 316	Axle lamp, (XO-901 Sea Blue Metallic)
1	AN-784	Page 316	Axle lamp, (XO-900 XO Grey, RAL 7047-80)
1	AN-786	Page 316	Axle lamp, (XO-903 Silver Metallic)
2	AN-128	Page 317	Bearing block lamp
3	AN-599	Page 335	PCB, Lamp sensor
4	AO-637	Page 328	XO daylight led, lamp head
5	MG-490		Lense, sensor, lamp

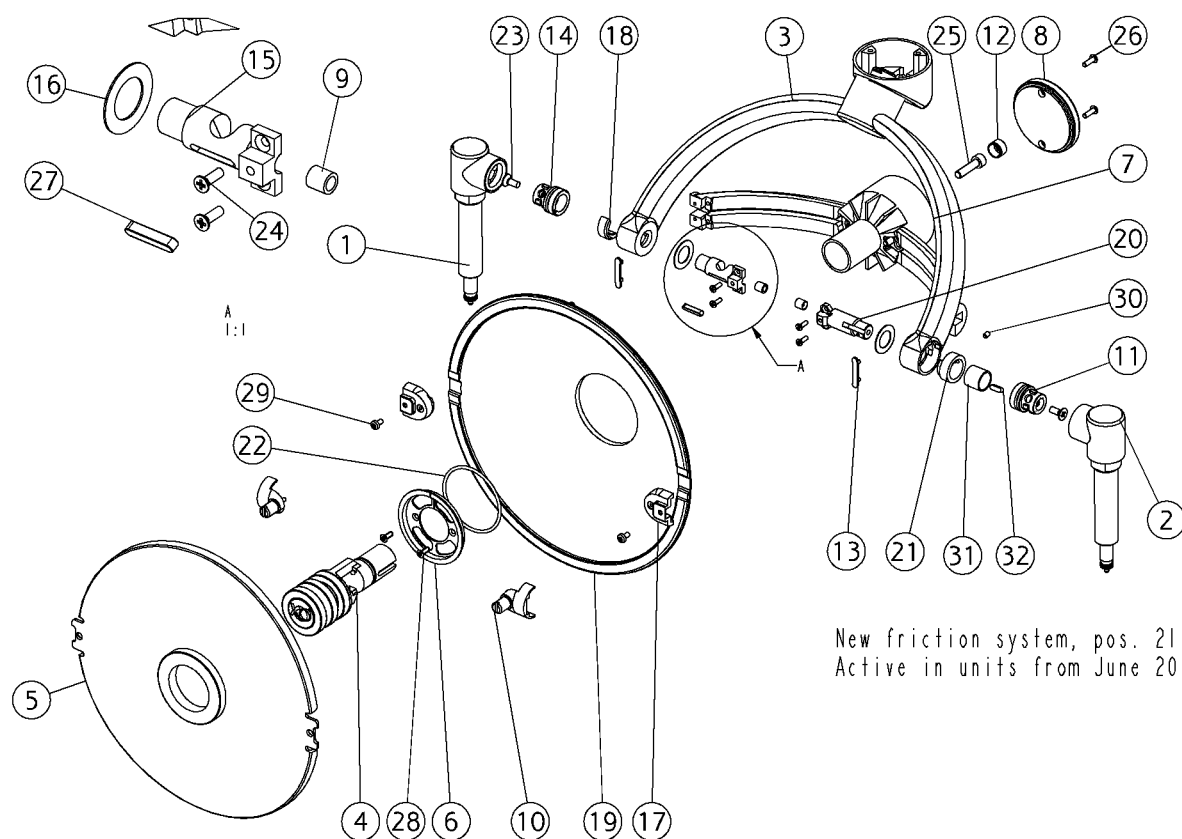

AO-637

XO daylight led, lamp head

Position	Part NO.	Assembly	Description
1	AO-343	Page 334	Handle, left, (XO-906 XO Titanium)
1	AO-341	Page 334	Handle, left, (XO-905 XO White,(RAL7047-30))
1	AN-859	Page 334	Handle, left, (XO-900 XO Grey, RAL 7047-80)
1	AP-495	Page 334	Handle, left, (CTBS)
1	AN-752	Page 334	Handle, left, (XO-903 Silver Metallic)
1	AN-691	Page 334	Handle, left, (XO-901 Sea Blue Metallic)
2	AO-340	Page 333	Handle, right, (XO-905 XO White,(RAL7047-30))
2	AN-692	Page 333	Handle, right, (XO-901 Sea Blue Metallic)
2	AN-753	Page 333	Handle, right, (XO-903 Silver Metallic)
2	AN-860	Page 333	Handle, right, (XO-900 XO Grey, RAL 7047-80)
2	AO-342	Page 333	Handle, right, (XO-906 XO Titanium)
2	AP-494	Page 333	Handle, right, (CTBS)
3	AO-623	Page 326	Swivel lamp 3 axis
4	AO-638	Page 331	Led light f. XO daylight
5	AO-639		Splash screen, daylight led
6	MG-495		End ring
7	MG-496		Not a spare part
8	MG-679		Not a spare part
9	MG-680		Not a spare part
10	MG-681		Screw mount. f. splash screen, lamp
11	MG-682		Reciever, handle, right
12	MG-683		Not a spare part
13	MG-684		Not a spare part
14	MG-685		Reciever, handle, left
15	MG-686		Not a spare part
16	MG-703		Not a spare part
17	MG-718		Not a spare part
18	MG-847		Not a spare part
19	AO-472		Kit, reflector led, complete
20	MN-466		Not a spare part
21	MN-467		Not a spare part
22	MR-210		O-ring 45 x 2


AO-637

XO daylight led, lamp head

Position	Part NO.	Assembly	Description
23	UG-046		Screw ux M4x12 din965
24	UG-047		Screw ux M2.5x8 din965
25	UG-050		Not a spare part
26	UG-051		Not a spare part
27	UG-470		Not a spare part
28	UH-081		Not a spare part
29	UH-083		Not a spare part
30	UH-294		Not a spare part
31	UH-295		Not a spare part
32	UH-296		Not a spare part

AO-638


Led light f. XO daylight

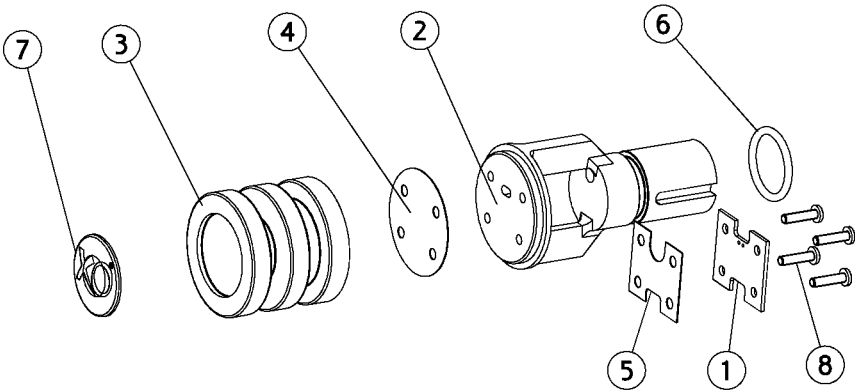

Position	Part NO.	Assembly	Description
1	AD-665		Not a spare part
2	MG-970		Not a spare part
3	MG-971		Not a spare part
4	MG-975		Not a spare part
5	MG-976		Not a spare part
6	MR-228		O-ring Ø16x2
7	UG-442		XO logo, Ø22
8	UH-084		Not a spare part

AO-639


Splash screen, daylight led

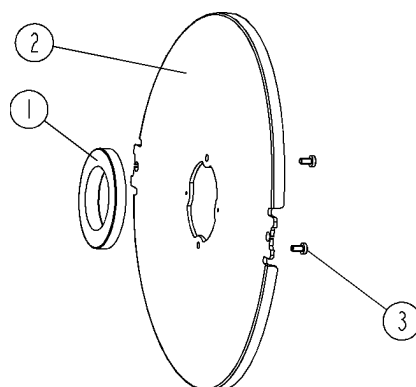

Position	Part NO.	Assembly	Description
1	MG-972		Not a spare part
2	MG-977		Splash screen
3	UH-083		Not a spare part

AN-549


Handles right

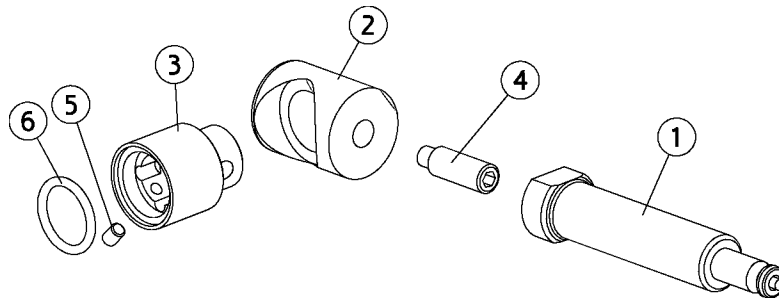

Position	Part NO.	Assembly	Description
1	AN-084	<i>Page 158</i>	Receiver, handles
2	AN-874		Angle handle, (XO-901 Sea Blue Metallic)
2	AN-875		Angle handle, (XO-903 Silver Metallic)
2	AN-873		Angle handle, (XO-900 XO Grey, RAL 7047-80)
3	AN-878		Bush handle right, (XO-901 Sea Blue Metallic)
3	AN-877		Bush handle right, (XO-900 XO Grey, RAL 7047-80)
3	AN-879		Bush handle right, (XO-903 Silver Metallic)
4	UG-125		Not a spare part
5	UG-134		Hex screw M4 x 6
6	UG-333		O-ring Ø16.6x2.4,lamp

AN-548


Handles left

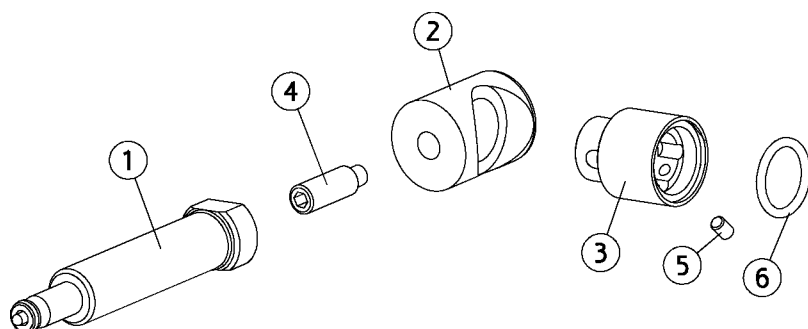

Position	Part NO.	Assembly	Description
1	AN-084	<i>Page 158</i>	Receiver, handles
2	AN-874		Angle handle, (XO-901 Sea Blue Metallic)
2	AN-875		Angle handle, (XO-903 Silver Metallic)
2	AN-873		Angle handle, (XO-900 XO Grey, RAL 7047-80)
3	AN-883		Bush handle left, (XO-903 Silver Metallic)
3	AN-881		Bush handle left, (XO-900 XO Grey, RAL 7047-80)
3	AN-882		Bush handle left, (XO-901 Sea Blue Metallic)
4	UG-125		Not a spare part
5	UG-134		Hex screw M4 x 6
6	UG-333		O-ring Ø16.6x2.4,lamp


AN-599

PCB, Lamp sensor

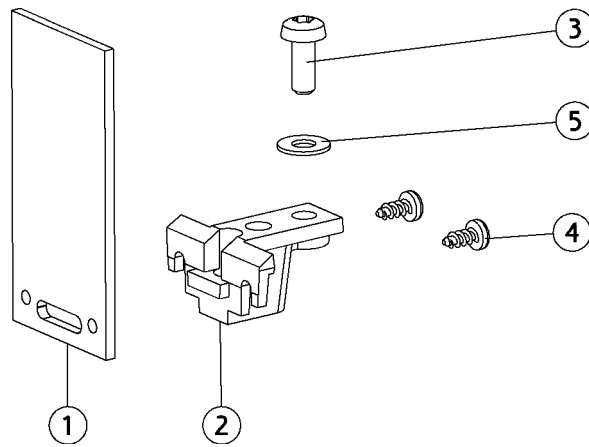

Position	Part NO.	Assembly	Description
1	AD-615		Not a spare part
2	MG-442		House for led
3	UG-139		Screw. M3x8
4	UG-241		Not a spare part
5	UG-282		Washer, Ø3


AP-897 Suspension, Joint, screen, lamp

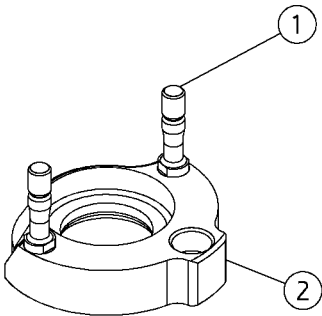

Position	Part NO.	Assembly	Description
1	MN-144		Not a spare part
2	MN-465		Not a spare part

AP-898

Suspension, Joint, screen, lamp, complete

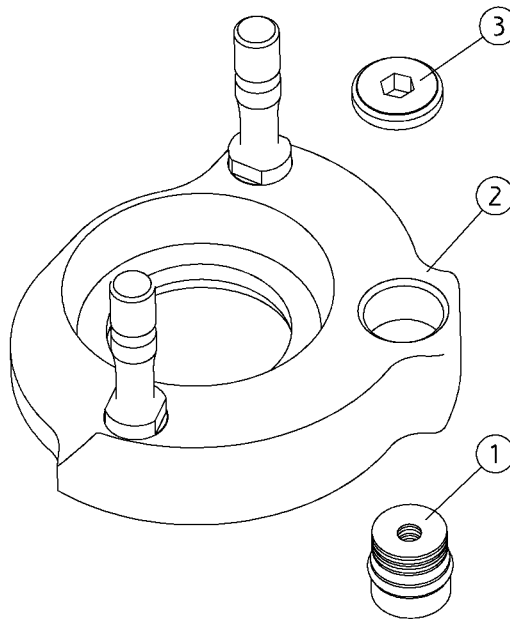

Position	Part NO.	Assembly	Description
1	AN-019	Page 183	Brake block, instrument bridge
2	AP-897		Suspension, Joint, screen, lamp
3	MG-350		Not a spare part


AP-898

Suspension, Joint, screen, lamp, complete

XO 4 Screen and Arm system

AN-284


Arm w. screen

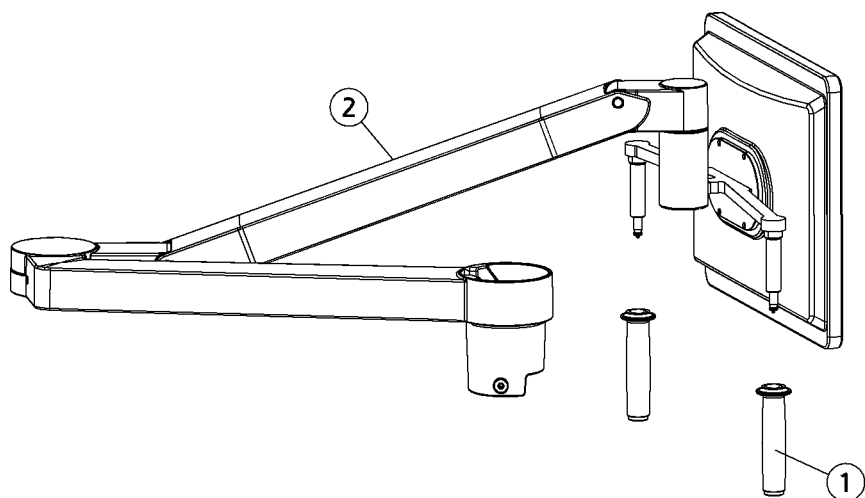

Position	Part NO.	Assembly	Description
1	AN-110		Handle XO 4
2	AN-763	<i>Page 341</i>	Arm, tft, w/o monitor, (XO-903 Silver Metallic)
2	AO-011	<i>Page 341</i>	Arm, tft, w/o monitor, (XO-905 XO White,(RAL7047-30))
2	AO-231	<i>Page 341</i>	Arm, tft, w/o monitor, (XO-906 XO Titanium)
2	AN-357	<i>Page 341</i>	Arm, tft, w/o monitor, (XO-901 Sea Blue Metallic)
2	AN-144	<i>Page 341</i>	Arm, tft, w/o monitor, (XO-900 XO Grey, RAL 7047-80)

AN-285


Arm w.screen w/o handle

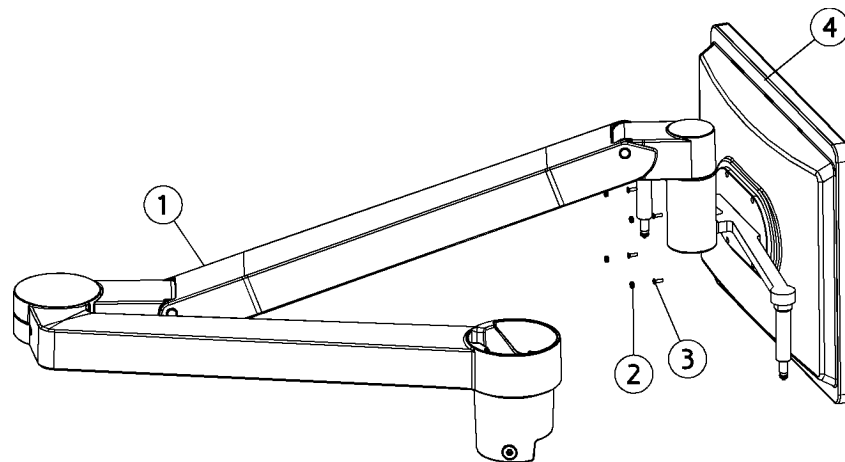

Position	Part NO.	Assembly	Description
1	AN-286	Page 342	Arm w/o screen (Unknown colour)
2	MG-640		Not a spare part
3	UG-114		Screw threadforming, M3x10
4	UG-913		XO screen 15 inch tft, unit

AN-286


Arm w/o screen

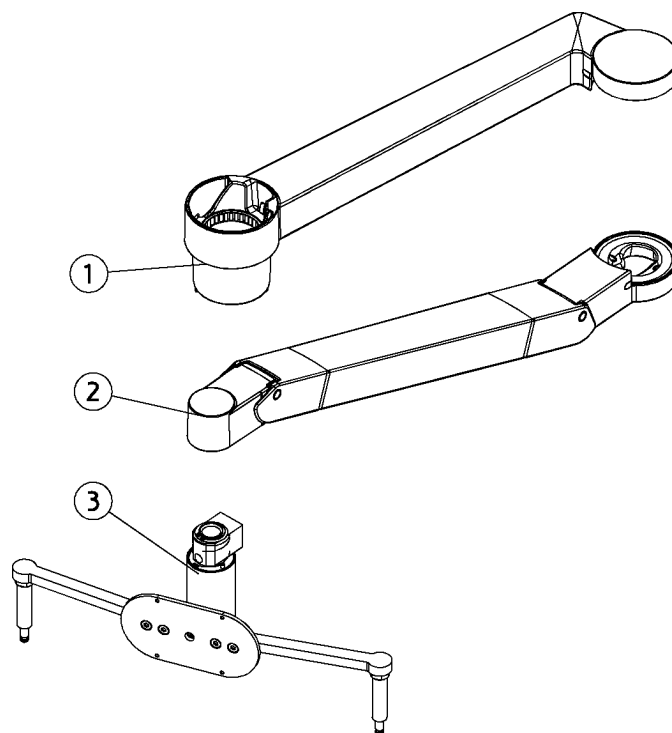

Position	Part NO.	Assembly	Description
1	AN-886	<i>Page 343</i>	Rear arm screen, (XO-903 Silver Metallic)
1	AN-595	<i>Page 343</i>	Rear arm screen, (XO-900 XO Grey, RAL 7047-80)
1	AN-596	<i>Page 343</i>	Rear arm screen, (XO-901 Sea Blue Metallic)
2	AN-889	<i>Page 345</i>	Forearm screen, (XO-903 Silver Metallic)
2	AN-887	<i>Page 345</i>	Forearm screen, (XO-900 XO Grey, RAL 7047-80)
2	AN-888	<i>Page 345</i>	Forearm screen, (XO-901 Sea Blue Metallic)
3	AN-890	<i>Page 349</i>	Suspension screen, (XO-900 XO Grey, RAL 7047-80)
3	AN-892	<i>Page 349</i>	Suspension screen, (XO-903 Silver Metallic)
3	AN-891	<i>Page 349</i>	Suspension screen, (XO-901 Sea Blue Metallic)

AN-122

Rear arm screen

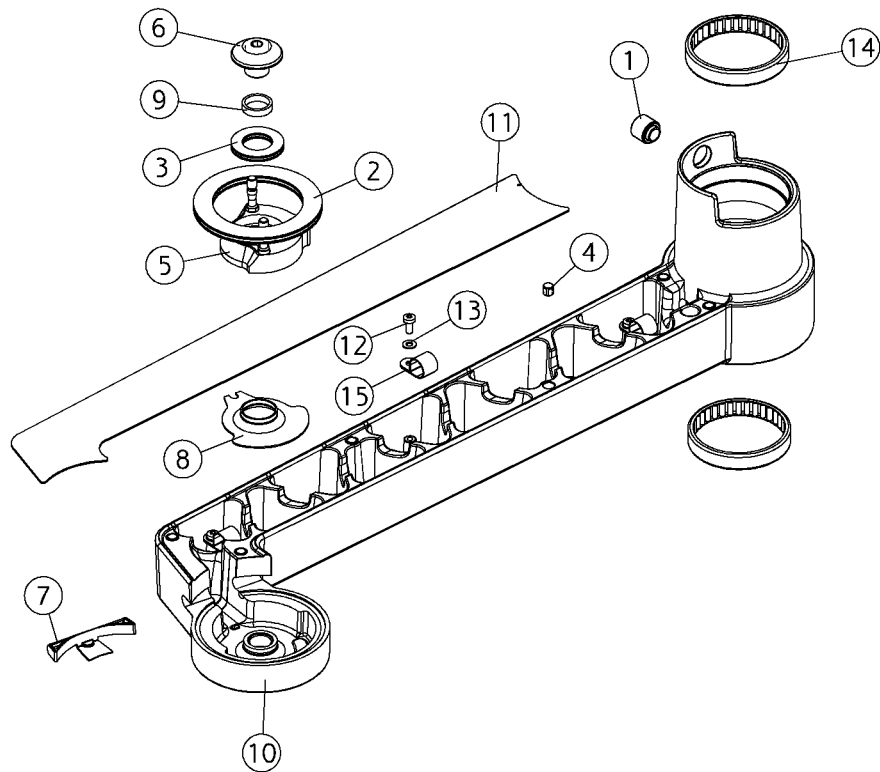


AN-122

Rear arm screen

Position	Part NO.	Assembly	Description
1	AN-018	<i>Page 182</i>	Friction brake, arms
2	AN-094	<i>Page 203</i>	Axle bearing 60x85x5
3	AN-095	<i>Page 201</i>	Axle bearing 20x35x4
4	AN-127	<i>Page 198</i>	Magnet Ø6,35 with spring ring
5	AP-898	<i>Page 337</i>	Suspension, Joint, screen, lamp, complete
6	MG-290		Not a spare part
7	MG-291		Coverplug rear arm lamp/screen
8	MG-328		Not a spare part
9	MG-329		Not a spare part
10	AN-572		Rear arm, screen, (XO-901 Sea Blue Metallic)
10	AN-571		Rear arm, screen, (XO-900 XO Grey, RAL 7047-80)
10	AN-893		Rear arm, screen, (XO-903 Silver Metallic)
10	AN-957		Rear arm, screen, (XO-905 XO White,(RAL7047-30))
10	AO-292		Rear arm, screen, (XO-906 XO Titanium)
10	AP-438		Rear arm, screen, (CTBS)
11	AP-441		Cover plate, rear arm, screen, (CTBS)
11	AO-295		Cover plate, rear arm, screen, (XO-906 XO Titanium)
11	AN-576		Cover plate, rear arm, screen, (XO-901 Sea Blue Metallic)
11	AN-961		Cover plate, rear arm, screen, (XO-905 XO White,(RAL7047-30))
11	AN-575		Cover plate, rear arm, screen, (XO-900 XO Grey, RAL 7047-80)
11	AN-894		Cover plate, rear arm, screen, (XO-903 Silver Metallic)
12	UG-110		Screw, thread making, M4x10
13	UG-254		Disc, Ø04.3xØ09.0x0.8
14	UG-458		Not a spare part
15	UG-527		Not a spare part

AN-123

Forearm screen

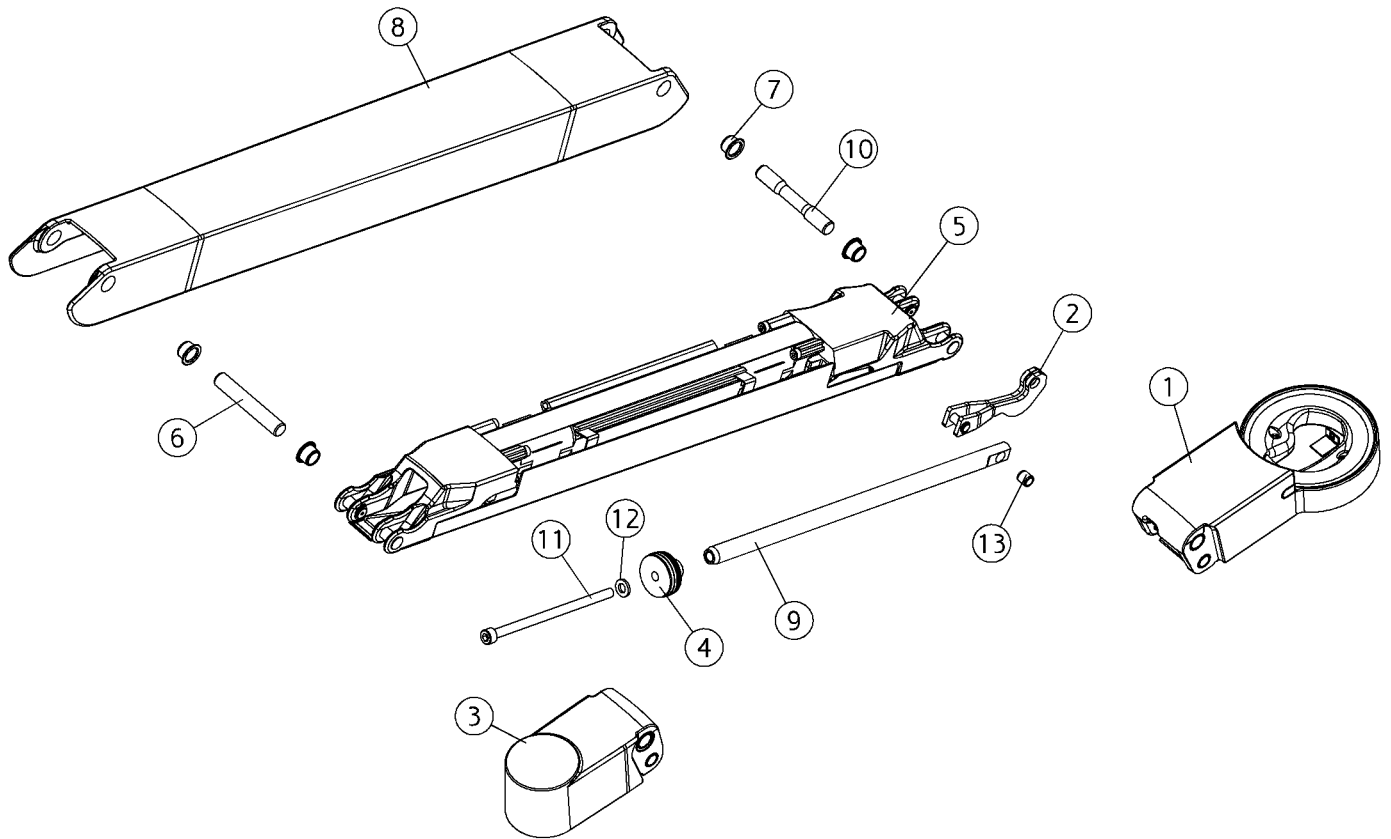


AN-123

Forearm screen

Position	Part NO.	Assembly	Description
1	AN-098	<i>Page 303</i>	Link1, forearm lamp
2	AN-104	<i>Page 312</i>	Lever lamp
3	AN-105	<i>Page 307</i>	Link2, forearm lamp
4	AN-118	<i>Page 313</i>	Control socket spring lamp arm
5	AN-896	<i>Page 347</i>	Bottom forearm screen, (XO-901 Sea Blue Metallic)
5	AN-897	<i>Page 347</i>	Bottom forearm screen, (XO-903 Silver Metallic)
5	AN-895	<i>Page 347</i>	Bottom forearm screen, (XO-900 XO Grey, RAL 7047-80)
6	MG-298		Axle Ø10x65
7	MG-299		Bushing Ø10x8 for u-profile
8	AN-287		Top forearm, screen, (XO-903 Silver Metallic)
8	AN-958		Top forearm, screen, (XO-905 XO White,(RAL7047-30))
8	AN-574		Top forearm, screen, (XO-901 Sea Blue Metallic)
8	AO-293		Top forearm, screen, (XO-906 XO Titanium)
8	AN-573		Top forearm, screen, (XO-900 XO Grey, RAL 7047-80)
8	AP-439		Top forearm, screen, (CTBS)
9	MG-522		Not a spare part
10	MN-145		Not a spare part
11	UG-031		Not a spare part
12	UG-256		Disc, Ø06.4xØ12.0x1.6
13	UG-459		Not a spare part

AN-124

Bottom forearm screen

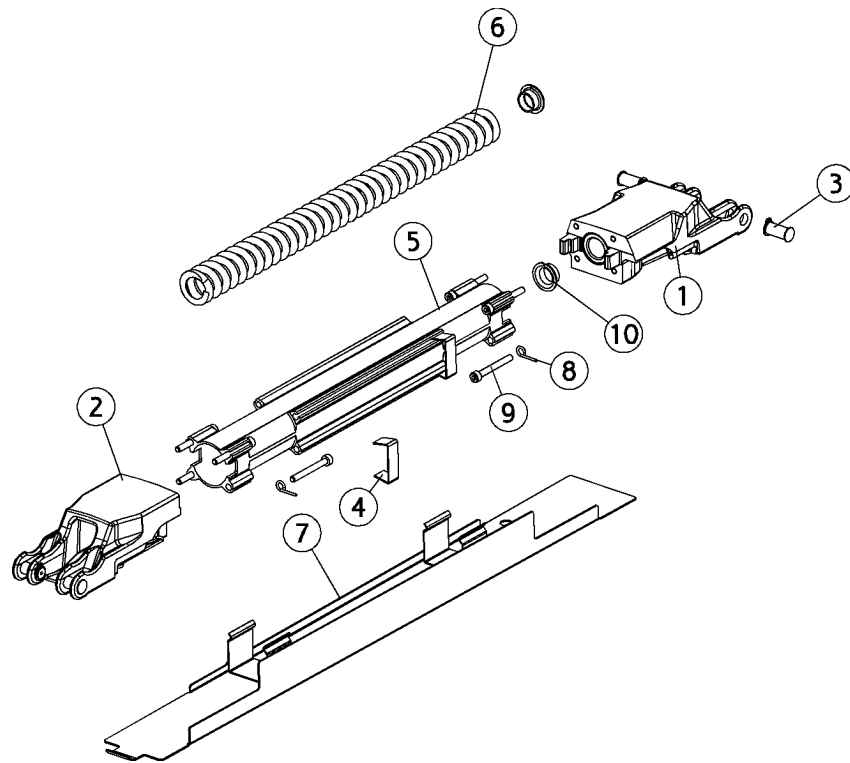


AN-124

Bottom forearm screen

Position	Part NO.	Assembly	Description
1	AN-590	<i>Page 309</i>	Suspension1 lamp, (XO-901 Sea Blue Metallic)
1	AN-589	<i>Page 309</i>	Suspension1 lamp, (XO-900 XO Grey, RAL 7047-80)
1	AN-898	<i>Page 309</i>	Suspension1 lamp, (XO-903 Silver Metallic)
2	AN-591	<i>Page 310</i>	Suspension2 lamp, (XO-900 XO Grey, RAL 7047-80)
2	AN-592	<i>Page 310</i>	Suspension2 lamp, (XO-901 Sea Blue Metallic)
2	AN-899	<i>Page 310</i>	Suspension2 lamp, (XO-903 Silver Metallic)
3	AN-103	<i>Page 200</i>	Axle 8x17
4	MG-464		Not a spare part
5	MG-467		Not a spare part
6	MG-521		Not a spare part
7	AN-546		Cover plate, forearm, screen, (XO-901 Sea Blue Metallic)
7	AO-294		Cover plate, forearm, screen, (XO-906 XO Titanium)
7	AN-959		Cover plate, forearm, screen, (XO-905 XO White,(RAL7047-30))
7	AN-545		Cover plate, forearm, screen, (XO-900 XO Grey, RAL 7047-80)
7	AP-440		Cover plate, forearm, screen, (CTBS)
7	AN-272		Cover plate, forearm, screen, (XO-903 Silver Metallic)
8	MG-669		Not a spare part
9	UG-013		Not a spare part
10	UG-463		Not a spare part

AN-126


Suspension, screen

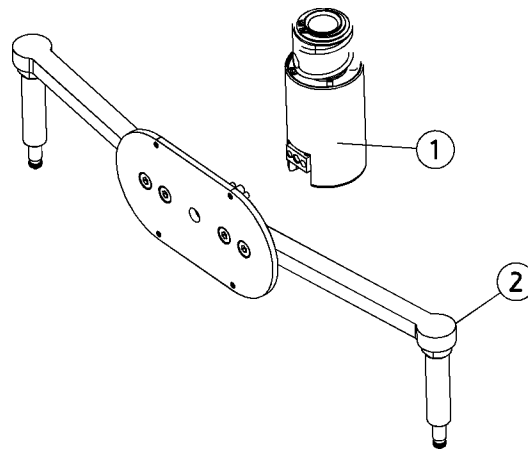

Position	Part NO.	Assembly	Description
1	AN-274	<i>Page 352</i>	Housing link3 screen, (XO-901 Sea Blue Metallic)
1	AN-275	<i>Page 352</i>	Housing link3 screen, (XO-903 Silver Metallic)
1	AN-273	<i>Page 352</i>	Housing link3 screen, (XO-900 XO Grey, RAL 7047-80)
2	AN-276	<i>Page 350</i>	Arm operation screen, (XO-900 XO Grey, RAL 7047-80)
2	AN-277	<i>Page 350</i>	Arm operation screen, (XO-901 Sea Blue Metallic)
2	AN-278	<i>Page 350</i>	Arm operation screen, (XO-903 Silver Metallic)


AN-134

Arm control screen

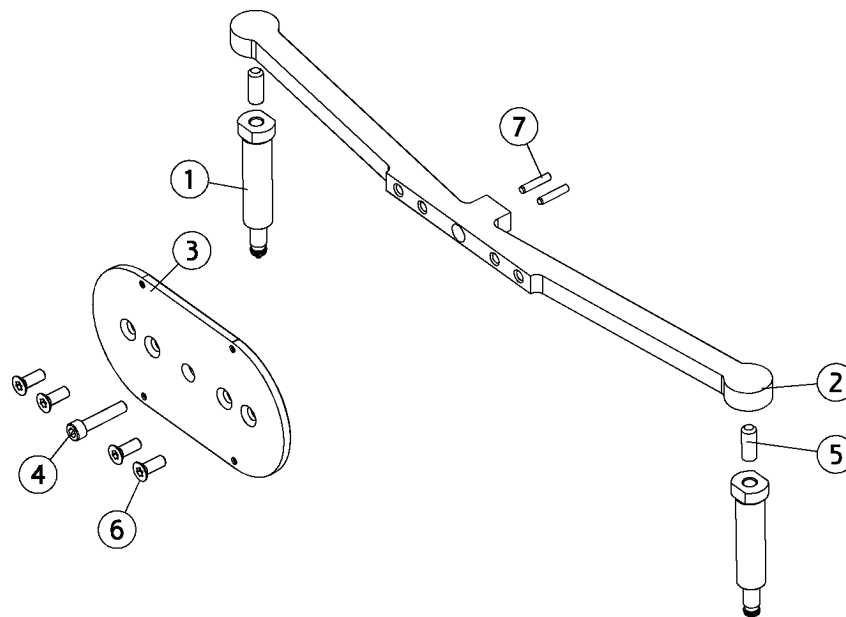

Position	Part NO.	Assembly	Description
1	AN-084	<i>Page 158</i>	Receiver, handles
2	AN-582		Arm f. handles, screen, (XO-901 Sea Blue Metallic)
2	AO-297		Arm f. handles, screen, (XO-906 XO Titanium)
2	AN-279		Arm f. handles, screen, (XO-903 Silver Metallic)
2	AP-443		Arm f. handles, screen, (CTBS)
2	AN-963		Arm f. handles, screen, (XO-905 XO White,(RAL7047-30))
2	AN-581		Arm f. handles, screen, (XO-900 XO Grey, RAL 7047-80)
3	AN-280		Adapter plate, screen, (XO-903 Silver Metallic)
3	AO-298		Adapter plate, screen, (XO-906 XO Titanium)
3	AN-584		Adapter plate, screen, (XO-901 Sea Blue Metallic)
3	AN-583		Adapter plate, screen, (XO-900 XO Grey, RAL 7047-80)
3	AN-965		Adapter plate, screen, (XO-905 XO White,(RAL7047-30))
4	UG-006		Not a spare part
5	UG-144		Not a spare part
6	UG-147		Not a spare part
7	UG-419		Not a spare part


AN-129 Axle screen

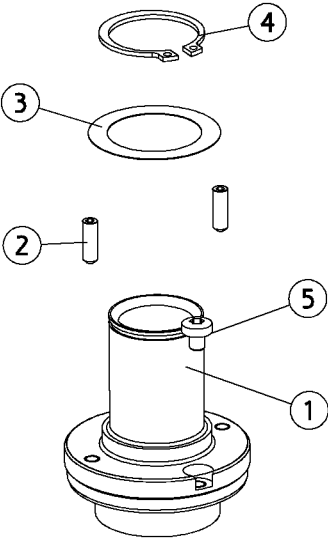

Position	Part NO.	Assembly	Description
1	MG-532		Not a spare part
2	UG-124		Pointed screw M4 x 12
3	UG-267		Not a spare part
4	UG-358		Not a spare part
5	UH-151		Not a spare part


AN-131

Housing link3 screen

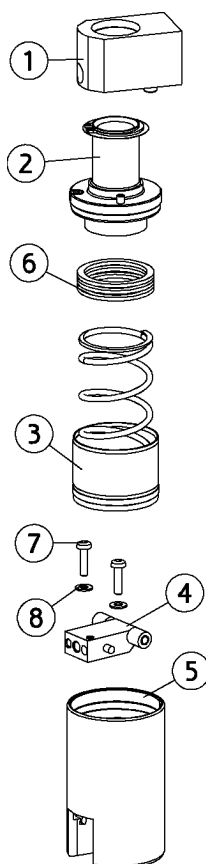

Position	Part NO.	Assembly	Description
1	AN-128	<i>Page 317</i>	Bearing block lamp
2	AN-129	<i>Page 351</i>	Axle screen
3	AN-132	<i>Page 353</i>	Piston spring screen
4	AN-133	<i>Page 354</i>	Lever arm, screen
5	AO-296		Housing link3 screen, (XO-906 XO Titanium)
5	AN-577		Housing link3 screen, (XO-900 XO Grey, RAL 7047-80)
5	AN-281		Housing link3 screen, (XO-903 Silver Metallic)
5	AP-442		Housing link3 screen, (CTBS)
5	AN-962		Housing link3 screen, (XO-905 XO White,(RAL7047-30))
5	AN-578		Housing link3 screen, (XO-901 Sea Blue Metallic)
6	MG-543		Not a spare part
7	UG-103		Screw, thread making, M4x16
8	UG-254		Disc, Ø04.3xØ09.0x0.8


AN-132 Piston spring screen

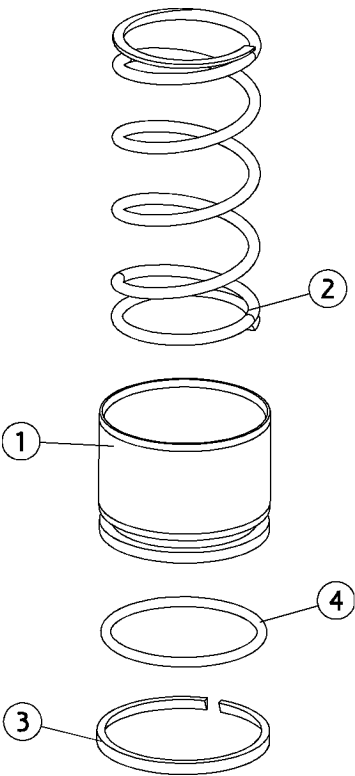

Position	Part NO.	Assembly	Description
1	MG-529		Not a spare part
2	MG-530		Not a spare part
3	MG-531		Not a spare part
4	UG-324		Not a spare part

AN-133


Lever arm, screen

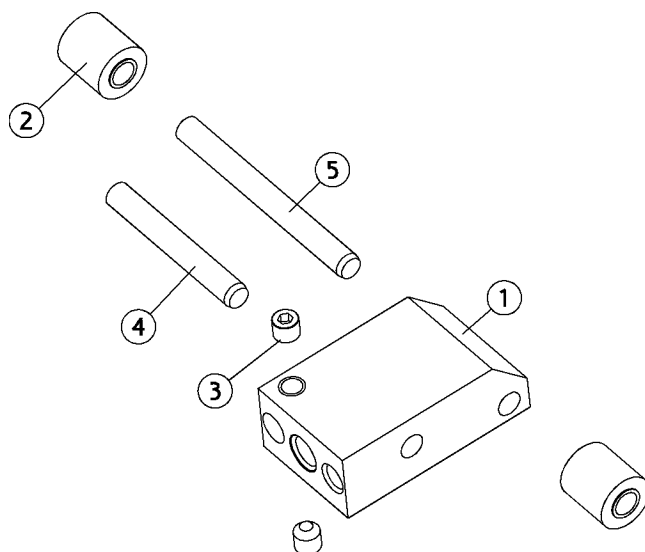

Position	Part NO.	Assembly	Description
1	MG-527		Not a spare part
2	MG-528		Not a spare part
3	UG-126		Pointed screw, M4x4
4	UG-434		Not a spare part
5	UG-435		Not a spare part

AO-037


Arm for screen-1

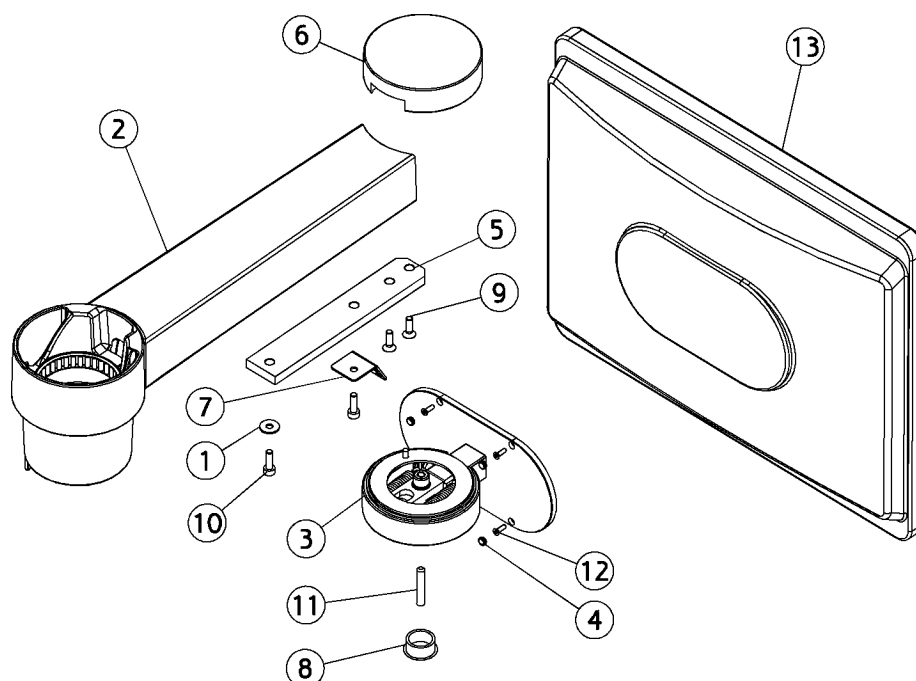

Position	Part NO.	Assembly	Description
1	000598		Not a spare part
2	AO-038	Page 356	Reararm XO 4-1 w/o colour (Unknown colour)
3	AO-039	Page 357	Link 2, screen arm-1 w/o color (Unknown colour)
4	MG-640		Not a spare part
5	MG-831		Not a spare part
6	AO-069		Not a spare part (XO-903 Silver Metallic)
6	AO-043		Not a spare part (XO-905 XO White,(RAL7047-30))
6	AP-454		Not a spare part (CTBS)
6	AO-310		Not a spare part (XO-906 XO Titanium)
7	MG-839		Not a spare part
8	UC-375		Not a spare part
9	UG-064		Not a spare part
10	UG-066		Not a spare part
11	UG-106		Not a spare part
12	UG-114		Screw threadforming, M3x10
13	UG-913		XO screen 15 inch tft, unit

AO-038


Reararm XO 4-1 w/o colour

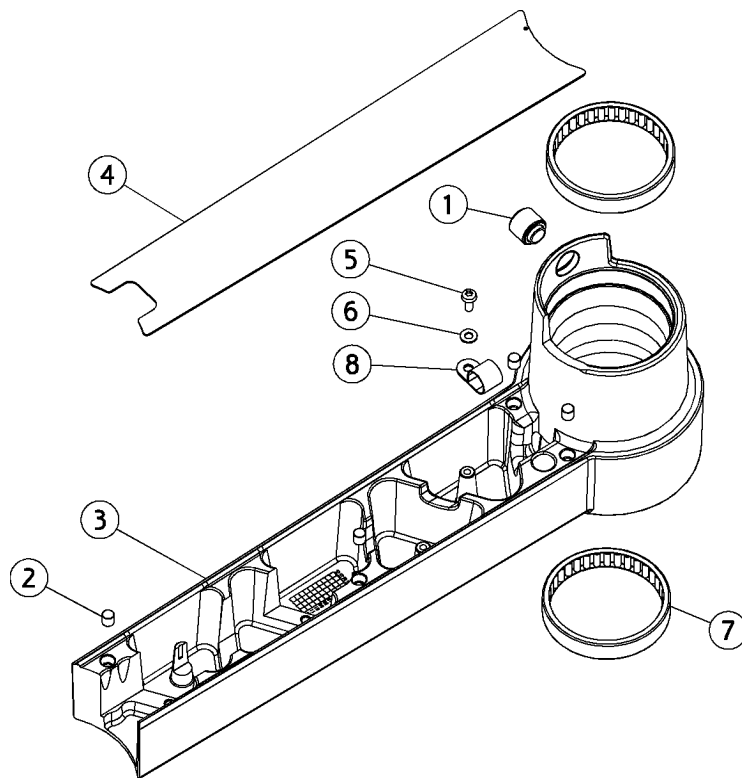

Position	Part NO.	Assembly	Description
1	AN-018	Page 182	Friction brake, arms
2	AN-127	Page 198	Magnet Ø6,35 with spring ring
3	AO-308		Rear arm, single, screen, (XO-906 XO Titanium)
3	AP-452		Rear arm, single, screen, (CTBS)
3	AO-067		Rear arm, single, screen, (XO-903 Silver Metallic)
3	AO-041		Rear arm, single, screen, (XO-905 XO White,(RAL7047-30))
4	AP-453		Cover, arm, single, screen, (CTBS)
4	AO-042		Cover, arm, single, screen, (XO-905 XO White,(RAL7047-30))
4	AO-068		Cover, arm, single, screen, (XO-903 Silver Metallic)
4	AO-309		Cover, arm, single, screen, (XO-906 XO Titanium)
5	UG-110		Screw, thread making, M4x10
6	UG-254		Disc, Ø04.3xØ09.0x0.8
7	UG-458		Not a spare part
8	UG-527		Not a spare part

AO-039


Link 2, screen arm-1 w/o color

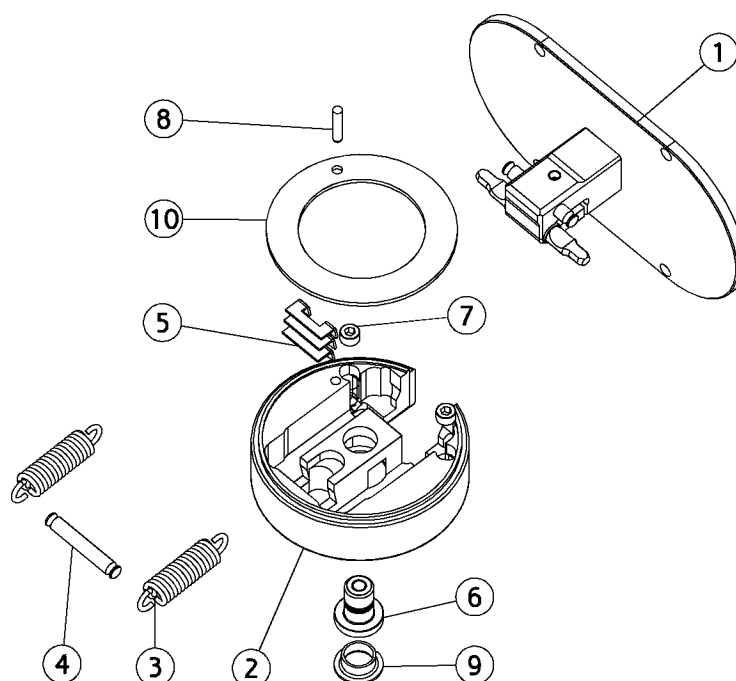

Position	Part NO.	Assembly	Description
1	AO-040	Page 358	Lever arm-1 screen, w/o col. (Unknown colour)
2	AO-044		Not a spare part (XO-905 XO White,(RAL7047-30))
2	AP-455		Not a spare part (CTBS)
2	AO-311		Not a spare part (XO-906 XO Titanium)
2	AO-070		Not a spare part (XO-903 Silver Metallic)
3	MG-837		Not a spare part
4	MG-838		Not a spare part
5	MG-840		Not a spare part
6	MG-841		Not a spare part
7	UG-121		Not a spare part
8	UG-419		Not a spare part
9	UG-463		Not a spare part
10	UG-473		Not a spare part

AO-040


Lever arm-1 screen, w/o col.

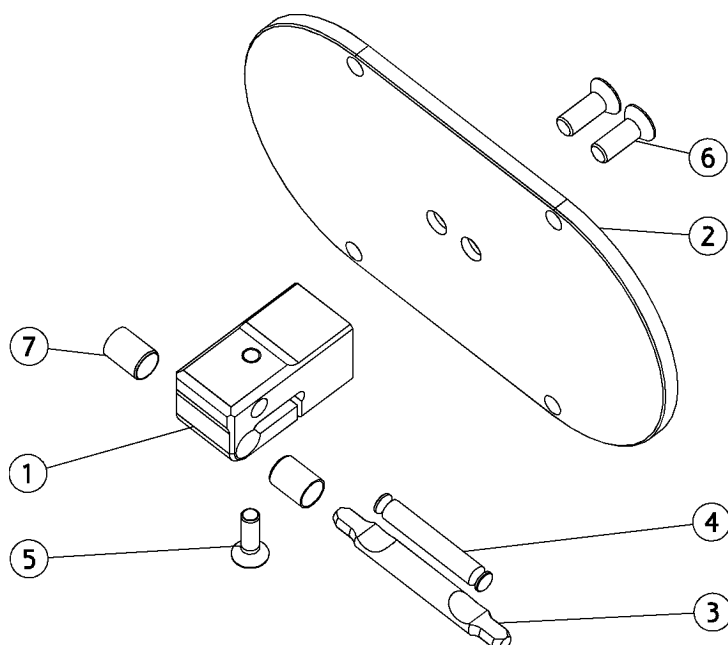

Position	Part NO.	Assembly	Description
1	AO-071		Not a spare part (XO-903 Silver Metallic)
1	AO-045		Not a spare part (XO-905 XO White,(RAL7047-30))
1	AP-456		Not a spare part (CTBS)
1	AO-312		Not a spare part (XO-906 XO Titanium)
2	AO-072		Adaptor plate, screen XO 4-1, (XO-903 Silver Metallic)
2	AO-046		Adaptor vesa 4-1 screen, (XO-905 XO White,(RAL7047-30))
2	AO-313		Not a spare part (XO-906 XO Titanium)
2	AP-457		Adaptor vesa 4-1 screen, (CTBS)
3	MG-836		Not a spare part
4	MG-838		Not a spare part
5	UG-064		Not a spare part
6	UG-147		Not a spare part
7	UG-474		Not a spare part

AO-985

XO wide screen, double arm

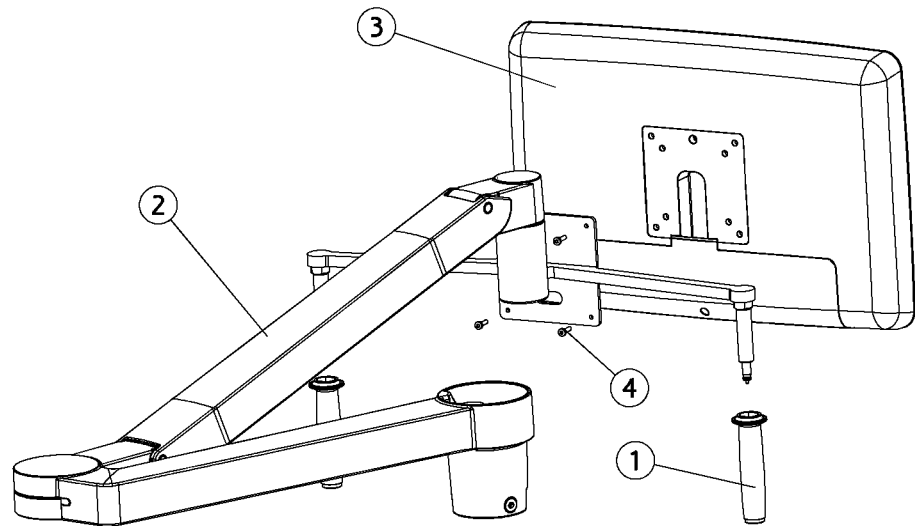


CABLE ASSEMBLY AO-973 NOT SHOWN ON DRAWING

Position	Part NO.	Assembly	Description
1	AN-110		Handle XO 4
2	AO-796	Page 360	Long arm f. XO wide screen (Unknown colour)
3	MG-982		XO4 widescreen for existing widescreen arm
4	UG-069		Screw M4x16 torx

AO-796


Long arm f. XO wide screen

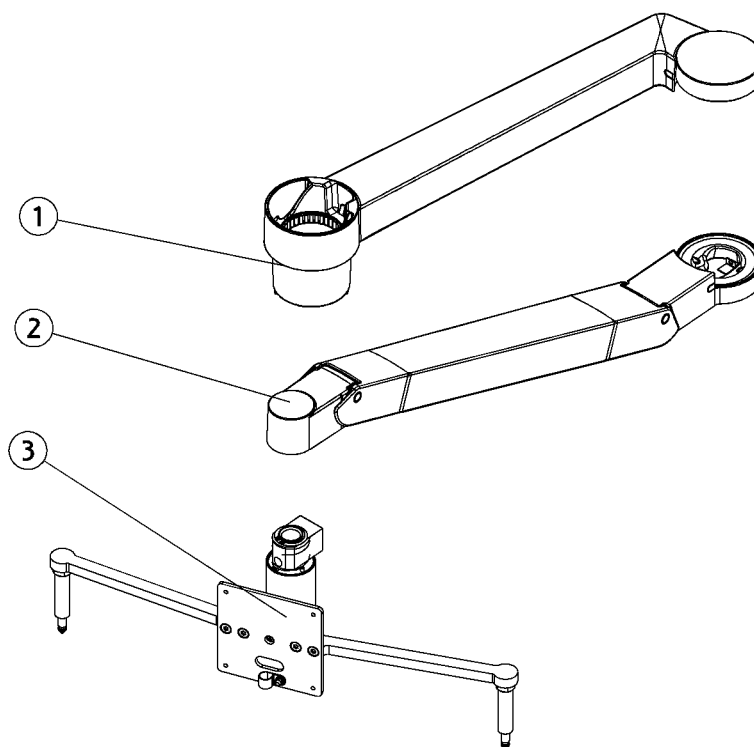

Position	Part NO.	Assembly	Description
1	AN-122	Page 343	Rear arm screen
2	AO-976	Page 361	Forearm XO wide screen
3	AO-977	Page 365	Suspension, XO wide screen

AO-976

Forearm XO wide screen

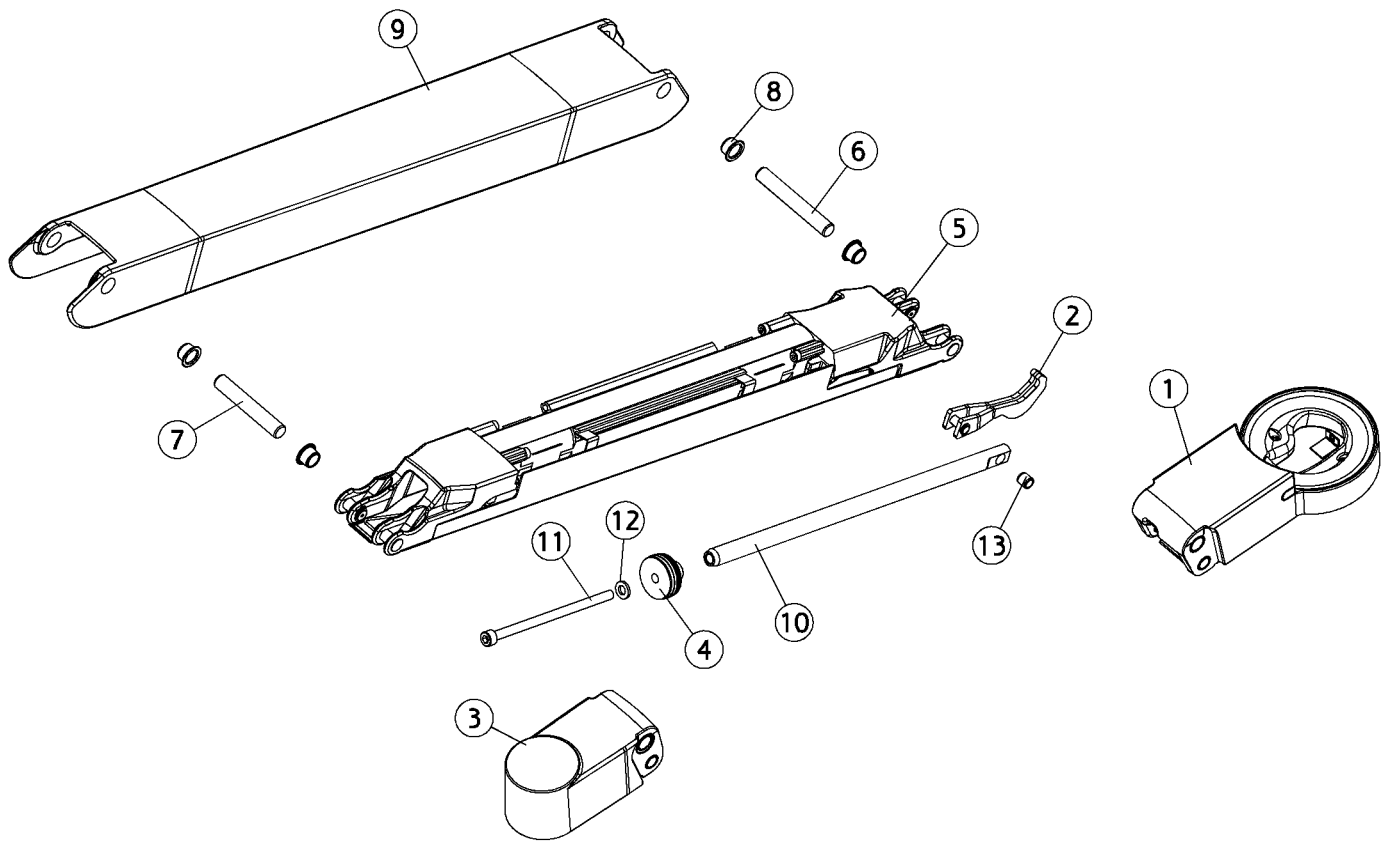


AO-976

Forearm XO wide screen

Position	Part NO.	Assembly	Description
1	AN-098	<i>Page 303</i>	Link1, forearm lamp
2	AN-104	<i>Page 312</i>	Lever lamp
3	AN-105	<i>Page 307</i>	Link2, forearm lamp
4	AN-118	<i>Page 313</i>	Control socket spring lamp arm
5	AO-799	<i>Page 363</i>	Bottom forearm XOwide screen
6	MG-297		Not a spare part
7	MG-298		Axle Ø10x65
8	MG-299		Bushing Ø10x8 for u-profile
9	AN-287		Top forearm, screen, (XO-903 Silver Metallic)
9	AN-958		Top forearm, screen, (XO-905 XO White,(RAL7047-30))
9	AN-574		Top forearm, screen, (XO-901 Sea Blue Metallic)
9	AO-293		Top forearm, screen, (XO-906 XO Titanium)
9	AN-573		Top forearm, screen, (XO-900 XO Grey, RAL 7047-80)
9	AP-439		Top forearm, screen, (CTBS)
10	MG-522		Not a spare part
11	UG-031		Not a spare part
12	UG-256		Disc, Ø06.4xØ12.0x1.6
13	UG-459		Not a spare part

AO-799

Bottom forearm XOWide screen

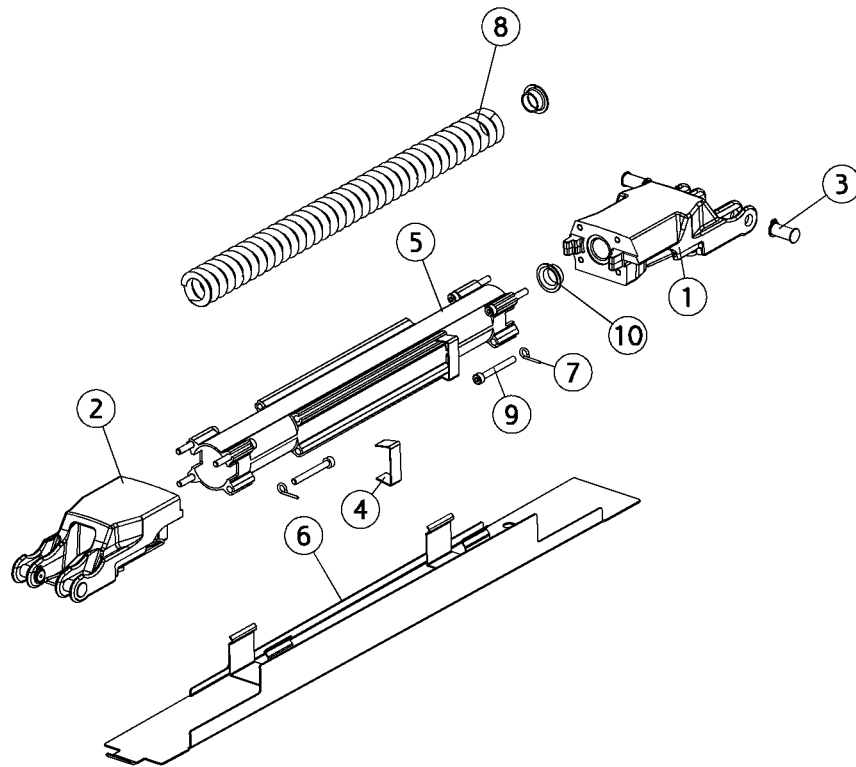


AO-799

Bottom forearm XOwide screen


Position	Part NO.	Assembly	Description
1	AN-100	<i>Page 309</i>	Suspension1 forearm lamp
2	AN-102	<i>Page 310</i>	Suspension2 forearm lamp
3	AN-103	<i>Page 200</i>	Axle 8x17
4	MG-464		Not a spare part
5	MG-467		Not a spare part
6	AN-546		Cover plate, forearm, screen, (XO-901 Sea Blue Metallic)
6	AO-294		Cover plate, forearm, screen, (XO-906 XO Titanium)
6	AN-959		Cover plate, forearm, screen, (XO-905 XO White,(RAL7047-30))
6	AN-545		Cover plate, forearm, screen, (XO-900 XO Grey, RAL 7047-80)
6	AP-440		Cover plate, forearm, screen, (CTBS)
6	AN-272		Cover plate, forearm, screen, (XO-903 Silver Metallic)
7	MG-669		Not a spare part
8	MG-973		Not a spare part
9	UG-013		Not a spare part
10	UG-463		Not a spare part

AO-977

Suspension, XO wide screen

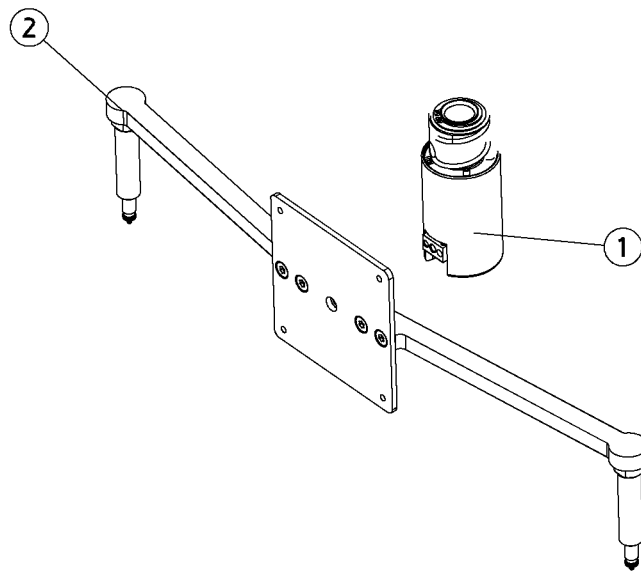


Position	Part NO.	Assembly	Description
1	AN-131	Page 352	Housing link3 screen
2	AO-899	Page 366	Arm control XO wide screen, hd display

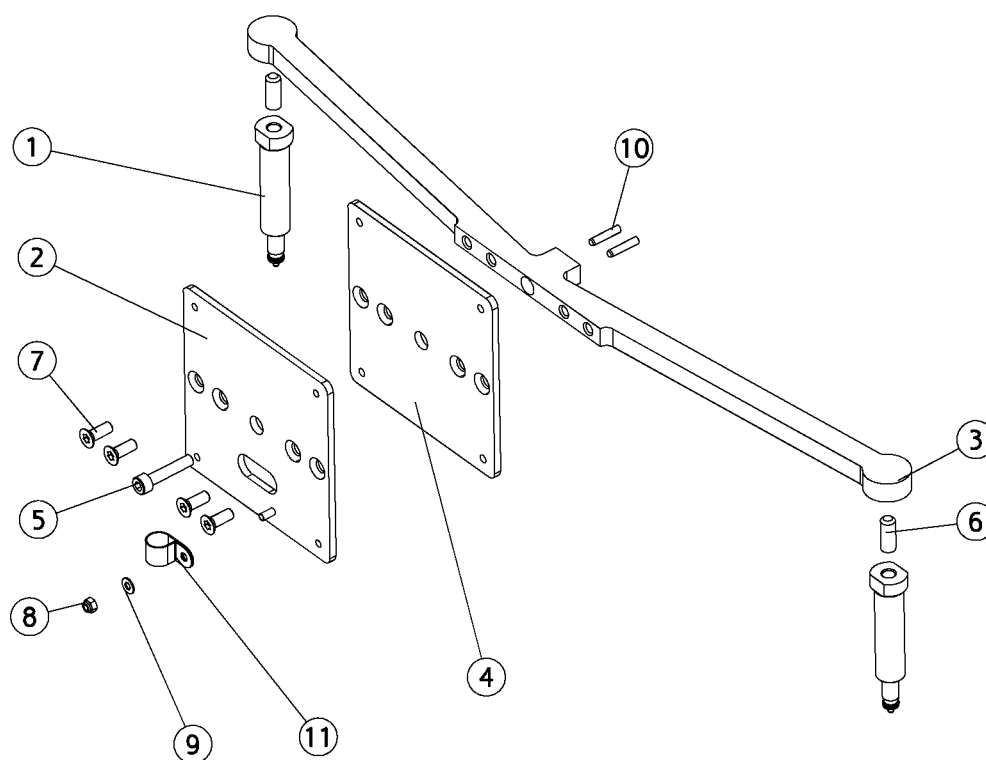

AO-899

Arm control XO wide screen, hd display


Position	Part NO.	Assembly	Description
1	AN-084	<i>Page 158</i>	Receiver, handles
2	AO-898		Not a spare part (XO-905 XO White,(RAL7047-30))
3	AP-461		Not a spare part (CTBS)
3	AO-896		Not a spare part (XO-905 XO White,(RAL7047-30))
4	MN-056		Not a spare part
5	UG-006		Not a spare part
6	UG-144		Not a spare part
7	UG-147		Not a spare part
8	UG-209		Not a spare part
9	UG-254		Disc, Ø04.3xØ09.0x0.8
10	UG-419		Not a spare part
11	UG-527		Not a spare part

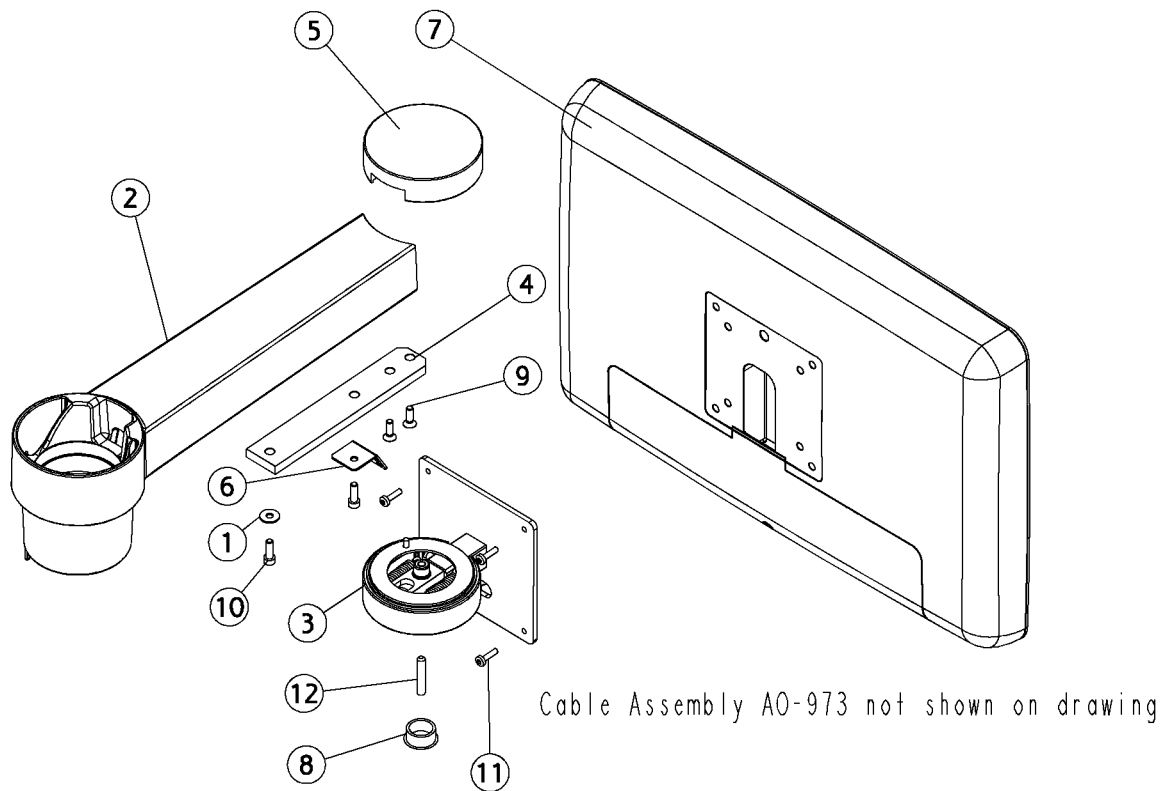

AO-798

Screen arm, parts, short f. wide screen

Position	Part NO.	Assembly	Description
1	000598		Not a spare part
2	AO-038	<i>Page 356</i>	Reararm XO 4-1 w/o colour
3	AO-975	<i>Page 370</i>	Joint2 single arm, XO wide screen
4	MG-831		Not a spare part
5	AO-069		Not a spare part (XO-903 Silver Metallic)
5	AO-043		Not a spare part (XO-905 XO White,(RAL7047-30))
5	AP-454		Not a spare part (CTBS)
5	AO-310		Not a spare part (XO-906 XO Titanium)
6	MG-839		Not a spare part
7	MG-982		XO4 widescreen for existing widescreen arm
8	UC-375		Not a spare part
9	UG-064		Not a spare part
10	UG-066		Not a spare part
11	UG-069		Screw M4x16 torx
12	UG-106		Not a spare part

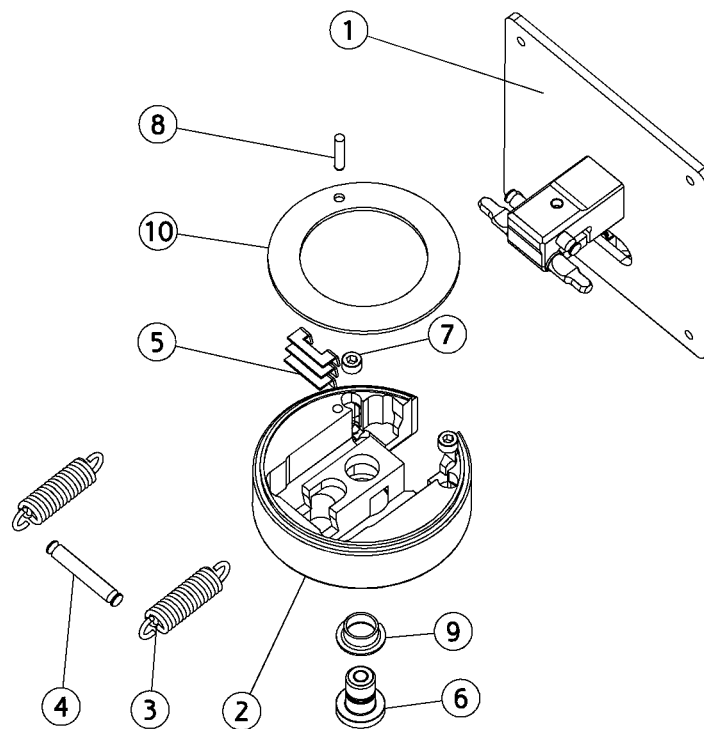


AO-975

Joint2 single arm, XO wide screen

Position	Part NO.	Assembly	Description
1	AO-978	<i>Page 372</i>	Lever arm, 4-1 XO wide screen, w/o colour
2	AO-044		Not a spare part (XO-905 XO White, (RAL7047-30))
2	AP-455		Not a spare part (CTBS)
2	AO-311		Not a spare part (XO-906 XO Titanium)
2	AO-070		Not a spare part (XO-903 Silver Metallic)
3	MG-837		Not a spare part
4	MG-838		Not a spare part
5	MG-840		Not a spare part
6	MG-841		Not a spare part
7	UG-121		Not a spare part
8	UG-419		Not a spare part
9	UG-463		Not a spare part
10	UG-473		Not a spare part

AO-978

Lever arm, 4-1 XO wide screen, w/o colour

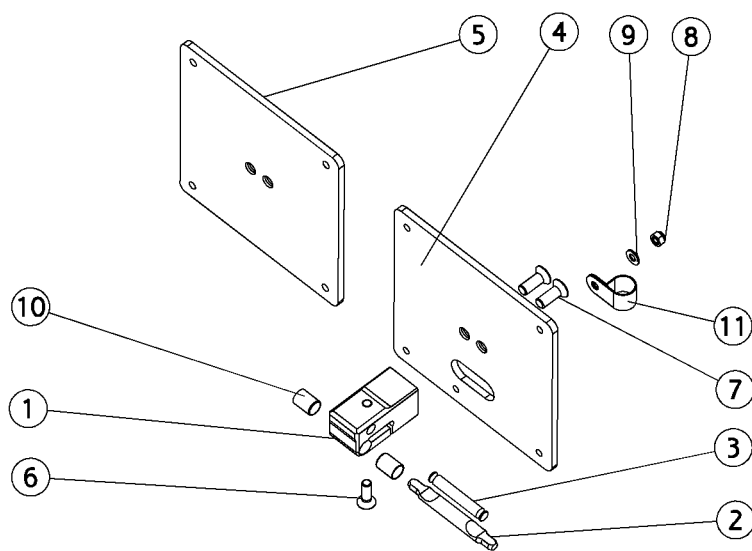


AO-978

Lever arm, 4-1 XO wide screen, w/o colour

Position	Part NO.	Assembly	Description
1	AO-071		Not a spare part (XO-903 Silver Metallic)
1	AO-045		Not a spare part (XO-905 XO White,(RAL7047-30))
1	AP-456		Not a spare part (CTBS)
1	AO-312		Not a spare part (XO-906 XO Titanium)
2	MG-836		Not a spare part
3	MG-838		Not a spare part
4	MG-983		Not a spare part (Unknown colour)
5	MN-055		Not a spare part
6	UG-064		Not a spare part
7	UG-147		Not a spare part
8	UG-209		Not a spare part
9	UG-254		Disc, Ø04.3xØ09.0x0.8
10	UG-474		Not a spare part
11	UG-527		Not a spare part

AP-517

XO HD display, double arm,

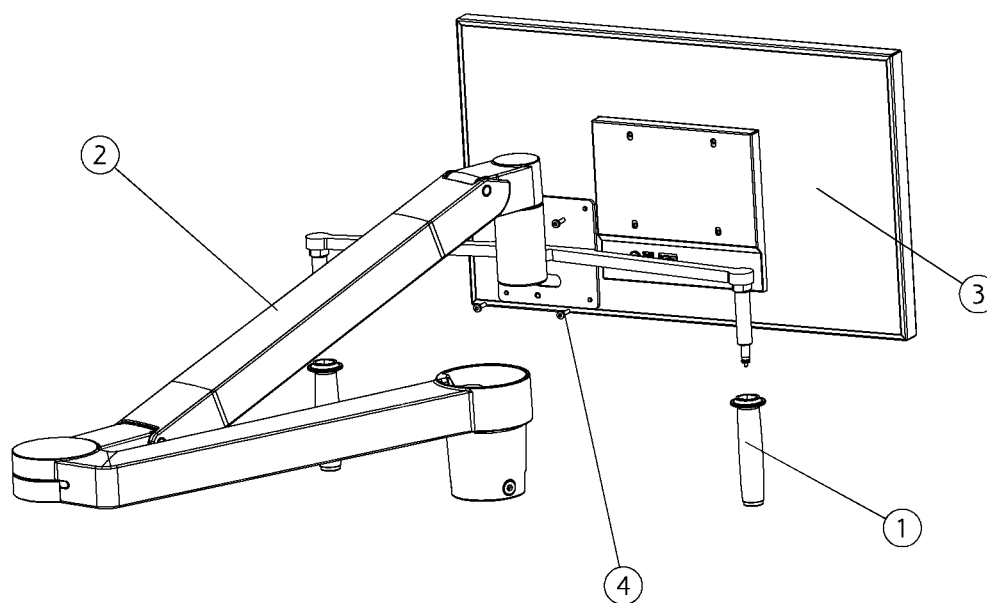


Cable assembly AP-625 not shown on drawing

Position	Part NO.	Assembly	Description
1	AN-110		Handle XO 4
2	AO-796	Page 360	Long arm f. XO wide screen (Unknown colour)
3	AP-619		XO HD display, (CTBS)
4	UG-069		Screw M4x16 torx

AP-518

XO HD display, single arm,

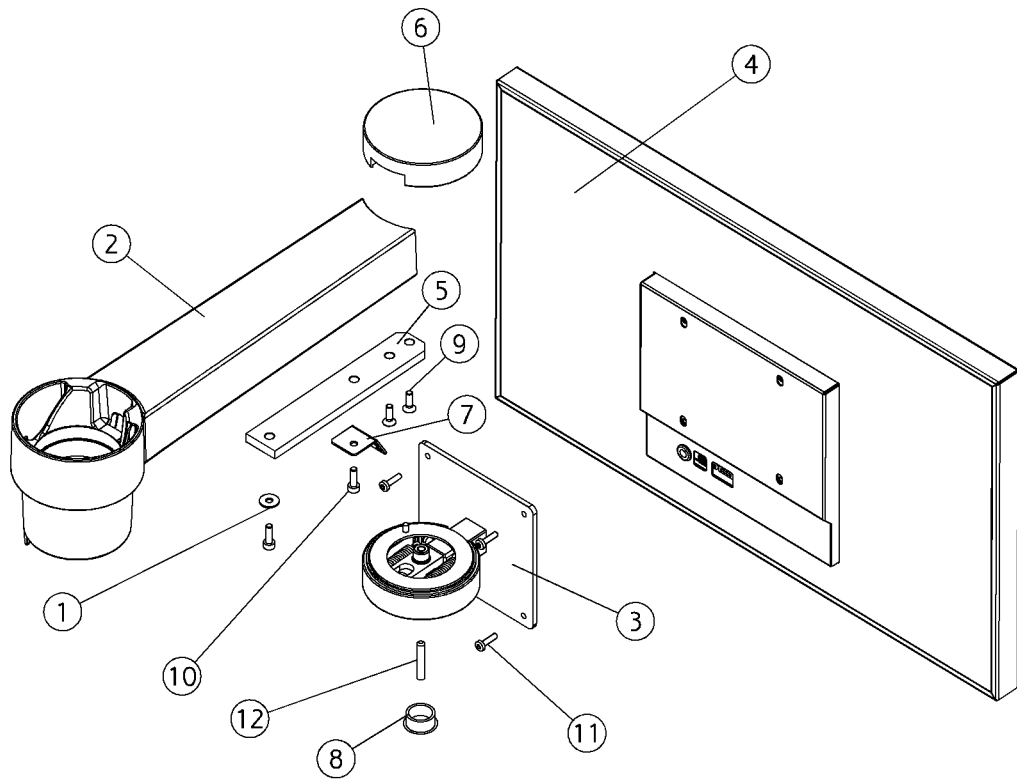

Cable Assembly AP-625 not shown on drawing

AP-518


XO HD display, single arm,

Position	Part NO.	Assembly	Description
1	000598		Not a spare part
2	AO-038	<i>Page 356</i>	Reararm XO 4-1 w/o colour
3	AO-975	<i>Page 370</i>	Joint2 single arm, XO wide screen
4	AP-619		XO HD display, (CTBS)
5	MG-831		Not a spare part
6	AO-069		Not a spare part (XO-903 Silver Metallic)
6	AO-043		Not a spare part (XO-905 XO White,(RAL7047-30))
6	AP-454		Not a spare part (CTBS)
6	AO-310		Not a spare part (XO-906 XO Titanium)
7	MG-839		Not a spare part
8	UC-375		Not a spare part
9	UG-064		Not a spare part
10	UG-066		Not a spare part
11	UG-069		Screw M4x16 torx
12	UG-106		Not a spare part

XO 4 Operator's Stool

AN-765

Chair, seat (saddle)

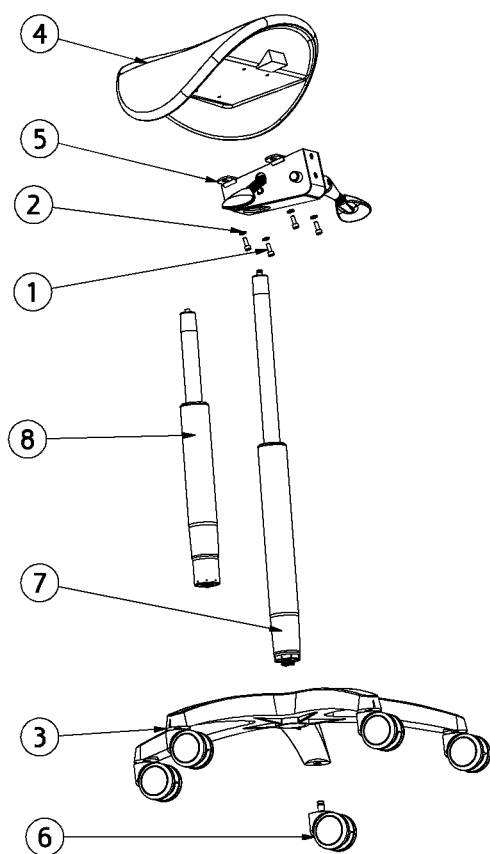

AN-765

Chair, seat (saddle)

Position	Part NO.	Assembly	Description
1	UG-063		Bolt i6kt din912 M6x16
2	UG-256		Disc, Ø06.4xØ12.0x1.6
3	UH-079		Alu foot, seat/stool, (XO-908 Iron Grey (RAL 7011-5))
3	UH-288		Alu foot, seat/stool, (CTBS)
4	UH-100		Upholstery, scuba, seat, (XO-970 Cocoon)
4	UG-611		Upholstery, seat, XO seat, (XO-954 Beige)
4	UH-109		Upholstery, scuba, seat, (XO-979 Charcoal)
4	UH-162		Upholstery, scuba, seat, (XO-991 Truffle (025))
4	UG-790		Upholstery, seat, XO seat, (BA-209 Sea Blue)
4	UG-788		Upholstery, seat, XO seat, (BA-407 Crystal White)
4	UG-793		Upholstery, seat, XO seat, (XO-951 Red)
4	UG-786		Upholstery, seat, XO seat, (BA-211 Dark Grey)
4	UH-102		Upholstery, scuba, seat, (XO-972 Walnut)
4	UH-063		Upholstery, seat, XO seat, (XO-959 Silk Grey)
4	UG-787		Upholstery, seat, XO seat, (WB-230 Grey)
4	UG-789		Upholstery, seat, XO seat, (WB-234 Black)
4	UG-610		Upholstery, seat, XO seat, (BA-374 Orange)
4	UH-101		Upholstery, scuba, seat, (XO-971 Dune)
4	UH-104		Upholstery, scuba, seat, (XO-974 Dolphin)
4	UH-161		Upholstery, scuba, seat, (XO-992 Brick (021))
4	UH-163		Upholstery, scuba, seat, (XO-993 Mahogany (006))
4	UH-107		Upholstery, scuba, seat, (XO-977 Autumn)
4	UH-042		Upholstery, seat, XO seat, (XO-961 Scarlet Red)
4	UH-108		Upholstery, scuba, seat, (XO-978 Cinnamon)
4	UG-614		Upholstery, seat, XO seat, (XO-958 Golden Honey)
4	UG-612		Upholstery, seat, XO seat, (XO-955 Bordeaux)
4	UH-164		Upholstery, scuba, seat, (XO-994 Sapphire (016))
4	UG-791		Upholstery, seat, XO seat, (XO-950 Light Blue)
4	UH-160		Upholstery, scuba, seat, (XO-990 Coral (022))
4	UH-105		Upholstery, scuba, seat, (XO-975 Sage)
4	UH-103		Upholstery, scuba, seat, (XO-973 Jet)
4	UG-613		Upholstery, seat, XO seat, (XO-956 Light Green)


AN-765

Chair, seat (saddle)

Position	Part NO.	Assembly	Description
4	UH-321		Upholstery, comfort, seat (CTBS) (CTBS)
4	UH-106		Upholstery, scuba, seat, (XO-976 Tourmaline)
4	UG-792		Upholstery, seat, XO seat, (BA-243 Safran Yellow)
5	UG-971		Supporting plate 41.2 left
6	UG-973		Double wheel Ø65,soft, black
7	UH-077		Gas spring, seat/stool, long, 37 cm, (CTBS) (XO-908 Iron Grey (RAL 7011-5))
7	UH-286		Gas spring, seat/stool, long, 37 cm, (CTBS)
8	UH-078		Gas spring, seat/stool, short, 30 cm, (CTBS) (XO-908 Iron Grey (RAL 7011-5))
8	UH-287		Gas spring, seat/stool, short, 30 cm, (CTBS)

AN-769

Chair, stool

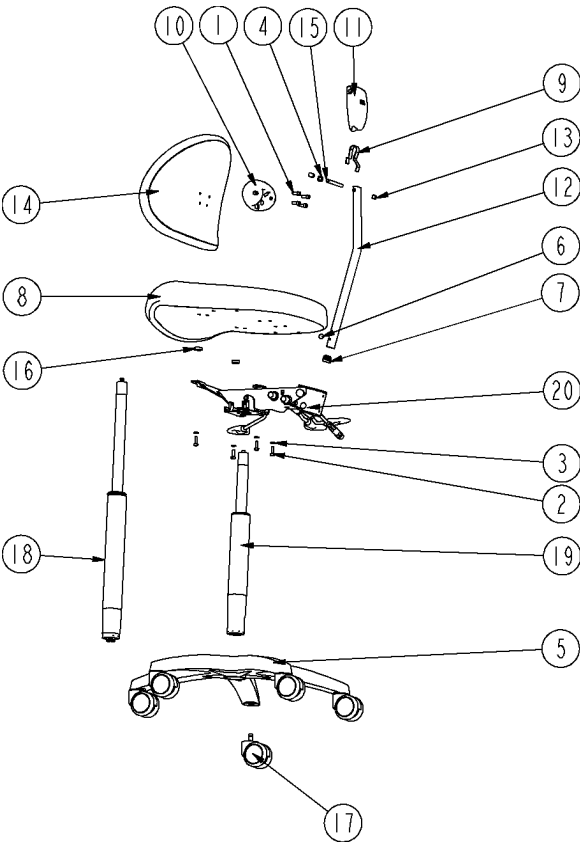

AN-769

Chair, stool

Position	Part NO.	Assembly	Description
1	UG-063		Bolt i6kt din912 M6x16
2	UG-159		Screw phx 6x20
3	UG-286		Disc M6
4	UG-287		Starlock washer Ø8
5	UH-079		Alu foot, seat/stool, (XO-908 Iron Grey (RAL 7011-5))
5	UH-288		Alu foot, seat/stool, (CTBS)
6	UG-958		Plastic plug for stop on back
7	UG-959		Cover plug for back bar uni21
8	UG-769		Upholstery, seat, XO stool, (XO-957 Sap Green)
8	UH-117		Upholstery, seat, XO stool, scuba, (XO-977 Autumn)
8	UH-110		Upholstery, seat, XO stool, scuba, (XO-970 Cocoon)
8	UH-168		Upholstery, seat, XO stool, scuba, (XO-993 Mahogany (006))
8	UG-765		Upholstery, seat, XO stool, (BA-374 Orange)
8	UH-113		Upholstery, seat, XO stool, scuba, (XO-973 Jet)
8	UG-774		Upholstery, seat, XO stool, (BA-209 Sea Blue)
8	UH-118		Upholstery, seat, XO stool, scuba, (XO-978 Cinnamon)
8	UH-115		Upholstery, seat, XO stool, scuba, (XO-975 Sage)
8	UG-767		Upholstery, seat, XO stool, (XO-955 Bordeaux)
8	UH-064		Upholstery, seat, XO stool, (XO-958 Golden Honey)
8	UH-119		Upholstery, seat, XO stool, scuba, (XO-979 Charcoal)
8	UH-116		Upholstery, seat, XO stool, scuba, (XO-976 Tourmaline)
8	UH-165		Upholstery, seat, XO stool, scuba, (XO-990 Coral (022))
8	UG-771		Upholstery, seat, XO stool, (WB-230 Grey)
8	UG-772		Upholstery, seat, XO stool, (BA-407 Crystal White)
8	UG-775		Upholstery, seat, XO stool, (XO-950 Light Blue)
8	UG-776		Upholstery, seat, XO stool, (BA-243 Safran Yellow)
8	UG-766		Upholstery, seat, XO stool, (XO-954 Beige)
8	UG-770		Upholstery, seat, XO stool, (BA-211 Dark Grey)
8	UH-166		Upholstery, seat, XO stool, scuba, (XO-992 Brick (021))
8	UH-114		Upholstery, seat, XO stool, scuba, (XO-974 Dolphin)
8	UH-169		Upholstery, seat, XO stool, scuba, (XO-994 Sapphire (016))
8	UG-777		Upholstery, seat, XO stool, (XO-951 Red)

AN-769

Chair, stool

Position	Part NO.	Assembly	Description
8	UH-319		Upholstery, seat, XO stool, comfort, (CTBS)
8	UG-768		Upholstery, seat, XO stool, scuba, (XO-956 Light Green)
8	UH-112		Upholstery, seat, XO stool, scuba, (XO-972 Walnut)
8	UH-167		Upholstery, seat, XO stool, scuba, (XO-991 Truffle (025))
8	UH-111		Upholstery, seat, XO stool, scuba, (XO-971 Dune)
8	UG-773		Upholstery, seat, XO stool, (WB-234 Black)
9	UG-962		Leaf spring for base back hing
10	UG-963		Base back hinge
11	UG-964		Back hinge
12	UG-965		Back clamp 21 v.54 420mm
13	UG-966		Cover plug for back hinge base
14	UH-120		Upholstery, back rest, XO stool, scuba, (XO-970 Cocoon)
14	UH-124		Upholstery, back rest, XO stool, scuba, (XO-974 Dolphin)
14	UG-783		Upholstery, back rest, XO stool, (XO-950 Light Blue)
14	UG-781		Upholstery, back rest, XO stool, (WB-234 Black)
14	UH-320		Upholstery, back rest, XO stool, comfort, (CTBS)
14	UG-782		Upholstery, back rest, XO stool, (BA-209 Sea Blue)
14	UH-121		Upholstery, back rest, XO stool, scuba, (XO-971 Dune)
14	UH-171		Upholstery, back rest, XO stool, scuba, (XO-992 Brick (021))
14	UH-049		Upholstery, back rest, XO stool, (XO-959 Silk Grey)
14	UG-780		Upholstery, back rest, XO stool, (BA-407 Crystal White)
14	UG-796		Upholstery, back rest, XO stool, (XO-955 Bordeaux)
14	UG-778		Upholstery, back rest, XO stool, (BA-211 Dark Grey)
14	UG-795		Upholstery, back rest, XO stool, (XO-954 Beige)
14	UH-172		Upholstery, back rest, XO stool, scuba, (XO-991 Truffle (025))
14	UG-797		Upholstery, back rest, XO stool, (XO-956 Light Green)
14	UH-174		Upholstery, back rest, XO stool, scuba, (XO-994 Sapphire (016))
14	UH-123		Upholstery, back rest, XO stool, scuba, (XO-973 Jet)
14	UH-128		Upholstery, back rest, XO stool, scuba, (XO-978 Cinnamon)
14	UG-794		Upholstery, back rest, XO stool, (BA-374 Orange)
14	UH-125		Upholstery, back rest, XO stool, scuba, (XO-975 Sage)
14	UG-779		Upholstery, back rest, XO stool, (WB-230 Grey)

AN-769


Chair, stool

Position	Part NO.	Assembly	Description
14	UH-074		Upholstery, back rest, XO stool, (BA-376 Pastel Blue)
14	UH-127		Upholstery, back rest, XO stool, scuba, (XO-977 Autumn)
14	UH-173		Upholstery, back rest, XO stool, scuba, (XO-993 Mahogany (006))
14	UH-129		Upholstery, back rest, XO stool, scuba, (XO-979 Charcoal)
14	UG-784		Upholstery, back rest, XO stool, (BA-243 Safran Yellow)
14	UH-126		Upholstery, back rest, XO stool, scuba, (XO-976 Tourmaline)
14	UH-170		Upholstery, back rest, XO stool, scuba, (XO-990 Coral (022))
14	UH-122		Upholstery, back rest, XO stool, scuba, (XO-972 Walnut)
14	UG-785		Upholstery, back rest, XO stool, (XO-951 Red)
14	UH-047		Upholstery, back rest, XO stool, (XO-957 Sap Green)
15	UG-968		Axle for back hinge
16	UG-969		Spacer, plastic 20x10,5x6
17	UG-973		Double wheel Ø65,soft, black
18	UH-077		Gas spring, seat/stool, long, 37 cm, (CTBS) (XO-908 Iron Grey (RAL 7011-5))
18	UH-286		Gas spring, seat/stool, long, 37 cm, (CTBS)
19	UH-078		Gas spring, seat/stool, short, 30 cm, (CTBS) (XO-908 Iron Grey (RAL 7011-5))
19	UH-287		Gas spring, seat/stool, short, 30 cm, (CTBS)
20	UG-976		Supporting plate 21.2 right

XO 4 Care, cleaning and tools

AN-210

Installation kit

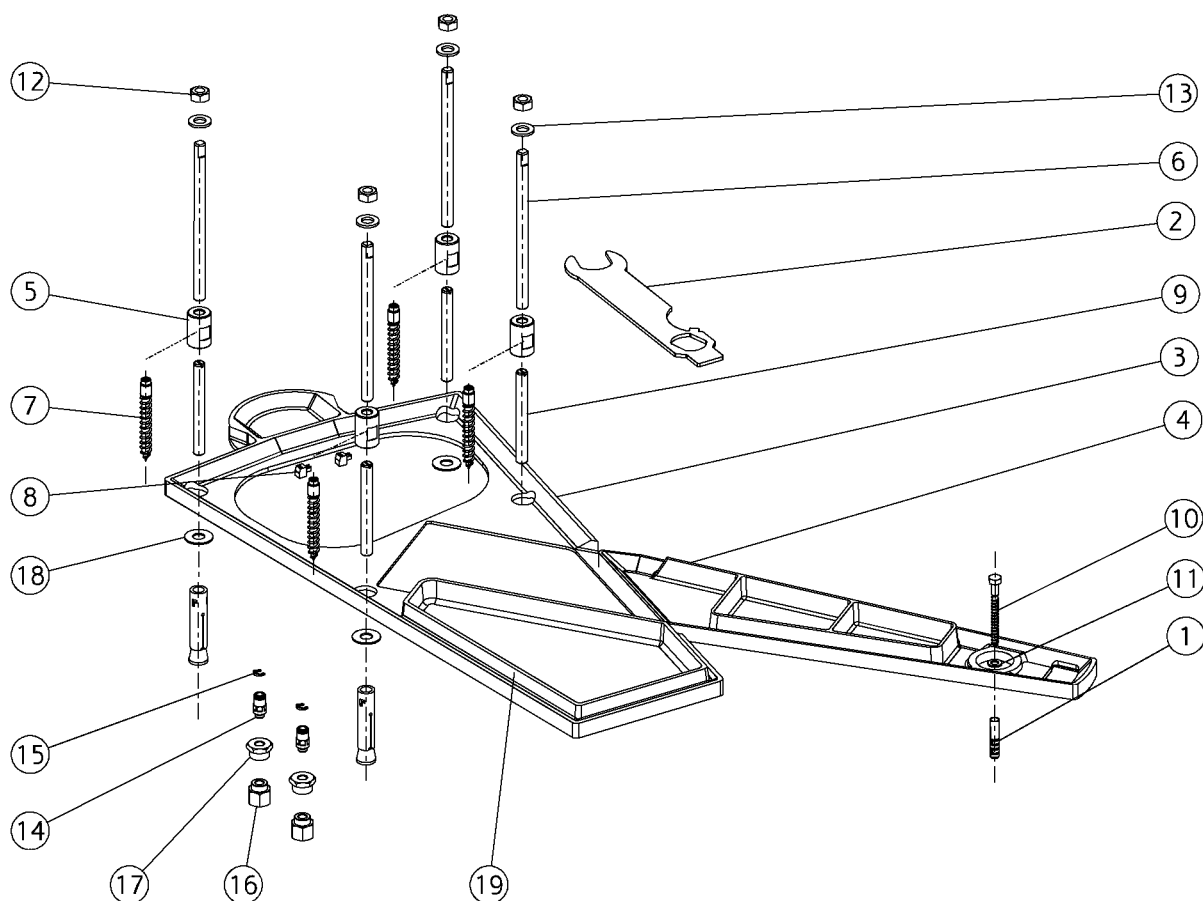


AN-210

Installation kit

Position	Part NO.	Assembly	Description
1	HA-178		Not a spare part
2	MG-416		Special tool, installation XO 4
3	MG-417		Gasket, socket - floor
4	MG-418		Gasket, supporting leg
5	MG-535		Threaded bush, floor
6	MG-536		Threaded pin, M12
7	MG-553		Attachment bolt XO 4
8	ML-660		Plug, 2 poles, screw
9	MN-459		Not a spare part
10	UA-510		Not a spare part
11	UA-511		Not a spare part
12	UG-203		Nut, M12
13	UG-253		Disc Ø13.0xØ24.0x2.5
14	UG-662		Quick connect, 6/4-R1/8
15	UG-663		Not a spare part
16	UG-664		Not a spare part
17	UG-688		Fitting 3/8 -1/8
18	UG-930		Not a spare part
19	UH-290		Not a spare part
20	XA-321		Not a spare part


AN-181 Carrying handle for unit stand

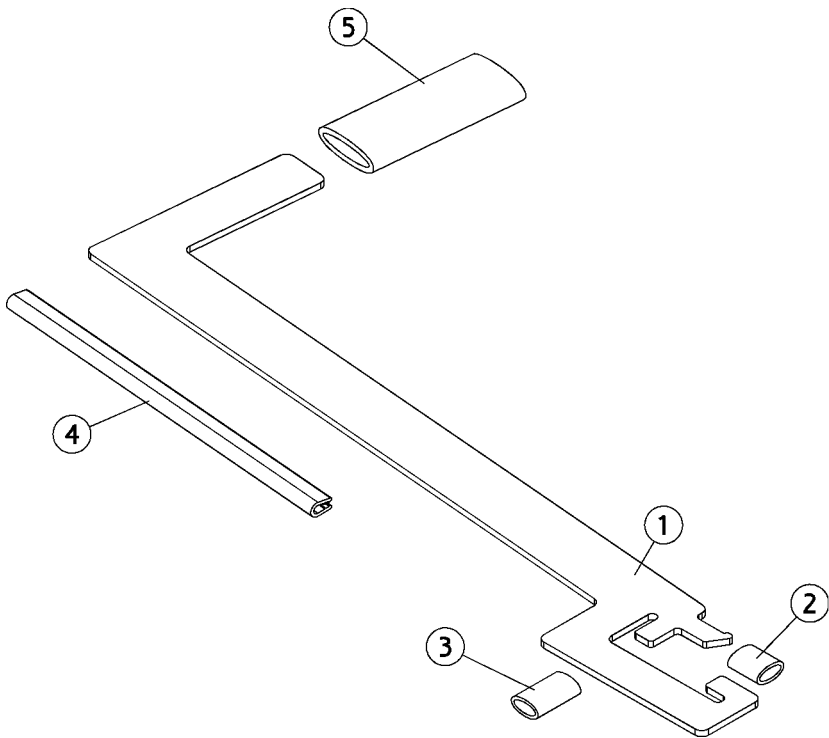

Position	Part NO.	Assembly	Description
1	MG-578		Not a spare part
2	UC-829		Not a spare part
3	UC-829		Not a spare part
4	UG-403		Not a spare part
5	UG-921		Not a spare part


AN-683

Mounting plate 12 steel XO 4

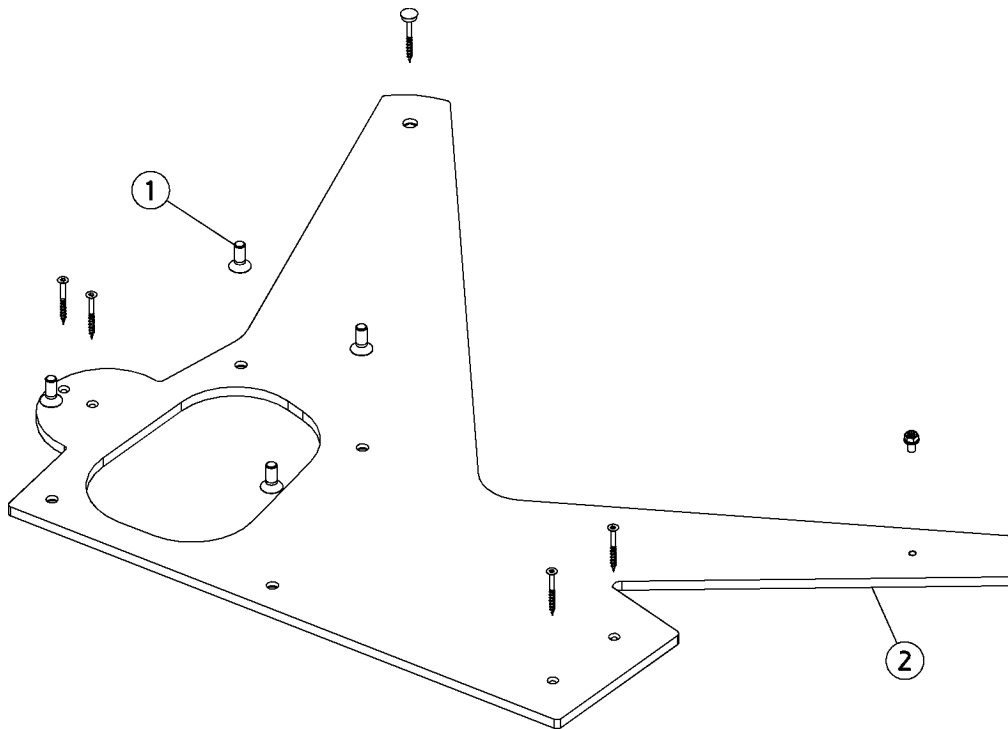

Position	Part NO.	Assembly	Description
1	AN-522	<i>Page 391</i>	Install. kit, mounting plate
2	AO-701		Mounting plt 12 steel, (XO-908 Iron Grey (RAL 7011-5))
2	AN-642		Mounting plt 12 steel, (XO-909 Iron Grey (RAL 7011-30))


AP-919 Drilling template 1:1, 1mm steel

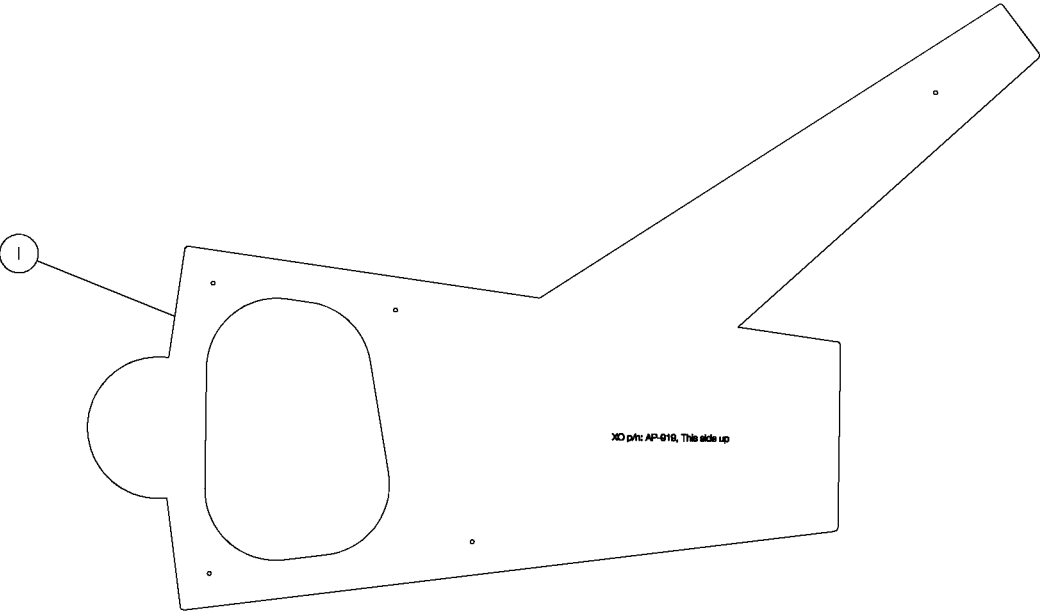

Position	Part NO.	Assembly	Description
1	AP-919		Drilling template 1:1, 1mm steel


AN-522

Install. kit, mounting plate

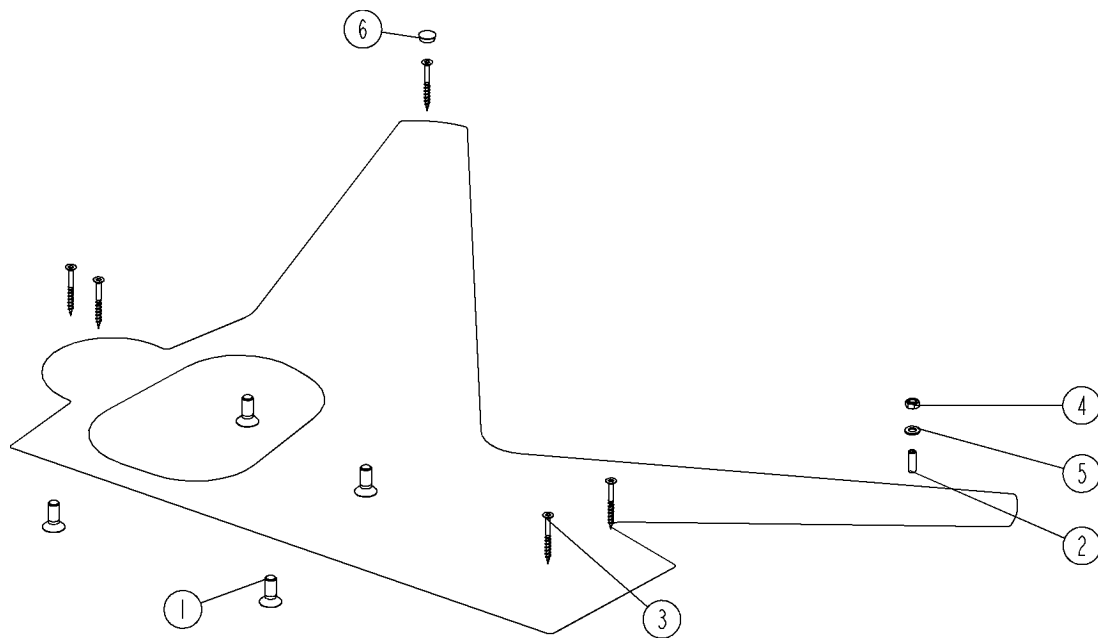

Position	Part NO.	Assembly	Description
1	UG-043		Bolt, M12x30
2	UG-123		Not a spare part
3	UG-155		Not a spare part
4	UG-204		Not a spare part
5	UG-252		Not a spare part
6	UG-941		Cover plug, Ø15,7 black

AP-682 Mounting plate 12 steel

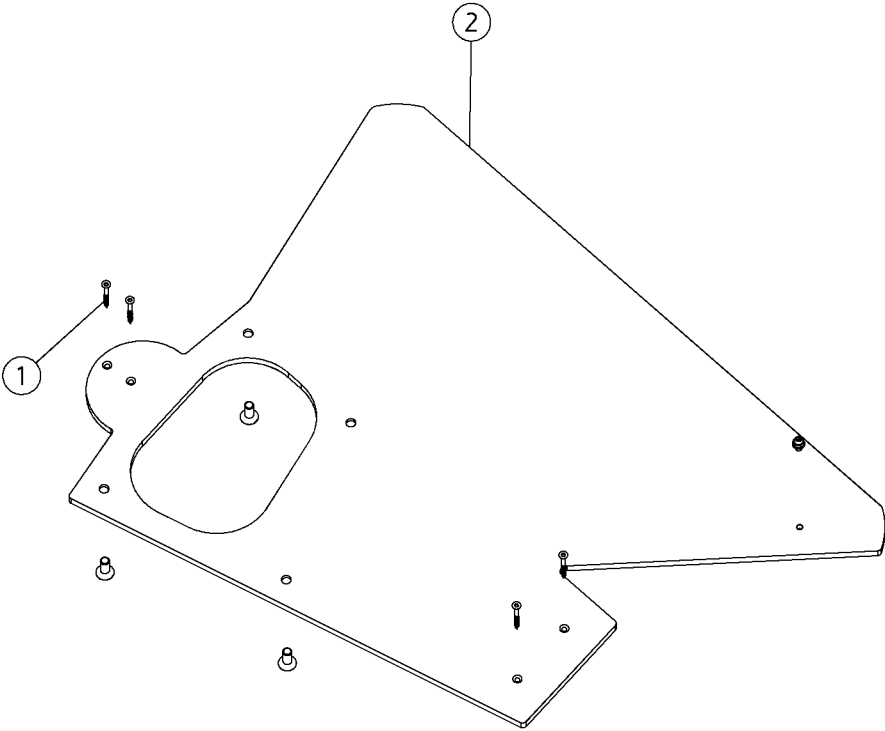

Position	Part NO.	Assembly	Description
1	AN-522	Page 391	Install. kit, mounting plate
2	AP-683		Mounting plt 12 steel, (CTBS)

TX-001 **Repair paint**

Position	Part NO.	Assembly	Description
1	UH-190		Repair paint, (XB-932 Midnight black)
1	UH-156		Repair paint, (XB-931 Traffic White)
1	UG-943		Repair paint, (XO-901 Sea Blue Metallic)
1	UH-191		Repair paint, (XB-933 Bright Aluminium)
1	UG-942		Repair paint, (XO-900 XO Grey, RAL 7047-80)
1	UH-001		Repair paint, (XO-906 XO Titanium)
1	UG-945		Repair paint, (XO-909 Iron Grey (RAL 7011-30))
1	UG-994		Repair paint, (XO-905 XO White,(RAL7047-30))
1	UH-189		Repair paint, (XB-934 Calm Beige)
1	UG-944		Repair paint, (XO-903 Silver Metallic)
1	UH-192		Repair paint, (XO-908 Iron Grey (RAL 7011-5))

TX-004

Lubrication parts

Position	Part NO.	Assembly	Description
1	YR-002		Silicone grease, 100 ml
2	YR-026		Darina r2 grease, 100 ml
3	YR-032		Paraliq grease, 100 ml
4	YR-033		Sunaplex 781 grease 100 ml
5	FA-385		Sunaplex 781 grease 1000 ml

TX-005

Upholstery cleaning

Position	Part NO.	Assembly	Description
1	AP-831		XO Quick Disinfection, in box with 6 x 1 liter
2	AP-832		XO Gentle Disinfection, in box with 6 x 1 liter
3	AP-833		XO Fabric Make Up, in box with 1 x 0,5 liter