

XO® 4

Retrofit of Auto Return Holder
suction on bridge.




YB-993
Version 1.00

XO CARE A/S
Usserød Mølle
Håndværkersvinget 6
DK 2970 Hørsholm
Denmark


+45 70 20 55 11
info@xo-care.com
www.xo-care.com




1. Remove the top cover

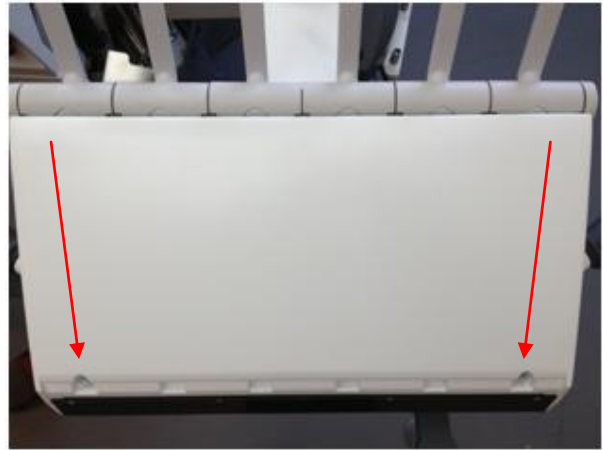


2. Unscrew the three bolts

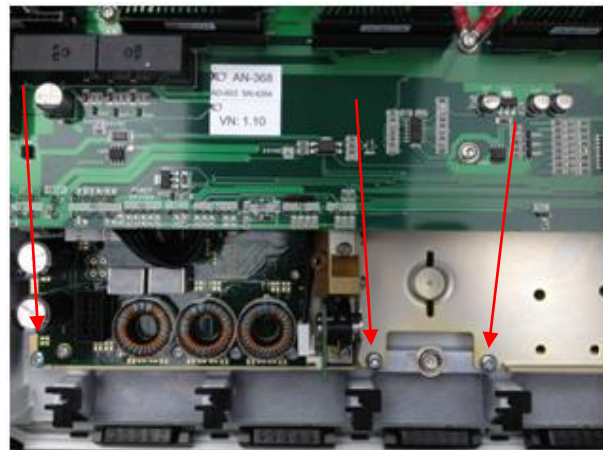


3. Install the position mechanism as shown on the picture.

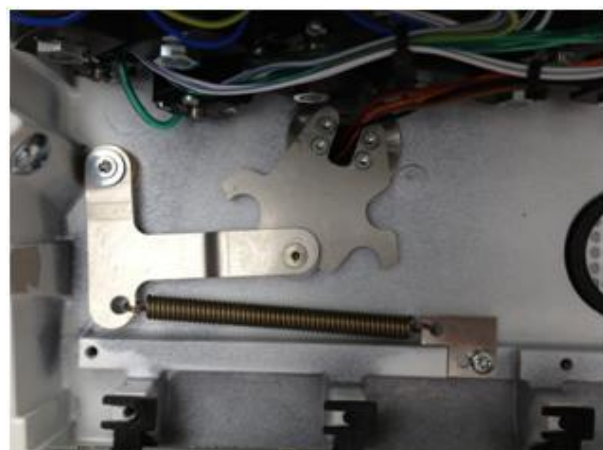


4. Install the cables from the suction holder as shown on the picture.

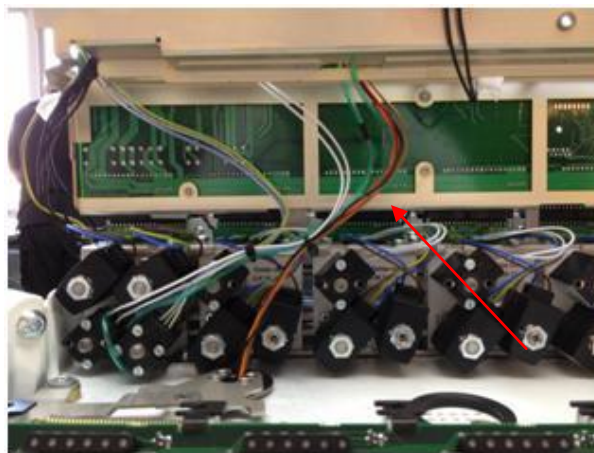

5. Install the plug on position J8 on the bridge PCB.

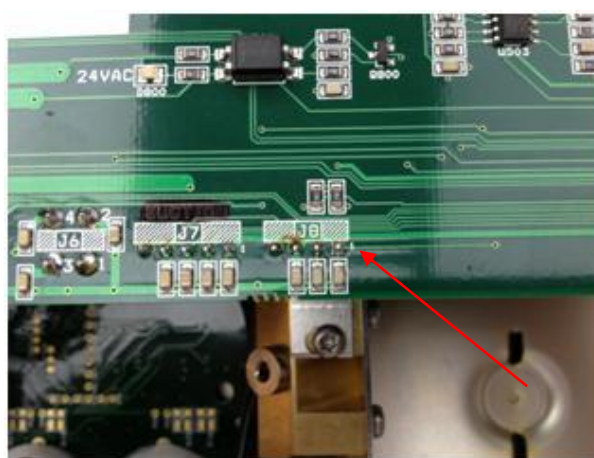

6. Make sure the plug is in the correct position.

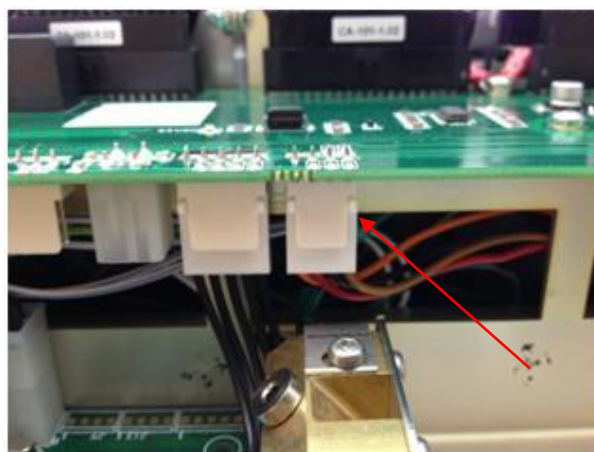

7. Close the bridge in opposite order.
Make sure that all cables are in position
before the screws are installed.

