

XO 4 & XO FLEX CHAIR SYNCHRONISATION

YB-755
Version 1.00



After replacement of the PCB AN-368 (main PCB in bridge of XO4 & XO FLEX) or replacement of its Lithium battery or upgrading the firmware, it is necessary to synchronise the unit chair, as the displacement limits stored in AN-368 will be missing.

To synchronise the chair, proceed as follows:

- 1 Power the unit on and wait until it is ready. Continue with step 2 or see alternative procedure.
- 2 Press C (configuration switch) twice. Display shows "**LIGHT**"
- 3 Press foot control joystick west (left). Display shows "**UNIT**"
- 4 Press foot control joystick south (against yourself). Display shows "**CHAIR**"
- 5 Press foot control joystick east (right). Display shows "**RINSE=0**"
- 6 Press foot control joystick south. Display shows "**SYNC**"
- 7 Press foot control joystick east. Display shows "**NO**"
- 8 Press foot control pedal down. Display shows "**YES**"
- 9 Press foot control joystick west. Display shows "**SYNC**" and the chair moves.
- 10 Press configuration switch. Display shows same info as in 1, and calibration has ended when the chair stops.

Note that this description refers to firmware version 1.51 or later.

Alternative procedure:

Same as above, but after step 1 move the chair (even slightly) and execute step 2 quickly after. Display shows "SYNC". Continue from step 7.



CE 0470
Subject to change.
Usserød Mølle
Håndværkersvinget 6
DK 2970 Hørsholm
+45 70 20 55 11
info@xo-care.com