

XO[®] 4

RETROFIT XO HD DISPLAY ON EXISTING SHORT ARM

YB-985
Version 1.10

XO CARE A/S
Usserød Mølle
Håndværkersvinget 6
DK 2970 Hørsholm
Denmark

+45 70 20 55 11
info@xo-care.com
www.xo-care.com



This instruction covers:
XO-618 Retrofit XO HD Display for existing single arm on XO 4.

WARNING:

When making changes and handling electronics in the XO 4 Unit – always use ESD antistatic wristband (UC-600).



HD cable set for XO HD Display.



XO® HD Display
Installation Guide

Disconnect the OP lamp cables in unit stand and remove carefully the OP lamp arm.
Disconnect existing screen cables in unit stand and remove complete screen arm.




Doc. no. YB-985
Version 1.10

Remove the existing cables in the arm.



Install the new cable set.

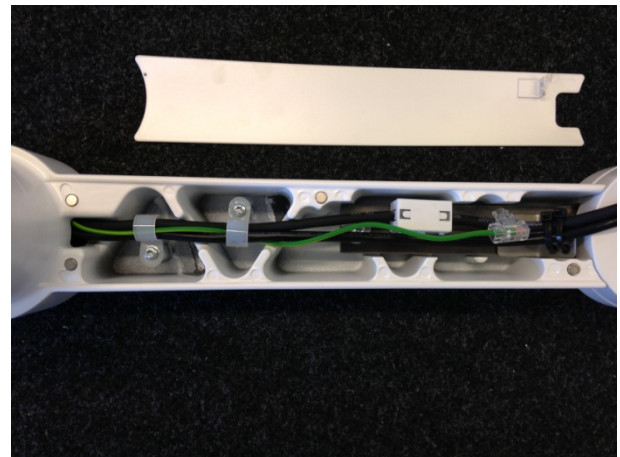

Install the new VESA bracket using 2 ea UG-147 bolts.


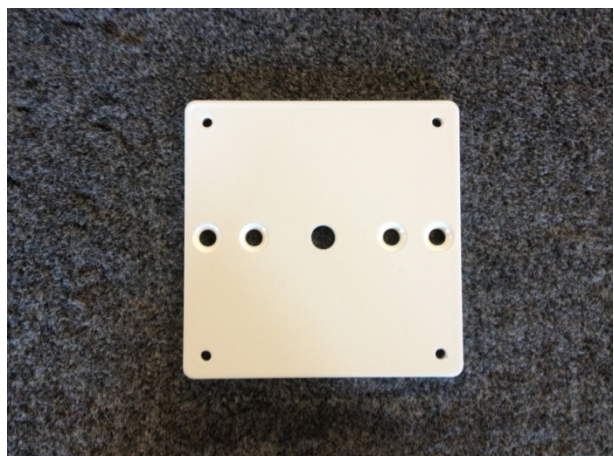




Fig. 41 (12-04)

Install the arm in accordance with instruction YB-990.

Install the display and connect the wiring in accordance with YB-990.

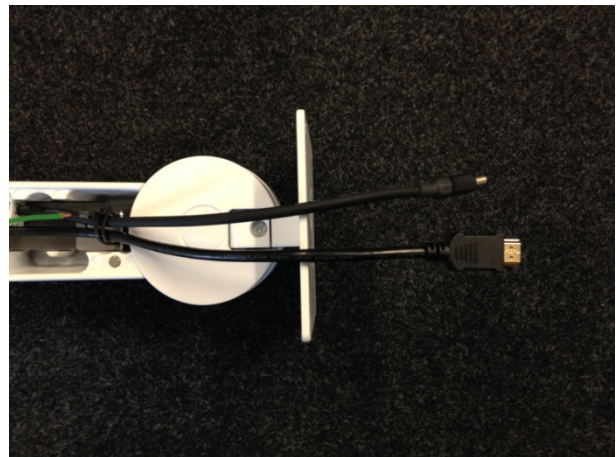

Adjust the balance if necessary.

