

XO[®] 4/ XO FLEX

RETROFIT OF QUICK CONNECTORS

YB-672
Version 1.10

XO CARE A/S
Usserød Mølle
Håndværkersvinget 6
DK 2970 Hørsholm
Denmark

+45 70 20 55 11
info@xo-care.com
www.xo-care.com



WARNING:

When making changes and handling electronics in the XO 4 Unit – always use ESD antistatic wristband (UC-600).




Tools:

Torx 20
Hose Cutter



Remove the 4 torx20 screws and remove the panel on the service hatch.

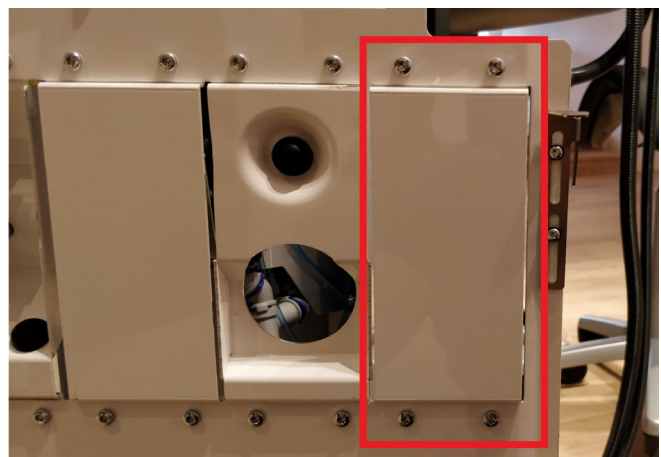


Fit the new panel with the quick connections on the service hatch.




Connect the supplied 4mm air and water hose to the quick connectors.

Top (Blue): Air

Bottom (White): Water

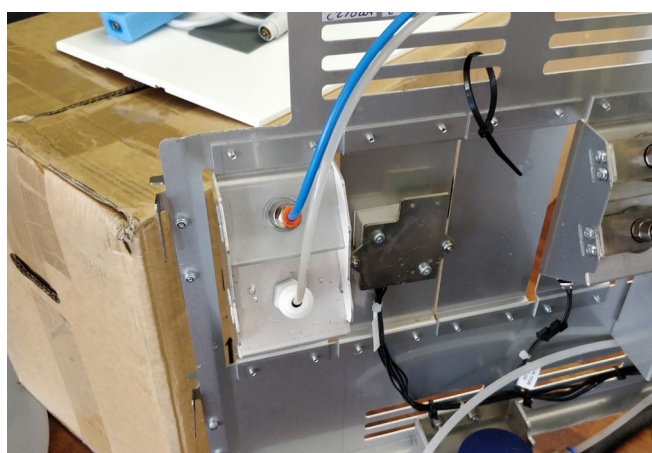

Route the 4mm water and air hose as illustrated.

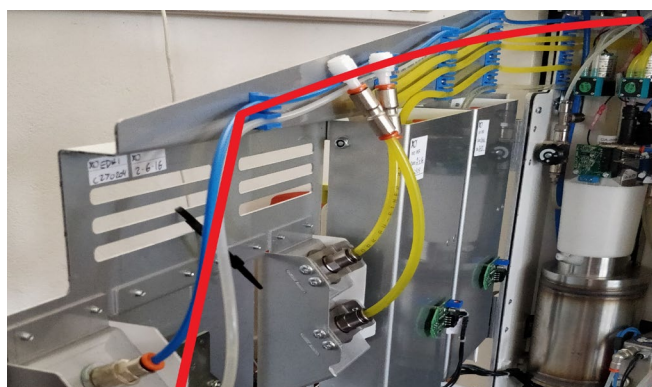


Route 4mm water and air hose as illustrated.




Cut the 4mm water hose to desired length, and connect the water hose to the T-connector on the water regulator.



Cut the 4mm air hose to desired length, and connect the air hose to the air regulator as illustrated.

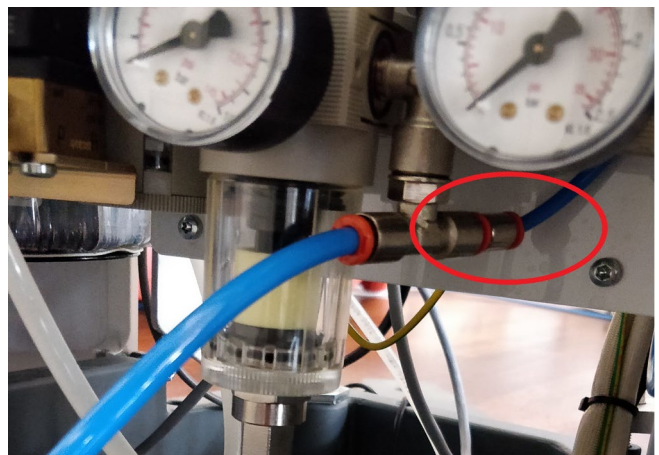

Be sure to remove the couplings before switching on the unit.

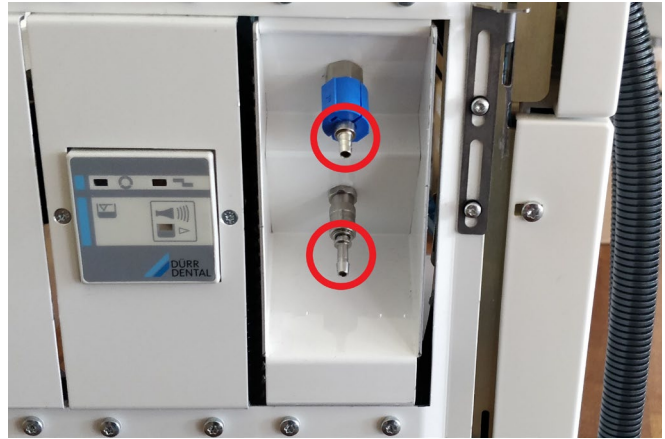

Switch on the unit and test the newly installed product. Check that there are no leaks and that the system functions properly.